


Instructions for Use and Technical Description

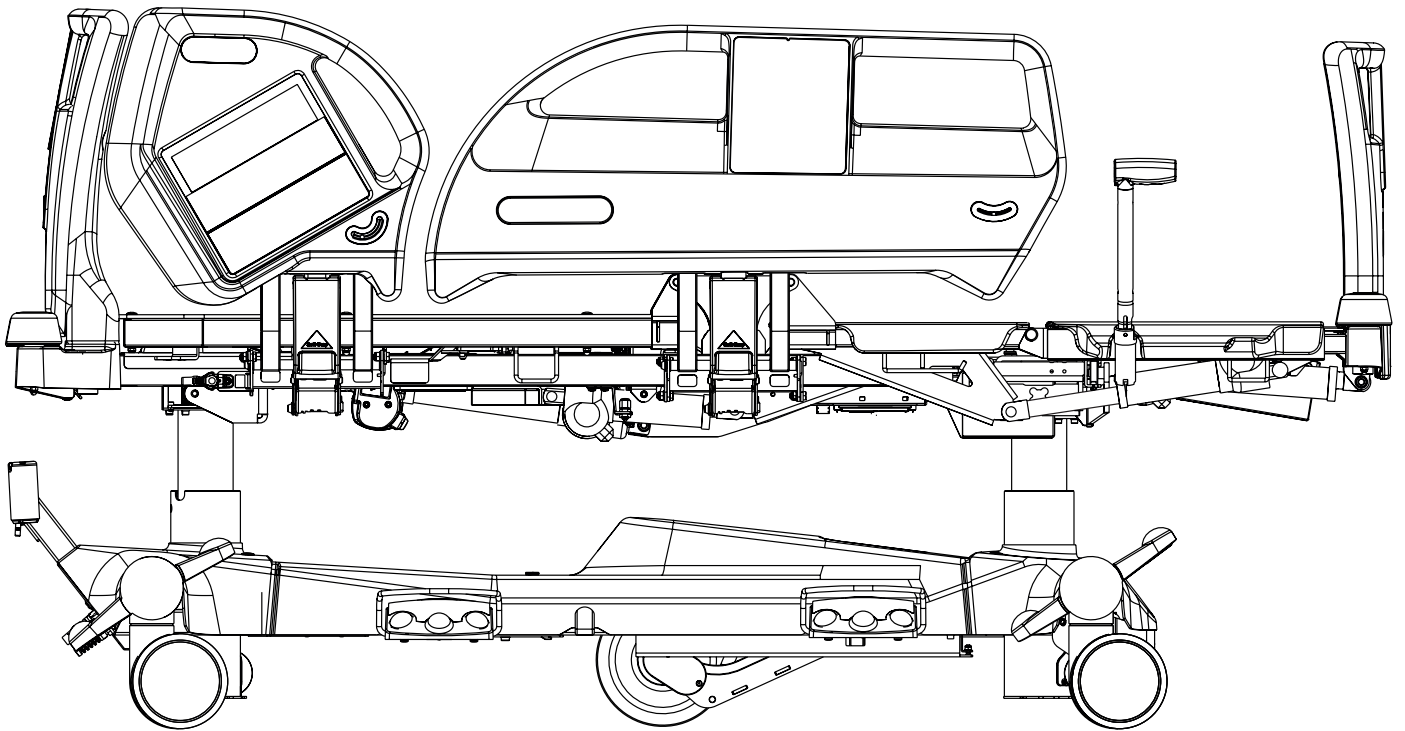

Eleganza 5

Positionable Bed for Intensive Care

version with scales and without scales



D9U001GE5-0101

Version: 20

Publication Date: 2025-12

Manufacturer:

L I N E T spol. s r.o.
Želevčice 5
274 01 Slaný

Tel.: +420 312 576 111
Fax: +420 312 522 668

E-mail: info@linet.cz
<http://www.linet.com>
Service department: service@linetgroup.com

Authorized Representative in Great Britain:

LINET UK Ltd
11 Brunel Way
Segensworth East
Fareham
Hampshire
PO15 5TX
United Kingdom

Authorized Representative in Switzerland:

Bigla Care AG
Bernstrasse 3
CH-3421 Lyssach
Switzerland

Authorized Representative in Malaysia:

Interscience Sdn. Bhd.
2, Jalan Sg. Kayu Ara 32/38
Berjaya Industrial Park
40460 Shah Alam
Selangor, Malaysia

Authorized Representative in Peru:

Drogería Emergo Peru S.R.L.
Calle Las Orquídeas Nro. 585, Int. 1301
San Isidro, Lima
Director Técnico Renato Delgado Rivera
RUC: 205 52 75 65 35
Peru

Authorized Representative in Philippines:

Biomedica Healthcare Inc.
Unit 2103 City land 10, Tower 1
Ayala Corner, Dela Costa Street
Makati City, Metro Manila
Philippines

Authorized Representative in Australia:

LINET Australia
1/30 Heathcote Road
Moorebank NSW 2170
Australia

Eleganza 5

Positionable bed for intensive care
version with scales and without scales

Author: L I N E T, s.r.o.
Related links: www.linnet.com

D9U001GE5-0101

Version: 20
Publication Date: 2025-12

Copyright © L I N E T, s.r.o., 2025
Translation © L I N E T, 2025
All rights reserved.

All trademarks and brands are the property of the appropriate owners. The manufacturer reserves the right to changes in the contents of the instructions for use that relate to the product's technical regulations. It is for this reason that the contents of the instructions for use may indicate differences from the current manufacture of the product. Reproduction, also excerpts, only with prior permission of the publisher. Subject to changes due to technical developments. All technical data are rated data and are subject to construction and manufacturing tolerances.

Table of Contents

1 Symbols and Definitions	8	11.4.5 Handset (optional)	67
1.1 Warning Notices	8	11.4.6 Patient Control Panels	68
1.1.1 Types of Warning Notices	8	11.4.7 Bed Height Foot Control (optional)	69
1.1.2 Structure of Warning Notices	8	11.4.8 Lateral Tilt Foot Control (standard)	69
1.2 Instructions	8	11.5 Bed Positioning	70
1.3 Lists	8	11.5.1 Backrest	70
1.4 Symbols on the Package	9	11.5.2 Thighrest	72
1.5 Symbols and Labels on the Bed	10	11.5.3 Calfrest	74
1.6 Acoustic signalisation	14	11.5.4 Bed Height	75
1.7 Audible Messages	15	11.5.5 Autocontour	77
1.8 Night Light	15	11.5.6 Emergency Trendelenburg Position	78
1.9 Red Light of the Bed Exit Alert	15	11.5.7 Antitrendelenburg and Trendelenburg Tilt	80
1.10 Visual signalisation	16	11.5.8 Examination Position	82
1.10.1 Mains Power LED (iBoard Standard, Attendant Control Panel)	16	11.5.9 Bed Extension	83
1.10.2 Accumulator indicator (iBoard Standard, Attendant Control Panel)	16	11.5.10 CPR Position	84
1.10.3 Lock LED (Attendant Control Panel)	16	11.5.11 Cardiac Chair Position	85
1.11 Definitions	17	11.5.12 Lateral Tilt	87
1.12 Abbreviations	18	11.5.13 Mobilisation Position	89
2 Safety Instructions	19	11.5.14 Low Flat Position	90
3 Intended use of Eleganza 5 (1GE5)	22	11.5.15 Ergoframe	91
3.1 User population	22	12 Scales Control (only version with scales)	92
3.2 Use environment	22	12.1 Preparation	92
3.3 Contraindications	22	12.2 Taring	93
3.4 Operator	22	12.3 Displaying	93
4 Product Description	23	12.4 Hold Mode	93
5 Technical Specification	24	12.5 Memory (only iBoard Standard)	93
5.1 Identification of Applied Parts (Type B)	24	12.6 Bed Overload	94
5.2 Scales (only version with scales)	24	12.7 Bed Underload	94
5.3 Mechanical Specifications (Eleganza 5)	24	12.8 Weighing in tilt	94
5.4 Environment conditions (Eleganza 5)	25	12.9 Zeroing Scales	95
5.5 Electrical Specifications (Eleganza 5)	25	13 Bed Exit Monitoring (only version with scales)	96
5.6 Electromagnetic Compatibility	26	13.1 Preparation	97
5.6.1 Manufacturer instructions - electromagnetic emissions	26	13.2 Activation of Bed Exit Monitoring	97
5.6.2 Manufacturer instructions - electromagnetic susceptibility	27	13.3 Monitored Zone	97
6 Use and Storage Conditions	28	13.4 BED EXIT ALERT	98
7 Scope of Delivery and Bed Variants	29	13.5 Alert Volume	98
7.1 Delivery	29	13.6 PAUSE	99
7.2 Scope of Delivery	29	13.7 Deactivation of Bed Exit Monitoring	99
7.3 Eleganza 5 Variants	29	14 Equipment	100
8 Putting into Service	30	14.1 Accessory Rail with plastic hooks (optional)	100
8.1 Accumulator Activation	30	14.2 Brake Signal (optional)	100
8.2 Head Board and Foot Board	31	14.3 i-Brake® (optional)	100
8.3 Mattress Support Platform	32	14.4 Retractable Fifth Castor (optional)	100
8.4 Potential Equalisation	32	14.5 LINIS SafetyPort (optional)	100
8.5 Before Use	33	14.6 i-Drive Power (optional)	101
8.6 Transport	33	14.6.1 i-Drive Power Activation/Deactivation	103
8.7 Firmware	33	14.6.2 Powered Drive	103
9 Power Supply Cord (Mains Power Cable)	33	14.6.3 Braking	104
10 Accumulator	34	14.6.4 Free Drive	104
10.1 Replacing the accumulator	35	14.7 Mobi-Lift® (optional)	106
10.2 Removing the Bed from Use	36	14.8 Safestop (optional)	106
10.3 Deactivating the Accumulator	36	14.9 X-Ray Lung Examination (optional)	107
11 Manipulation	37	14.10 Nurse Call	109
11.1 Siderails	37	14.11 USB Connector	110
11.2 Castor Control	43	15 Mattress	111
11.3 CPR Backrest Release	44	15.1 Passive Mattress	111
11.4 Control Elements	45	15.1.1 Straps with side release buckles	111
11.4.1 iBoard Standard	46	15.2 Active Mattress (not integrated)	113
11.4.2 Nurse Control Panel (optional)	56	15.3 OptiCare (integrated mattress)	114
11.4.3 iBoard Basic (optional)	60	15.3.1 Mattress Screen	114
11.4.4 Attendant Control Panel	64	16 Accessories	115
		16.1 Lifting Pole	117
		16.2 Infusion Stand	118
		16.3 Oxygen Bottle Holder	119
		16.4 Ventilation Circuit Holder	121
		16.5 Writing Shelf	121
		16.6 Monitor Shelf	122
		16.7 Protector	122

16.8 Urinary Bag Holder	124
16.9 Traction Frame	124
16.10 Safety Monitor	126
16.11 Pushing Handles	129
16.12 Pushing Handles with Holder of the i-Drive Power Control Panel	131
16.13 IDock (size S and size M)	133
17 Cleaning/Disinfection	135
17.1 Cleaning (Eleganza 5)	136
17.1.1 Daily Cleaning	136
17.1.2 Cleaning before Changing Patients	136
17.1.3 Complete Cleaning and Disinfection	137
18 Troubleshooting	138
19 Maintenance	139
19.1 Regular maintenance	139
19.2 Spare Parts	139
19.3 Electrical Safety Check and Preventive Maintenance	139
19.3.1 Electrical Safety Check	139
19.3.2 Preventive Maintenance	140
20 Disposal	141
20.1 Environment Protection	141
20.2 Disposal	141
20.2.1 Within Europe	141
20.2.2 Outside Europe	141
21 Warranty	142
22 Standards and Regulations	142

List of Figures

Fig. Bed Overview (Eleganza 5)	23
Fig. Isolating Foil Placement	30
Fig. Foot Board Lock	31
Fig. Detail of Isolating Foil	31
Fig. Potential equalisation connector - female	32
Fig. 4-part Mattress support platform	32
Fig. Potential equalisation - male	32
Fig. Deactivation of the accumulator (Attendant Control Panel).....	36
Fig. Siderail version with iBoard Standard.....	38
Fig. Siderail version with iBoard Basic.....	39
Fig. Siderail version with Nurse Control Panel and iBoard Basic.....	40
Fig. Siderail version with Nurse Control Panel	41
Fig. Siderail version without scales	42
Fig. Positions of Castor Control Lever	43
Fig. CPR lever.....	44
Fig. iBoard Standard without Mattress Keyboard	46
Fig. iBoard Standard with Mattress Keyboard	46
Fig. Display and Keyboard of the Settings Section	47
Fig. Display and Keyboard of the Positioning Section	51
Fig. Bed statuses on the iBoard Standard display	53
Fig. Bed icon with bed statuses (orange - alerts/insecure statuses).....	53
Fig. Nurse Control Panel.....	56
Fig. iBoard Basic.....	60
Fig. Attendant Control Panel.....	64
Fig. Handset	67
Fig. Patient Control Panel in the foot siderail	68
Fig. Patient Control Panel in the head siderails (optional)	68
Fig. Bed Height Foot Control	69
Fig. Lateral Tilt Foot Control	69
Fig. Backrest Angle on iBoard Standard Display	70
Fig. Backrest Adjustment Button (iBoard Standard, Patient Control Panels)	71
Fig. Backrest Adjustment Buttons (Attendant Control Panel).....	71
Fig. Backrest Adjustment Buttons (Handset).....	71
Fig. Thighrest Adjustment Button (iBoard Standard, Patient Control Panels)	72
Fig. Thighrest Adjustment Buttons (Attendant Control Panel)	73
Fig. Thighrest Adjustment Buttons (Handset)	73
Fig. Calfrest Adjustment Button (iBoard Standard).....	74
Fig. Calfrest Adjustment Buttons (Attendant Control Panel)	74
Fig. Bed Height Adjustment Button (iBoard Standard)	75
Fig. Bed Height Pedals (Bed Height Foot Control)	76
Fig. Bed Height Adjustment Buttons (Attendant Control Panel).....	76
Fig. Bed Height Adjustment Button (Handset).....	76
Fig. Mobi-Lift Bed Height Button	76
Fig. Autocontour Adjustment Button (iBoard Standard, Patient Control Panels)	77
Fig. Autocontour Adjustment Buttons (Handset)	78
Fig. Trendelenburg Position	78
Fig. Emergency Trendelenburg Position Button (Nurse Control Panel).....	79
Fig. Emergency Trendelenburg Position Button (Attendant Control Panel).....	79
Fig. Antitrendelenburg Position	80
Fig. Trendelenburg Tilt Button (iBoard Standard)	80
Fig. Antitrendelenburg Tilt Button (iBoard Standard)	80
Fig. Tilt Angle on iBoard Standard Display	80
Fig. Trendelenburg Tilt Button (Nurse Control Panel).....	81
Fig. Anti-Trendelenburg Tilt Button (Nurse Control Panel).....	81
Fig. Longitudinal Tilt Buttons (Attendant Control Panel).....	81
Fig. Examination Position Pedal (Bed Height Foot Control).....	82
Fig. Bed Extension Adjustment Buttons (Attendant Control Panel).....	83
Fig. CPR Button (iBoard Standard, Attendant Control Panel).....	84
Fig. CPR Position Button (Nurse Control Panel)	84
Fig. Cardiac Chair Position Button (iBoard Standard)	85
Fig. Cardiac Chair Position Button (Nurse Control Panel).....	86
Fig. Cardiac Chair Position Button (Attendant Control Panel).....	86
Fig. Lateral Tilt Buttons (iBoard Standard).....	87
Fig. Lateral Tilt Angle on iBoard Standard Display	87
Fig. Lateral Tilt Pedals (Lateral Tilt Foot Control).....	88

Fig. Lateral Tilt Buttons (Attendant Control Panel)	88
Fig. Mobilisation Position Button (iBoard Standard).....	89
Fig. Low Flat Position Pedal on the Lateral Tilt Foot Control.....	90
Fig. Mobilisation Position Button (Nurse Control Panel).....	90
Fig. Scales Section (iBoard Standard) - Display and Keyboard	92
Fig. Scales Section (iBoard Basic) - display and keyboard	92
Fig. HOLD Mode (iBoard Basic).....	93
Fig. Five steps of the countdown	94
Fig. High Load (iBoard Basic).....	94
Fig. Bed Underload (iBoard Basic).....	94
Fig. Bed Exit Monitoring Section (iBoard Standard) - Display and Keyboard	96
Fig. Bed Exit Monitoring Section (iBoard Basic) - display and keyboard	97
Fig. Volume Icon.....	98
Fig. Visual signalisation of Bed Exit Alert (iBoard Basic).....	98
Fig. PAUSE Countdown (with remaining minutes)	99
Fig. Bed Exit OFF (iBoard Basic).....	99
Fig. Activation Panel (4)	102
Fig. Main control panel (2)	102
Fig. Position of Main Control Panel.....	102
Fig. i-Drive Power Mains Switch with Label	103
Fig. i-Drive Power Emergency Retraction Button Label	103
Fig. Accumulator Charge Status	104
Fig. Mobi-Lift Handle.....	106
Fig. X-Ray Cassette Holder	108
Fig. Nurse Call Control Element (inner side of head siderail) - version 1	109
Fig. Nurse Call Control Element (inner side of head siderail) - version 2	109
Fig. Nurse Call Control Element (outer side of head siderail).....	109
Fig. USB Connector on the right side of Backrest	110
Fig. Bottom of Passive Mattress	111
Fig. Fixation of the passive mattress with straps on the mattress support platform of Eleganza 5 bed	112
Fig. Bottom of Active Mattress (not integrated).....	113
Fig. Mattress Display and Keyboard (iBoard Standard).....	114
Fig. Accessories.....	116
Fig. Places for lifting pole (sleeve fittings on accessory adapter).....	117
Fig. Lifting pole	117
Fig. Places for infusion stand (sleeve fittings on accessory adapter)	118
Fig. Infusion stand	118
Fig. Oxygen Bottle Holder (on the head end).....	119
Fig. Oxygen Bottle Holder in the adapter (position 1).....	120
Fig. Oxygen Bottle Holder in the adapter (position 3).....	120
Fig. Oxygen Bottle Holder in the adapter (position 2).....	120
Fig. Oxygen Bottle Holder in the adapter (position 4).....	120
Fig. Writing Shelf.....	121
Fig. Ventilation Circuit Holder.....	121
Fig. Monitor Shelf.....	122
Fig. Placement of Protector	123
Fig. Fixing element on the telescopic profile of the bed extension	123
Fig. Eleganza 5 with Traction Frame (side view).....	124
Fig. Placement of the Urinary Bag Holder	124
Fig. Safe Working Load of the hooks (Infusion Holder).....	124
Fig. Safe Working Load of the pulley	124
Fig. Positions to insert Traction Frame (undercarriage of Eleganza 5)	125
Fig. Eleganza 5 with Traction Frame (top view)	125
Fig. Bed Exit Section (iBoard Standard) - Display and Keyboard	127
Fig. Bed icon with bed statuses (orange - alerts/insecure statuses).....	127
Fig. Bed icon with bed statuses (orange - alerts/insecure statuses).....	128
Fig. Bed Exit Section (iBoard Basic) - Display and Keyboard.....	128
Fig. Placement of pushing handles on the head end crossbar (positions of nuts).....	129
Fig. Distance between bushing holder and the bar that is at right angle to the head end crossbar	129
Fig. Pushing Handles on the Eleganza 5 bed.....	130
Fig. Placement of pushing handles on the head end crossbar (positions of nuts).....	131
Fig. Distance between bushing holder and the bar that is at right angle to the head end crossbar	131
Fig. Pushing Handles with Holder of the i-Drive Power Control Panel on the Eleganza 5 bed.....	132
Fig. IDock docking-cart	133
Fig. Installation of the Transport Bracket on the bed foot end crossbar	134

1 Symbols and Definitions

1.1 Warning Notices

1.1.1 Types of Warning Notices

Warning notices are differentiated by the type of danger using the following key words:

- ▶ **CAUTION** warns about the risk of material damage.
- ▶ **WARNING** warns about the risk of physical injury.
- ▶ **DANGER** warns about the risk of fatal injury.

1.1.2 Structure of Warning Notices



SIGNAL WORDS!

Type and source of danger!

- ▶ Measures to avoid the danger.

1.2 Instructions

Structure of instructions:



- ▶ Perform this step.
- Results, if necessary.

1.3 Lists

Structure of bulleted lists:



- List level 1
 - List level 2
 - List level 3




1.4 Symbols on the Package

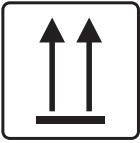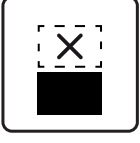	FRAGILE, HANDLE WITH CARE
	THIS WAY UP
	KEEP DRY (PROTECT FROM HUMIDITY)
	PAPER RECYCLING SYMBOL
	DO NOT STACK DURING STORAGE
	DO NOT USE HAND TRUCK HERE
	OVERSEAS PACKAGE: STACKING LIMIT BY NUMBER (3 PACKAGES FOR TRANSPORT)
	OVERSEAS PACKAGE: STACKING LIMIT BY NUMBER (5 PACKAGES FOR STORAGE)

1.5 Symbols and Labels on the Bed

	READ INSTRUCTIONS FOR USE
	GO BUTTON (PRESS TO ACTIVATE CONTROL ELEMENT)
	STOP BUTTON (PRESS TO INTERRUPT BED POSITIONING)
	MANUFACTURER
	MANUFACTURING DATE
	REFERENCE NUMBER (PRODUCT TYPE DEPENDING ON CONFIGURATION)
	UK CONFORMITY ASSESSED (UKCA) MARKING (ONLY Eleganza 5 WITHOUT SCALES NORMATIVELY HARMONIZED FOR GREAT BRITAIN ECONOMIC AREA)
	AUTHORIZED REPRESENTATIVE IN GREAT BRITAIN
	AUTHORIZED REPRESENTATIVE IN SWITZERLAND

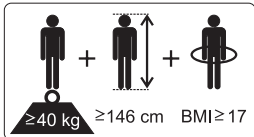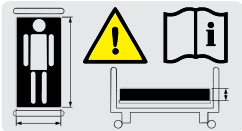 250 kg	SAFE WORKING LOAD
	PHYSICAL DESCRIPTION OF AN ADULT (DESIGNATION OF MEDICAL BED FOR ADULTS)
	USE MATTRESS RECOMMENDED BY MANUFACTURER
	DO NOT PUT ANY OBJECTS ON UNDERCARRIAGE
	CALFREST LOAD LIMIT
	WARNING AGAINST CRUSHING OR TRAPPING OF HANDS
	JACK FOR ATTACHMENT OF CONDUCTOR FOR POTENTIAL EQUALISATION
	CPR LEVER
	GENERAL WARNING SIGN
	CAUTION
	TYPE B APPLIED PARTS
	THERMAL PROTECTION FOR TRANSFORMER
	SAFETY ISOLATING TRANSFORMER (GENERAL)

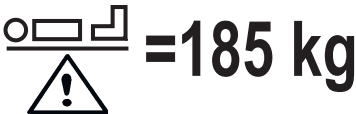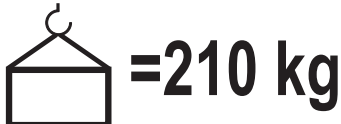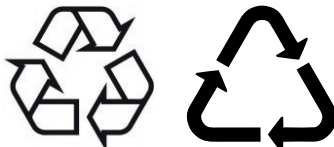	FOR INDOOR USE ONLY
	MAXIMUM PATIENT WEIGHT
	WEIGHT OF BED
	CE MARKING FOR Eleganza 5 WITH SCALES
	CE MARKING FOR Eleganza 5 WITHOUT SCALES AND FOR OPTICARE
	WEEE SYMBOL (RECYCLE AS ELECTRONIC WASTE, DO NOT PUT INTO THE HOUSEHOLD WASTE)
	RECYCLING SYMBOL
	DO NOT POLLUTE THE ENVIRONMENT
	DO NOT OPEN
	SERIAL NUMBER
	BATCH NUMBER (ACCESSORIES)

 Li-ion	THIS COMPONENT CONTAINS LITHIUM - DO NOT PUT INTO THE HOUSEHOLD WASTE (If Eleganza 5 is equipped with Integration Module (SafetyMonitor System), it contains lithium battery.)
	MEDICAL DEVICE (compatible with Medical Device Regulation)
	MASS OF MOBILE HOSPITAL BED (MASS OF EMPTY BED + SAFE WORKING LOAD)

1.6 Acoustic signalisation

SOUND	MEANING
CONTINUOUS SOUND	overheating
	accumulator overcurrent
	scales overload (only version with scales)
	actuator overload
	Safestop notification
BEEP lasting 0,3s + CONTINUOUS SOUND	Siderail Signal (lateral tilt + head siderail or foot siderail down)
REPEATED BEEP: 0,6s sound / 2,6s silence	STOP error (all STOP buttons are disabled)
MELODY: 3 beeps, pause, 2 beeps, longer pause, 3 beeps, pause, 2 beeps	Bed Exit Alert (only version with scales, the Bed Exit Alert melody is accompanied by spoken word and red light signalling)
BEEP lasting 0,3s	Siderail Signal (head siderail or foot siderail down + bed positioning)
	lateral tilt 15° achieved
	transition from tilt (lateral tilt, Trendelenburg, Antitrendelenburg) to horizontal position
ECHO TONE lasting 0,3s	confirmation
DESCENDING BEEP lasting 0,3s	blocked function
BEEP lasting 0,5s	start of service mode or end of service mode
	keyboard error (positioning blocked)
BEEP lasting 3s	system error
BEEP lasting 5s	SCU disconnected (only if integrated mattress is used)
	scale module disconnected (only version with scales)
REPEATED BEEP during 3 minutes: 1,1s sound / 1,1s silence	Brake Signal (signalling the unbraked bed in version with Brake Signal only)

1.7 Audible Messages

Eleganza 5 bed is equipped with four audio notifications designed to provide instructions to caregivers or patients. These notifications are available in 16 languages.

AUDIBLE MESSAGES	MEANING
Please remain in the bed, assistance is coming.	Announcement of the Bed Exit Alert related to the Inner Zone Bed Exit Monitoring or to the Outer Zone Bed Exit Monitoring.
Bed Exit Alert ON.	Bed Exit Monitoring is activated.
Bed Exit Alert OFF.	Bed Exit Monitoring is deactivated.
The bed is not connected to power.	Bed is disconnected from the mains.
	The notification sounds anytime a positioning button was pressed on the accumulator powered bed.

1.8 Night Light

Bed illumination helps the nursing staff as well as the patient to orientate. The lowered intensity of lightning is set up after turning the bed on. The night light is turned off during accumulator operation.

The bed is equipped with three-phase illumination:

1. Lowered intensity of illumination
2. Full intensity of illumination
3. Illumination is turned off

After pressing any button:

- The bed illumination will light up at full intensity for 10 minutes. After 10 minutes the bed illumination will be lowered.
- After disconnection of the bed from the mains illumination lights up for few seconds.

Turning off bed illumination:

- Disconnect bed from the mains.
- After disconnection of the bed from the mains illumination lights up for few seconds.

1.9 Red Light of the Bed Exit Alert

Red LED indicators of the bed indicate the Bed Exit Alert is triggered.

- 4 sets of 2 red LED indicators are situated as follows:
- 2 sets on the 2 undercarriage sides,
 - 2 sets at the foot end corners of the bed.

When Bed Exit Alert is triggered, these red LED indicators pulse. The red LED indicators go out when the Bed Exit Alert is canceled.

1.10 Visual signalisation

1.10.1 Mains Power LED (iBoard Standard, Attendant Control Panel)

MAINS POWER LED 	MEANING
lit	connected to the mains
flashing: 0,6s lit / 0,6s not lit	keyboard error (flashing inverted to Lock LED) error (first fault)
flashing: 0,1s lit / 0,1s not lit	service mode
not lit	disconnected from the mains power transformer switching error

1.10.2 Accumulator indicator (iBoard Standard, Attendant Control Panel)

ACCUMULATOR INDICATOR  	MEANING
lit	accumulator disconnected or faulty
flashing: 1,6s lit / 0,2s not lit	accumulator deeply discharged
flashing: 0,1s lit / 0,1s not lit	accumulator discharged
flashing: 0,2s lit / 1,6s not lit	accumulator is charging
not lit	accumulator charged

1.10.3 Lock LED (Attendant Control Panel)

 LOCK LED VISUAL SIGNALISATION	lit	flashing: 0,6s lit / 0,6s not lit			not lit
Thighrest, Calfrest and Bed Extension Lock LED	locked	lock error	keyboard error	motion blocked	unlocked
Backrest Lock LED	locked	lock error	keyboard error	motion blocked	unlocked
Bed Height, Lateral Tilt, Trendelenburg and Antitrendelenburg Tilt Lock LED	locked	lock error	keyboard error	motion blocked	unlocked
Foot Control Lock LED	locked	lock error	keyboard error	motion blocked	unlocked

1.11 Definitions

Basic Bed Configuration	The pricelist model configuration, not including a mattress.
Bed Weight	The value depends on the product configuration, accessories or customer adjustments.
Clearance of Undercarriage	The height from the floor to the lowest point of the undercarriage between the castors, for the manipulation of accessories under a braked bed in the standard position.
Duty Cycle	Cycle of operation of the motor: time of activity/time of rest.
Ergoframe	Ergoframe is the kinematic system of Mattress support platform Adjustment whose effect is the elimination of pressure on the patient's abdomen and pelvic area and frictional forces on the patient's back and legs.
Maximum Patient Weight	Maximum Patient Weight depends on the application environment according to IEC 60601-2-52. For application environment 1 (intensive/critical care) and 2 (acute care) reduce Safe working Load by 65 kg. For application environment 3 (long-term care) and 5 (ambulatory care) reduce Safe working Load by 35 kg.
Safe Working Load	The highest allowable load on the bed (patient, mattress, accessories and the load supported by those accessories).
Siderail Height	The height of the upper crossbar or the edges of the siderails (not the highest point of the siderail controls) from the patient surface.
Standard Bed Position	- The height of the patient surface with regard to the floor is 400 mm. - The mattress support platform, including the individual parts, has to be in a horizontal position (0°). - The siderails are always locked in the upper position. - The basic position of the integrated extension.
Adult	Patient having a physical size equal to or more than 146 cm, a mass equal to or more than 40 kg and a body mass index (BMI) equal to or more than 17 (according to IEC 60601-2-52).
Mass of mobile hospital bed	Sum of empty bed mass and Safe Working Load.

1.12 Abbreviations

AC (~)	Alternating Current
CE	European Conformity
CPR	Cardiopulmonary Resuscitation
dB	Sound Intensity Unit
DC (—)	Direct Current
CUC	Configuration number
EMC	Electromagnetic Compatibility
FET	Field-effect transistor
HF	High Frequency
HPL	High Pressure Laminate
ICU	Intensive Care Unit
INT.	Duty Cycle
IP	Ingress Protection
IV	Intravenous
LED	Light Emitting Diodes
ME	Medical Electrical (Equipment)
ON	Activation
OFF	Deactivation
ppm	parts per million, millionth (1000 ppm = 0,1%)
REF	Reference Number (product type depending on configuration)
SCU	System Control Unit (integrated mattress)
SN	Serial Number
SWL	Safe Working Load
UDI	Unique Device Identification (for medical devices)
USB	Universal Serial Bus
WEEE	Waste Electrical and Electronic Equipment

2 Safety Instructions



WARNING!

Eleganza 5 bed should be left in its lowest position when the patient is unattended in order to reduce risk of injury due to falls!



WARNING!

Siderails of Eleganza 5 should be located in the „up“ position to reduce the risk of the patient accidentally slipping or rolling off the mattress!



WARNING!

Incompatible siderails and mattresses can cause an entrapment hazard!



WARNING!

Inappropriate handling of the power supply cord, e. g. by kinking, shearing or other mechanical damages is hazardous!



WARNING!

When routing cables from other equipment in the Eleganza 5 bed avoid squeezing those between parts of the Eleganza 5 bed!



WARNING!

Eleganza 5 bed should not be used with bed hoists and bed lifts!



WARNING!

The bed is intended for adults.
► Follow the chapter **Intended use**.



WARNING!

Incompatible mattresses can create hazards.



WARNING!

To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.



WARNING!

No modification of this equipment is allowed.



WARNING!

Do not modify this equipment without authorization of the manufacturer.



WARNING!

If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of the equipment.



WARNING!

An additional multiple socket-outlet or extension cord shall not be connected to the medical electrical system.



WARNING!
During specific investigations or treatments the significant risks of reciprocal interference posed by ME equipment may occur.



WARNING!
Staff expert assessment is needed to consider all individual cases of contraindications!



WARNING!
Certain bed positions are not suitable for specific diagnosis/medical conditions. Fowler position is not suitable for spinal cord injuries! Trendelenburg position is not suitable for patients with higher intracranial pressure!



WARNING!
Length adjustment of the bed must be proportional to the height of patient!
Risk of trapping or squeezing because of patient's body constitution disproportionate to the size of mattress support platform!



WARNING!
Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established!



WARNING!
Only authorised and trained person using the tool is allowed to change fuses and power supplies!



WARNING!
This medical device is not intended for oxygen enriched environment!



WARNING!
This medical device is not intended for use with flammable substances!



WARNING!
This medical device is not portable medical electrical equipment!



WARNING!
Make sure the duty cycle (2 min ON/18 min OFF) is not exceeded during bed positioning!



WARNING!
Patient is allowed to use selected control elements only if hospital personnel had assessed that the patient's physical and psychological state is in accordance with use of them and only if the hospital personnel had trained the patient in accordance with the instructions for use!



WARNING!
Hospital personnel is allowed to use the weighing system (scales) for weighing patients only if they had been trained according to the instructions for use!



WARNING!
No part of the Eleganza 5 ME equipment shall be serviced or maintained while in use with a patient!



WARNING!
Patient is allowed to use selected control elements of the ME equipment unless servicing and maintenance of the ME equipment is performed!


WARNING!
Patient must not perform any maintenance of the Eleganza 5 equipment!

- ▶ Follow the instructions carefully.
- ▶ Use the bed exclusively if it is in perfect working order.
- ▶ If necessary, check the bed functions daily or at each shift change.
- ▶ Ensure any user has read and understood the instructions for use completely before operating the product.
- ▶ Use the bed exclusively with the correct mains supply.
- ▶ Ensure that the bed is operated exclusively by qualified personnel who have been trained according to the instructions for use.
- ▶ Ensure that the patient (health permitting) has been informed about the operation of the bed and all applicable safety instructions.
- ▶ Move the bed exclusively on even, hard-surfaced floors.
- ▶ Contact service department of the manufacturer immediately to let it replace any damaged parts by the original spare parts.
- ▶ Ensure the bed is operated by qualified personnel trained in using the bed in accordance with the instructions for use by manufacturer or by person approved by the manufacturer, eventually by authorized representative or by person approved by the authorized representative.
- ▶ Ensure that maintenance and installation are performed exclusively by qualified personnel who have been trained by the manufacturer.
- ▶ During peak loads or unavoidable excess loads (CPR), adjust Mattress support platform to the lowest position.
- ▶ Ensure that only one adult patient lies on the bed at any time.
- ▶ To avoid injury or crushing, take extra caution when operating any moving parts of the bed.
- ▶ When using lifting poles or infusion stands, ensure that nothing will be damaged when you move or adjust the bed.
- ▶ Brake the castors when the bed is occupied.
- ▶ Keep the Mattress support platform in the lowest position at any time when the healthcare personnel are not treating the patient in order to prevent the patient from falling or injuries.
- ▶ Ensure that siderails are operated exclusively by healthcare personnel.
- ▶ Never use the bed in areas where there is a hazard of explosion.
- ▶ Enable or disable functions on Patient Control Panels using the Attendant Control Panel as appropriate for the patient's physical and mental state. Verify that the function is actually disabled.
- ▶ Never handle the mains plug with wet hands.
- ▶ Disconnect the product from the mains exclusively by pulling the mains plug.
- ▶ When pulling the mains plug, always hold the plug, not the cable.
- ▶ Position the mains cable so that there are no loops or kinks in the cable; protect the cable from mechanical wear and tear.
- ▶ Improper handling of mains cable can cause an electric shock hazard, other serious injuries or damage to the mattress replacement system.
- ▶ Ensure that the stipulated duty cycle of motor is not exceeded.
- ▶ To change fuses or cables contact service organisation authorized by manufacturer.
- ▶ To prevent failures, use exclusively the manufacturer's original accessories and mattresses.
- ▶ Ensure that the stipulated safe working load is not exceeded.
- ▶ If the patient's condition could lead to an entrapment, leave the mattress support platform in the flat position whilst unattended.
- ▶ Adjust bed height when transporting the bed in order to facilitate overcoming possible obstacles.
- ▶ Do not exceed maximum load of 80 kg (176.37 lbs) for mattress support platform extension.
- ▶ Do not exceed the maximum patient weight limit (see Mechanical Specifications).
- ▶ Do not hang anything on any cable.
- ▶ Never use the mattress replacement system near radiators or other heat sources.
- ▶ Select a suitable location for the placement of bed accessories and other objects to prevent involuntary activation of buttons or controls which may result in the adjustment of bed positioning.
- ▶ Do not use the bed when its parts have been removed (e.g. parts of mattress support platform) unless these parts are designed to be removed.
- ▶ Never place any accessories or handset on the siderails where keyboards are located.
- ▶ After each emergency situation always check if any of the controllers (controllers in siderails, handset or ACP) is not involuntarily pressed by the bed accessories or by the mattress.
- ▶ The weighing system must be calibrated at regular intervals and in accordance with the metrological regulations of the relevant country. All testing and certification must be carried out by qualified personnel. The healthcare provider is responsible for ensuring the required testing frequency and testing procedure of the weighing system is carried out.
- ▶ To avoid unintended activation of moving parts during any use of the bed always check that none of the control elements of the bed is pressed by persons, mattress or other objects.

3 Intended use of Eleganza 5 (1GE5)

The intended use is the hospitalization of the patient in the intensive and acute care units, which includes above all the following aspects:

- ▶ Adjustment of the specific positions needed for the preventive reasons, routine nursing, treatments, mobilization, physiotherapy, examinations, sleeping, and relaxation. These positions are further specified and described in the clinical evaluation of this device, together with their potential clinical outcomes and benefits.
- ▶ Providing the safe environment for the patient during all relevant procedures. The particular requirements on patient safety are the subject of the clinical evaluation, including evaluation of the risk/benefit ratio. The relevant safety issues are the part of the risk management file.
- ▶ Patient in-bed indoor transport out of the patient room.
- ▶ Providing the suitable working conditions for the caregivers to perform the routine and specific tasks during the patient hospitalization.
- ▶ Indicative measurement of the patient weight, used as supportive feature without direct diagnostic effect. It helps staff to assess the general patient status and apply the nutrition and medicaments (valid for the version of the beds with in-bed scales).

3.1 User population

- ▶ Adult patients (weight ≥ 40 kg, height ≥ 146 cm, BMI ≥ 17) in the intensive and acute care units (Application Environment 1 and 2 as in IEC 60601-2-52)
- ▶ Caregivers (nurses, doctors, technical personnel, transport personnel, cleaning personnel)

3.2 Use environment

In Application Environment 1 as in IEC 60601-2-52:

- ▶ intensive/critical care provided in a hospital where 24 h medical supervision and constant monitoring is required and provision of life support system/equipment used in medical procedures is essential to maintain or improve the vital functions of the PATIENT

In Application Environment 2 as in IEC 60601-2-52:

- ▶ acute care provided in a hospital or other medical facility where medical supervision and monitoring is required and bed used in medical procedures is often provided to help maintain or improve the condition of the PATIENT

3.3 Contraindications

- ▶ The medical device is not intended for the pediatric patients use.
- ▶ Certain positions are not suitable for specific diagnoses/medical conditions (e.g. spinal cord injuries vs. Fowler position, higher ICP patients vs. Trendelenburg). Staff expert assessment / nursing consideration is needed in all individual case of contraindication.

3.4 Operator

- ▶ Caregiver
- ▶ Patient (based on individual patient status assessment by caregiver the patient can utilize dedicated device functions)

4 Product Description

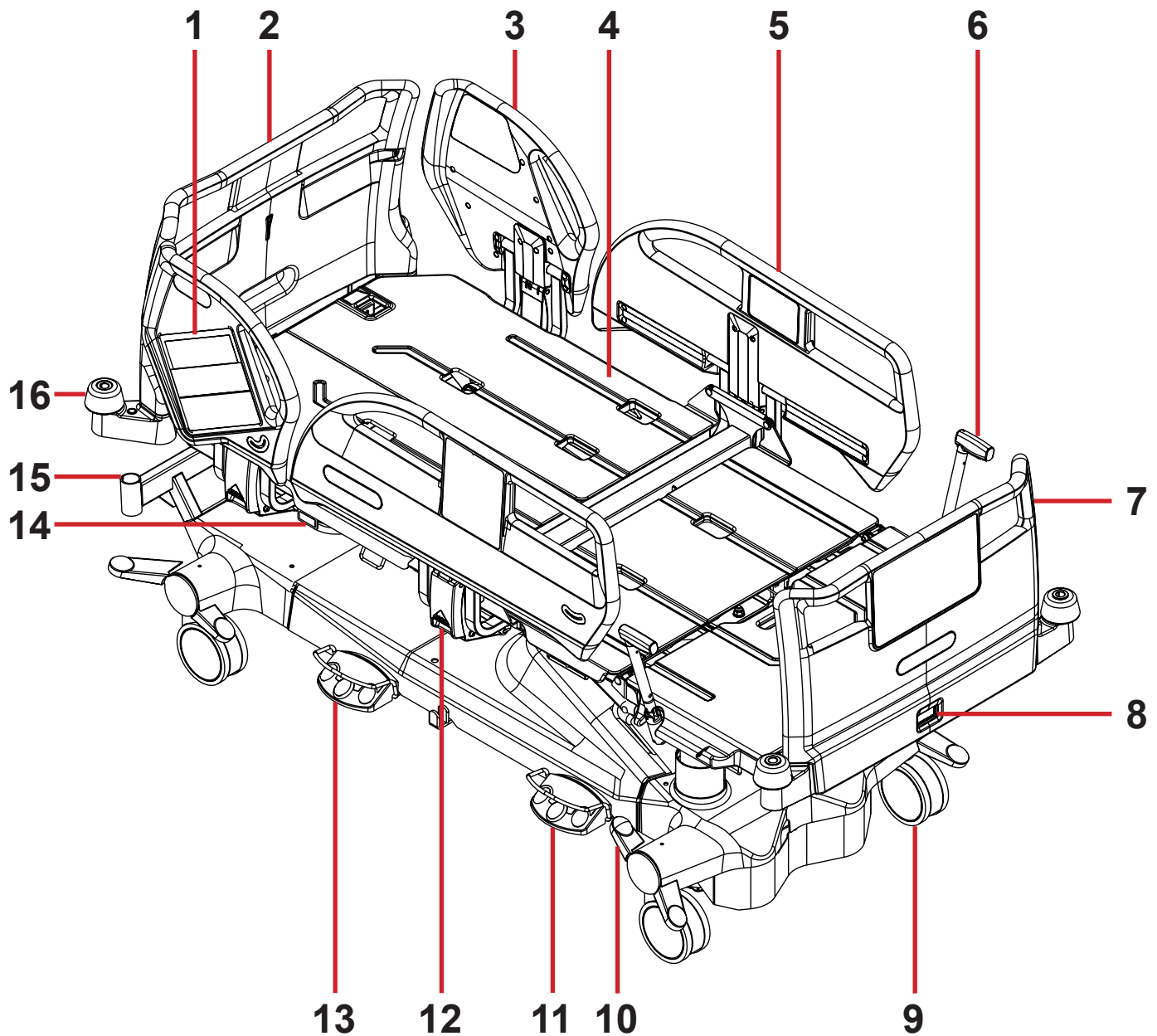


Fig. Bed Overview (Eleganza 5)

1. iBoard Standard (or optionally iBoard Basic)
2. Removable Head Board
3. Split Siderail – Head Siderail with Patient Control Panel
4. Four-part Mattress support platform with Ergoframe® System (under the mattress)
5. Split Siderail – Foot Siderail with Patient Control Panel
6. Mobi-Lift® Handle
7. Removable Foot Board
8. Foot Board Safety Lock
9. Castor
10. Castor Control Lever
11. Bed Height Foot Control
12. Siderail Release Lever
13. Lateral Tilt Foot Control
14. CPR Lever – Backrest Release
15. Accessory Adaptor
16. Corner bumper

5 Technical Specification

All technical data are rated data and are subject to construction and manufacturing tolerances.



WARNING!

If Eleganza 5 bed is used with OptiCare integrated mattress system respect values of mechanical and electrical specifications which can harm none of them!

5.1 Identification of Applied Parts (Type B)

All part of the bed (and accessories) the patient can reach are type B Applied Parts.

- Mattress support platform frame, Covers and all Movable Parts
- Head Board and Foot Board
- Siderails
- Mobi-Lift Handles
- Handset

5.2 Scales (only version with scales)

Accuracy of displayed weight values:

- 0,5 kg (1,1 lbs)
- Scales Class III

5.3 Mechanical Specifications (Eleganza 5)

Parameter	Value
External Dimensions in Standard Bed Position (length x width)	219 cm x 100 cm
Siderail Height above Mattress support platform	14,2 cm (minimum), 45 cm (maximum)
Dimensions of Head Siderail (length x height)	51,1 cm x 46,8 cm
Dimensions of Foot Siderail (length x height)	100,2 cm x 43,1 cm
Bed Extension	0 cm - 22 cm
Dimensions of Mattress (length x width)	208 cm x 90 cm
Maximum Mattress Height	23 cm
Clearance in Standard Position	14,2 cm (with fifth retractable castor 11,3 cm)
Castor diameter	15 cm
Minimum-Maximum Mattress support platform Height above floor (without Mattress)	43,5 cm - 81,5 cm
Ergoframe (Backrest/Thighrest)	7,4 cm / 4 cm
Maximum Backrest Angle	65°
Maximum Thighrest Angle	30°
Maximum Calfrest Angle	30°
Angle between Calfrest and Thighrest	120°
Lateral Tilt Adjustment	+15°/-15°
Trendelenburg/Antitrendelenburg Position (Angle)	+14°/-14°
Bed Weight (without mattress and depending on configuration)	210 Kg - 248 Kg
SWL (Bed Safe Working Load)	250 Kg
Mass Of Mobile Hospital Bed (Mass Of Empty Bed + Safe Working Load)	525 kg
SWL (Lifting Pole Safe Working Load)	75 Kg
Maximum Patient Weight	185 Kg
Application Environment in accordance with IEC 60601-2-52	1, 2

5.4 Environment conditions (Eleganza 5)

Parameter	Value
Use Conditions	
Ambient Temperature	10°C - 40°C
Relative Humidity	30% - 75 %
Atmospheric Pressure	795 hPa - 1060 hPa
Storage and Transport Conditions	
Ambient Temperature	-20°C- 50°C
Relative Humidity	20% - 90 %
Atmospheric Pressure	795 hPa - 1060 hPa

5.5 Electrical Specifications (Eleganza 5)

Parameter	Value
Input Voltage, Frequency	230 V AC, 50/60 Hz 127 V AC, 50/60 Hz 120 V AC, 50/60 Hz 110 V AC, 50/60 Hz 100 V AC, 50/60 Hz
Maximum Power Input	370 VAC
Ingress Protection (EN 60529)	IP X4
Protection Class	Class I
Electrical Motor Duty Cycle	2 minutes ON /18 minutes OFF
Accumulator	Pb AKU 2 x 12 V / 1,2 Ah / Fuse 15A
Fuse Version 230 V Version 127 V Version 120 V Version 110 V Version 100 V	2 x T2.0A L 250 V 2 x T4.0A L 250 V 2 x T4.0A L 250 V 2 x T4.0A L 250 V 2 x T4.0A L 250 V
Plug Type of the Power Supply Cord	E/F (CEE 7/7)

NOTE Upon request, LINET ® can deliver hospital beds with electrical specifications that comply with regional standards (custom voltage, different mains plugs).

5.6 Electromagnetic Compatibility

Bed is intended for hospitals except for near active HF surgical equipment and the RF shielded room of a medical system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Bed has defined no essential performance.



WARNING!

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

List of used cables:

- ▶ **Mains cable**, maximum length 6 m
- ▶ **Attendant Control Panel**, maximum length 3m
- ▶ **Handset**, maximum length 3m



WARNING!

Use of the accessories, converters and other cables, than specified and provided by manufacturer of this bed could lead to increase of electromagnetic emission or lower the electromagnetic immunity of this bed and lead to improper operation.



WARNING!

Mobile RF communication device (including end use devices like antenna cables and external antenna) should not be used closer than 30 cm (12 inches) from any part of this bed Eleganza 5, including cables specified by manufacturer. Otherwise this could lead to deterioration of functionality of this bed.



WARNING!

Do not overload the bed (SWL), respect the duty cycle (INT.) and consider 19 Maintenance chapter in order to maintain the basic safety with regard to electromagnetic disturbances for the expected service life.

5.6.1 Manufacturer instructions - electromagnetic emissions

Emission Test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Class A
Voltage fluctuations / flicker emissions IEC 61000-3-3	Complies

5.6.2 Manufacturer instructions - electromagnetic susceptibility

Immunity Tests	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV for contact discharge ± 15 kV for contact discharge
Radiated RF IEC 61000-4-3 Proximity fields from RF wireless communications equipment IEC 61000-4-3	3 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz See Table 1
Fast electrical transients / burst IEC 61000-4-4	±2 kV for power line repetition frequency 100 kHz
Surge IEC 61000-4-5	± 1 kV Line-to-line ± 2 kV Line-to-ground
Conducted RF IEC 61000-4-6	3 V (0,15 MHz – 80 MHz) 6 V in ISM bands between 0,15 MHz and 80 MHz 80 % AM at 1 kHz
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m
Voltage dips, short interruptions on power supply input lines IEC 61000-4-11	0 % UT; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % UT; 1 cycle and 70 % UT; 25/30 cycle Single phase: at 0° 0 % UT; 250/300 cycle

Table 1 - IMMUNITY to RF wireless communications equipment

Test frequency (MHz)	Band (MHz)	Service	Modulation	Immunity Test Level V/m
385	380 - 390	TETRA 400	Pulse modulation 18 Hz	27
450	430 - 470	GMRS 460, FRS 460	FM ± 5 kHz deviation 1 kHz sine	28
710 745 780	704 - 787	LTE band 13, 17	Pulse modulation 217 Hz	9
810 870 930	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE band 5	Pulse modulation 18 Hz	28
1 720 1 845 1 970	1 700 - 1 990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	28
2 450	2 400 - 2 570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE band 7	Pulse modulation 217 Hz	28
5 240 5 500 5 785	5 100 - 5 800	WLAN 802.11 a/n	Pulse modulation 217 Hz	9

NOTE There are applied no deviations to requirements of IEC 60601-1-2 ed. 4.1.

NOTE There are no known other measures for keeping the basic safety based on EMC phenomena.

NOTE Beds equipped with integration module meet standard for IEEE 802.11 b/g/n (2400,0 MHz – 2483,5 MHz, modulation DSSS (IEEE 802.11 b), OFDM (IEEE 802.11 g/n) 20MHz bandwidth, EIRP = 0,34 W).

6 Use and Storage Conditions



DANGER!

Danger to life due to electric shock!

To ensure the bed's class I protection against electric shocks:

- ▶ Ground the mains.
- ▶ Use exclusively Hospital Grade or Hospital Only receptacles for grounding.

Eleganza 5 and OptiCare are designed for use in rooms for medical purposes. Electrical installations must therefore meet local norms laying down the necessary conditions for electrical installations.

- ▶ Disconnect the bed from the mains in exceptional cases (i.e. lightnings, earthquake).

Eleganza 5 and OptiCare are not suitable for indoor environments containing flammable gases (except oxygen cylinders).

7 Scope of Delivery and Bed Variants

7.1 Delivery

- ▶ Upon receipt, check that the shipment is complete as specified on the delivery note.
- ▶ Notify the carrier and supplier of any deficiencies or damages immediately as well as in writing or make a note on the delivery note.

7.2 Scope of Delivery

- Eleganza 5 medical bed
- Instructions for use

7.3 Eleganza 5 Variants

s = standart
o = optional

Variable bed features:

- Integrated Mattress
 - bed with OptiCare mattress (complete set) (o)
 - OptiCare ready bed without mattress and without System Control Unit (o)
 - OptiCare ready bed without mattress and with System Control Unit (o)
- Scales
 - without scales (without Bed Exit Monitoring) (o)
 - with scales (with Bed Exit Monitoring or with SafetyMonitor) (s)
- Castors
 - Tente Integral 150 mm (5.9 in.) single castors (s)
 - Tente Integral 150 mm (5.9 in.) double castors (o)
 - Tente Integral 150 mm (5.9 in.) single castors + 5th castor (o)
 - Tente Integral 150 mm (5.9 in.) double castors + 5th castor (o)
 - Tente Integral 150 mm (5.9 in.) single castors + Retractable 5th castor (o)
 - Tente Integral 150 mm (5.9 in.) double castors + Retractable 5th castor (o)
 - Tente Integral 150 mm (5.9 in.) single castors + i-Drive Power® (o)
 - Tente Integral 150 mm (5.9 in.) double castors + i-Drive Power® (o)
- Control Elements
 - iBoard Standard in both head siderails (s)
 - iBoard Basic in both head siderails (o)
 - Attendant Control Panel (s)
 - Handset with illuminated buttons and adapter for simple connection Plug and Play (o)
 - Foot Control for Lateral Tilt (s)
 - Foot Control for Height Adjustment (o)
 - Patient Control Panels integrated in foot siderails (s)
 - Patient Control Panels integrated in head siderails (o)
- 1 pair of Mobi-Lift® handles (o)
- i-Brake® (o)
- x-ray cassette holder (o)
- EMR ready bed (o)
- Nurse call (o)
- Safestop (o)
- USB (o)
- LINIS SafetyPort
 - without LINIS SafetyPort (s)
 - basic hardware preparation for LINIS SafetyPort (CE06: without Integration Module) (o)
 - complete hardware preparation for LINIS SafetyPort (CE31: with Integration Module) (o)

NOTE Type of undercarriage cover (one part undercarriage cover or split undercarriage cover) depends on configuration.

8 Putting into Service



WARNING!

Risk of injury when working on the bed!

- ▶ Ensure that the bed is disconnected from the mains connection prior to putting into service, putting out of service and maintenance.
- ▶ Ensure that the castors are locked prior to putting into service, putting out of service and maintenance.



CAUTION!

Material damage due to incorrect putting into service!

- ▶ Ensure that putting into service is performed exclusively by manufacturer's customer service or trained hospital personnel.

NOTE For safe, easy handling, LINET ® recommends having two technicians assemble the bed.

Set up the bed as follows:

- ▶ Unpack the bed.
- ▶ Check the delivery (see Scope of Delivery and Bed Variants).
- ▶ Remove isolating foil from the mains control box (see Accumulator Activation).
- ▶ Install equipment and accessories.
- ▶ In case of delivery with dismantled head board and foot board, mount the head board and foot board (see Head Board and Foot Board).
- ▶ Set up the bed exclusively on a suitable floor surface (see Transport).
- ▶ Ensure that the mains cable does not collide or get stretched when adjusting the bed.
- ▶ Check that the plug is inserted correctly.
- ▶ Do not leave any extension cords or power strips loose on the floor.
- ▶ Ensure that all of the required mechanical and electrical prevention mechanisms are available on site.
- ▶ There is no mains switch on the bed, i.e. the mains cable is the only means to isolate the bed from the mains. Ensure that the mains cable is always accessible.
- ▶ Have the separable plug of the mains cable changed and maintained exclusively by qualified and trained service technicians authorised by the manufacturer.



CAUTION!

Material damage due to temperature difference!

- ▶ If there is a considerable temperature difference between the bed and the place of operation (after transport/storage), leave bed unconnected for 24 for the difference to balance itself.

8.1 Accumulator Activation

Control Section Placement

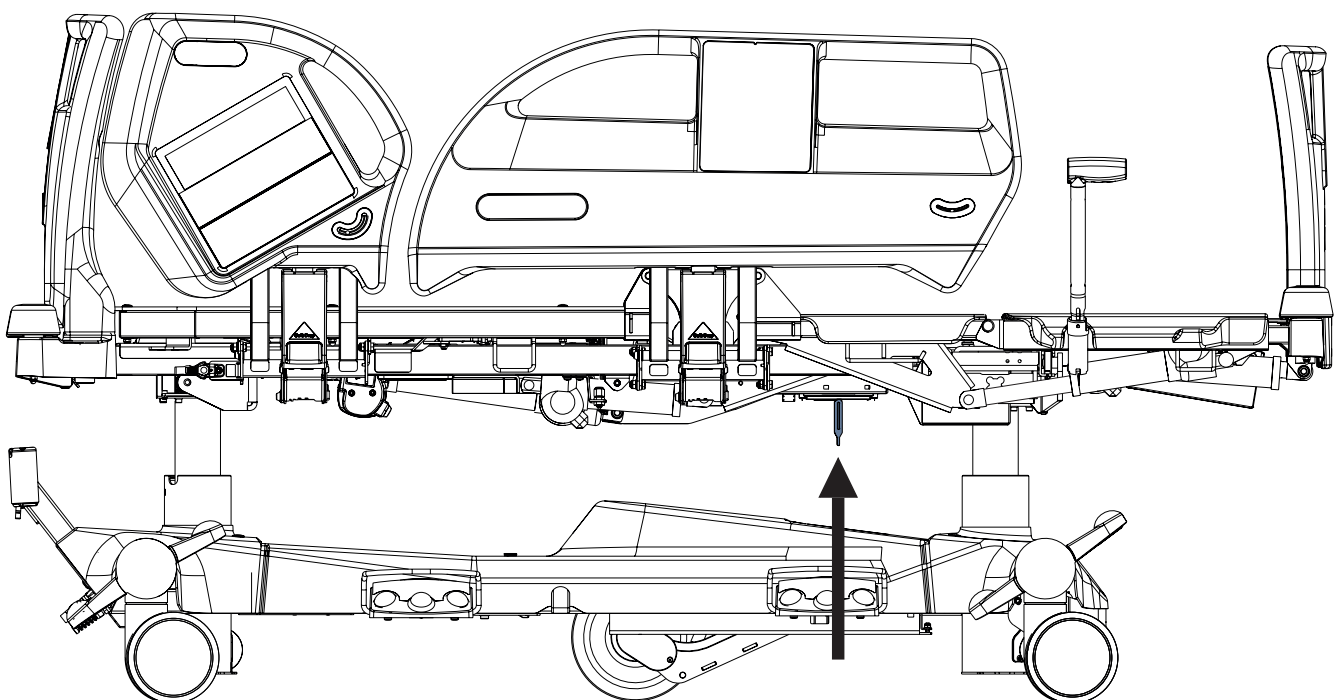


Fig. Isolating Foil Placement

Removing the Isolating Foil

To remove isolating foil:

- ▶ Remove isolating foil from mains control box by pulling strap.
- ▶ Check if isolating foil is complete and undamaged as shown in figure Detail of Isolating Foil.
- ▶ If isolating foil is damaged, contact the manufacturer's service department immediately.

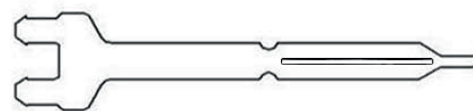


Fig. Detail of Isolating Foil

NOTE Isolating Foil is sharp-edged. Remove it carefully not to cut yourself.

8.2 Head Board and Foot Board

Dismount the head board or foot board as follows:

- ▶ Unlock head board lock or foot board lock.
- ▶ Pull head board or foot board from sleeve fitting.
- ▶ Lock head board lock or foot board lock.

Install the head board or foot board as follows:

- ▶ Unlock head board lock or foot board lock.
- ▶ Slide head board or foot board into sleeve fitting.
- ▶ Lock head board lock or foot board lock.

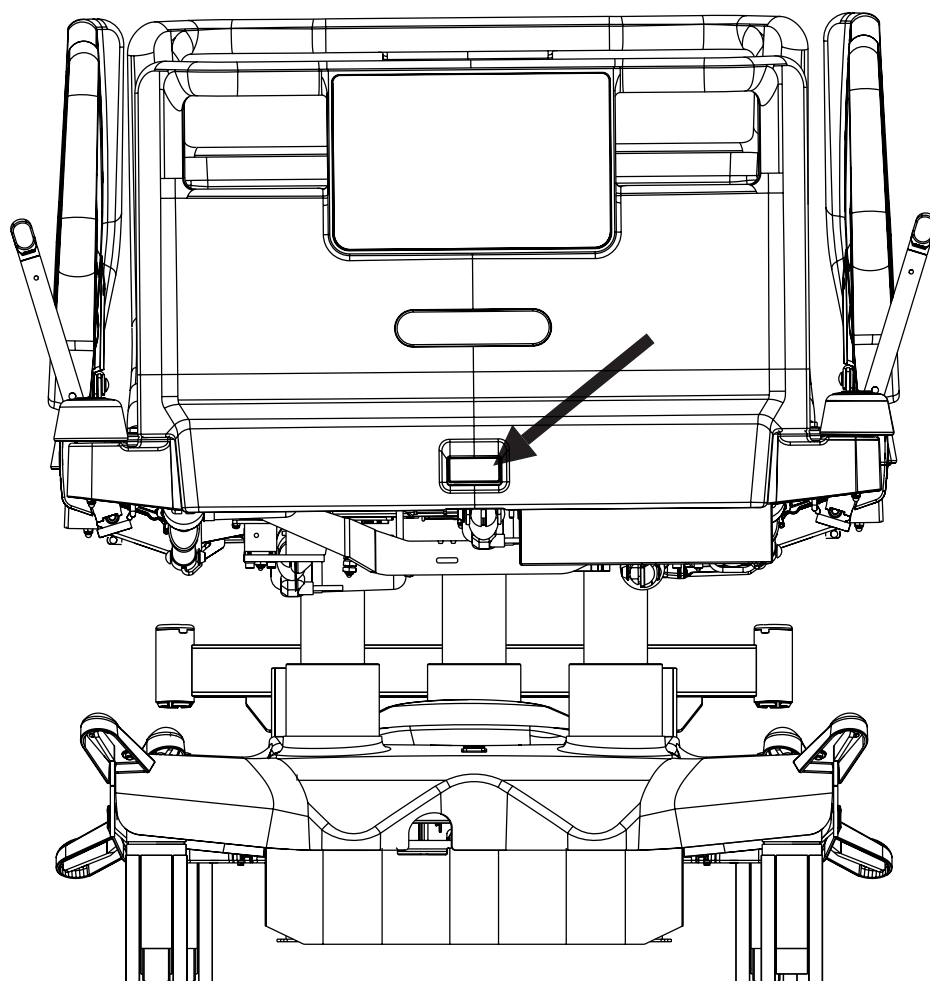


Fig. Foot Board Lock

8.3 Mattress Support Platform

Eleganza 5 bed has 4-part Mattress support platform consisting of Backrest, Seat section, Thighrest and Calfrest. The Mattress support platform without x-ray cassette holder has 4 removable covers of Mattress support platform (1, 2, 3 and 4).

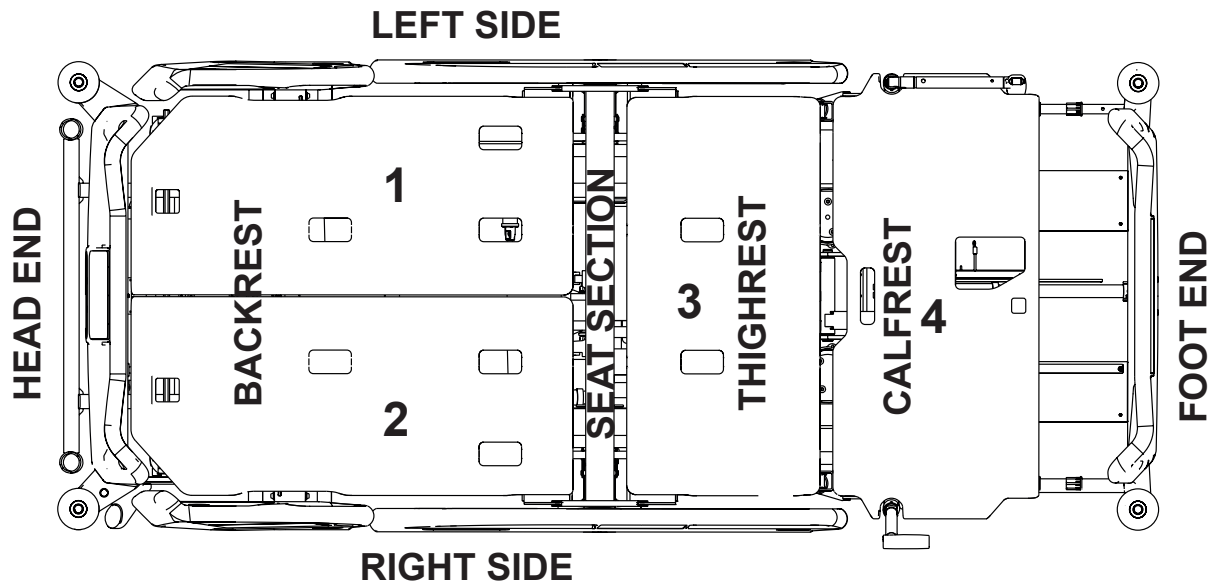


Fig. 4-part Mattress support platform

8.4 Potential Equalisation

The bed is equipped with a standard protective connector. This connector is used for potential equalisation between the bed and any intravascular or intracardiac device connected to the patient to protect the patient from static electric hocks.

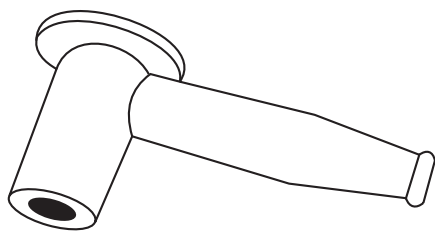


Fig. Potential equalisation connector - female

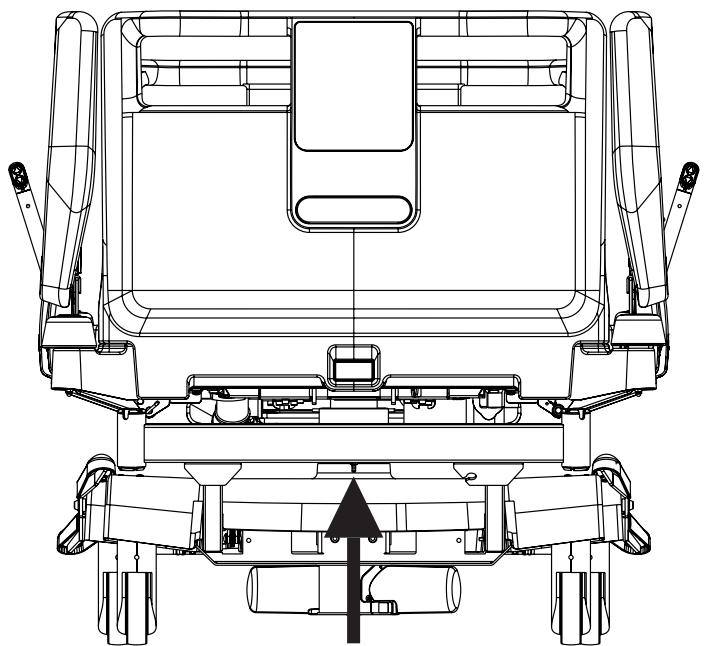


Fig. Potential equalisation - male

Use equalisation connector if:

- the patient is connected to any intravascular or intracardiac device.

Before connecting the patient to an intravascular/intracardiac device:

- ▶ Connect the ground wire of the device to the potential equalisation connector on the bed on which the patient in question is lying.
- ▶ Use a standard hospital connector (female).
- ▶ Make sure that the connectors match.
- ▶ Make sure that there is no possibility for inadvertent disconnection.

Before moving the bed:

- ▶ Disconnect the patient from the intravascular or intracardiac device.
- ▶ Disconnect the potential equalisation connector.

8.5 Before Use

Prepare the bed for use as follows:

- ▶ Connect the bed to the mains.
- ▶ Charge the accumulator.
- ▶ Raise and tilt the mattress support platform to the highest position.
- ▶ Lower and tilt the mattress support platform to the lowest position.
- ▶ Check that the castors as well as main brake work correctly.
- ▶ Check that the bed extension works correctly.
- ▶ Check that it is possible to remove the head board and foot board.
- ▶ Check all of the functions on the control elements.
- ▶ Check that the siderails function properly.
- ▶ Dispose of all packaging (see Disposal).

8.6 Transport

For a safe transport, observe the following:

- ▶ Ensure that no cables are run over when moving a bed.
- ▶ Ensure that the mains cable is attached with a hook (at the head end of the bed).
- ▶ Ensure that the castors are unlocked before moving the bed during the loading/unloading process (see Castor Control).
- ▶ Adjust bed height to at least 20 cm below maximum height.
- ▶ Push bed by handles on head board or foot board.
- ▶ Move the bed exclusively on suitable floor surfaces.
- ▶ Ensure the bed is braked when it is not transported.
- ▶ For longer distances, ensure that the castor steering function (main control) is activated.
- ▶ Ensure that the brakes are released while moving the bed.

Suitable surfaces:

- Tile
- Hard linoleum
- Poured flooring

Unsuitable surfaces:

- Too soft, unsealed or defective flooring
- Soft wooden flooring
- Soft and porous stone floors
- Carpeted floors with underlay
- Soft linoleum

8.7 Firmware

The bed includes firmware that can be updated only by an authorised service technician.

This firmware is protected against unauthorised access by mechanical housing (tool is needed to access), by seal (components with processor are sealed), by exclusive compatibility with an authorised software tool and by check of compatibility of the new firmware with the bed.

9 Power Supply Cord (Mains Power Cable)

Eleganza 5 medical bed is ME equipment with non-detachable Power Supply Cord.

Attachment plug is means of connecting and disconnecting bed from the mains.

Power supply cable (mains power cable) must be attached with a hook at the head end of the bed during transport.



CAUTION!

Disconnecting bed from the mains does not stop motions of the bed!

- ▶ Stop the bed before disconnection bed from the mains.

Where the integrity of the external protective conductor in the installation or its arrangement is in doubt

- ▶ operate the bed from internal accumulator only.

10 Accumulator



WARNING!

When the bed is not connected to the mains and accumulator is not sufficiently charged all electrical functions of the bed are blocked!

Purpose

The accumulator serves as a backup during power failures or for emergency bed positioning.

- ▶ Use only accumulator approved by the manufacturer.
- ▶ Check the functionality of the accumulator at least once a month and have the accumulator changed if necessary.

The manufacturer will assume no responsibility for any damage to the bed or the accumulator caused by:

- non-observance of the manufacturer's instructions in the instructions for use
- using accumulator not approved by the manufacturer

Warranty

The manufacturer provides a **6-month** warranty for the full function of the accumulator.

Accumulator lifetime could be up to 5 years if operated under optimum conditions.

Accumulator capacity can be significantly reduced if:

- ▶ too high ambient temperature
- ▶ many accumulator charge/discharge cycles
- ▶ recurrence of deep discharge
- ▶ bed is often powered only by the accumulator

Charging

The accumulator supplied with the bed is delivered insufficiently charged.

Charging of the accumulator before use of the bed takes approximately 4 hours.

Charging of the accumulator runs automatically when the power cable is connected to the mains.

To charge the accumulator:

- ▶ Connect the bed to the mains.

Storage

For declared lifetime period of leaded accumulators is recommended during storage:

- ▶ To prevent accumulator from deep discharging and to keep accumulator at least partly charged by regular recharging
- ▶ To store accumulator on the dry places with temperature from 10°C to 40°C
- ▶ To prevent accumulators from being in the sunshine

Signalisation

The LED (on iBoard Standard or Attendant Control Panel) indicates the accumulator charge status:

Yellow LED	Accumulator charge status
Not lit	Accumulator is charged.
Short flashing (shortly lit, longer not lit) (circa 1.8s)	Accumulator is charging - continue charging until the LED is extinguished. If LED is still flashing after 12 hours of charging or stops flashing, but you can not position with bed, accumulator is defective or broken. Contact manufacturer's service department.
Short flashing (0,2s lit, 0,2s not lit)	Only CPR function can be used.
Long flashing (longer lit, shortly not lit) (circa 0.2s)	Low accumulator voltage - accumulator can not be used as a backup power supply even for a short period; accumulator is completely discharged or defective (If this type of signalisation persists, after charging it is necessary to replace the accumulator. Contact manufacturer's service department.)
Lit continuously, when bed is connected to the mains.	Accumulator absence or failure condition (accumulator is connected incorrectly, line between the power supply and accumulator is broken or accumulator fuses are faulty); contact service department of the manufacturer in case of such signalisation.

10.1 Replacing the accumulator



CAUTION!

Damage to the bed due to incorrect accumulator replacement!

- ▶ Have the accumulator replaced exclusively by qualified personnel.
- ▶ Exclusively use the accumulator approved by the manufacturer.



CAUTION!

Material damage due to overheating!

If the accumulator is faulty, degassing may occur. In rare cases this might cause deformations of the accumulator case, control panel housing or cable.

- ▶ Stop using the bed immediately (see Removing the Bed from Use).
- ▶ Inform the manufacturer's service department.



CAUTION!

Risk of reducing accumulator durability due to incorrect use!

- ▶ Use bed on accumulator only in crisis situations (e.g.: power blackout, patient complications during transport, etc.).
- ▶ After reconnecting bed to the mains charge accumulator to full capacity (see chart Accumulator charge status).
- ▶ Have the accumulator replaced exclusively by a qualified service organisation of the manufacturer.
- ▶ The manufacturer recommends to replace the accumulator by qualified service organization after 2 years of use.

After this period the supposed service life of accumulator ends and the manufacturer cannot guarantee the accumulator service life after this period.

Faulty accumulator

The accumulator is regarded as faulty if at least one of the following conditions applies:

- ▶ Accumulator charging constantly
- ▶ Low voltage on accumulator
- ▶ Low charging current of accumulator

This status is indicated by the accumulator status indicator being constantly lit. These statuses are summarised and written to black box in the control unit.

To cancel this status:

- ▶ Press button  .

Discharged accumulator

The accumulator is regarded as discharged if the following condition is met:

- ▶ Defined decrease of voltage depending on discharging current
- ▶ This status is indicated by the accumulator status indicator flashing quickly.
- ▶ The electric CPR position is the only possible position.
- ▶ This status will be cancelled automatically when the bed switches to sleep mode.

To cancel this status:

- ▶ Press button  .

10.2 Removing the Bed from Use

Remove the bed from use as follows:

- ▶ Disconnect the bed from the mains.
- ▶ Disconnect the ground wire.
- ▶ Deactivate the accumulator.
- ▶ Remove accessories.

To prevent damage during storage:

- ▶ Pack or cover the bed and accessories.
- ▶ Ensure that storage conditions are the same as the operating conditions.

10.3 Deactivating the Accumulator

To avoid damaging the bed and the environment during storage:

- ▶ Deactivate the accumulator on the Attendant Control Panel.

To deactivate the accumulator:

- ▶ Disconnect the bed from the mains.
 - ▶ Disconnect the ground wire.
 - ▶ Activate the keypad by pressing the button  on the Attendant Control Panel.
 - ▶ Press the Thighrest Up + Thighrest Down + Emergency Trendelenburg Position buttons at the same time and hold them for three seconds.
- The accumulator is deactivated.

To activate the accumulator again:

- ▶ Connect Power Cable to the mains.

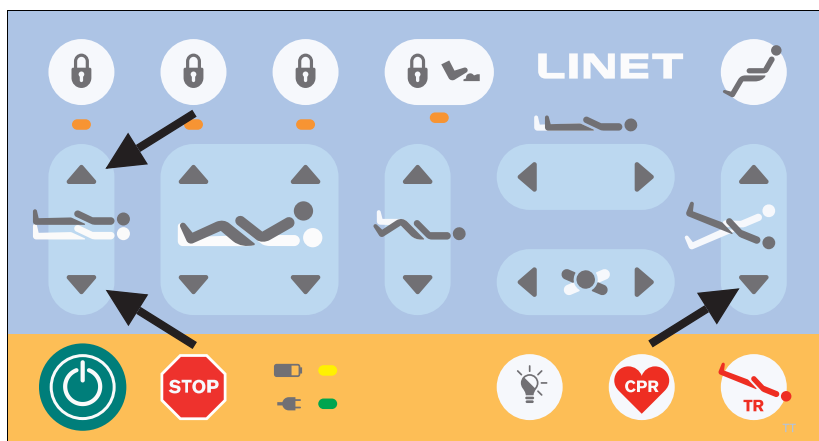


Fig. Deactivation of the accumulator (Attendant Control Panel)

11 Manipulation

**WARNING!****Risk of injury when adjusting the bed!**

- ▶ Ensure that there are no body parts between the mattress support platform elements and the mattress support platform frame when adjusting the bed.
- ▶ Ensure that there are no body parts below the mattress support platform frame before adjusting the bed.
- ▶ Do not position Eleganza 5 ME equipment to make it difficult to operate the attachment plug of the power supply cord!

11.1 Siderails

The split siderails are components of the bed in contact with patient. A pneumatic spring supports the operation of the split siderails. The nursing personnel are responsible for the siderails being raised up while the patient is in bed.

**WARNING!****Risk of injury, damaging or unintentional movement of the bed due to incorrect placement of accessories or Handset!**

- ▶ Never place any accessories or Handset on the siderails in the area where keyboards are located.
- ▶ Never place Handset on the edge of siderail.

The correct placement of Handset is shown at following pictures.

**WARNING!****Risk of injury due to incorrectly latched siderail!**

- ▶ Ensure that siderail is secured in the upper or lower position.

**WARNING!****Risk of injury due to incorrect position of siderails!**

- ▶ Ensure that siderails are folded up while the patient is in bed.

SIDERAIL DESCRIPTION OF THE VERSION WITH iBOARD STANDARD

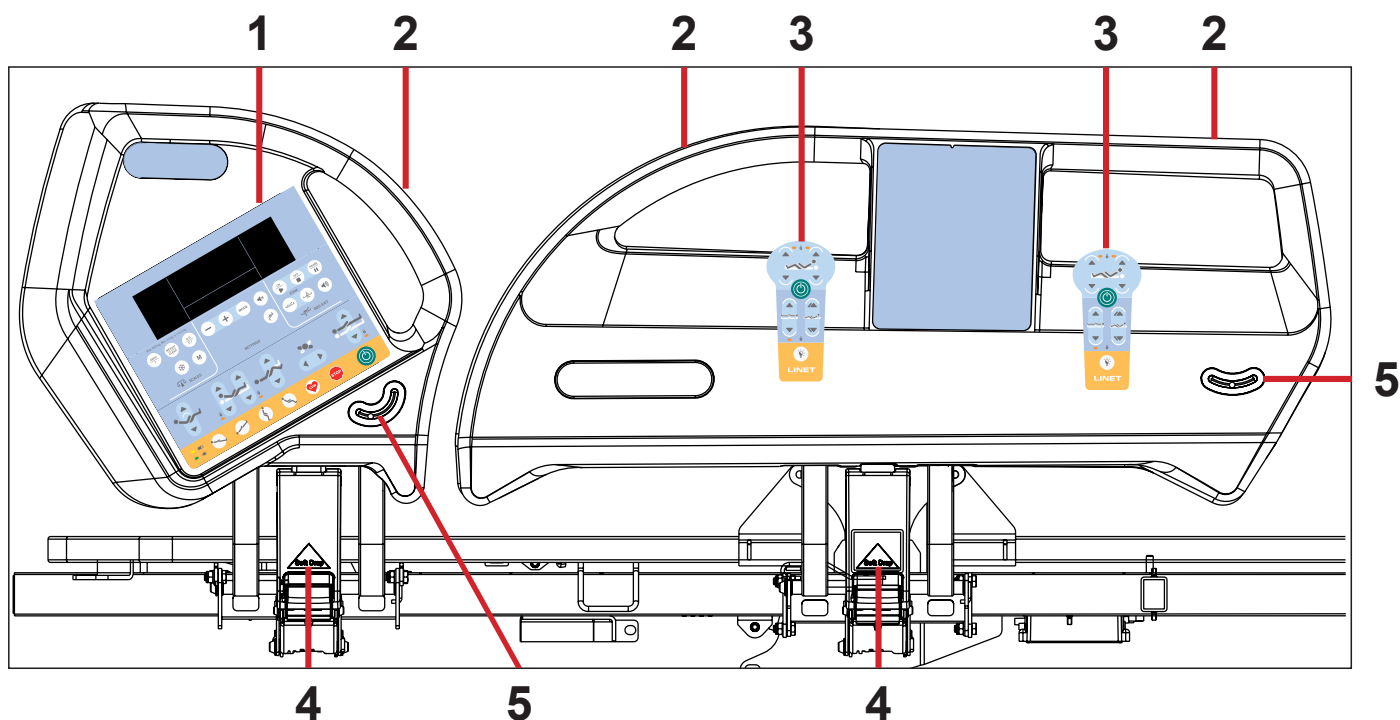


Fig. Siderail version with iBoard Standard

- 1. iBoard Standard
- 2. Siderail Handle
- 3. Correct Placement of Handset
- 4. Siderail Release Handle
- 5. Angle Indicator

MANIPULATION

To raise siderails up:

- ▶ Grab siderail by Siderail Handle (2).
- ▶ Pull siderail up until it latches. You will hear audible „click“.

To release siderails down:

- ▶ Grab siderail by Siderail Handle (2).
- ▶ Unlock siderail by pulling Siderail Release Handle (4) to yourself.
- ▶ Fold down siderail slowly.

SIDERAIL DESCRIPTION OF THE VERSION WITH iBOARD BASIC

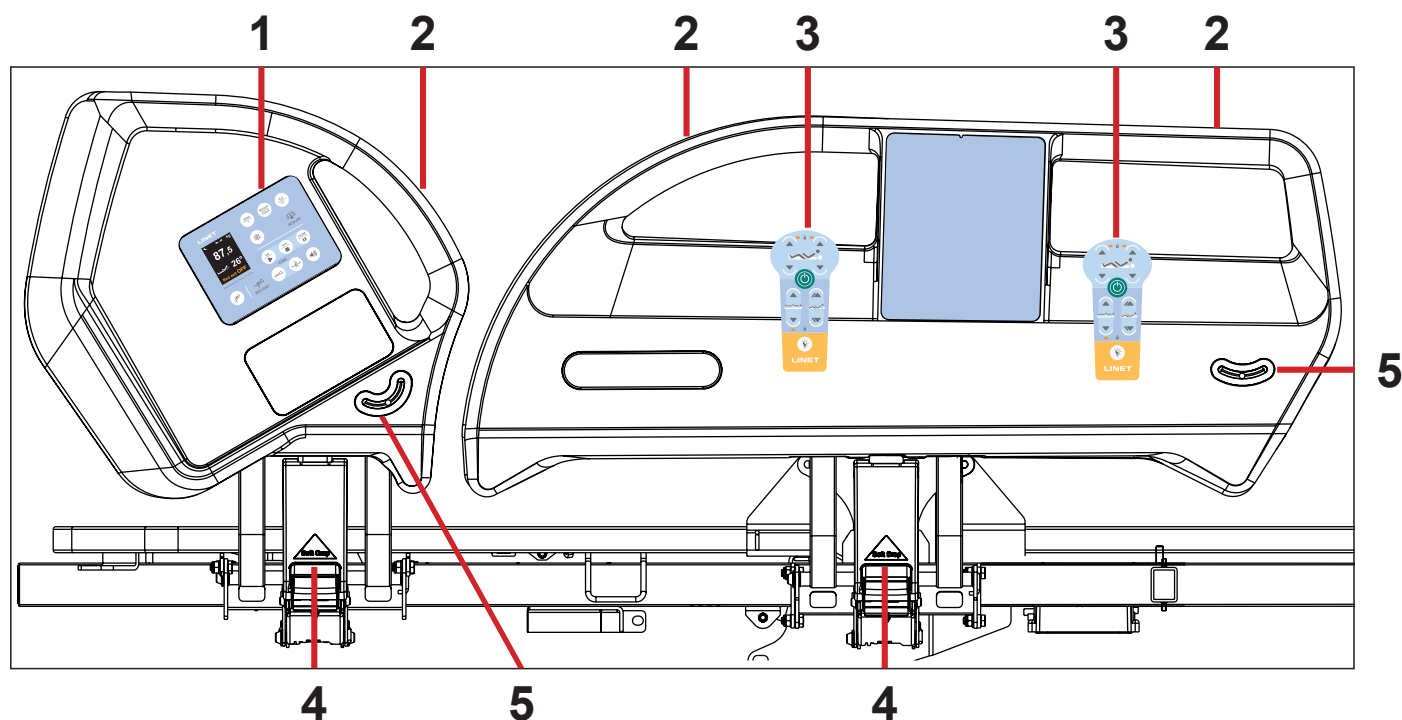


Fig. Siderail version with iBoard Basic

- 1. iBoard Basic
- 2. Siderail Handle
- 3. Correct Placement of Handset
- 4. Siderail Release Handle
- 5. Angle Indicator

MANIPULATION

To raise siderails up:

- ▶ Grab siderail by Siderail Handle (2).
- ▶ Pull siderail up until it latches. You will hear audible „click“.

To release siderails down:

- ▶ Grab siderail by Siderail Handle (2).
- ▶ Unlock siderail by pulling Siderail Release Handle (4) to yourself.
- ▶ Fold down siderail slowly.

SIDERAIL DESCRIPTION OF THE VERSION WITH NURSE CONTROL PANEL AND iBOARD BASIC

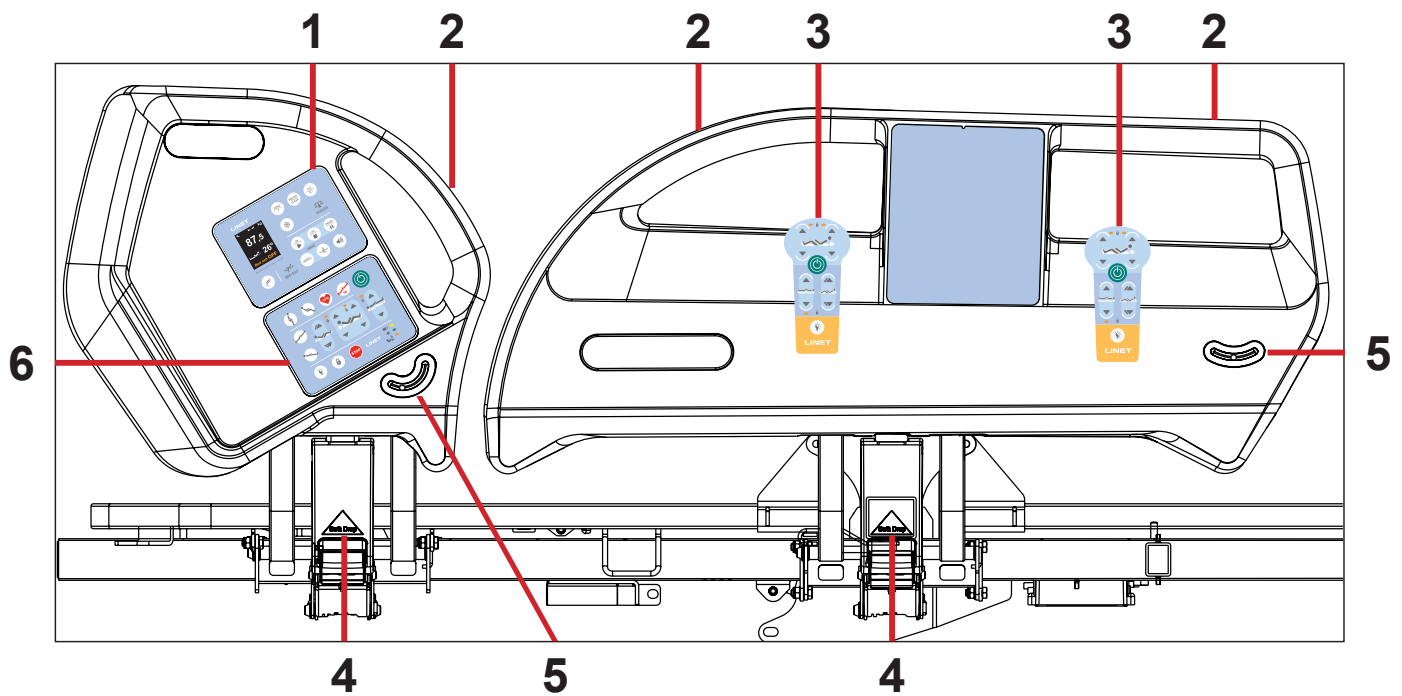


Fig. Siderail version with Nurse Control Panel and iBoard Basic

- 1. iBoard Basic
- 2. Siderail Handle
- 3. Correct Placement of Handset
- 4. Siderail Release Handle
- 5. Angle Indicator
- 6. Nurse Control Panel

MANIPULATION

To raise siderails up:

- ▶ Grab siderail by Siderail Handle (2).
- ▶ Pull siderail up until it latches. You will hear audible „click“.

To release siderails down:

- ▶ Grab siderail by Siderail Handle (2).
- ▶ Unlock siderail by pulling Siderail Release Handle (4) to yourself.
- ▶ Fold down siderail slowly.

SIDERAIL DESCRIPTION OF THE VERSION WITH NURSE CONTROL PANEL

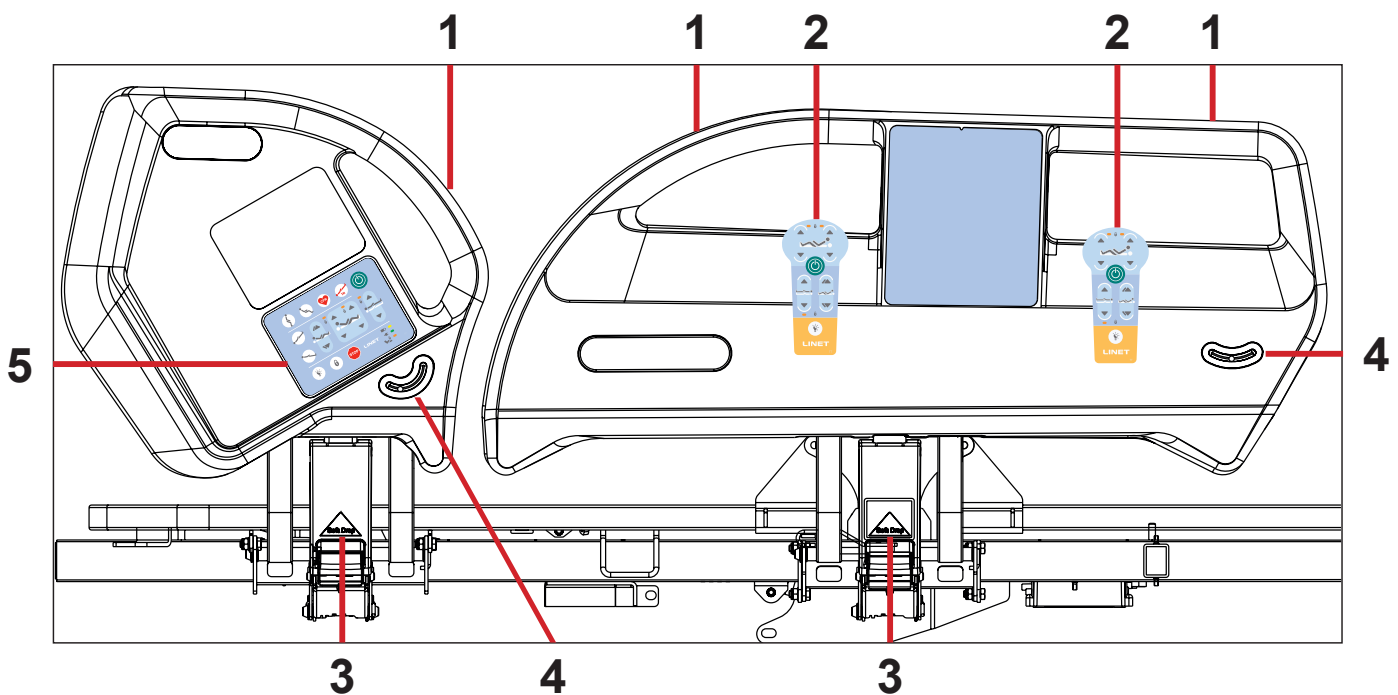


Fig. Siderail version with Nurse Control Panel

- 1. Siderail Handle
- 2. Correct Placement of Handset
- 3. Siderail Release Handle
- 4. Angle Indicator
- 5. Nurse Control Panel

MANIPULATION

To raise siderails up:

- ▶ Grab siderail by Siderail Handle (1).
- ▶ Pull siderail up until it latches. You will hear audible „click“.

To release siderails down:

- ▶ Grab siderail by Siderail Handle (1).
- ▶ Unlock siderail by pulling Siderail Release Handle (3) to yourself.
- ▶ Fold down siderail slowly.

SIDERAIL DESCRIPTION OF THE VERSION WITHOUT SCALES

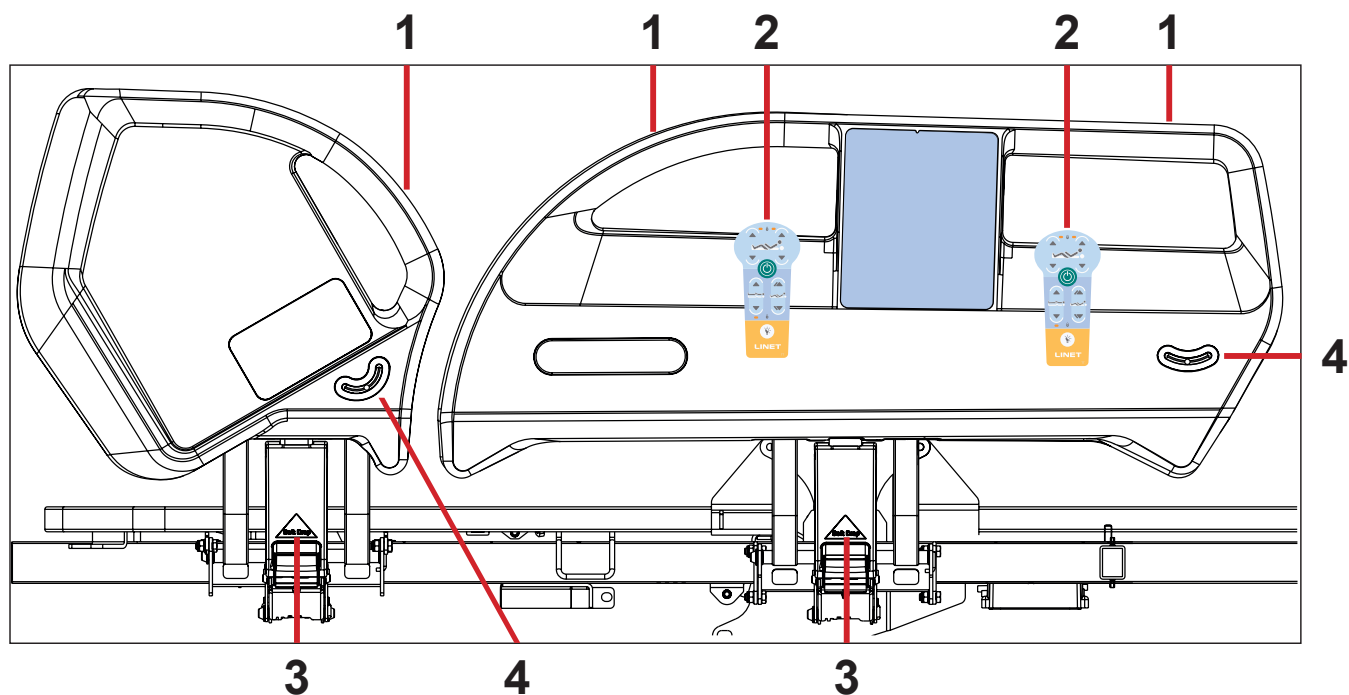


Fig. Siderail version without scales

- 1. Siderail Handle
- 2. Correct Placement of Handset
- 3. Siderail Release Handle
- 4. Angle Indicator

MANIPULATION

To raise siderails up:

- ▶ Grab siderail by Siderail Handle (1).
- ▶ Pull siderail up until it latches. You will hear audible „click“.

To release siderails down:

- ▶ Grab siderail by Siderail Handle (1).
- ▶ Unlock siderail by pulling Siderail Release Handle (3) to yourself.
- ▶ Fold down siderail slowly.

11.2 Castor Control



CAUTION!

Material damage due to incorrect transport and involuntary movement!

- ▶ Prior to transport, ensure that the bed is disconnected from the mains.
- ▶ Ensure that the castors are braked prior to assembly, disassembly and maintenance.
- ▶ Ensure that the castors are braked when the bed is occupied.
- ▶ Hang the mains cable on the transport hook on the bed during transport.
- ▶ Have the bed transported exclusively by nursing personnel and by at least 2 persons.



CAUTION!

Minimal clearance underneath the bed (standard version with 15 cm castors) is 11,3 cm!

- ▶ Observe the path for any obstacles and avoid collisions and possible damages of any bed's part on the undercarriage.
- ▶ Do not use bed lifts and hoists for lifting the bed.

The bed is equipped with central castor's control and brake system.
The control levers are located in the four corners of the undercarriage.

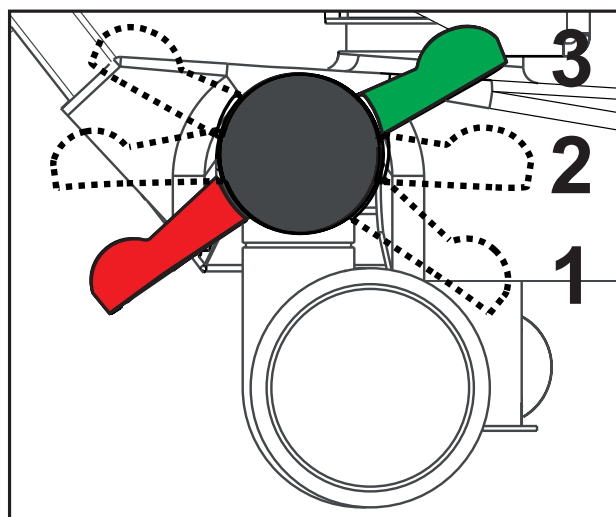


Fig. Positions of Castor Control Lever

Castor control lever positions:

1. Forward Movement - Steering (GREEN PEDAL DOWN)

An arrested front castor determines the direction of movement. If the bed is equipped with a fifth castor, this castor determines the direction of movement.

2. Unrestricted Movement

All four castors are unlocked.

3. Braked (RED PEDAL DOWN)

All four castors are braked.

11.3 CPR Backrest Release



WARNING!

Risk of injury due to lowering the backrest too quickly!

- ▶ Ensure that the siderails are in the low position.
- ▶ Ensure that there are no body parts between any movable parts of the bed.
- ▶ Push the Backrest down using the mattress guard handle only.

The bed allows quick, mechanical lowering of the backrest for emergency procedures (CPR).

Set the position as follows:

- ▶ Pull and hold release handle.
- ▶ Push Backrest down.

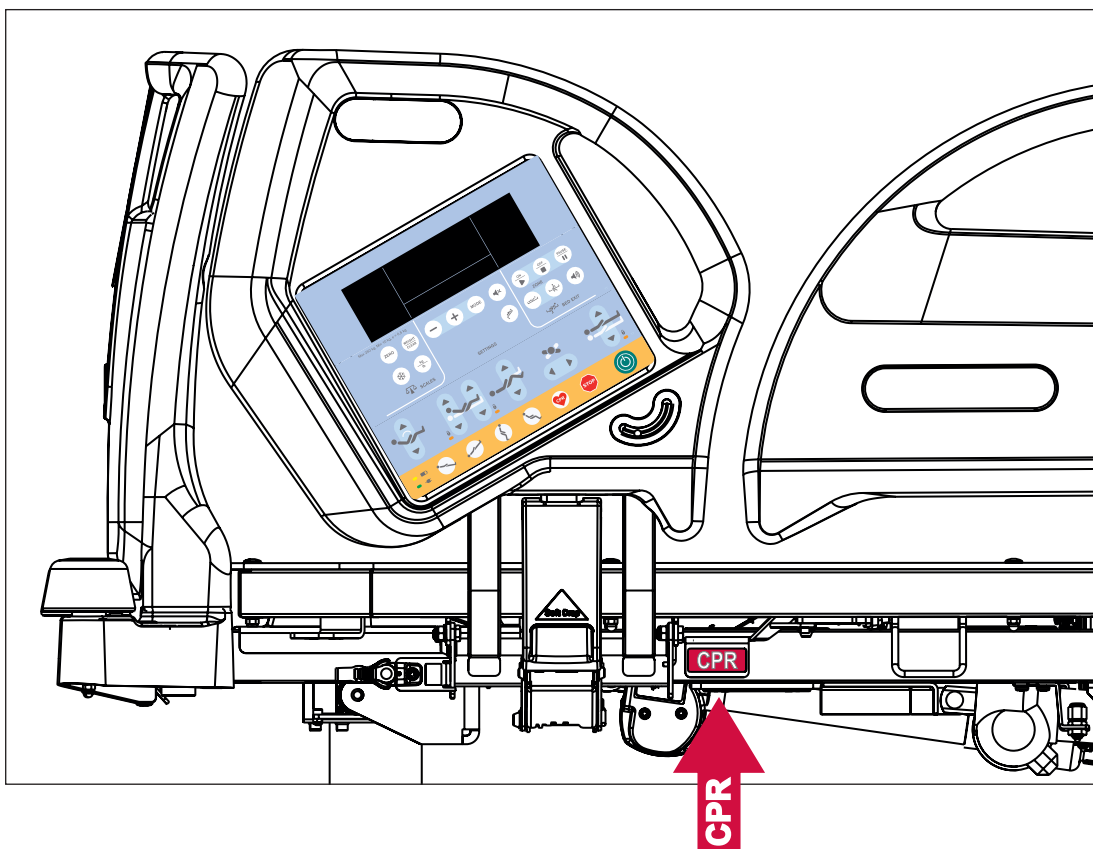


Fig. CPR lever

11.4 Control Elements

The bed is operated by different control elements.

Control elements depending on the model:

- iBoard Standard in both head siderails
- iBoard Basic in both head siderails
- Nurse Control Panel
- Patient Control Panels integrated in both foot siderails (illuminated)
- Patient Control Panels integrated in both head siderails
- Attendant Control Panel
- Handset with illuminated buttons and with adapter for easy connection (Plug and Play)
- Lateral Tilt Foot Control
- Bed Height Foot Control

Disabling individual functions on the Attendant Control Panel will affect all control elements.

If the bed does not react to individual position settings:

- Check whether the function is disabled on the Attendant Control Panel.

POSITIONING	iBoard Standard	Attendant Control Panel	Handset	Patient Control Panels (head siderail, foot siderail)	Bed Height Foot Control	Lateral Tilt Foot Control
Backrest	✓	✓	✓	✓		
Thighrest	✓	✓	✓	✓		
Calfrest	✓	✓				
Bed Height	✓	✓	✓		✓	
Autocontour	✓		✓	✓		
Examination Position					✓	
Emergency Trendelenburg Position		✓				
Antitrendelenburg and Trendelenburg Tilt	✓	✓				
Bed Extension		✓				
CPR Position	✓	✓				
Cardiac Chair Position	✓	✓				
Mobilization Position	✓					
Lateral Tilt	✓	✓				✓
Low Flat Position						✓

11.4.1 iBoard Standard

The iBoard Standard is the main control element for the caregivers. It is integrated in the outside of both head siderails. Only version with scales can be equipped with iBoard Standard.

- Ensure that exclusively trained nursing staff operates the iBoard Standard.

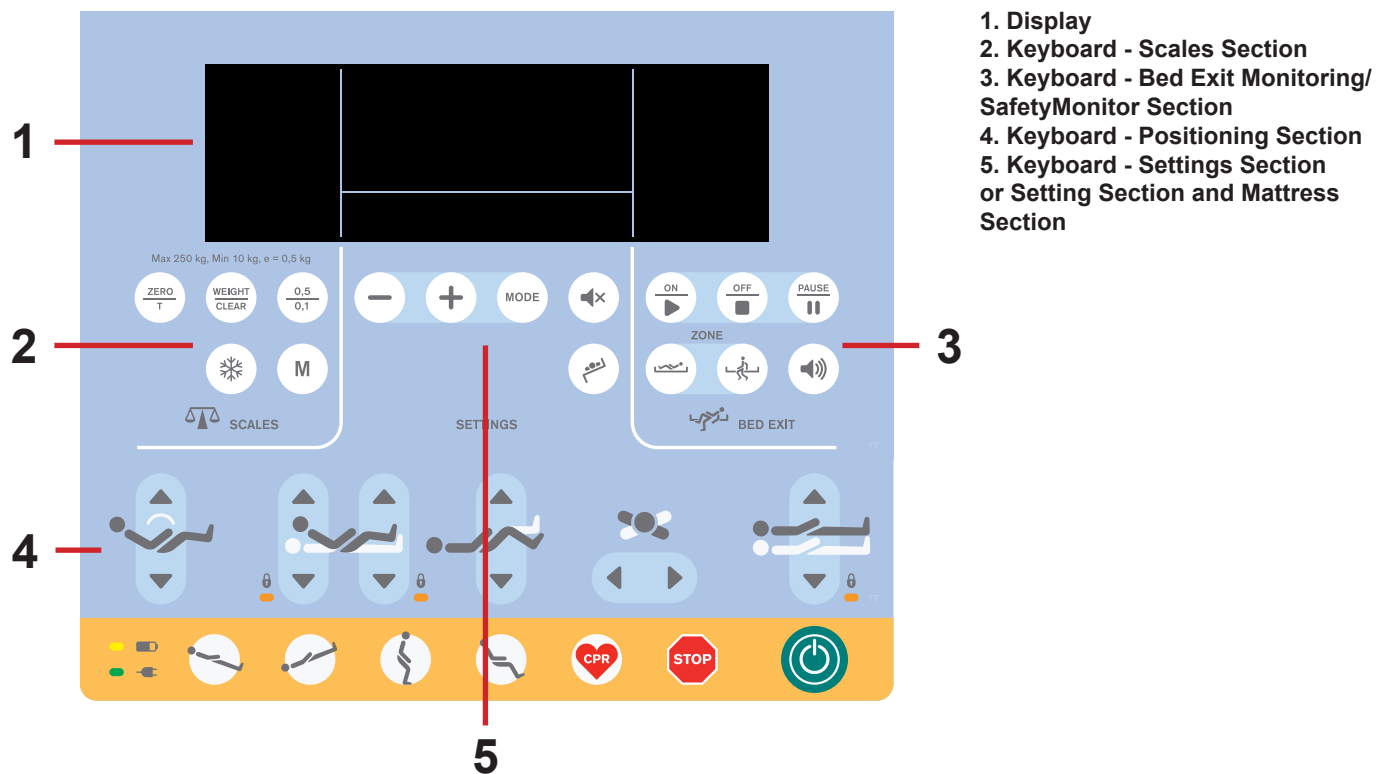


Fig. iBoard Standard without Mattress Keyboard

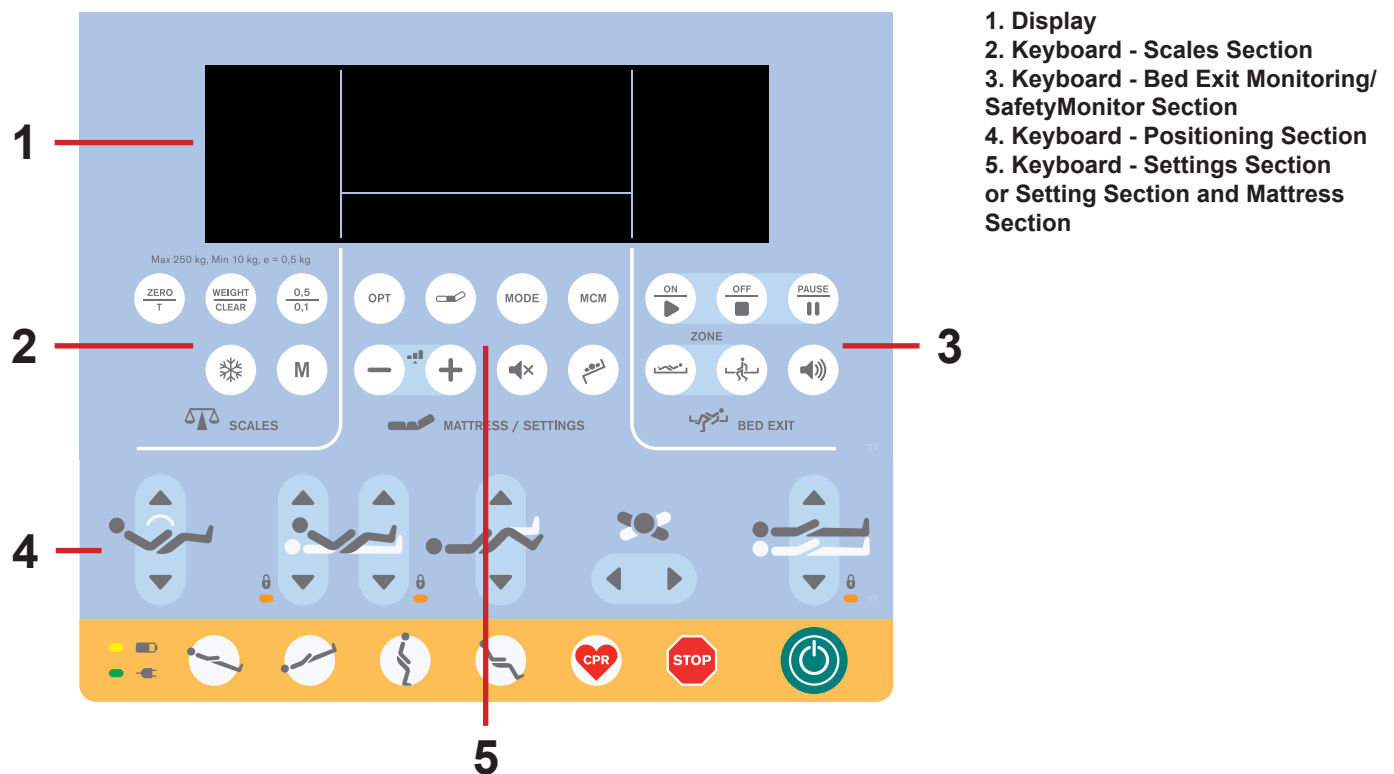


Fig. iBoard Standard with Mattress Keyboard

Settings Section

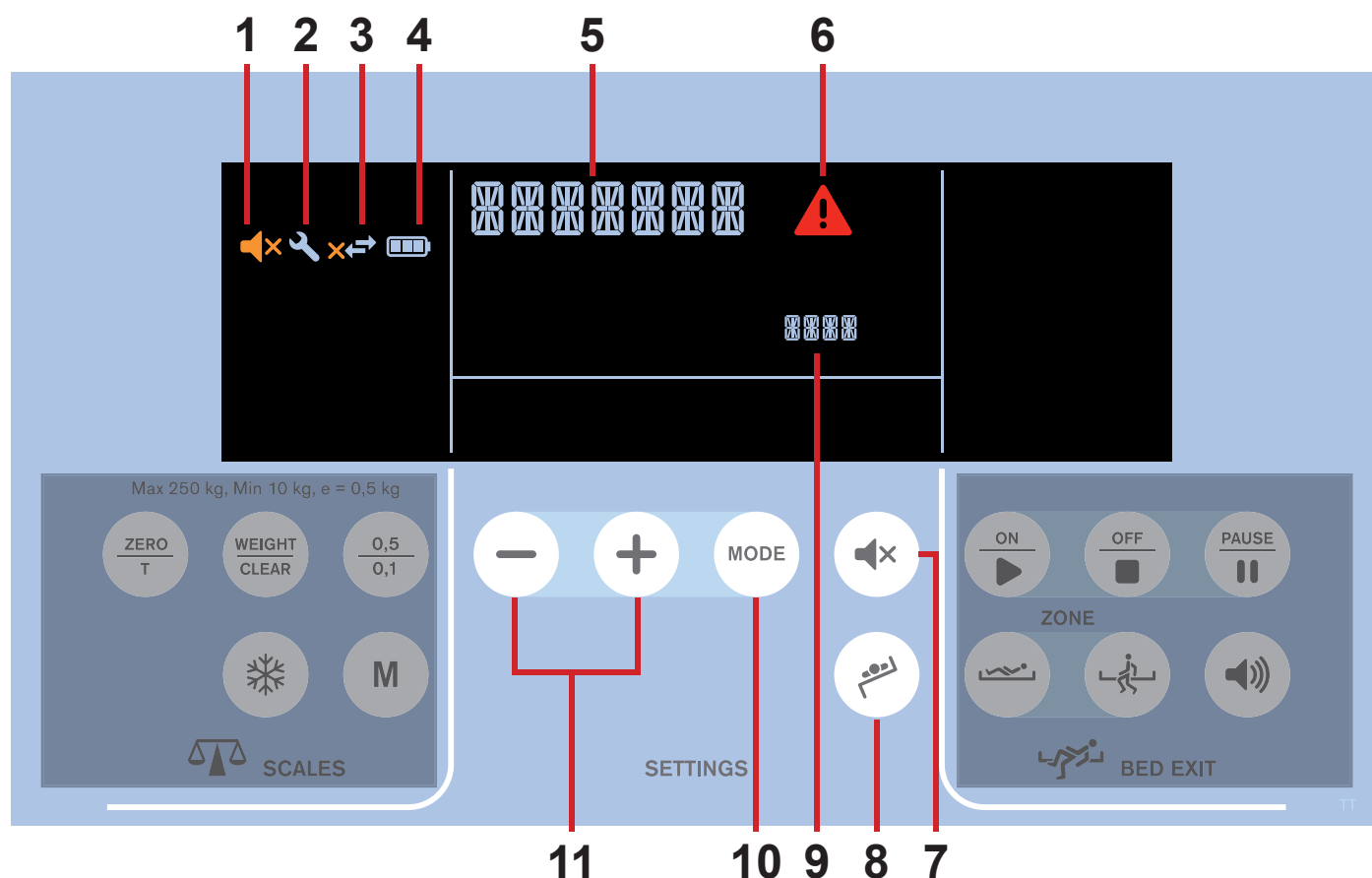


Fig. Display and Keyboard of the Settings Section

1. MUTE Icon
2. Service required Icon
3. Connection to server Icon (without cross - connected, with cross - disconnected)
4. Accumulator charge status
5. Status
6. ALERT Icon
7. MUTE Button
8. Transfer Mode Button
9. Value to be set
10. MODE Button
11. MINUS (—) / PLUS (+) Buttons (previous/next item or reduce/increase value)

MUTE (1)

Function MUTE can mute Siderail Signal, Brake Signal or sounds of the integrated mattress for 3 minutes.

To mute Siderail Signal or Brake Signal:

► Press button .

Icon  indicates activated Mute Mode.

SERVICE REQUIRED (2)

Icon  appears on the left part of iBoard Standard Display when service is required.

► Contact authorized service organization.

Icon  is displayed with name of status (5).

CONNECTION TO SERVER (3)

This icon appears on the display if SafetyMonitor option is ordered and Integration Module is installed.


Icon  appears on the iBoard Standard Display when bed is connected to server.

Icon  appears on the iBoard Standard Display when bed is disconnected from server.

STATE OF CHARGE (4)



WARNING!
Disabled functions due to critically discharged accumulator!
► Connect bed immediately to the mains.

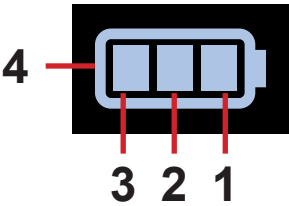

Accumulator segments indicate state of charge (4 levels).

The more segments of accumulator icon  the higher level of charge.

Status	Indication
Charging	Lit segment 3, segment 2 and 1 are flashing alternately
Faulty Accumulator or Accumulator Not Installed	Empty frame 4 is flashing
Critically Discharged Accumulator	Segment 3 is flashing, Bed Positioning Disabled

TRANSFER MODE (8)

Transfer Mode enables Lateral Tilt with a siderail in down position for 5 minutes.

To set Transfer Mode:

► Press  button.

Lateral Tilt icon on the iBoard Basic display or iBoard Standard display is flashing.

SETTINGS

During normal use icon **5** indicates time (**hours : minutes**).

To enter Settings Mode:

- ▶ Press and hold button  .

Icon **5** indicates option and icon **8** indicates its actual value.

Available options are: YEAR / MONTH / DAY / HOUR / MINUTE / WEIGHT TIMER / **VOICE**.

It is not possible to set value for option WEIGHT TIMER in the version of Eleganza 5 bed without scales.

NOTE: Options are displayed in this order. YEAR follows after WEIGHT TIMER again.

NOTE: WEIGHT TIMER means automatic disappearance of weight value.

To exit the Settings Mode:

- ▶ Press and hold button  in the Settings Mode.

Icon **5** indicates time (**hours : minutes**).

To set year:

- ▶ Press button  in the Settings Mode when icon **5** indicates „**YEAR**“.
- ▶ To set the value use buttons  .
- ▶ Press button  to save the value and continue to the setting of the other option.

The value is saved by exiting.

Icon **5** indicates another option automatically and icon **8** indicates its actual value.

To set month:

- ▶ Press button  in the Settings Mode when icon **5** indicates „**MONTH**“.
- ▶ To set the value use buttons  .
- ▶ Press button  to save the value and continue to the setting of the other option.

The value is saved by exiting.

Icon **5** indicates another option automatically and icon **8** indicates its actual value.

To set day:

- ▶ Press button  in the Settings Mode when icon **5** indicates „**DAY**“.
- ▶ To set the value use buttons  .
- ▶ Press button  to save the value and continue to the setting of the other option.

The value is saved by exiting.

Icon **5** indicates another option automatically and icon **8** indicates its actual value.

To set hour:

- ▶ Press button  in the Settings Mode when icon **5** indicates „**HOUR**“.
- ▶ To set the value use buttons  .
- ▶ Press button  to save the value and continue to the setting of the other option.

The value is saved by exiting.

Icon **5** indicates another option automatically and icon **8** indicates its actual value.

To set minute:

- Press button  in the Settings Mode when icon **5** indicates „**MINUTE**“.
- To set the value use buttons  .
- Press button  to save the value and continue to the setting of the other option.

The value is saved by exiting.

Icon **5** indicates another option automatically and icon **8** indicates its actual value.

To set weight timer:

- Press button  in the Settings Mode when icon **5** indicates „**WEIGHT TIMER**“.
- To set the value use buttons  .
- Press button  to save the value and continue to the setting of the other option.

The value is saved by exiting.

Icon **5** indicates another option automatically and icon **8** indicates its actual value.

To set language of the audible messages:

- Press  button in the Settings Mode when icon **5** indicates „**VOICE**“.
- To set the value, use   buttons.

List of the available languages:

EN	English
DE	German
CZ	Czech
FR	French
IT	Italian
ES	Spanish
SE	Swedish
NL	Dutch
PT	Portuguese
FI	Finnish
RO	Romanian
NO	Norwegian
LV	Latvian
HU	Hungarian
LT	Lithuanian
PL	Polish
OFF	function deactivated

- Press  button to save the value and continue to the setting of the other option.

The value is saved by exiting.

Icon **5** indicates another option automatically and icon **8** indicates its actual value.

Positioning Section

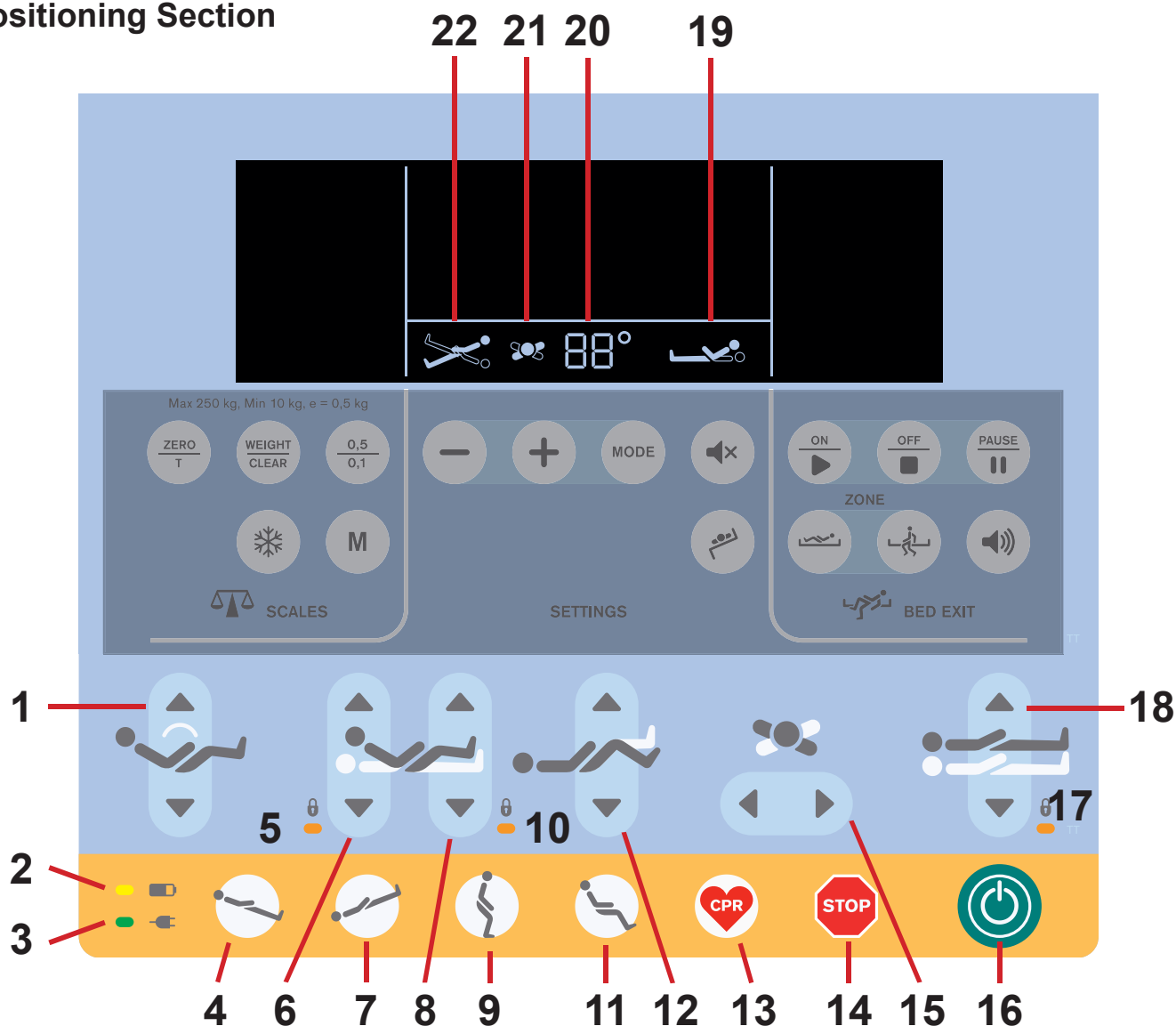


Fig. Display and Keyboard of the Positioning Section

1. Autocontour Adjustment Buttons (simultaneous movement of the Backrest and Thighrest)
2. Accumulator charge status LED
3. Mains Power LED
4. Antitrendelenburg Tilt Button
5. Locked Backrest LED
6. Backrest Adjustment Buttons
7. Trendelenburg Tilt Button
8. Thighrest Adjustment Buttons
9. Mobilization Position Button
10. Locked Thighrest, Calfrest and Bed Extension LED
11. Cardiac Chair Position Button
12. Calfrest Adjustment Buttons
13. CPR (Resuscitation) Position Button
14. Central STOP Button
15. Lateral Tilt Buttons
16. GO Button
17. Locked Bed Height, Lateral Tilt, Trendelenburg Tilt and Antitrendelenburg Tilt LED
18. Bed Height Adjustment Buttons
19. Backrest Position Indicator
20. Angle Value
21. Lateral Tilt Indicator
22. Trendelenburg/Antitrendelenburg Position Indicator

Positioning buttons 1, 4, 6, 7, 8, 9, 11, 12, 13, 15 and 18 are explained in chapter **Bed Positioning**.

GO BUTTON

The button  activates the keyboard of all Control Elements.

Pressing button  will keep the keyboard active for 3 minutes.

Pressing a button will keep the keyboard active for another 3 minutes.

During this time the following is possible:

- ▶ Adjusting individual Mattress support platform elements by pressing the corresponding positioning buttons.
- ▶ Disabling individual functions with the lock buttons.

NOTE: To activate CPR function ( button) the  button is not needed.

STOP BUTTON

Pressing button  immediately stops all electronic bed movements.

LOCKED FUNCTION SIGNALISATION

If **LED 5** is lit, Backrest Adjustment is locked.

If **LED 5** is not lit, Backrest Adjustment is unlocked.

If **LED 10** is lit, Thighrest, Calfrest and Bed Extension Adjustment are locked.

If **LED 10** is not lit, Thighrest, Calfrest and Bed Extension Adjustment are unlocked.

If **LED 17** is lit, Bed Height and Lateral Tilt, Trendelenburg Tilt and Antitrendelenburg Tilt are locked.

If **LED 17** is not lit, Bed Height and Lateral Tilt, Trendelenburg Tilt and Antitrendelenburg Tilt are unlocked.

ACCUMULATOR INDICATOR

Signalisation of Accumulator LED  is described in chapter **Accumulator**.

MAINS POWER LED

Status	Meaning
lit LED	connected to the mains
unlit LED	disconnected from the mains
flashing LED	system error

Scales Section


Eleganza 5 is optionally equipped with a weighing system that allows weighing the patient in bed. There are control buttons and display for the weighing system on the Scales section of iBoard Standard. Scales functions are described in chapter **Scales Control**.

Bed Exit Monitoring Section (only version with scales)

Eleganza 5 is optionally equipped with a Bed Exit Monitoring system that monitors patient's presence in bed and triggers alerts when patient is not present in bed. There are control buttons and display for the Bed Exit Monitoring on the Bed Exit section of iBoard Standard. Functions of Bed Exit Monitoring are described in chapter **Bed Exit Monitoring**.



Fig. Bed statuses on the iBoard Standard display

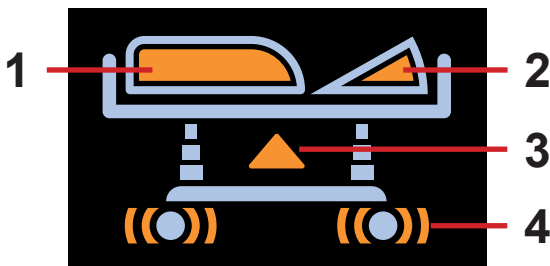

1. siderail status (orange - siderail down)
2. backrest angle (orange - backrest in less than 30°)
3. bed height (orange - bed not in the lowest position)
4. brake status (orange - unbraked castors)

Fig. Bed icon with bed statuses (orange - alerts/insecure statuses)

Statuses (iBoard Standard)

Status	Meaning	Required Action
LOCK	Function locked.	Unlock function if required!
X-RAY	Incorrectly inserted X-Ray Cassette Holder.	Insert X-Ray Cassette Holder correctly!
GO	GO Button not activated.	Press GO Button!
SIDERAIL	Lateral Tilt disabled when siderail folded down.	Raise siderail up to enable additional Lateral Tilting.
0°	Horizontal position was reached during tilting.	Press corresponding button to continue in positioning.
COLLISION	Antitrendelenburg Tilt and Trendelenburg Tilt disabled during Lateral Tilt.	Undo the Lateral Tilt to continue with Antitrendelenburg Tilt or Trendelenburg Tilt.
	Positioning blocked to avoid collision of the bed with floor or collision of the bed with bed equipments.	To continue in positioning, adjust the bed so that there is no collision.
OVERLOAD + 	Safe Working Load exceeded (more than 10 kg over Safe Working Load).	Remove load!
	Lateral Tilt disabled (Load more than 200 kg).	Remove load to enable Lateral Tilting again!
PUMP DISCONNECTED + 	System Control Unit (OptiCare) disconnected.	Contact service department approved by manufacturer.
USE MANUAL CPR + 	The mattress deflating failed.	Use manual CPR! (System Control Unit (OptiCare) is disconnected or automatic deflation is not available.)
STOP SERVICE +  + 	System Fatal Error.	Contact service department approved by manufacturer.
SAVE WEIGHT	Confirmation of rewriting memory.	Press M button to save weight or WEIGHT/CLEAR button to cancel.
SCALE/BEA DISCONNECTED + 	Scale module disconnected and Bed Exit Monitoring disabled.	Contact service department approved by manufacturer.

FAULT COLUMN + 	Column Unit Error.	Contact service department approved by manufacturer.
SAFE STOP + 	Movement of the Mattress support platform stopped by function Safestop.	Remove an object from undercarriage to continue in adjusting the bed height.
CLOSE MANUAL CPR + 	The mattress inflating failed.	Close manual CPR to enable inflation.
BEDEXIT ALERT + 	The patient has left the bed (Outer Zone monitoring) or the patient has left the Inner Zone (Inner Zone monitoring).	Check the patient and turn off the Bed Exit Alert.
AUTOMATIC CALIBRATION	Regular automatic process (repeated after 10 hours if it is interrupted).	Wait until the Automatic Calibration is finished.
MIN 35 KG	Insufficient load for Bed Exit Monitoring.	Place patient on the bed to enable the Bed Exit Monitoring.
POWER	Disconnected from the mains power supply.	Connect bed to the mains power.
Hi	Safe Working Load exceeded (from 4,5 kg to 10 kg over Safe Working Load).	Remove load!
Lo	The bed is underloaded.	Ensure the mattress support platform is not lifted inconveniently by something and the scales are appropriately tared.

11.4.2 Nurse Control Panel (optional)

The Nurse Control Panel is the main Control Element for the caregivers. It is integrated in the outside of both head siderails.

- Ensure that exclusively trained nursing staff operates the Nurse Control Panel.

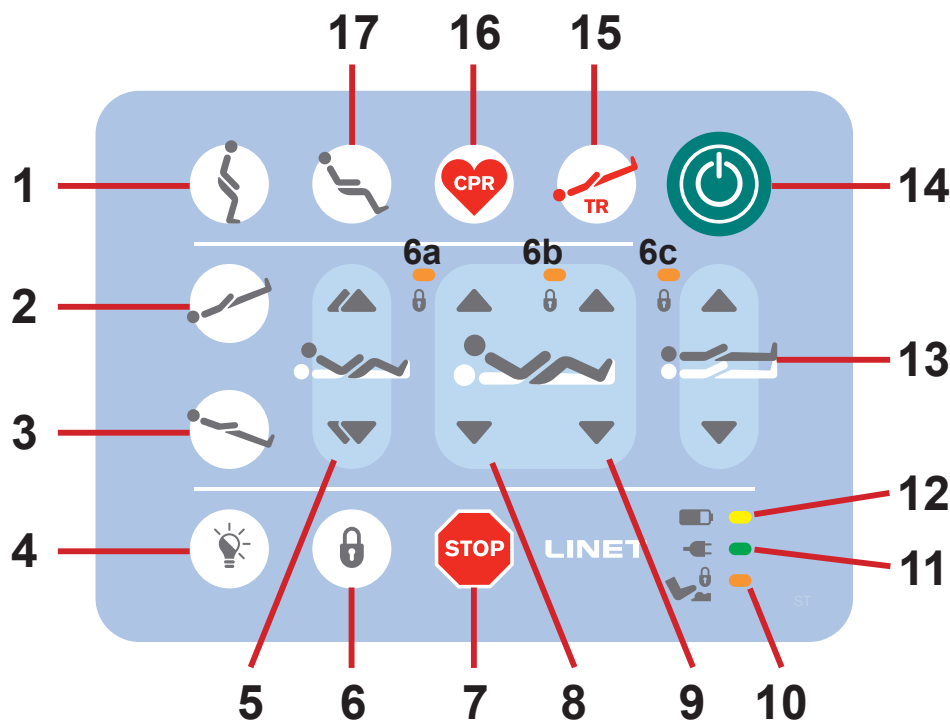


Fig. Nurse Control Panel

1. Mobilization Position Button
2. Trendelenburg Tilt Button
3. Anti-Trendelenburg Tilt Button
4. LIGHT CONTROL Button
5. Autocontour Adjustment Buttons (simultaneous movement of the Backrest and Thighrest)
6. LOCK Button
- 6a. Locked Backrest LED
- 6b. Locked Thighrest LED
- 6c. Locked Bed Height, Trendelenburg Tilt and Anti-Trendelenburg Tilt LED
7. Central STOP Button
8. Backrest Adjustment Buttons
9. Thighrest Adjustment Buttons
10. Foot Control Lock LED
11. Mains Power LED
12. Accumulator LED
13. Bed Height Adjustment Buttons
14. GO Button
15. Emergency Trendelenburg Position Button
16. CPR (Resuscitation) Position Button
17. Cardiac Chair Position Button

Positioning buttons 1, 2, 3, 5, 8, 9, 13, 15, 16 and 17 are explained in **Bed Positioning** chapter.

GO BUTTON

The button  activates the keyboard of all Control Elements.

Pressing button  will keep the keyboard active for 3 minutes.

Pressing a button will keep the keyboard active for another 3 minutes.

During this time the following is possible:

- ▶ Adjusting individual Mattress support platform elements by pressing the corresponding positioning buttons.
- ▶ Disabling individual functions with the lock buttons.

NOTE: To activate CPR function ( button) the  button is not needed.

STOP BUTTON

Pressing button  immediately stops all electronic bed movements.

LOCK

To lock Backrest Adjustment:


- ▶ Press button .

LED Lock **6a**, **6b** and **6c** are flashing.

- ▶ During flashing of the LEDs press any button **8**.

LED Lock **6a** is lit.

Backrest Adjustment is disabled using any Control Element.

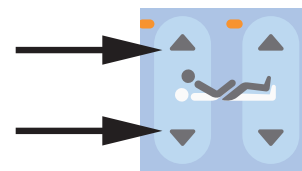

To lock Thighrest:


- ▶ Press button .

LED Lock **6a**, **6b** et **6c** are flashing.

- ▶ During flashing of the LEDs press any button **9**.

LED Lock **6b** is lit.

Thighrest is disabled using any Control Element.

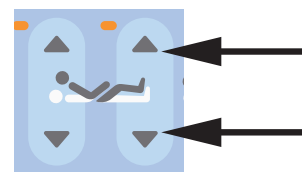

To lock Bed Height, Trendelenburg Tilt and Anti-Trendelenburg Tilt :


- ▶ Press button .

LED Lock **6a**, **6b** et **6c** are flashing.

- ▶ During flashing of the LEDs press any button **12**.

LED Lock **6c** is lit.

Bed Height, Trendelenburg Tilt and Anti-Trendelenburg Tilt are disabled using any Control Element.

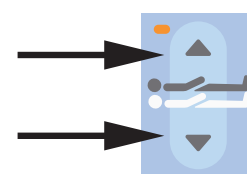

UNLOCK

To unlock Backrest Adjustment:


- ▶ Press button .

LED Lock **6a**, **6b** et **6c** are flashing.

- ▶ During flashing of the LEDs press any button **8**.

LED Lock **6a** is unlit.

Backrest Adjustment is enabled again.

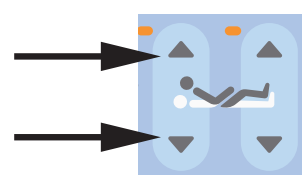

To unlock Thighrest:


- ▶ Press button .

LED Lock **6a**, **6b** et **6c** are flashing.

- ▶ During flashing of the LEDs press any button **9**.

LED Lock **6b** is unlit.

Thighrest is enabled again.

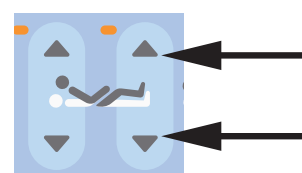

To unlock Bed Height, Trendelenburg Tilt and Anti-Trendelenburg Tilt:


- ▶ Press button .

LED Lock **6a**, **6b** et **6c** are flashing.

- ▶ During flashing of the LEDs press any button **12**.

LED Lock **6c** is unlit.

Bed Height, Trendelenburg Tilt and Anti-Trendelenburg Tilt are enabled again.

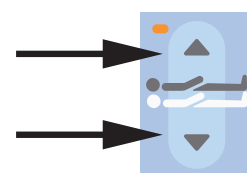

LOCKED FUNCTION SIGNALISATION

If **LED 6a** is lit, Backrest Adjustment is locked.

If **LED 6a** is not lit, Backrest Adjustment is unlocked.

If **LED 6b** is lit, Thighrest is locked.

If **LED 6b** is not lit, Thighrest is unlocked.

If **LED 6c** is lit, Bed Height and Trendelenburg Tilt and Anti-Trendelenburg Tilt are locked.

If **LED 6c** is not lit, Bed Height and Trendelenburg Tilt and Anti-Trendelenburg Tilt are unlocked.

ACCUMULATOR INDICATOR

Signalisation of Accumulator LED  is described in chapter **Accumulator**.

MAINS POWER LED

Status	Meaning
lit LED	connected to the mains
unlit LED	disconnected from the mains
flashing LED	system error

LIGHT CONTROL BUTTON

To turn off lights of the control elements:

- Press button .

To turn on lights of the control elements again:

- Press button .

11.4.3 iBoard Basic (optional)

The iBoard Basic is the optional Control Element for the caregivers. It is integrated in the outside of both head siderails. Only version with scales can be equipped with iBoard Basic. The iBoard Basic serves for control of scales and control of Bed Exit Monitoring.

- Ensure that exclusively trained nursing staff operates the iBoard Basic.

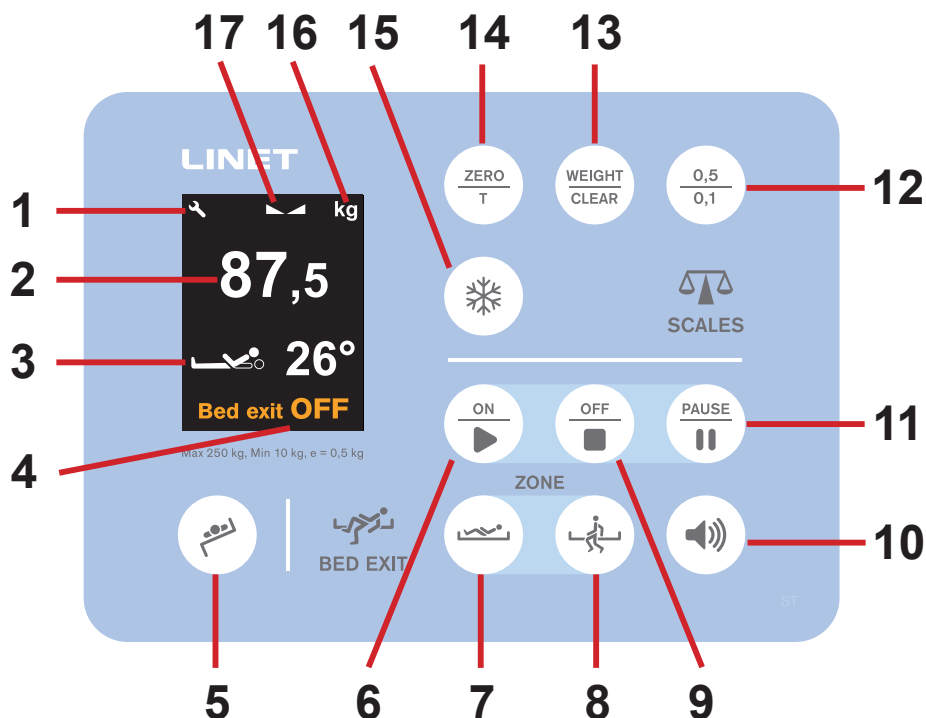


Fig. iBoard Basic

1. Service check required Icon
2. Weight value
3. Backrest Angle Indicator
4. OFF icon (Bed Exit Monitoring is OFF)
5. Transfer Mode Button
6. ON Button - Bed Exit Monitoring
7. Inner Zone Button - Bed Exit Monitoring
8. Outer Zone Button - Bed Exit Monitoring
9. OFF Button - Bed Exit Monitoring
10. Volume Icon (3 levels)
11. PAUSE Button - Bed Exit Monitoring
12. Scale interval switch button (0,5kg/0,1kg)
13. WEIGHT/CLEAR Button (cancel)
14. ZERO/T Button (tare or zero scales)
15. HOLD Button
16. Unit of weight (kg)
17. Stabilized Scales Icon

TRANSFER MODE (5)

Transfer Mode enables Lateral Tilt with a siderail in down position for 5 minutes.

To set Transfer Mode:

- Press  button.

Lateral Tilt icon on the iBoard Basic display or iBoard Standard display is flashing.

AUDIBLE MESSAGES (VOICE)

To set language of the audible messages on the iBoard Basic:

- ▶ Press and hold ZERO button and HOLD button at the same time. Setting menu will appear on the display.
- ▶ Press VOLUME icon to select VOICE.
- ▶ Press ZERO button or HOLD button to select a required language.

List of the available languages:

EN	English
DE	German
CZ	Czech
FR	French
IT	Italian
ES	Spanish
SE	Swedish
NL	Dutch
PT	Portuguese
FI	Finnish
RO	Romanian
NO	Norwegian
LV	Latvian
HU	Hungarian
LT	Lithuanian
PL	Polish
OFF	function deactivated

- ▶ Confirm the selection by pressing the WEIGHT / CLEAR button.

Scales

Scales functions are described in chapter **Scales Control**.

Bed Exit Monitoring

Functions of Bed Exit Monitoring are described in chapter **Bed Exit Monitoring**.

Statuses (iBoard Basic)

Signalisation	Meaning	Required Action
	Function locked.	Unlock function if required!
	GO Button not activated.	Press GO Button!
SIDERAIL	Lateral Tilt disabled when siderail folded down.	Raise siderail up to enable additional Lateral Tilting.
COLLISION	Antitrendelenburg Tilt and Trendelenburg Tilt disabled during Lateral Tilt.	Undo the Lateral Tilt to continue with Antitrendelenburg Tilt or Trendelenburg Tilt.
	Positioning blocked to avoid collision of the bed with floor or collision of the bed with bed equipments.	To continue in positioning, adjust the bed so that there is no collision.
0°	Horizontal position was reached during tilting.	Press corresponding button to continue in positioning.
X-RAY	Incorrectly inserted X-Ray Cassette Holder.	Insert X-Ray Cassette Holder correctly!
OVERLOAD + 	Safe Working Load exceeded (more than 10 kg over Safe Working Load).	Remove load!
	Lateral Tilt disabled (Load more than 200 kg).	Remove load to enable Lateral Tilting again!
STOP SERVICE +  + 	System Fatal Error.	Contact service department approved by manufacturer.
FAULT COLUMN + 	Column Unit Error.	Contact service department approved by manufacturer.
SCALE / BEA DISCONNECTED +  + 	Scale module disconnected and Bed Exit Monitoring disabled.	Contact service department approved by manufacturer.

Signalisation	Meaning	Required Action
SAFE STOP + 	Movement of the Mattress support platform stopped by function Safestop.	Remove an object from undercarriage to continue in adjusting the bed height.
	Insufficient load for Bed Exit Monitoring.	Place patient on the bed to enable the Bed Exit Monitoring.
	Disconnected from the mains power supply.	Connect bed to the mains power.
BED EXIT +  ALERT + 	The patient has left the bed (Outer Zone monitoring) or the patient has left the Inner Zone (Inner Zone monitoring).	Check the patient and turn off the Bed Exit Alert.
HIGH	Safe Working Load exceeded (from 4,5 kg to 10 kg over Safe Working Load).	Remove load!
LOW	The bed is underloaded.	Ensure the mattress support platform is not lifted inconveniently by something and the scales are appropriately tared.

11.4.4 Attendant Control Panel

The Attendant Control Panel is a standard Control Element. The Attendant Control Panel can be hung on the foot board or on side-rails if required. It is possible to hold the Attendant Control Panel in the hand while operating.

- Ensure that exclusively trained nursing staff operates the Attendant Control Panel.

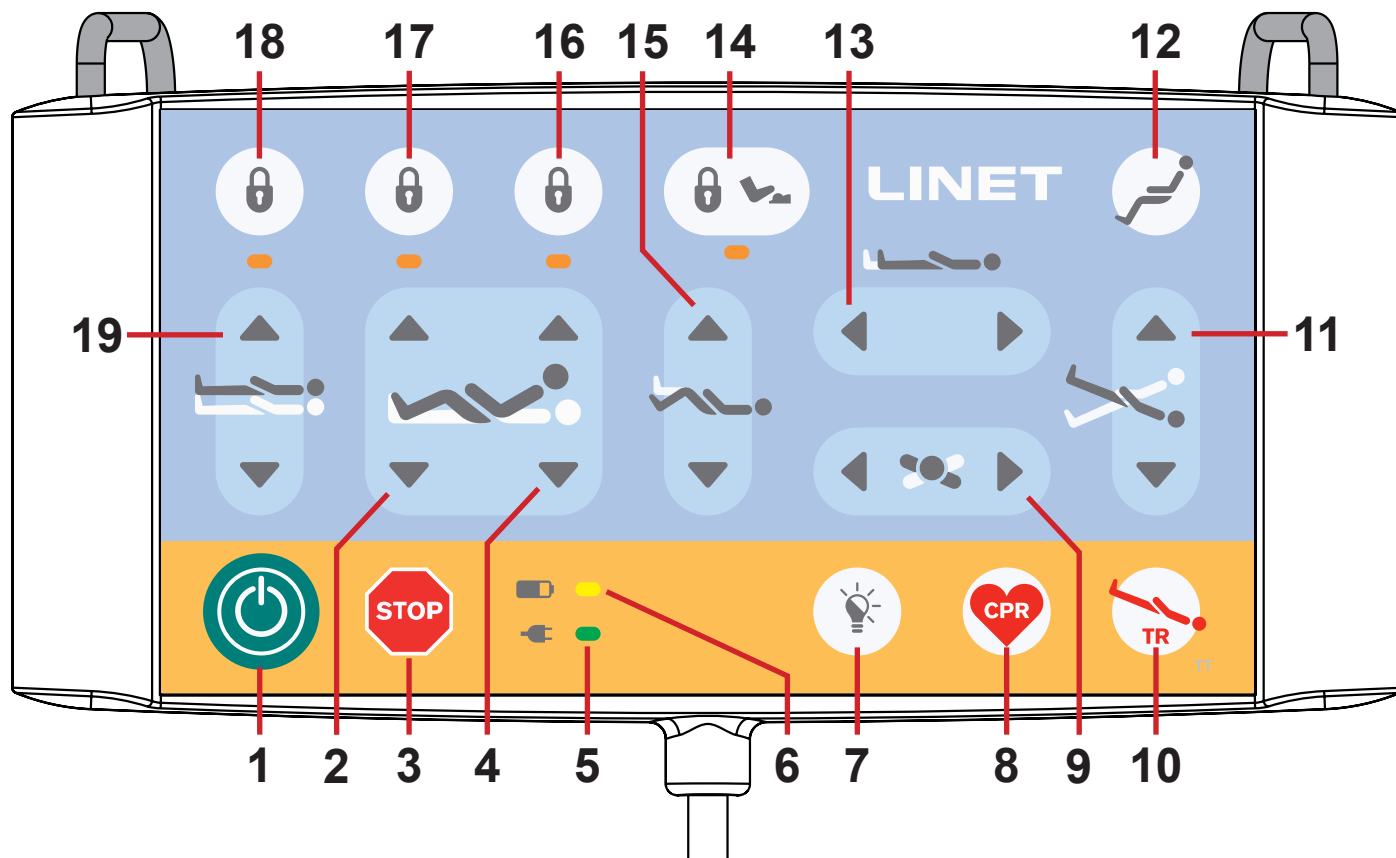


Fig. Attendant Control Panel

1. GO Button
2. Thighrest Adjustment Buttons
3. Central STOP Button
4. Backrest Adjustment Buttons
5. Mains Power LED
6. Accumulator Charge Status LED
7. Illumination Control Button
8. CPR Position Button
9. Lateral Tilt Buttons
10. Emergency Trendelenburg Position Button
11. Longitudinal Tilt Buttons
12. Cardiac Chair Position Button
13. Bed Extension Adjustment Buttons
14. Foot Control Lock Button and LED
15. Calfrest Adjustment Buttons
16. Backrest Lock Button and LED
17. Thighrest, Calfrest and Bed Extension Adjustment Lock Button and LED
18. Bed Height, Lateral Tilt, Trendelenburg Tilt and Antitrendelenburg Tilt Lock Button and LED
19. Bed Height Adjustment Buttons

Activating GO Button

The button  activates the keyboard of all control elements for 3 minutes.

The function of the GO button is identical on all control elements.

During this time the following is possible:

- ▶ Adjusting individual mattress support platform elements by pressing the corresponding function buttons.
- ▶ Disabling individual functions with the lock buttons.

Each time a function button is pressed, the keyboard will remain active for another 3 minutes.

Function Buttons

The function buttons **2, 4, 8, 9, 10, 11, 12, 13** and **15** are described in chapter **Bed Positioning**.

NOTE Pressing two function buttons at the same time will be recognized as an error by the controller. The controller will interrupt immediately all bed movements and display shows alert.

Lock

To lock Backrest Adjustment:

- ▶ Press button **16**.

Corresponding LED on Attendant Control Panel and on iBoard Standard (**5**) are lit.

Backrest Adjustment is disabled using any Control Element.

To lock Thighrest, Calfrest and Bed Extension Adjustment:

- ▶ Press button **17**.

Corresponding LED on Attendant Control Panel and on iBoard Standard (**10**) are lit.

Thighrest, Calfrest and Bed Extension Adjustment are disabled using any Control Element.

To lock Bed Height, Lateral Tilt, Trendelenburg Tilt and Antitrendelenburg Tilt Adjustment:

- ▶ Press button **18**.

Corresponding LED on Attendant Control Panel and on iBoard Standard (**17**) are lit.

Bed Height, Lateral Tilt, Trendelenburg Tilt and Antitrendelenburg Tilt Adjustment are disabled using any Control Element.

Unlock

To unlock Backrest Adjustment:

- ▶ Press button **16**.

Corresponding LED on Attendant Control Panel and on iBoard Standard (**5**) are not lit.

Backrest Adjustment is enabled again.

To unlock Thighrest, Calfrest and Bed Extension Adjustment:

- ▶ Press button **17**.

Corresponding LED on Attendant Control Panel and on iBoard Standard (**10**) are not lit.

Thighrest, Calfrest and Bed Extension Adjustment are enabled again.

To unlock Bed Height, Lateral Tilt, Trendelenburg Tilt and Antitrendelenburg Tilt Adjustment:

- ▶ Press button **18**.

Corresponding LED on Attendant Control Panel and on iBoard Standard (**17**) are not lit.

Bed Height, Lateral Tilt, Trendelenburg Tilt and Antitrendelenburg Tilt Adjustment are enabled again.

MAINS POWER LED



Status	Meaning
lit LED	connected to the mains
unlit LED	disconnected from the mains
flashing LED	system error

Bed and Keyboards Illumination

Bed and Keyboards Illumination helps the nursing staff as well as the patient to orientate. The illumination is turned off when the bed is powered by accumulator.

The Eleganza 5 bed is equipped with three-phase illumination:

1. Full intensity of illumination
2. Lowered intensity of illumination
3. Illumination is turned off

The lowered intensity of illumination is set up after the bed is connected to the mains.

After pressing the GO button:

- ▶ The Patient Control Panels and Handset keyboards and Bed Illumination will light up at full intensity.

After elapsing the GO period:

- ▶ The illumination intensity of Patient Control Panels and Handset keyboards will be lowered.

7 minutes after GO period has elapsed:

- ▶ The intensity of Bed Illumination will be lowered.

In the event any button is pressed on the Attendant Control Panel, Handset or Patient Control Panels outside of GO period:

- ▶ The Bed Illumination will light up at full intensity for 10 minutes and after that the illumination will be lowered.
- ▶ Simultaneously, the Patient Control Panels and Handset keyboards will light up at full intensity for 7 seconds. Then the illumination will be lowered.

After activating the bed via any Foot Control:

- ▶ The full intensity of Bed Illumination will be activated for 9 minutes and 40 seconds.

In case of pressing any Foot Control once outside the period of activation:

- ▶ All the Patient Control Panels and Handset keyboards will light up at full intensity for 7 seconds and then the illumination will be lowered.
- ▶ The full intensity of Bed Illumination will be activated for 10 minutes and then the illumination will be lowered.

Turning off all illumination:

- ▶ Press **Illumination Control Button (7)** on the Attendant Control Panel and all illumination on the bed (Patient Control Panels and Handset keyboards and Bed Illumination) will be turned off. This function is not blocked by the preceding activation by the GO button.

11.4.5 Handset (optional)

The handset is available with illuminated keyboard.

The illumination is activated for 7s if any button was pressed and the illumination is activated for 3 minutes if GO Button was pressed.

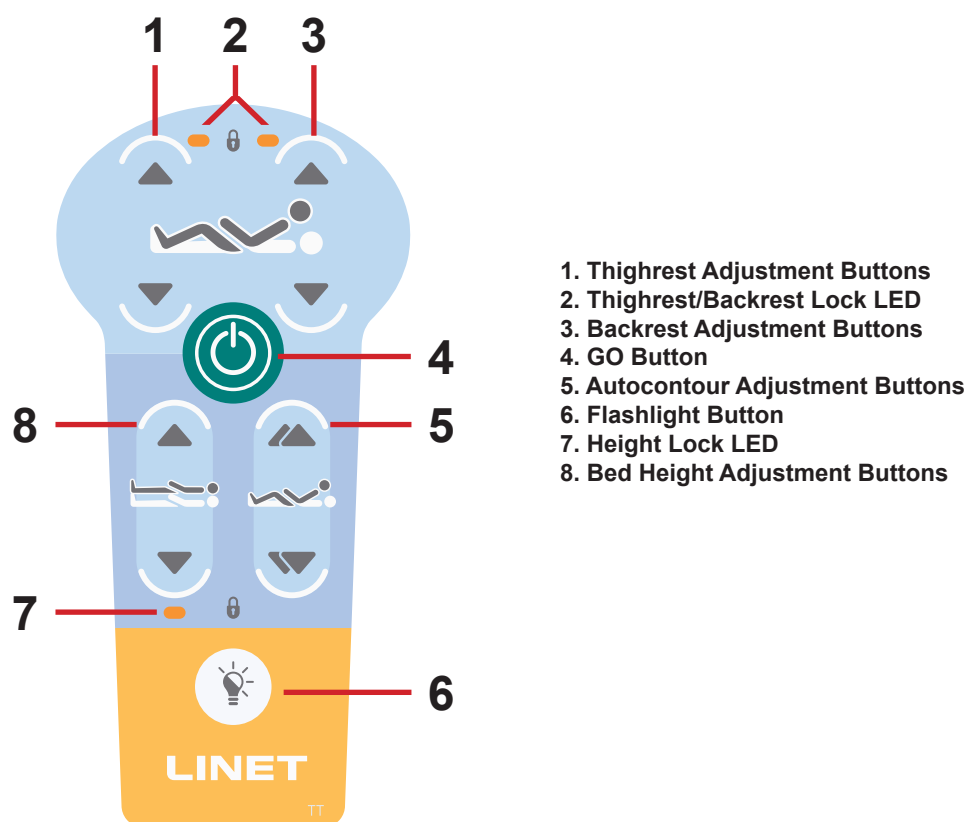


Fig. Handset

GO BUTTON

The button  activates the keyboard of all Control Elements, except for Lateral Tilt Foot Control.

Pressing button  will keep the keyboard active for 3 minutes.

Pressing a button will keep the keyboard active for another 3 minutes.

FUNCTION BUTTONS

The function buttons 1, 3, 5 and 8 are described in **Bed Positioning** chapter.

To switch on the flashlight:

- Press  flashlight button.

NOTE Depending on the patient's condition, the nursing staff decides whether the patient is allowed to adjust the bed's position.

If required, prevent the patient from adjusting the bed as follows:

- Disable functions.

NOTE An adapter for the handset is available. The adapter enables quick installation and removal (e.g. replacing a defective handset, using the handset for another bed).

11.4.6 Patient Control Panels

The Patient Control Panels integrated in the foot siderails allow the patient to adjust the positions of the Backrest, Thighrest and Autocontour. Optionally additional Patient Control Panel is situated in the inner side of head siderails.

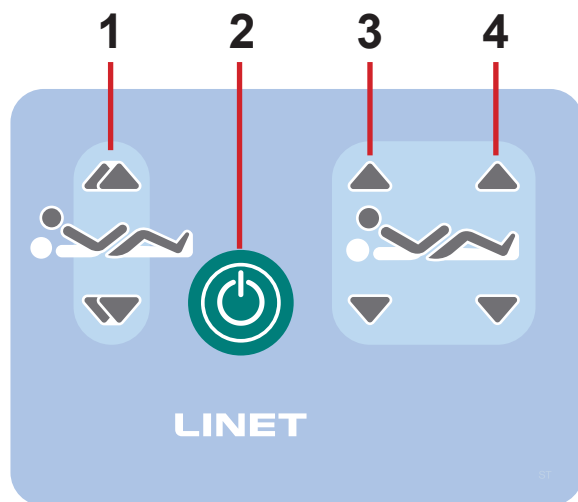


Fig. Patient Control Panel in the foot siderail

1. Autocontour Adjustment Button (simultaneous movement of the Backrest and Thighrest)
2. GO Button
3. Backrest Adjustment Button
4. Thighrest Adjustment Button

NOTE Keyboards are illuminated. The illumination is activated for 7s if any button was pressed and the illumination is activated for 3 minutes if GO Button was pressed.

NOTE Functions on the Patient Control Panels in the foot siderails are disabled when the foot siderail is in lower position.

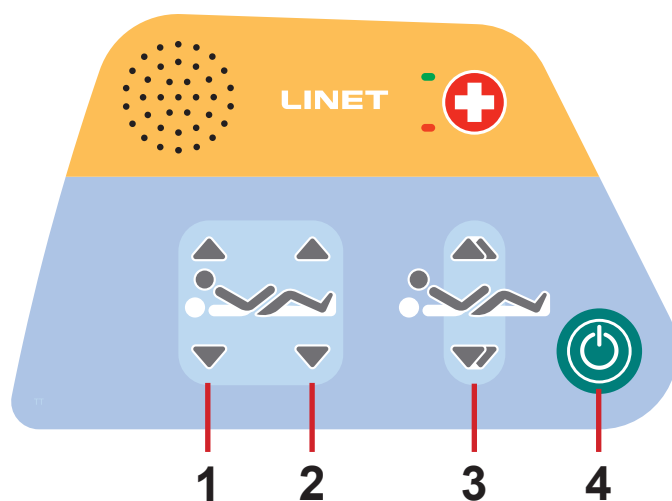


Fig. Patient Control Panel in the head siderails (optional)

1. Backrest Adjustment Button
2. Thighrest Adjustment Button
3. Autocontour Button (simultaneous movement of the Backrest and Thighrest)
4. GO Button

11.4.7 Bed Height Foot Control (optional)

The foot control is optional and allows setting the Height of the bed with one's feet.
Press the selected pedal twice in 3 seconds. Bed Height Foot Control is activated for 20s after this procedure.

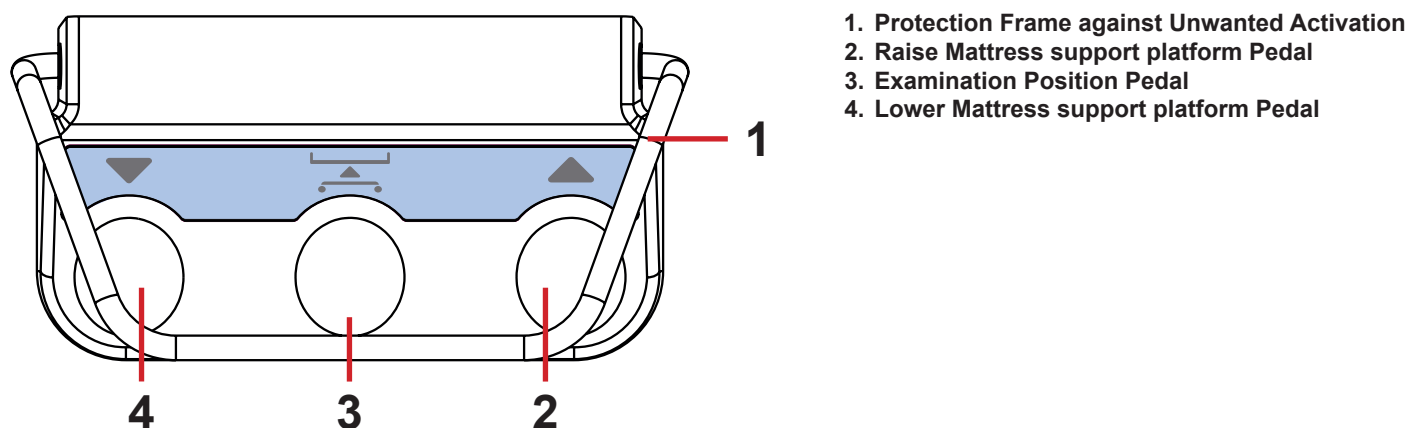


Fig. Bed Height Foot Control

The use of Bed Height Foot Control is described in the **Bed Positioning** chapter.

11.4.8 Lateral Tilt Foot Control (standard)

The foot control allows setting the Lateral Tilt of the bed with the feet.
Press GO pedal to activate the foot control.


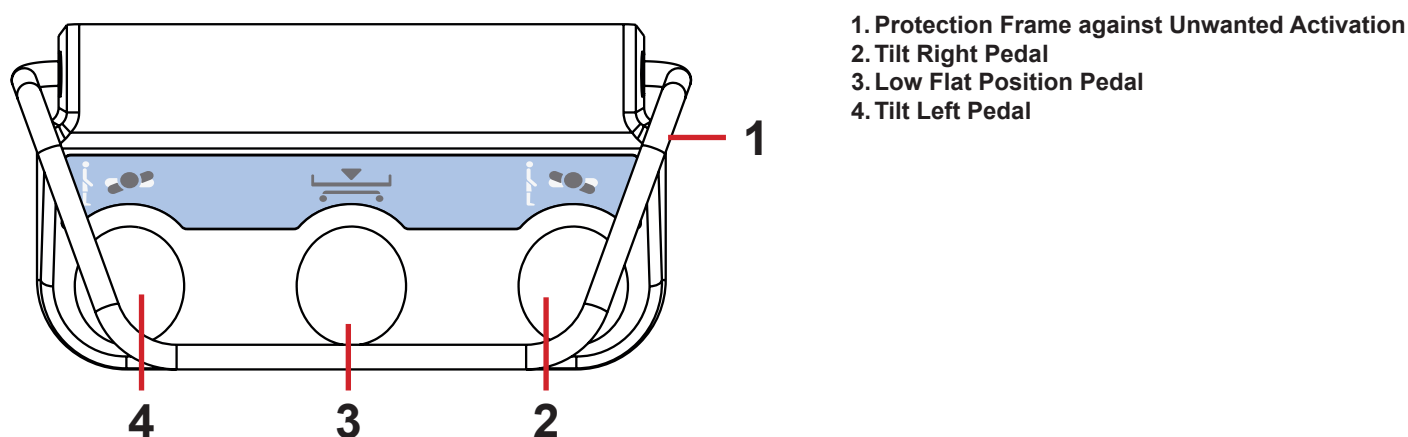


Fig. Lateral Tilt Foot Control

The use of Lateral Tilt Foot Control is described in the **Bed Positioning** chapter.

11.5 Bed Positioning

11.5.1 Backrest

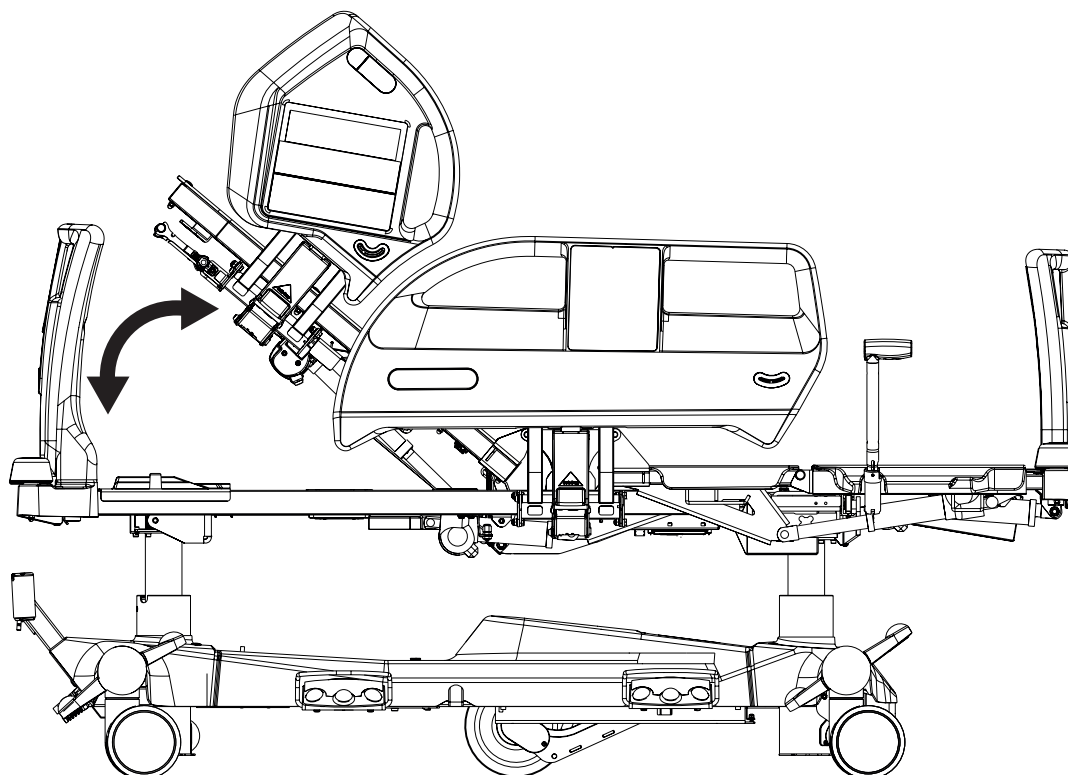

To position Backrest use:


- ▶ iBoard Standard
- ▶ Nurse Control Panel
- ▶ Attendant Control Panel
- ▶ Handset
- ▶ Patient Control Panel (in foot siderail)
- ▶ Patient Control Panel (in head siderail)



Fig. Backrest Angle on iBoard Standard Display

iBoard Standard Display shows Backrest Angle.

During continuous positioning Backrest stops automatically in 30 and 45 degrees (the beep will appear). To continue in positioning press corresponding button once more.

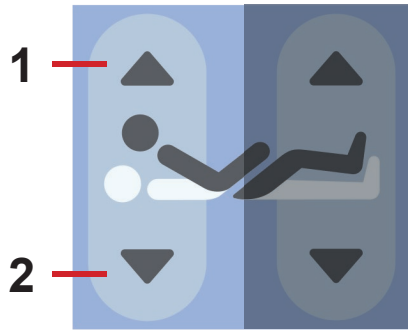

*Fig. Backrest Adjustment Button
(iBoard Standard, Patient Control Panels)*

1. Backrest Up

2. Backrest Down

iBoard Standard:


- ▶ Press button .
- ▶ Press selected part of Backrest Adjustment Button until intended position is reached.

Patient Control Panel (in foot siderail):

- ▶ Press button .
- ▶ Press selected part of Backrest Adjustment Button until intended position is reached.

Patient Control Panel (in head siderail):


- ▶ Press button .
- ▶ Press selected part of Backrest Adjustment Button until intended position is reached.

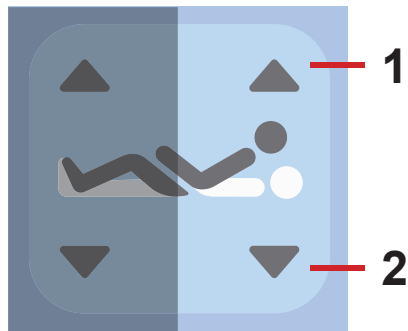

*Fig. Backrest Adjustment Buttons
(Attendant Control Panel)*

Attendant Control Panel:

- ▶ Press button .
- ▶ Press selected part of Backrest Adjustment Button until intended position is reached.

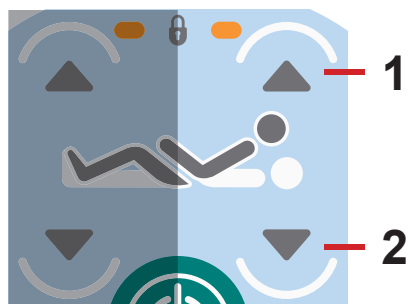


*Fig. Backrest Adjustment Buttons
(Handset)*

Handset:

- ▶ Press button .
- ▶ Press selected part of Backrest Adjustment Button until intended position is reached.

11.5.2 Thighrest

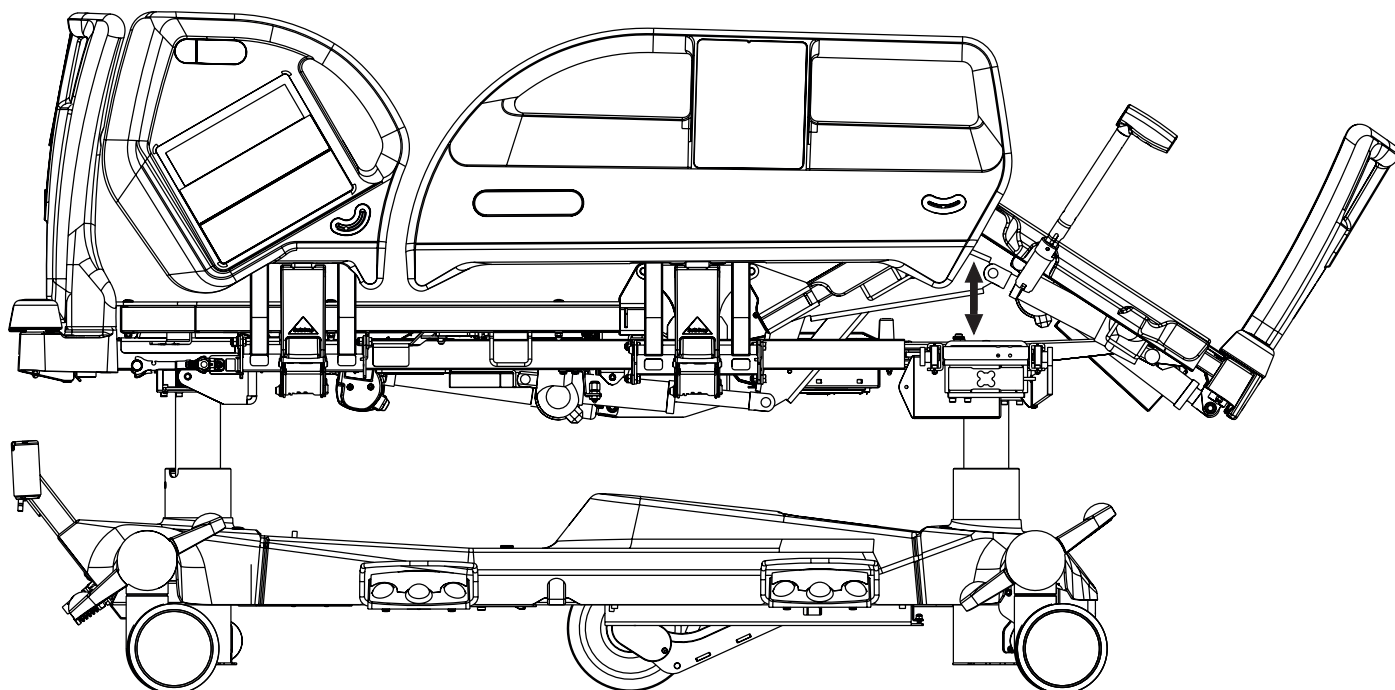

To position Thighrest use:

- ▶ iBoard Standard
- ▶ Nurse Control Panel
- ▶ Attendant Control Panel
- ▶ Handset
- ▶ Patient Control Panel (in foot siderail)
- ▶ Patient Control Panel (in head siderail)

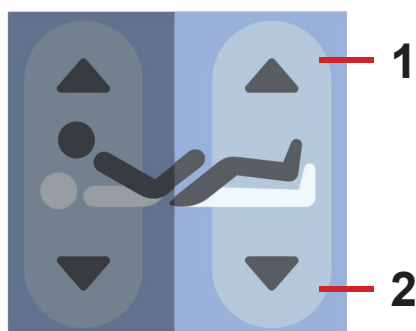


Fig. Thighrest Adjustment Button (iBoard Standard, Patient Control Panels)

1. Thighrest Up

2. Thighrest Down

iBoard Standard:


- ▶ Press button .
- ▶ Press selected part of Thighrest Adjustment Button until intended position is reached.

Patient Control Panel (in foot siderail):

- ▶ Press button .
- ▶ Press selected part of Thighrest Adjustment Button until intended position is reached.

Patient Control Panel (in head siderail):


- ▶ Press button .
- ▶ Press selected part of Thighrest Adjustment Button until intended position is reached.

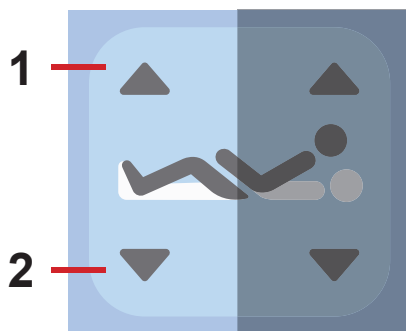

*Fig. Thighrest Adjustment Buttons
(Attendant Control Panel)*

Attendant Control Panel:

- ▶ Press button .
- ▶ Press selected part of Thighrest Adjustment Button until intended position is reached.

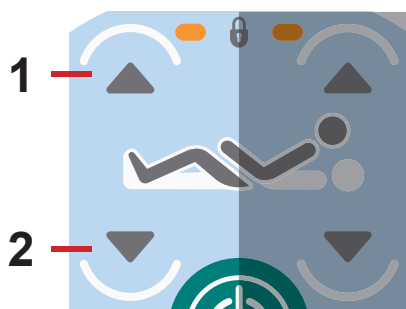

*Fig. Thighrest Adjustment Buttons
(Handset)*


Handset:

- ▶ Press button .
- ▶ Press selected part of Thighrest Adjustment Button until intended position is reached.

1. Thighrest Up

2. Thighrest Down

11.5.3 Calfrest

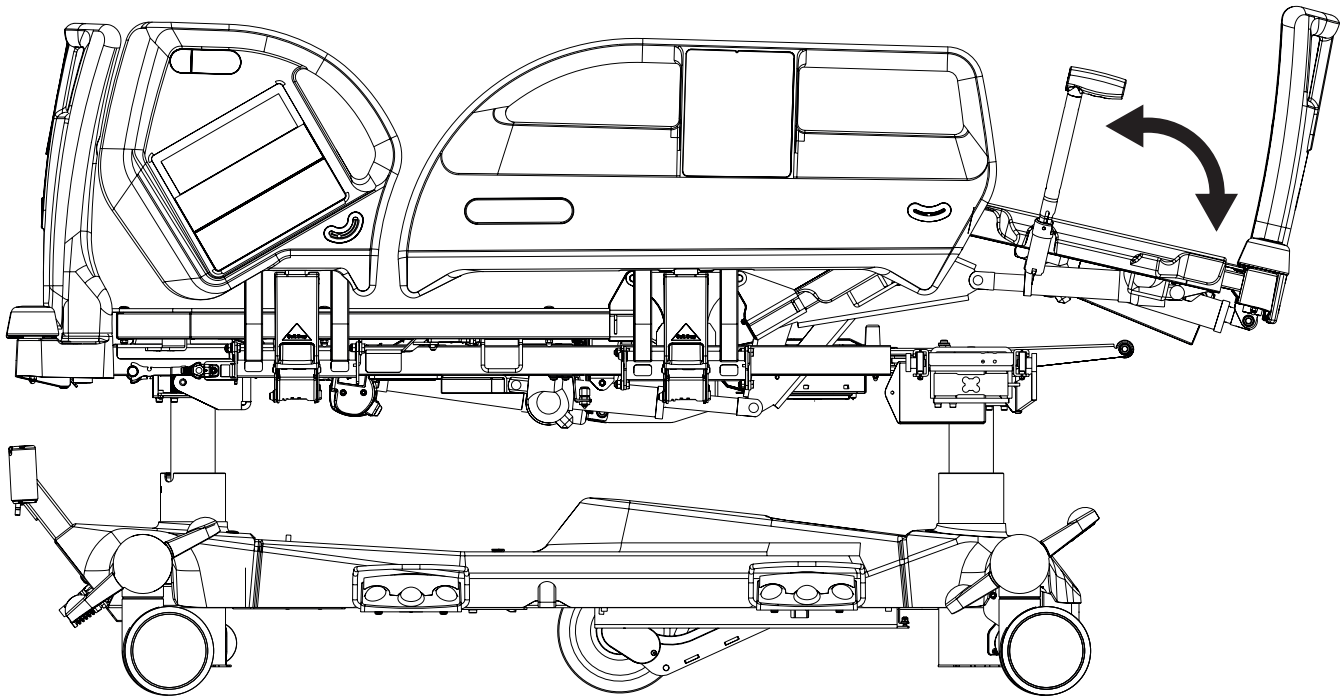

To position Calfrest position Thighrest firstly.

To position Calfrest use:

- iBoard Standard
- Attendant Control Panel

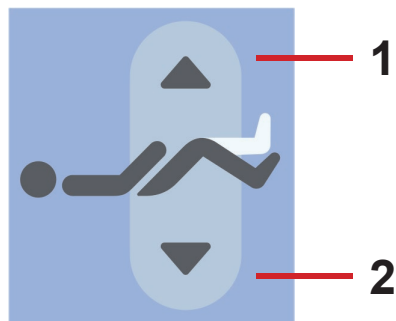


Fig. Calfrest Adjustment Button
(iBoard Standard)

1. Calfrest Up

2. Calfrest Down

iBoard Standard:

- Press button .
- Press selected part of Calfrest Adjustment Button until intended position is reached.


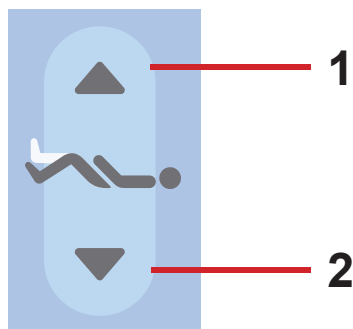


Fig. Calfrest Adjustment Buttons
(Attendant Control Panel)

Attendant Control Panel:

- Press button .
- Press selected part of Calfrest Adjustment Button until intended position is reached.

11.5.4 Bed Height

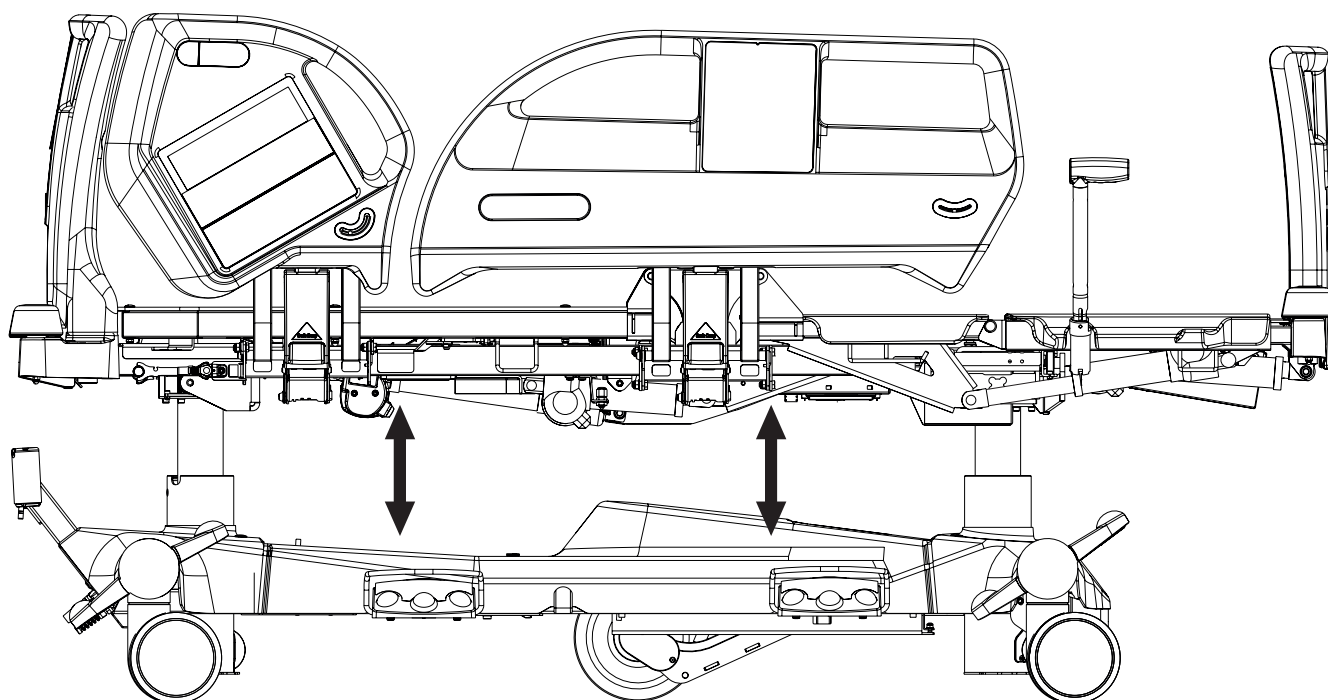

To position Bed Height use:

- ▶ iBoard Standard
- ▶ Nurse Control Panel
- ▶ Attendant Control Panel
- ▶ Hanset
- ▶ Bed Height Foot Control
- ▶ Mobi-Lift

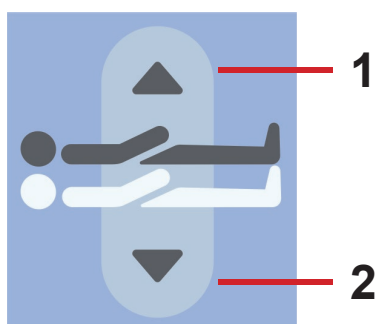


Fig. Bed Height Adjustment Button (iBoard Standard)

iBoard Standard:

- ▶ Press button .
- ▶ Press selected part of Bed Height Adjustment Button until intended position is reached.

1. Mattress support platform Up

2. Mattress Platform Down

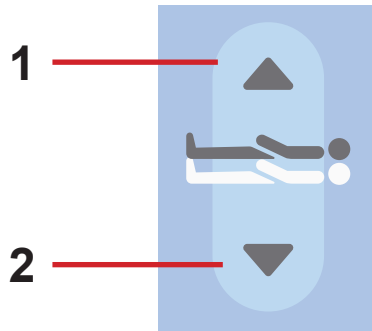


Fig. Bed Height Adjustment Buttons (Attendant Control Panel)

Attendant Control Panel:

- ▶ Press button .
- ▶ Press selected part of Bed Height Adjustment Button until intended position is reached.

1. Mattress support platform Up

2. Mattress Platform Down

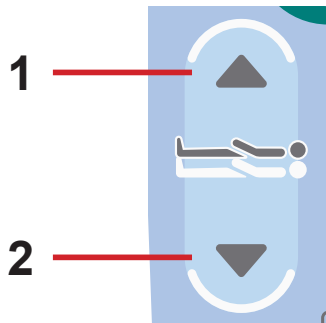


Fig. Bed Height Adjustment Button (Handset)

Handset:

- ▶ Press button .
- ▶ Press selected part of Bed Height Adjustment Button until intended position is reached.



Fig. Bed Height Pedals (Bed Height Foot Control)

Bed Height Foot Control:

- ▶ Press the selected Bed Height Pedal and release it.
- ▶ Press and hold selected Bed Height Pedal once more until intended position is reached.

NOTE: Press the selected pedal twice in 3 seconds. Bed Height Foot Control is activated for 20s after this procedure.


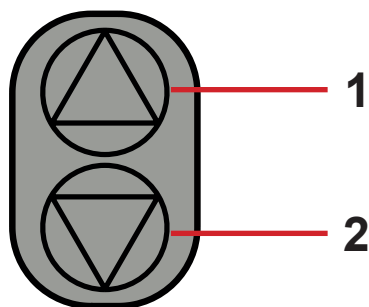


Fig. Mobi-Lift Bed Height Button

Mobi-Lift:

- ▶ Press button .
- ▶ Press selected part of Mobi-Lift Bed Height Button until intended position is reached.

11.5.5 Autocontour

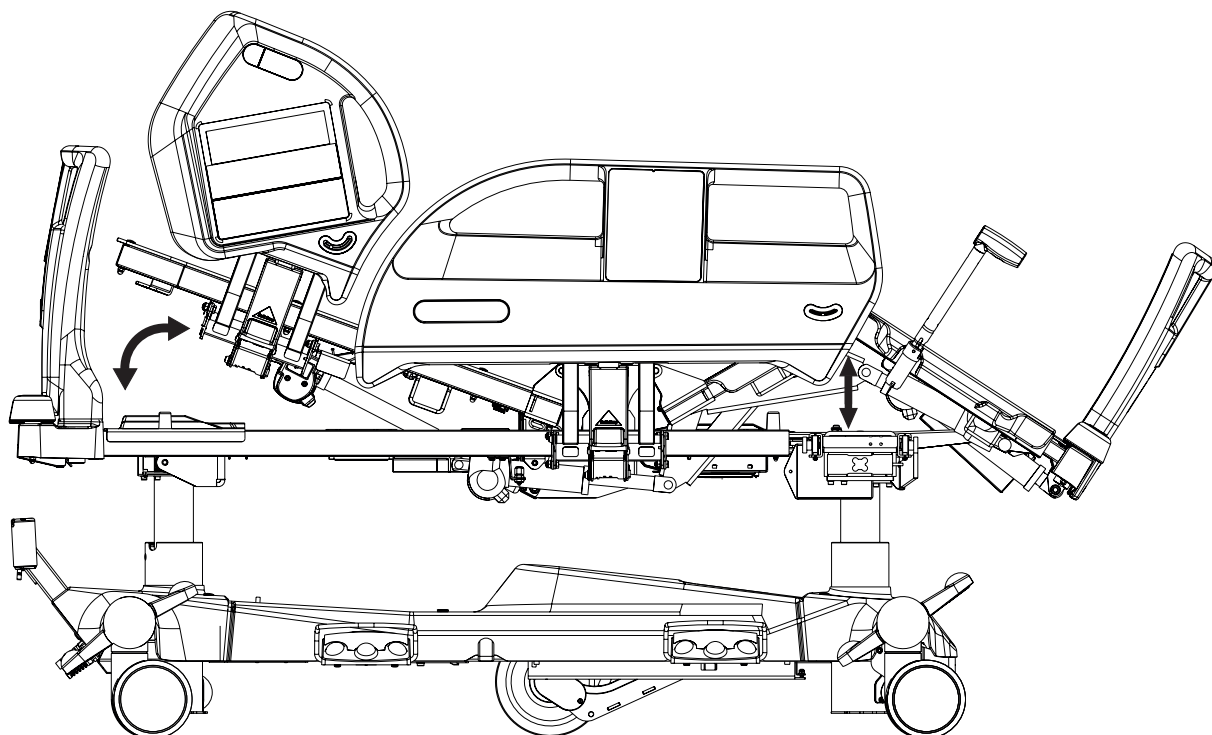

To position Autocontour use:

- ▶ iBoard Standard
- ▶ Nurse Control Panel
- ▶ Handset
- ▶ Patient Control Panel (in foot siderail)
- ▶ Patient Control Panel (in head siderail)

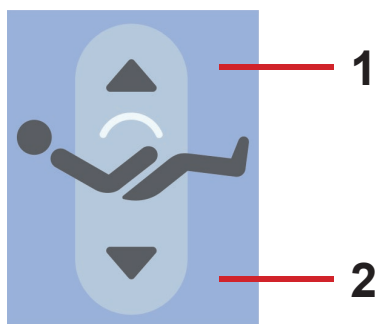


Fig. Autocontour Adjustment Button
(iBoard Standard, Patient Control
Panels)

1. Autocontour Up

2. Autocontour Down

iBoard Standard:


- ▶ Press button .
- ▶ Press selected part of Autocontour Adjustment Button until intended position is reached.

Patient Control Panel (in foot siderail):

- ▶ Press button .
- ▶ Press selected part of Autocontour Adjustment Button until intended position is reached.

Patient Control Panel (in head siderail):

- ▶ Press button .
- ▶ Press selected part of Autocontour Adjustment Button until intended position is reached.

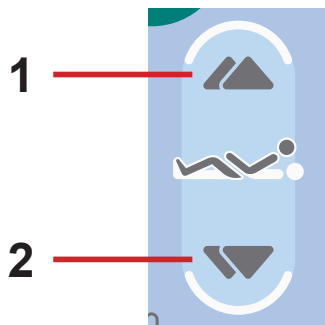

Handset:

- ▶ Press button .
- ▶ Press selected part of Autocontour Adjustment Button until intended position is reached.

Fig. Autocontour Adjustment Buttons (Handset)

1. Autocontour Up

2. Autocontour Down

11.5.6 Emergency Trendelenburg Position

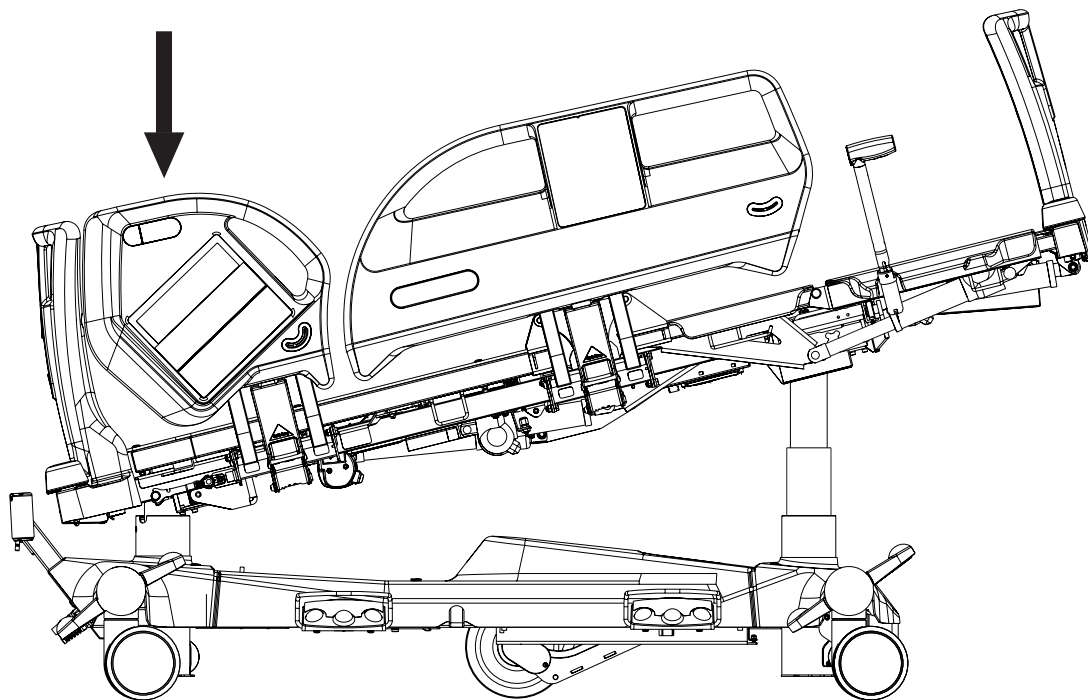


Fig. Trendelenburg Position

Trendelenburg position provides anti-shock conditions for the patient. During Trendelenburg Position Mattress support platform is straightened in the tilt.

To position Emergency Trendelenburg Position use:

- ▶ Nurse Control Panel
- ▶ Attendant Control Panel

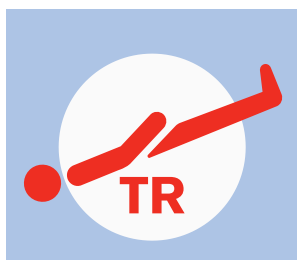


Fig. Emergency Trendelenburg Position Button (Nurse Control Panel)

Nurse Control Panel:

- ▶ Press button  .
- ▶ Press Emergency Trendelenburg Position Button until intended position is reached.



Fig. Emergency Trendelenburg Position Button (Attendant Control Panel)

Attendant Control Panel:

- ▶ Press button  .
- ▶ Press Trendelenburg Position Button until intended position is reached.

11.5.7 Antitrendelenburg and Trendelenburg Tilt

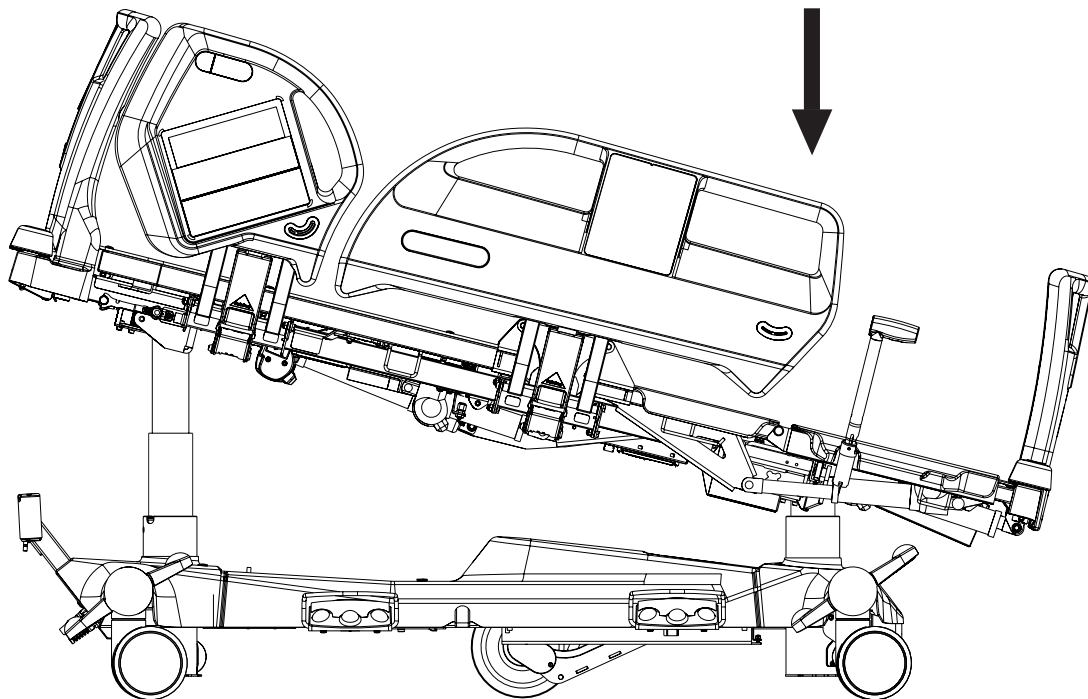


Fig. Antitrendelenburg Position

To position Trendelenburg or Antitrendelenburg Tilt use:

- ▶ iBoard Standard
- ▶ Nurse Control Panel
- ▶ Attendant Control Panel

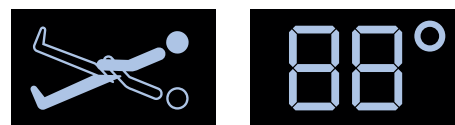


Fig. Tilt Angle on iBoard Standard Display

iBoard Standard Display shows Tilt Angle.



Fig. Trendelenburg Tilt Button (iBoard Standard)

iBoard Standard:

- ▶ Press button .
- ▶ Press Trendelenburg Tilt Button until intended position is reached.



Fig. Antitrendelenburg Tilt Button (iBoard Standard)

iBoard Standard:

- ▶ Press button .
- ▶ Press Antitrendelenburg Tilt Button until intended position is reached.

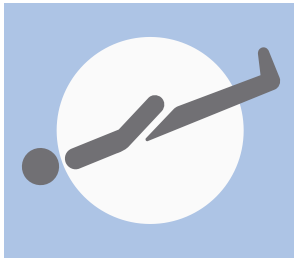


Fig. Trendelenburg Tilt Button
(Nurse Control Panel)

Nurse Control Panel:

- ▶ Press button .
- ▶ Press Trendelenburg Tilt Button until intended position is reached.

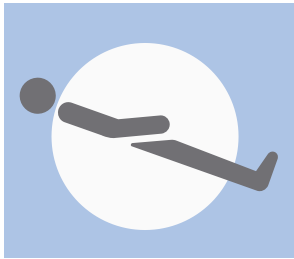


Fig. Anti-Trendelenburg Tilt Button
(Nurse Control Panel)

Nurse Control Panel:

- ▶ Press button .
- ▶ Press Anti-Trendelenburg Tilt Button until intended position is reached.

1. Antitrendelenburg Tilt

2. Trendelenburg Tilt


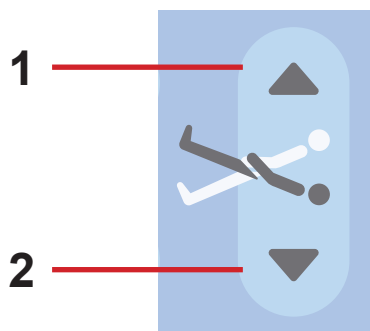


Fig. Longitudinal Tilt Buttons
(Attendant Control Panel)

Attendant Control Panel:

- ▶ Press button .
- ▶ Press selected part of Trendelenburg and Antitrendelenburg Tilt Button until intended position is reached.

11.5.8 Examination Position

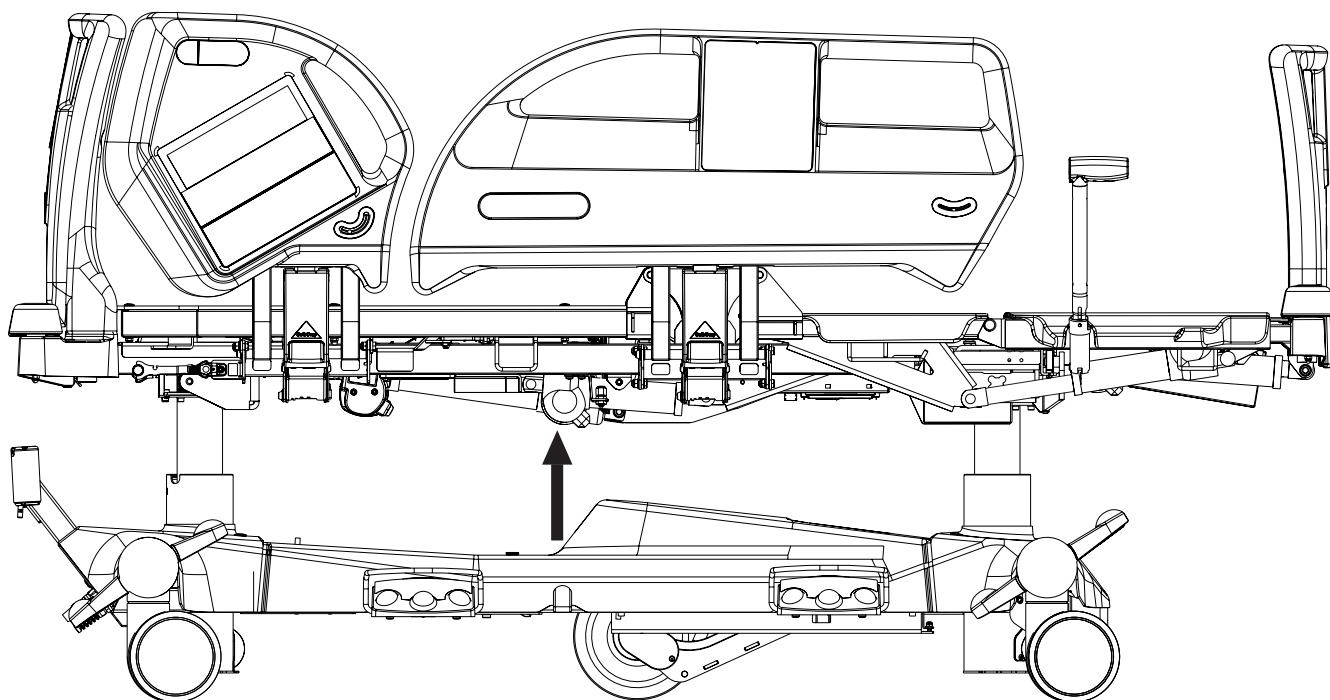

To position Examination Position use:

- Bed Height Foot Control




Fig. Examination Position Pedal
(Bed Height Foot Control)

Bed Height Foot Control:

- Press Examination Position Pedal and release it.
- Press and hold Examination Position Pedal once more until intended position is reached.


NOTE: Bed Height Foot Control is activated for 30s after this procedure.

11.5.9 Bed Extension

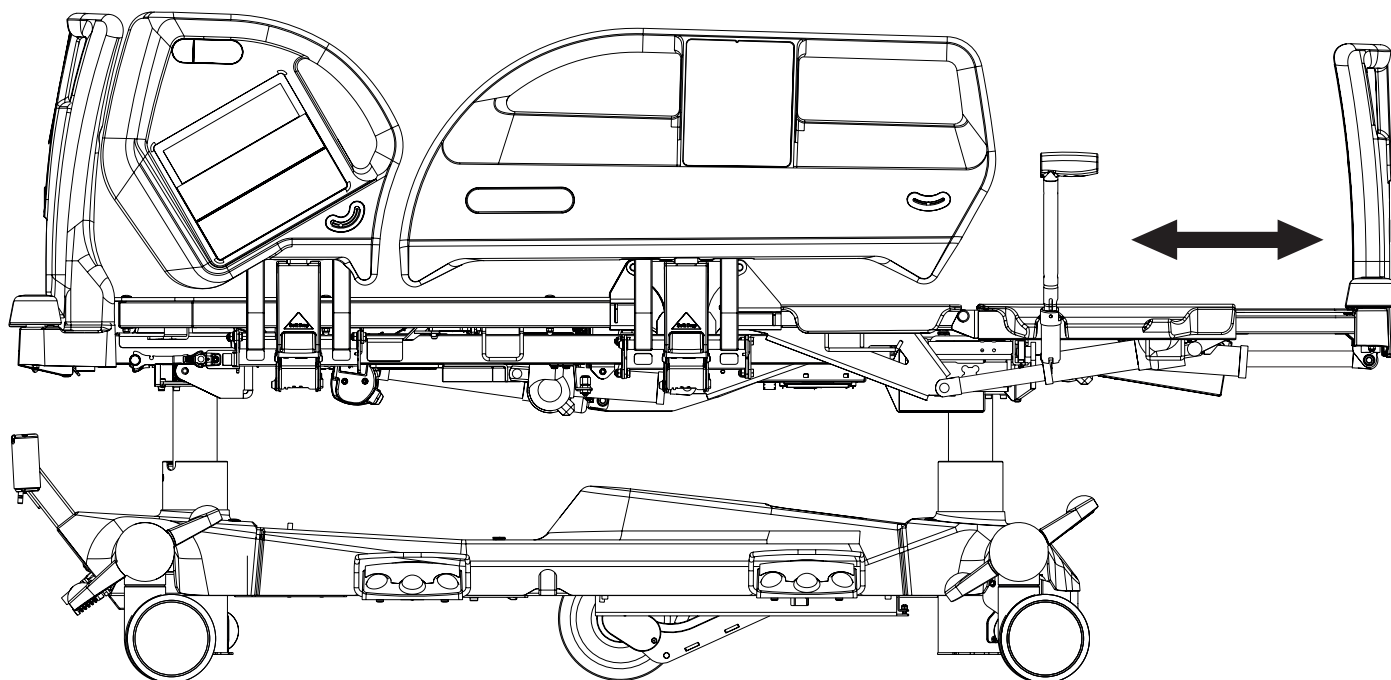

To position Bed Extension use:

- Attendant Control Panel

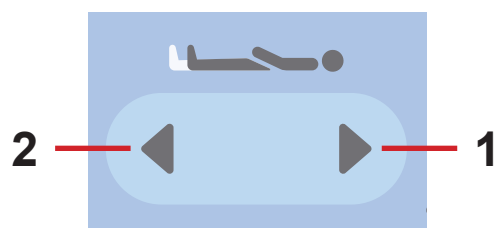

*Fig. Bed Extension Adjustment Buttons
(Attendant Control Panel)*


Attendant Control Panel:

- Press button .
- Press selected part of Bed Extension Adjustment Button until intended position is reached.

1. Longer Mattress Support Platform

2. Shorter Mattress Support Platform

11.5.10 CPR Position

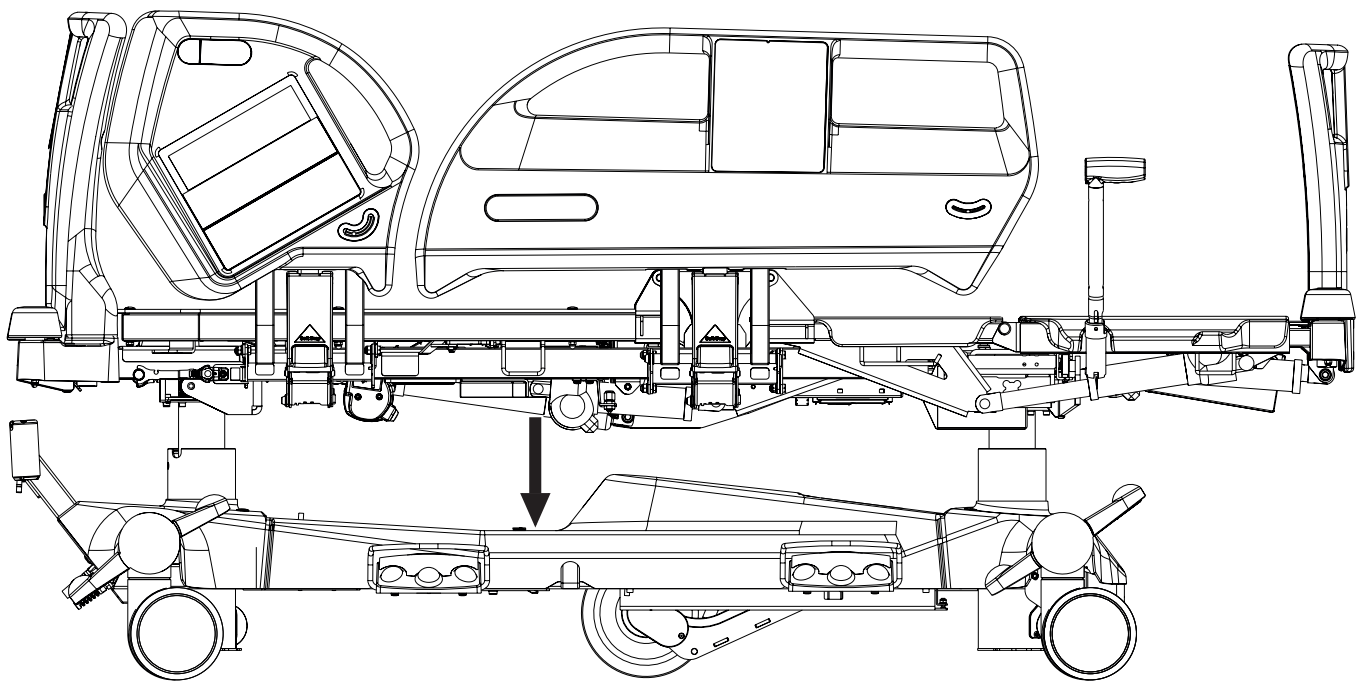

In CPR Position bed reaches flat Mattress support platform.

If the bed is equipped with Opticare mattress, pressing CPR Button will also deflate the mattress.

To position CPR Position use:

- ▶ iBoard Standard
- ▶ Nurse Control Panel
- ▶ Attendant Control Panel



Fig. CPR Button (iBoard Standard, Attendant Control Panel)

iBoard Standard:

- ▶ Press CPR Position Button until intended position is reached.

Attendant Control Panel:

- ▶ Press CPR Position Button until intended position is reached.

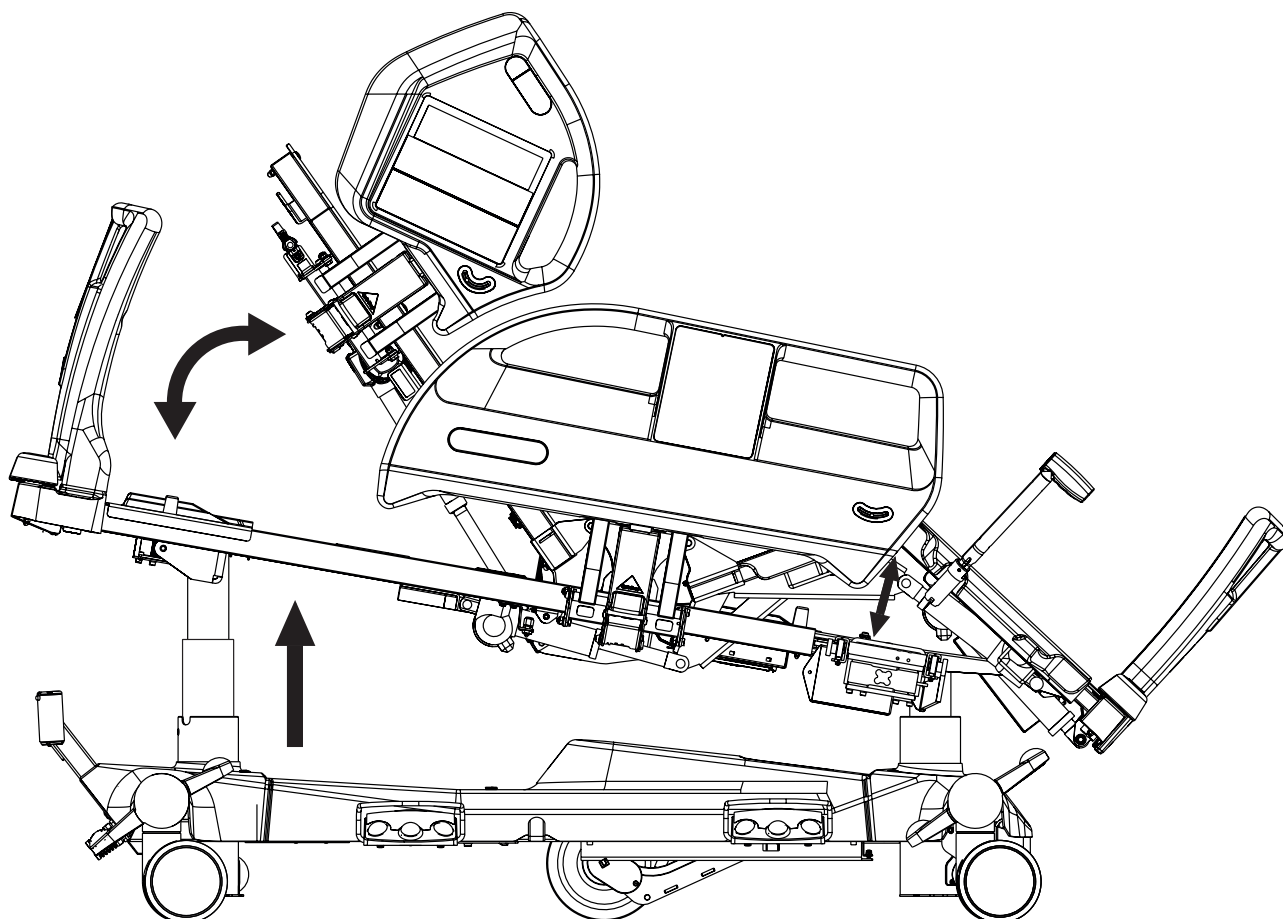


Fig. CPR Position Button (Nurse Control Panel)

Nurse Control Panel:

- ▶ Press CPR Position Button until intended position is reached.

11.5.11 Cardiac Chair Position



To position Cardiac Chair Position use:

- ▶ iBoard Standard
- ▶ Nurse Control Panel
- ▶ Attendant Control Panel



Fig. Cardiac Chair Position
Button (iBoard Standard)

iBoard Standard:

- ▶ Press button .
- ▶ Press Cardiac Chair Position Button until intended position is reached.



Fig. Cardiac Chair Position Button (Nurse Control Panel)

Nurse Control Panel:

- ▶ Press button .
- ▶ Press Cardiac Chair Position Button until intended position is reached.

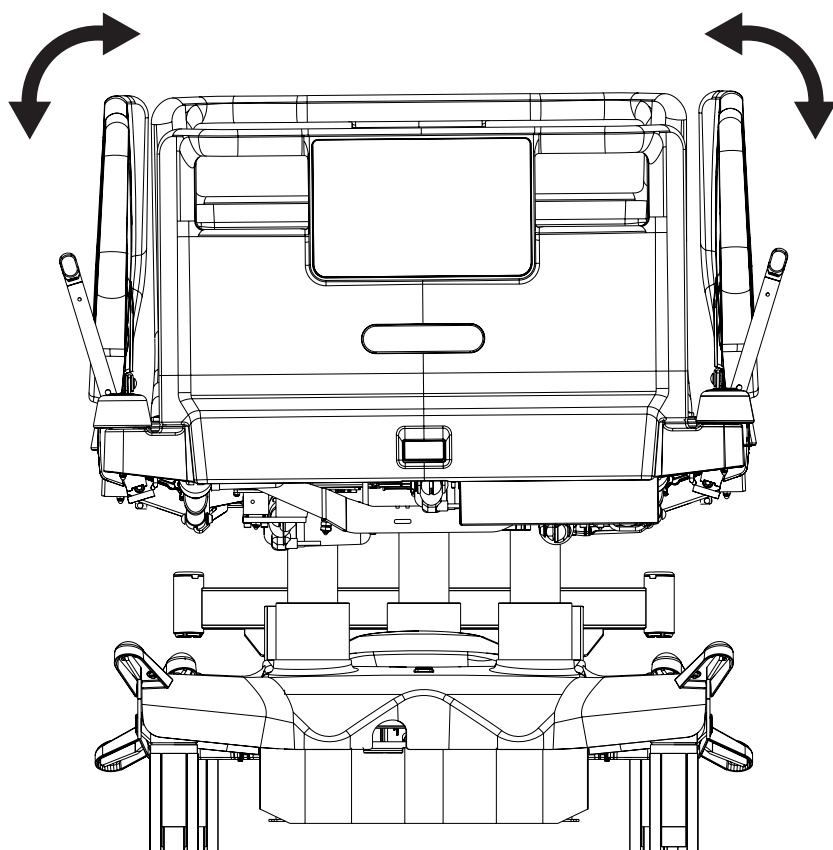


Fig. Cardiac Chair Position Button (Attendant Control Panel)

Attendant Control Panel:

- ▶ Press button .
- ▶ Press Cardiac Chair Position Button until intended position is reached.

11.5.12 Lateral Tilt



It is not possible to position Lateral Tilt with a siderail folded down without use of an additional function.

To position Lateral Tilt use:

- ▶ iBoard Standard
- ▶ Lateral Tilt Foot Control
- ▶ Attendant Control Panel

iBoard Standard Display shows Lateral Tilt Angle. Maximum Lateral Tilt Angle is 15 degrees.



Fig. Lateral Tilt Angle on iBoard Standard Display

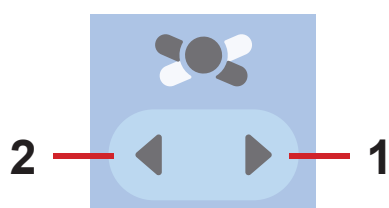


Fig. Lateral Tilt Buttons (iBoard Standard)

1. Tilt Right
2. Tilt Left

iBoard Standard:

- ▶ Press button .
- ▶ Press selected Lateral Tilt Button until intended position is reached.

To facilitate mobilisation of the patient:

- ▶ Press button  +  until intended position is reached.
- Lateral Tilt is adjusted although a siderail is folded down.



Fig. Lateral Tilt Pedals (Lateral Tilt Foot Control)

Lateral Tilt Foot Control:

- ▶ Press button .
- ▶ Press and hold selected Lateral Tilt Pedal until intended position is reached.

- 1. Tilt Right
- 2. Tilt Left

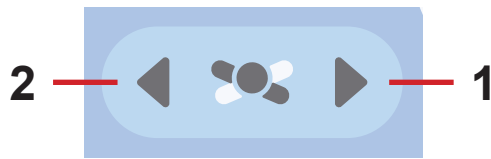


Fig. Lateral Tilt Buttons (Attendant Control Panel)

Attendant Control Panel:

- ▶ Press button .
- ▶ Press selected Lateral Tilt Button until intended position is reached.

TRANSFER MODE


Transfer Mode enables Lateral Tilt with a siderail in down position for 5 minutes.

To set Transfer Mode:

- ▶ Press  button on the iBoard Standard keyboard or the iBoard Basic keyboard.

Lateral Tilt icon on the iBoard Basic display or iBoard Standard display is flashing.


11.5.13 Mobilisation Position

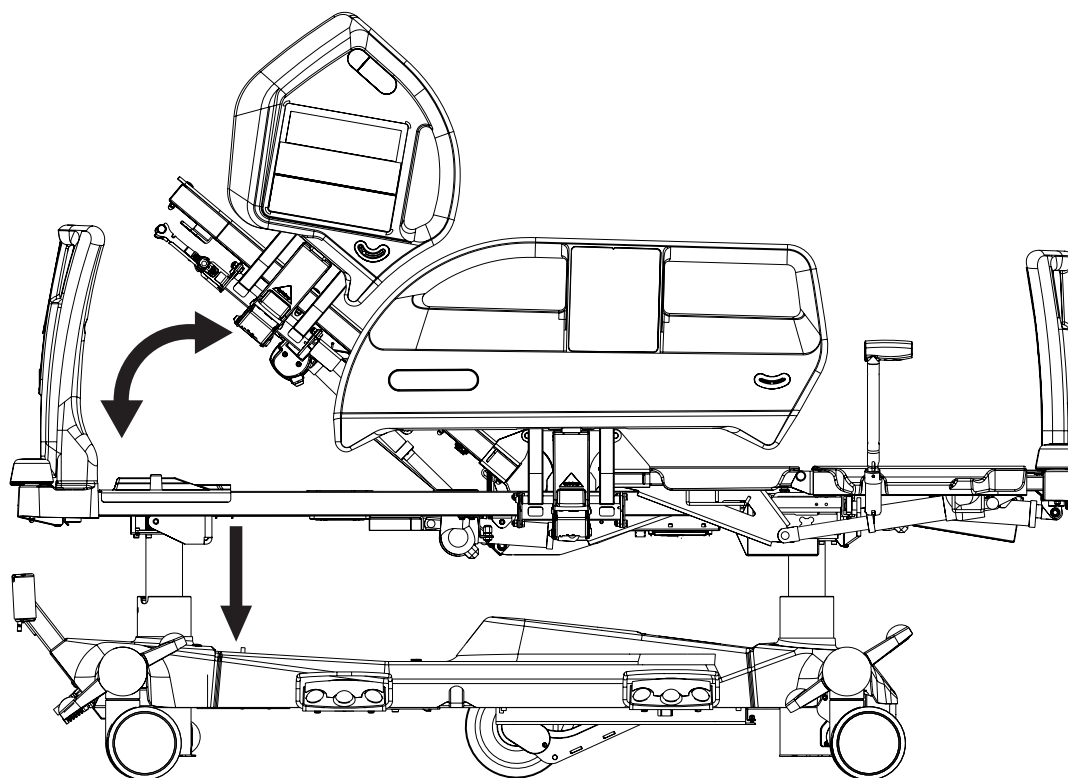

In Mobilisation Position bed is descending to the lowest Bed Height and Backrest reaches the maximum angle.

To position Mobilisation Position use:

- iBoard Standard
- Nurse Control Panel

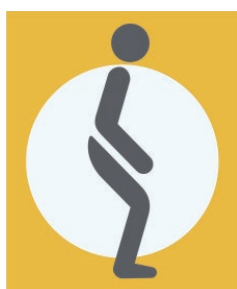

*Fig. Mobilisation Position
Button (iBoard Standard)*

iBoard Standard:

- Press button  .
- Press Mobilisation Position Button until intended position is reached.

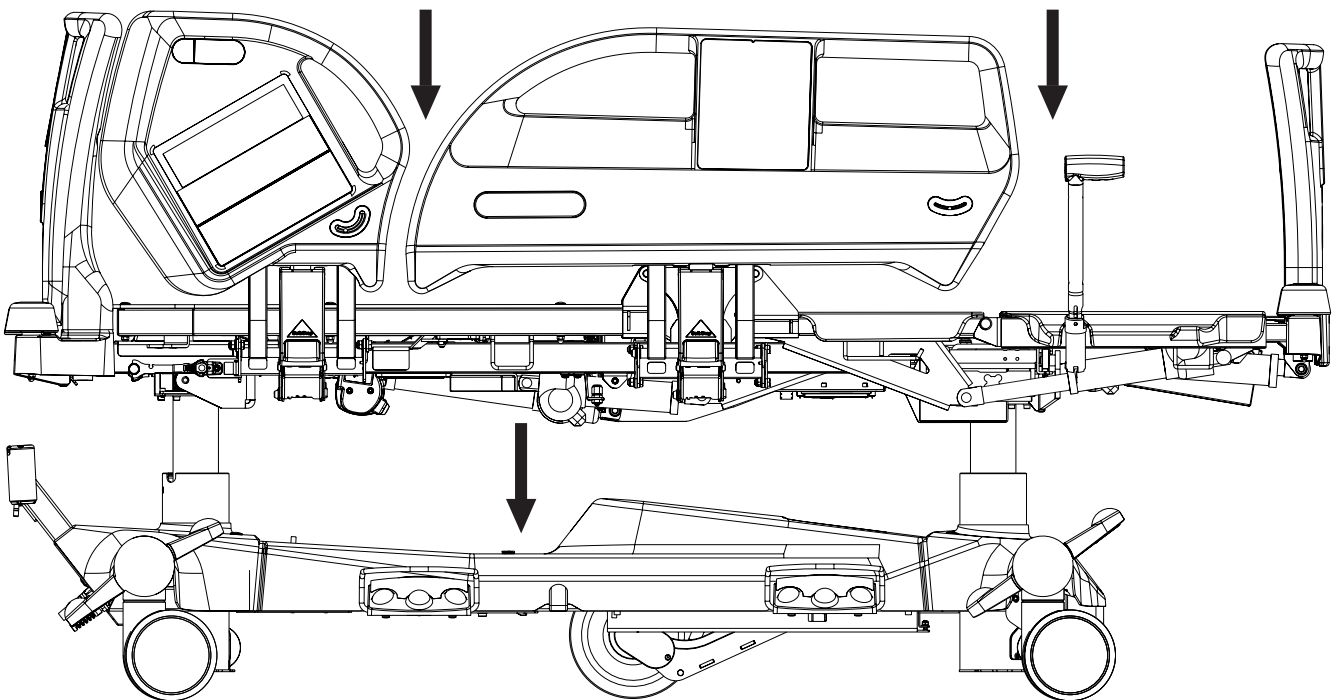


Fig. Mobilisation Position Button (Nurse Control Panel)

Nurse Control Panel:

- ▶ Press button .
- ▶ Press Mobilisation Position Button until intended position is reached.

11.5.14 Low Flat Position



In Low Flat Position the mattress support platform is firstly adjusted to flat position without any lateral tilt and then lowered to the minimum height.

To position Low Flat Position use:

- ▶ Lateral Tilt Foot Control

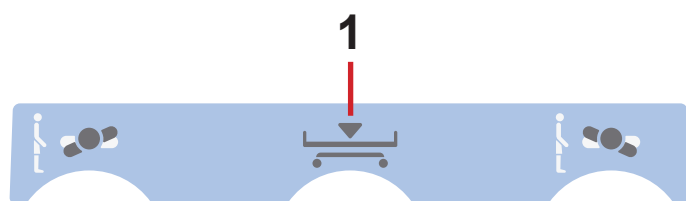


Fig. Low Flat Position Pedal on the Lateral Tilt Foot Control

Lateral Tilt Foot Control:

- ▶ Press Low Flat Position Pedal and release it.
- ▶ Press and hold the Low Flat Position Pedal until intended position is reached.

1. Low Flat Position

11.5.15 Ergoframe

Ergoframe® is the kinematic system of Backrest and Thighrest Adjustment resulting in extension of the Mattress support platform in the seat section.

Ergoframe® enlarges the space for pelvic area during Auto-contour. Because of increasement of the space the force applied results in decrease of the pressure that can cause pressure injuries in the pelvic area.

Ergoframe maintains a stable ergonomic position of the body and spine of the patient, thus limiting unwanted movement of the patient by moving down or up in beds. Unified movement eliminates the patient's shift over the mattress and thus maintains a uniform position of the patient's body that is not bound to the position of the bed parts.

12 Scales Control (only version with scales)

Use iBoard Standard or iBoard Basic to control the scales.

iBoard Standard

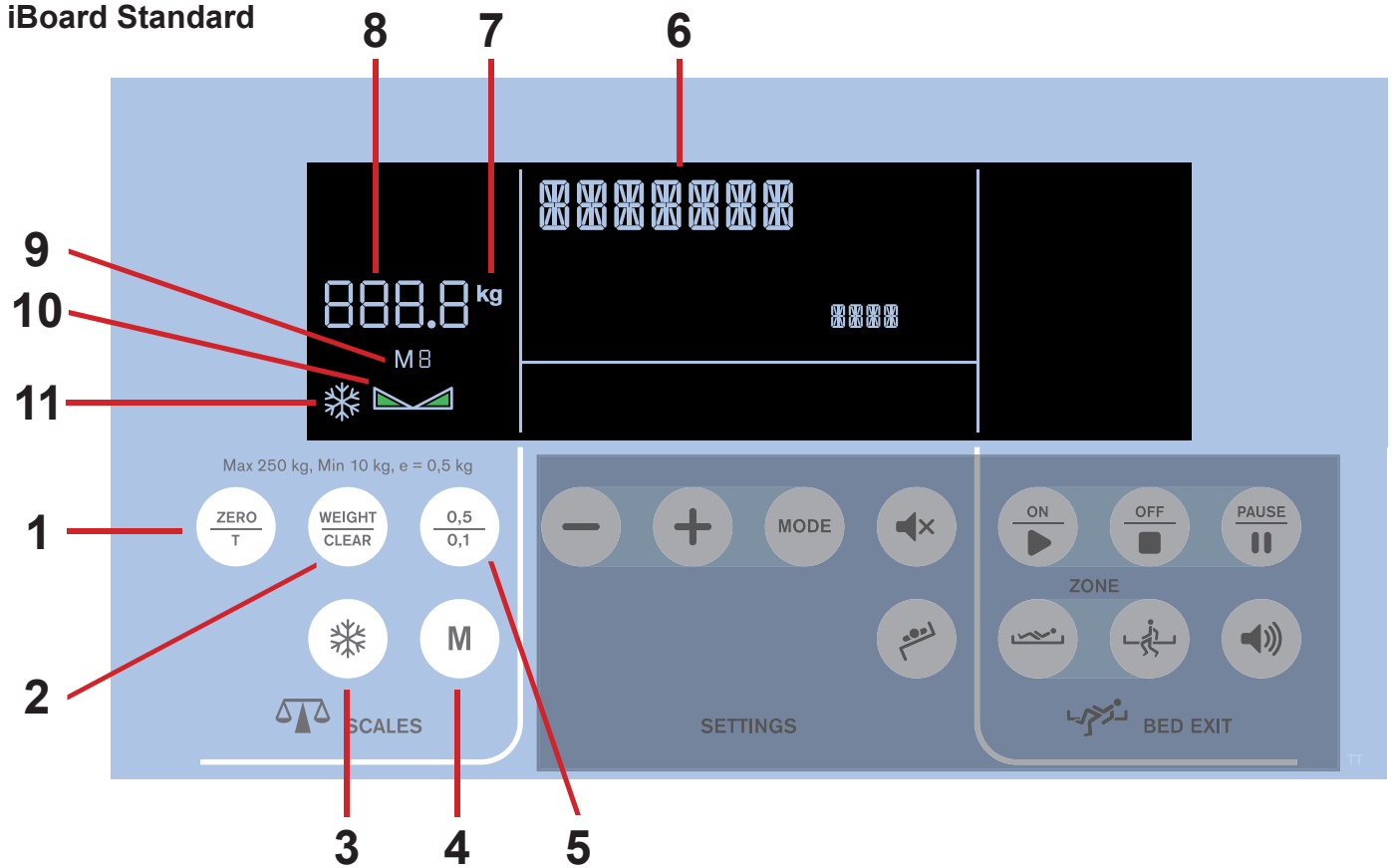


Fig. Scales Section (iBoard Standard) - Display and Keyboard

iBoard Basic

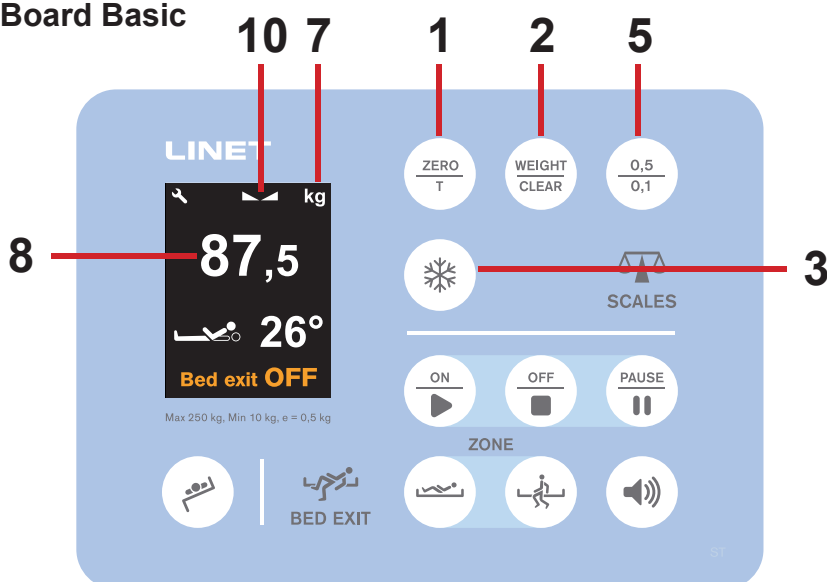


Fig. Scales Section (iBoard Basic) - display and keyboard

1. ZERO/T Button (tare or zero scales)
2. WEIGHT/CLEAR Button (cancel)
3. HOLD Button
4. Memory Button (save or display weight value)
5. Scale interval switch button (0,5kg/0,1kg)
6. Status Indicator
7. Unit of weight (kg)
8. Weight value
9. Memory Record Indicator
10. Stabilized Scales Icon
11. HOLD Icon

12.1 Preparation

- Install mattress and accessories to prepare bed before patient admission and using the scales.



CAUTION!
Incorrect use of scales due to incomplete preparation!
 ► Before each patient admission tare the scales.

12.2 Taring

Taring can be done in a range of 5kg to 249.5kg. Taring is used to set "0" on the display before placing the patient on the bed. Taring must be done with an unloaded bed with mattress, bed sheets, pillows and necessary accessories, without the patient. It is recommended to position Mattress support platform about 20 cm above the lowest horizontal position.

To tare weight:

- Ensure that nothing and nobody touches the bed except you.
- Press and hold button  until value (field 8) starts to flash. Release button .
- Press button  again to confirm taring. "0" is shown on the display.

Place the patient on the bed.

To cancel taring:

- Press button  while taring.

12.3 Displaying

Verification Scale Interval is 0.5 kg.

- Press button  to display value with actual scale interval 0,1 kg for 5s.

During this mode unit of weight (**kg**) and decimal point are flashing on the display.

Field 8 shows normally actual weight value if other functions are not activated.

NOTE Weight value automatically disappears after 1 minute. If it is needed, press button  to display it again.

12.4 Hold Mode

Hold Mode can be used only when scales are stabilized.

It allows adding or removing bed accessories and other items without changing the weight value.

To activate Hold Mode:

- Wait until the scales are stabilized. The icon  or  will be illuminated when the scales are stabilized.
- Press button  until snowflake icon  appears on the display.
- Add or remove required accessories.

To deactivate Hold Mode:

- After adding or removing accessories wait until the scales are stabilized (icon  or  is displayed).
- Press button .
- Display shows the original weight value.

To deactivate Hold Mode without fixing the weight value:

- Press button .

12.5 Memory (only iBoard Standard)

Saving patient's weight

To save patient's weight:

- Press and hold button  to save the value.

Value starts to flash before it is saved.



Fig. HOLD Mode (iBoard Basic)

NOTE To save another weight value repeat this procedure. During one day repeated savings mean rewriting the same record. During rewriting Status Indicator (6) shows **SAVE**.

To cancel saving patient's weight:

- Press button  during saving.

NOTE Maximum number of saved values is 9. Icon  shows records from **M1** to **M9**. It means 9 different values for 9 following days.

NOTE Each saved value is associated with time when it was saved. Values are related to the present day (today, day minus 1, day minus 2,...).

Displaying the saved patient's weight

To display saved value of patient's weight:

- Press button  to display the value.

Value is displayed for 5s (field 8) with indication of days past from the moment of saving (field 6) and with number of displayed record (field 9).

NOTE To display another saved weight value repeat this procedure.

To return to actual patient's weight:

- Press button  while displaying a saved value.

After 10s actual patient's weight appears instead of displayed memory record.

NOTE The last record is saved as the first (**M1**) in sequence of 9 records.

NOTE After pressing button  it is possible to hold the button additionally to save an actual patient's weight.

Erasing memory

To erase all memory records:

- Press button  and button  at the same time.

During pressing these buttons „NEW P“ with countdown is displayed.

12.6 Bed Overload

If load of the bed is over 254,5 kg:

- The „Hi“ is shown on the display.

If load of the bed is over 260 kg:

- The „OVERLOAD“ is shown on the display.

NOTE If the bed is overloaded, it is impossible to position or manipulate the bed until overloading is removed. Bed overloading always has higher priority than Hold Mode and Taring.

12.7 Bed Underload

If the bed is underloaded (factory zero is 5kg):

- Display shows the „Lo“.

12.8 Weighing in tilt

Accuracy is conditioned by the spirit level, which is located on the right head corner of the bed. If the bubble is in the highlighted circle then weighing is accurate.

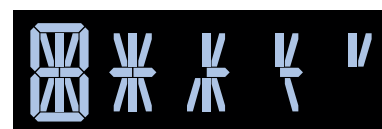


Fig. Five steps of the countdown



Fig. High Load (iBoard Basic)



Fig. Bed Underload (iBoard Basic)

12.9 Zeroing Scales

Zeroing is only possible in a range of $\pm 5\text{kg}$ from factory zero. Zeroing is used to reset weight on the display and set up user zero, which sets the maximum weight range of the weighing system. Zeroing must be done with an empty, unloaded bed, without the mattress and accessories. Zeroing is done after installation, weight verification or servicing.

To zero scales:

- ▶ Position the bed about 20 cm above the lowest position and set the mattress support platform to the horizontal position. Ensure that nothing touches the bed except you.
- ▶ Press and hold button  until weight value starts to flash.
- ▶ Press button  to confirm zeroing.

“0” is shown on the display and an acoustic signal confirms zeroing.

To cancel zeroing:

- ▶ Press button  while zeroing.

13 Bed Exit Monitoring (only version with scales)

Use iBoard Standard or iBoard Basic to control the Bed Exit Monitoring.

If the bed is equipped with SafetyMonitor (accessory), Bed Exit Monitoring function is included between the functions of the Safety-Monitor.

iBoard Standard

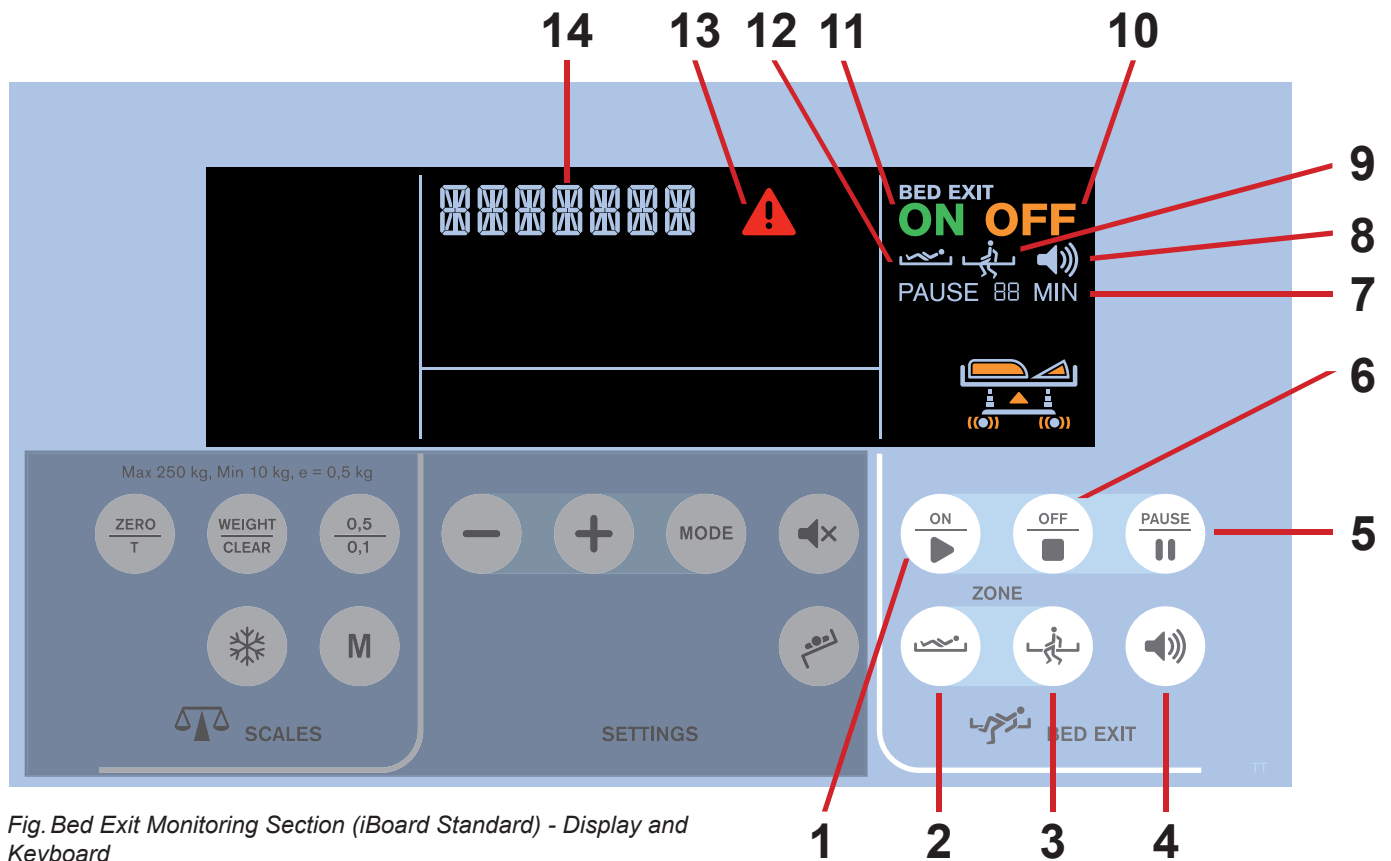


Fig. Bed Exit Monitoring Section (iBoard Standard) - Display and Keyboard

1. ON Button
2. Inner Zone Button
3. Outer Zone Button
4. VOLUME Button (3 levels)
5. PAUSE Button
6. OFF Button
7. PAUSE Countdown Icon (with remaining minutes)
8. Volume Icon (3 levels)
9. Bed Exit Monitoring Activated (Outer Zone)
10. OFF (Bed Exit Monitoring)
11. ON (Bed Exit Monitoring)
12. Bed Exit Monitoring Activated (Inner Zone)
13. Alert Icon
14. Status indicator

iBoard Basic

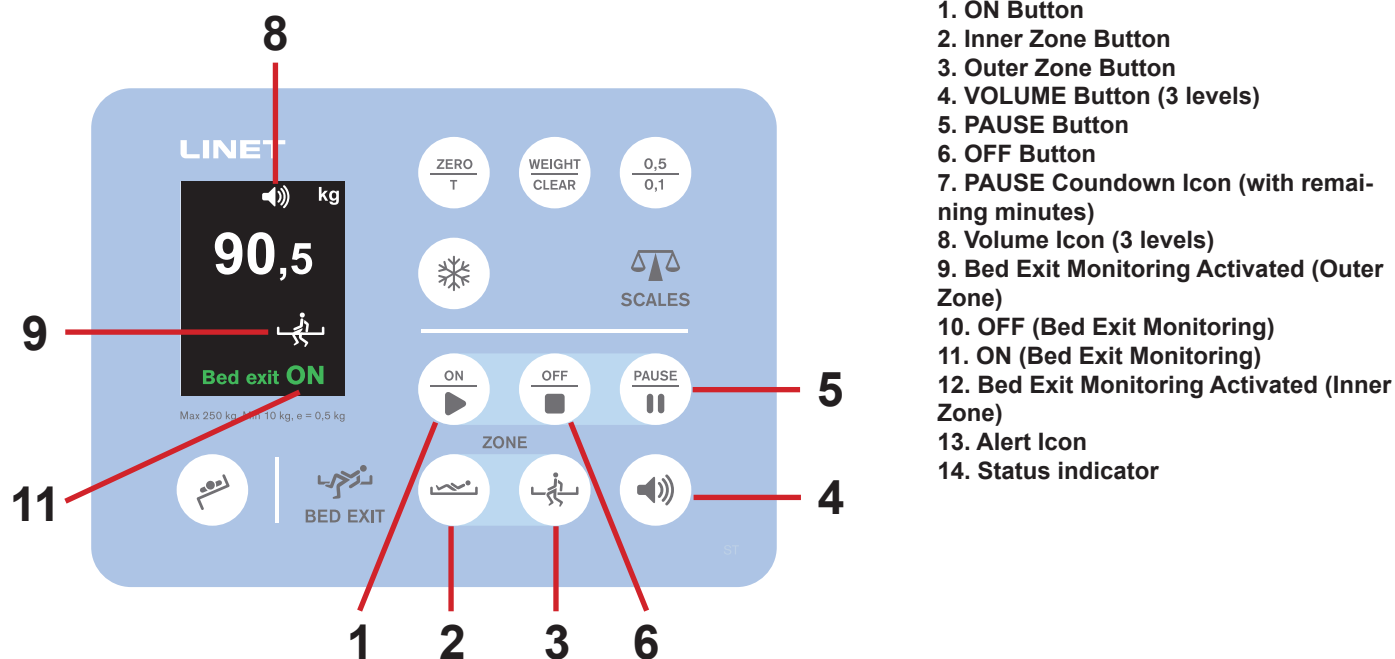


Fig. Bed Exit Monitoring Section (iBoard Basic) - display and keyboard

13.1 Preparation

- Place a patient on the bed with suitable mattress.

NOTE For the correct Bed Exit Monitoring in the Inner Zone patient's position in the middle of the bed is needed.

13.2 Activation of Bed Exit Monitoring

Bed Exit Monitoring is **OFF** and icon **OFF** is displayed by default.

To activate Bed Exit Monitoring:

- Press button .

Icon **ON** appears on the display.

When Bed Exit Monitoring is activated Inner Zone is set by default. Icon  therefore appears on the display.

NOTE Minimum patient weight for Bed Exit Monitoring is 35 kg.

13.3 Monitored Zone

To set Outer Zone:

- Press button .

Icon  appears on the display.

To set Inner Zone:

- Press button .

Icon  appears on the display.

13.4 BED EXIT ALERT

Alert is triggered when patient has left selected monitored zone or PAUSE period elapsed and patient is not in ordered position. The Bed Exit Alert consists of the specific melody, the spoken sentence 'Please remain in the bed, assistance is coming.' and the lit red LED indicators.

To stop Alert:

- ▶ Press button  .

Bed Exit Monitoring is deactivated and icon **OFF** appears on the display. The audible alert is muted.

To pause Alert:

- ▶ Press button  or button  .

Field 7 appears on the display with 15 minute countdown timer. The audible alert is muted.

13.5 Alert Volume

Maximum Alert Volume Level is set by default.

It is possible to set Alert Volume before and during triggered alert.

To lower Alert Volume Level:

- ▶ Press button  .

Icon  with lower Alert Volume Level appears on the display. Volume is lowered.

To return to Maximum Alert Volume Level:

- ▶ Press button  after Minimum Alert Volume Level has been reached.

Icon  with the 3 levels appears on the display.

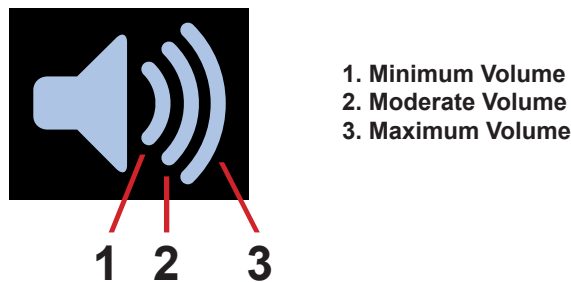


Fig. Volume Icon



Fig. Visual signalisation of Bed Exit Alert (iBoard Basic)

13.6 PAUSE

Bed Exit Monitoring function is temporarily interrupted during PAUSE Mode and audible alert signals are not activated.

To PAUSE Bed Exit Monitoring:

- ▶ Press button  or button .

Icon 7 appears on the display with 15 minute countdown timer.

After PAUSE period elapsed and patient is in ordered position Bed Exit Monitoring is reactivated.

To extend the PAUSE period:

- ▶ Press button  again to extend the countdown to 15 minutes period again.

To terminate the PAUSE period:

- ▶ Press button .

NOTE When Outer Zone monitoring is activated, PAUSE period is terminated when patient returns to the bed.



Fig. PAUSE Countdown
(with remaining minutes)

13.7 Deactivation of Bed Exit Monitoring

To deactivate Bed Exit Monitoring:

- ▶ Press button .

Icon **OFF** appears on the display.

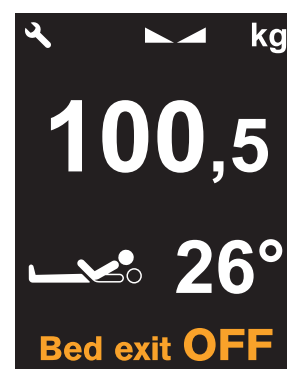


Fig. Bed Exit OFF (iBoard
Basic)

14 Equipment

14.1 Accessory Rail with plastic hooks (optional)

Accessory Rail with 2 plastic hooks is intended for hanging accessories. It is located on the sides of bed.

14.2 Brake Signal (optional)

If bed is equipped with Brake Signal and this bed is connected to the mains power, the Brake Signal sounds when the bed is not braked.

14.3 i-Brake® (optional)

It is possible to equip the bed with an automatic castor brake. The automatic castor brake prevents injuries of patients and staff due to an unbraked bed. The brakes are activated automatically 60 seconds after the bed is plugged in, and 60 seconds after they have been released if the bed is not being moved. It is possible to activate the brakes manually as well.

14.4 Retractable Fifth Castor (optional)

It is possible to equip the bed with Fifth Castor in the centre of undercarriage. The Fifth Castor helps to steer and manoeuvre the bed in long corridors and small rooms. If the bed is plugged in, the Fifth Castor automatically retracts. Retracted Fifth Castor does not obstruct access to any devices under the undercarriage.

To activate the 5th wheel i-Drive®:

- Disconnect the bed from the mains.
- Adjust the castor control so that the green lever points down.

14.5 LINIS SafetyPort (optional)

LINIS SafetyPort is a medical device data system for capturing and transferring data from LINET beds into SafetyPort Dashboard and third party systems, including nurse calls, EHR and digital whiteboards. Data collection and evaluation takes place at one central location for all beds connected to the system simultaneously. The records are completely anonymous and the system does not work with any personally identifiable information. The customer can decide which data will be sent to the 3rd party system and adjust their sending period. LINIS SafetyPort is intended to be used to increase efficiency of healthcare personnel workflows by saving their time spent on documentation and eliminating errors. This is achieved by automated recording of different parameters of medical beds and their subsequent transfer to various hospital systems in HL7 format. Optional feature LINIS SafetyPort Dashboard is intended to save time the healthcare personnel spends on checking different beds at their workspace and to provide them with both near-real-time data and their aggregation to be able to check the history of provided care. LINIS SafetyPort may be used in various healthcare environments, including both intensive and non-intensive care units as well as units providing speciality care to a broad population of patients. The product is intended to be used by variety of healthcare personnel who have the cognitive skills to operate the product and are trained to use the product. LINIS SafetyPort is not an alarm system and the use of this product for this purpose means incorrect use.

14.6 i-Drive Power (optional)

i-Drive Power System - Basic Description

It is possible to equip the bed with the i-Drive Power wheel. The i-Drive Power helps hospital staff to drive the bed during patient transport with minimal manpower.

The i-Drive wheel is located in the center of the bed under the undercarriage. i-Drive Power is equipped with its own accumulator and charger and it is not dependent on the bed functions so, if discharged you can still use the bed functions. The bed is equipped with one i-Drive controller. i-Drive is oriented in straight direction of the bed.

Safety instruction for i-Drive Power

- ▶ Follow the instructions carefully.
- ▶ Ensure that the bed is operated exclusively by qualified staff.
- ▶ Make sure the siderails are raised up during the transport.
- ▶ Never use bed positioning buttons during transport.
- ▶ Never use Fast forward button when descending. The Fast forward button is recommended for use when ascending as it is more efficient.
- ▶ Special precaution need to be considered when reversing. Always keep distance from the bed and never use reverse button when descending or ascending.
- ▶ Do not use Free Drive to transport on a slope over 1 degree unless adequate personnel are available to manage safe bed transport.
- ▶ The driving down the slope that exceeds 6 degrees will require adequate contribution of a manpower.
- ▶ Never leave the bed with an activated i-Drive Power system without supervision of the trained staff.
- ▶ Always use the regular mechanical brake system to brake and stabilize the bed.
- ▶ Pay increased attention when driving the bed using i-Drive Power. Be aware of people and objects in close proximity and avoid collision with them by careful driving, especially by appropriate speed control.
- ▶ Make sure the bed is unplugged and bed brakes are released before using i-Drive Power.
- ▶ Push the emergency stop drive button if immediate movement interruption is needed (e.g. to avoid collision with other persons or objects).
- ▶ Retract the i-Drive Power wheel to the undercarriage when parking. This will prevent misuse when unbraking and braking the bed.
- ▶ The i-Drive Power electromagnetic brake is designed just for temporary bed stop and not for the permanent parking.
- ▶ Switch off the i-Drive Power accumulator prior to long-term storage or transport.
- ▶ Push the emergency retraction button under the chassis cover to retract the i-Drive Power wheel
- ▶ in case an of i- Drive Power system failure. This will enable moving the bed to a safe area manually without using i-Drive Power.
- ▶ Retract the i-Drive Power wheel to the undercarriage every time you intend to move the bed sideways.
- ▶ Pay attention to the LED accumulator status indicator and plan your drive using the i-Drive Power accordingly. Insufficient accumulator capacity can cause unexpected complications and risks during the drive.
- ▶ Always plug the bed in when you finish your drive in order to recharge the accumulator and keep your bed ready to go using the i-Drive Power.
- ▶ The i-Drive Power accumulator must be replaced every 2 years to maintain proper functions of the i-Drive Power.

Specifications of Use



WARNING!

Risk of injury due to careless driving!

- ▶ Always drive safely and carefully.
- ▶ Observe the path for any obstacles and avoid collisions.
- ▶ Ensure there are no people in your way.
- ▶ Manipulate with the bed carefully not to drive over any staff or patients.



CAUTION!

Maximal clearance underneath the bed is 11,3 cm!

- ▶ Observe the path for any obstacles and avoid collisions.

Intended use:

- ▶ bed transport (with or without patient) by the hospital staff

Unintended use:

- ▶ riding the bed
- ▶ other usage than described in instructions for use
- ▶ by other person than the trained staff

NOTE Raising and lowering of the i-Drive wheel is electrically controlled by the i-Drive activation panel.

14.6.1 i-Drive Power Activation/Deactivation

To activate the i-Drive Power:

1. Check, if the mains switch of i-Drive Power is activated.
2. Press the Activation button  located on the Activation Panel. The i-Drive wheel will lower and the green indicator will flash.


To deactivate the i-Drive Power:

1. Retract the i-Drive wheel using the button  located on the Activation Panel.
2. Deactivate the i-Drive using the mains switch.

Emergency i-Drive Power wheel retraction:

1. Press any button  on the bed.
2. Deactivate the i-Drive Power using the mains switch.
3. Press the i-Drive Power Emergency Retraction Button situated on the bottom side of undercarriage under the label.

NOTE Use emergency retraction in case of accumulator discharge or drive malfunction to move the bed to a safe area manually without using i-Drive Power.

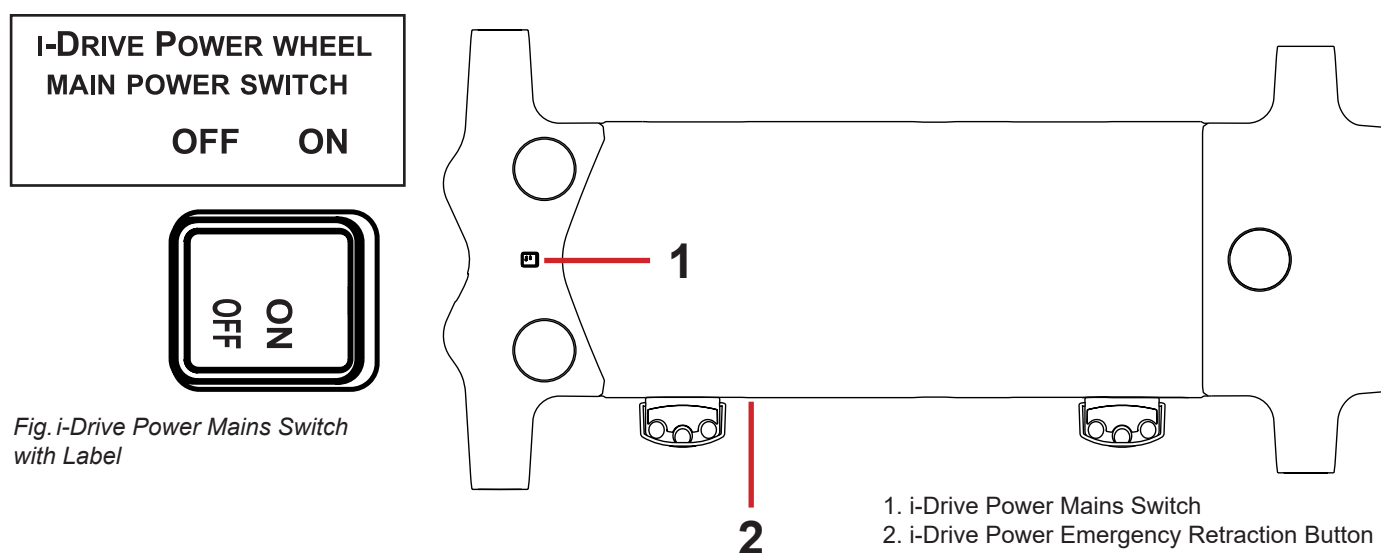

TO RETRACT THE I-DRIVE POWER WHEEL IN EMERGENCY:

1. PRESS ANY GO BUTTON ON THE BED
2. SWITCH OFF THE MAIN POWER SWITCH
3. HOLD THE EMERGENCY RETRACTION BUTTON UNDER THE CHASSIS COVER



Fig. i-Drive Power Emergency Retraction Button Label

14.6.2 Powered Drive



CAUTION!

Damage to property due to incorrect transport and involuntary movement!

- ▶ Prior to transport, ensure that the bed is disconnected from the mains.
- ▶ Prior to transport, ensure that the auxiliary outlet plug (if available) is disconnected from the mains.
- ▶ Ensure that the castors are locked prior to assembly, disassembly and maintenance (e.g.: i-Drive Power maintenance).
- ▶ Ensure that the castors are locked when the bed is occupied.
- ▶ Hang the mains cable on the appropriate hook on the bed during transport.

1. Check, if the mains switch of i-Drive Power is activated.
2. Press the button  on the Activation Panel. The i-Drive wheel will lower and the LED  will flash.
3. Place your hand on the Safety Sense touch sensor (1) and push the button  or button  or button . Your hand must be placed on the Safety Sense sensor to use the i-Drive Power, if released, the i-Drive Power will stop.
4. The i-Drive motor is immediately stopped and the electric brake is activated after pressing the red button  when braking or in emergency.
5. i-Drive Power control system is automatically deactivated and the electric brake is activated if no i-Drive function is used for 3 minutes. This is signaled by the green LED  which is extinguished after 3 minutes.

NOTE i-Drive Power is not designed for ascending or descending a slope greater than 6° or longer than 20 m. The support of personnel is needed when ascending or descending with a full SWL.

NOTE When i-Drive wheel is lowered, it is not possible to move the bed sideways. Press the button  to retract the wheel, release the castors to the neutral position and then move the bed to any direction required.

14.6.3 Braking

1. Press and hold the button  to brake immediately.
-or-
2. Press and hold the button  to brake slowly (Press the button  to brake when reversing).
-or-
3. Release your hand from the touch sensor area and i-Drive Power will brake automatically.

NOTE Always brake the bed by using the castor control lever when the transport is finished or interrupted. The i-Drive electromagnetic brake is not designed to permanently brake the bed.

NOTE In a crisis situation (e.g. acceleration when driving down a steep slope) i-Drive dual braking prevents acceleration and slows down bed movement. However, it is not guaranteed the bed will stop by itself without personnel support (using button  and castor control lever).

NOTE When descending, it is possible to actively brake using the opposite direction button to slow.

14.6.4 Free Drive

The i-Drive motor is equipped with free drive, which is active after pressing the forwards ( or ) or backwards () buttons (until user holds the touch sensor area).

Free Drive is deactivated and the brake is activated when the direction of motion is changed. This is feature for lowering the risks when going to a slope.

Accumulator

Accumulator charge status:

1. While this indicator is flashing, the accumulator is critically discharged. (LED1)
2. 50% (LED2)
3. 75% (LED3)
4. 100% - the accumulator is charged (LED4)

To charge the accumulator:

- Connect the bed main cable to mains power.
- i-Drive will be charged (with the accumulator discharged, the charging may take up to 9 hours).

NOTE Accumulator charge values are just informational. Accumulator life is reduced when the accumulator is allowed to discharge completely.

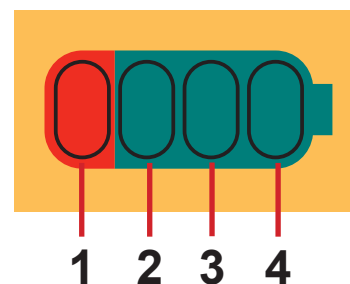


Fig. Accumulator Charge Status

Fault Signalization

The system is protected against failure states, by stopping and braking the drive system, and respective signalization. The fault indicator flashing briefly and the accumulator indicator shows the fault state. Some defects are cleared automatically (e.g.: drive overheating). When drive or electronics is overheated, an short acoustic signal occurs before the drive is blocked.

Error	LED1	LED2	LED3	LED4
Drive overheated	OFF	OFF	OFF	ON
Electronics overheated	OFF	OFF	ON	OFF
Brake error	OFF	OFF	ON	ON
Retraction not completed	OFF	ON	OFF	OFF
5 V OFF limits	OFF	ON	OFF	ON
FET closing penetrated	OFF	ON	ON	OFF
Control circuit overheated	OFF	ON	ON	ON
Controlcircuiterror	ON	OFF	OFF	OFF
Activation button stuck	ON	OFF	OFF	ON
Retraction button stuck	ON	OFF	ON	OFF
Active button after start	ON	OFF	ON	ON

Light Indicators

Indicator	Meaning
Go Indicator ► Constantly lit ► Flashing	Hand is on touch sensor; drive wheel is ready for use. Hand is not on touch sensor; i-Drive is not ready for use.
Fault Indicator ► Constantly lit ► Flashing	i-Drive cannot be activated (i-Drive wheel is not lowered, castor control lever is braked, bed is connected to the mains). System is faulty (indicated on accumulator status indicator) -or- i-Drive control box heat protection is activated

Technical Specifications

Parameter	Value
i-Drive wheel diameter	8,27 in.
Max. fast forward speed (flat ground, loaded)	4,43 Km/h (±15%)
Max. forward speed (flat ground, loaded)	2,16 Km/h (±15%)
Max. reverse speed (flat ground, loaded)	2,16 Km/h (±15%)
Max. angle of ascent	6°
Noise level (when retracting the drive wheel)	65 dB

Electrical specification

Parameter	Value
Input Voltage, Frequency	230 V AC, 50/60 Hz 127 V AC, 50/60 Hz 120 V AC, 50/60 Hz 110 V AC, 50/60 Hz 100 V AC, 50/60 Hz
Accumulator Voltage	36 V DC, Capacity: 12 Ah
Maximum Power Input	300 W
Fuse Version 230 V Version 127 V Version 120 V Version 110 V Version 100 V	2 x T1,6A L 250V 2 x T3,15A L 250V 2 x T3,15A L 250V 2 x T3,15A L 250V 2 x T3,15A L 250V

i-Drive Power Maintenance

Periodical maintenance of the i-Drive Power must be done by qualified service technician or authorized service organization at least once a year. To continue maintenance please see chapter Maintenance.

Service technician must check the following:

- ▶ accumulator status and eventual replacement of the accumulator (after maximum of three years of duty)
- ▶ gas spring – replace if necessary (after maximum of three years of duty)
- ▶ i-Drive Power wheel – replace if necessary
- ▶ lifting mechanism – grease if necessary
- ▶ cables, control elements – replace if necessary
- ▶ i-Drive Power function

14.7 Mobi-Lift® (optional)



WARNING!

Risk of injury due to slipping or falling when standing up!

- ▶ Ensure that the support handles are completely inserted in the sleeve fittings.
- ▶ Ensure that no bed linen is caught between the sleeve fitting and the support handle.

Mobi-Lift® is optional. It serves as a support handle to enhance the patient's safety when getting up. Mobi-Lift® is a support handle with a built-in Height Adjustment button. It allows the patient to raise and lower the Mattress support platform.

Using the Support Handles

To adjust the support handle:

- ▶ Lift the handle up towards the bed.
- ▶ Push the handle into the sleeve fitting as far as it will go.

To adjust the height of the mattress support platform:

- ▶ Press button  on any control element.
- ▶ Press the button to adjust the bed height.

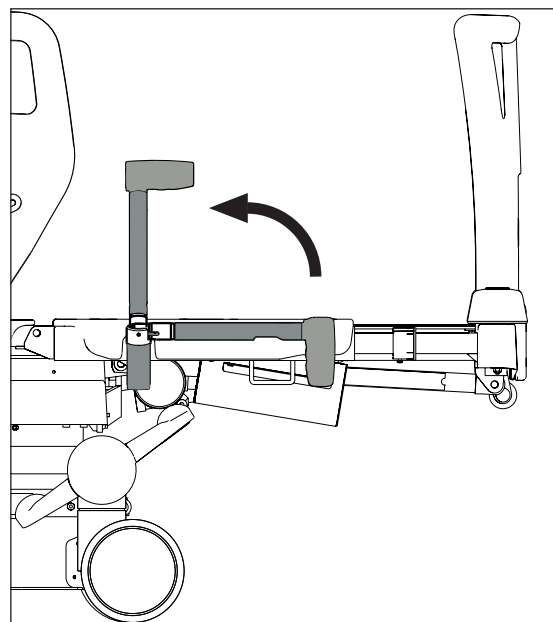


Fig. Mobi-Lift Handle

14.8 Safestop (optional)



CAUTION!

Risk of material damage due to incorrect use!

- ▶ Take extra care during the bed lowering to the Emergency CPR position.
- ▶ Make sure that there are no objects on the bed undercarriage in collision with mattress support platform during the bed lowering to the Emergency CPR position.

Safestop prevents user of the bed from injuries due to crushing by the lowered Mattress support platform.

When obstacle occurs on the undercarriage and Mattress support platform is going down, the motion is automatically stopped.

Safestop function does not interrupt the mattress support platform lowering during the Emergency CPR position adjustment.

iBoard Standard display shows **SAFE STOP** +  and beeping is performed.

14.9 X-Ray Lung Examination (optional)

**WARNING!****Risk of x-ray image misinterpretation due to its deterioration!**

- ▶ Ensure the distance between a lying patient and an inserted compatible x-ray cassette is not increased unnecessarily. A too much thick mattress or combination of mattresses could cause that undesirable state.
- ▶ Ensure there are no added x-ray contrast materials between patient and x-ray cassette. There are no x-ray contrast materials in the place of X-ray cassette Holder.
- ▶ Please note that the image quality may be affected by artifacts and these x-ray images are not intended solely for diagnostic purposes. The patient's clinical presentation also needs to be considered when interpreting the x-ray images.
- ▶ Ensure a compatible x-ray cassette is secured correctly in the X-ray cassette Holder!
- ▶ Take x-ray images when the bed is braked and no part of the bed is forced to move or the mattress is not forced to move considerably!

**CAUTION!****Risk of damaging a compatible x-ray cassette!**

- ▶ Do not leave any x-ray cassette in the X-Ray Cassette Holder if x-ray examination should not be performed immediately.

**CAUTION!****It is not possible to adjust bed positions unless x-ray cassette holder is correctly inserted to the bed!**

- ▶ Check regularly if the x-ray cassette holder is correctly inserted to the bed!

The X-ray cassette Holder is intended for a cassette needed for the x-ray examinations of patients while they are lying on a mattress. The X-ray examination via this X-ray cassette Holder inserted correctly into the bed is intended for indicative results only. More accurate X-ray images must be taken by the other clinical means. The X-ray examination via this X-ray cassette Holder is intended to support the confirmation of invasive lines, monitoring devices and tube placement and complications that may occur following these procedures.

The Backrest of the bed consists of HPL and is x-ray translucent. The bed is equipped with an x-ray cassette holder inserted under the Backrest left side. This design allows taking x-ray images of the patient's lungs without moving the patient manually.

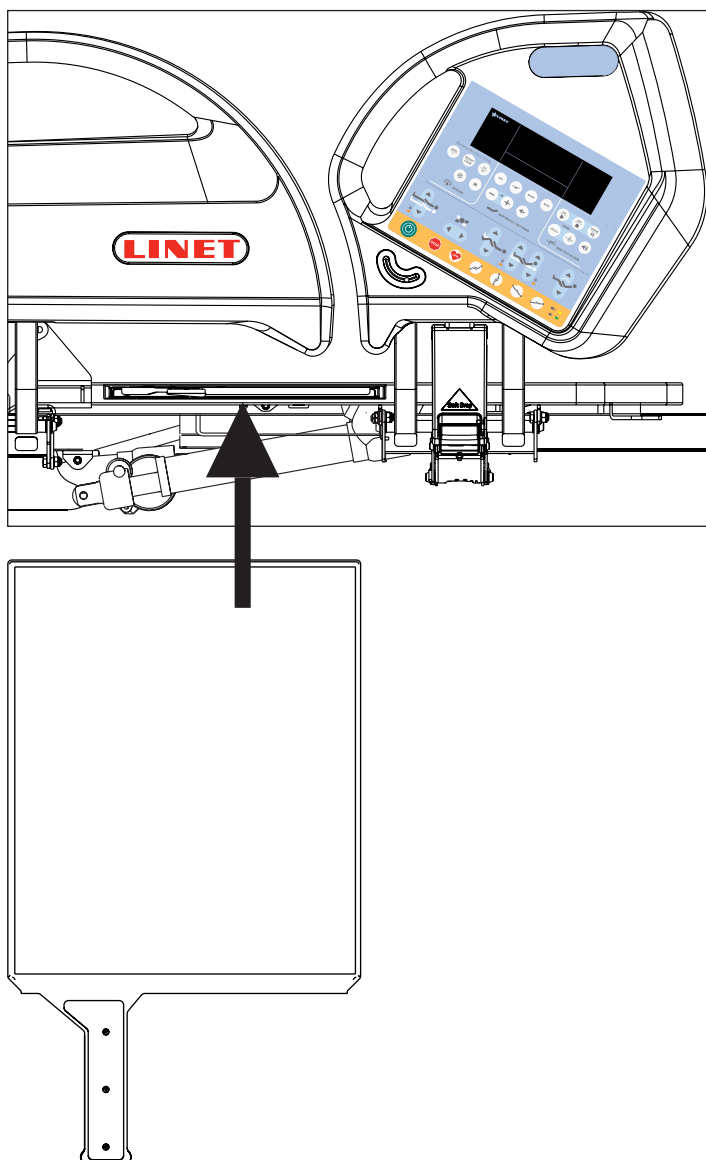


Fig. X-Ray Cassette Holder

Necessary Steps before the Examination

- ▶ Make sure that patient is in centre of bed.
- ▶ Make sure that backrest is in lowest position and siderails are raised up.
- ▶ Pull out x-ray cassette holder.
- ▶ Insert x-ray cassette (format 43×35 cm (16.93 in. x 13.78 in.)) in the horizontal position.
- ▶ Insert back x-ray cassette holder with x-ray cassette so that the cassette centre indicator is exactly under the edge of the mattress support platform.
- ▶ Correct position of x-ray cassette holder using the tooth mechanism so that the upper edge of the x-ray cassette is exactly under the patient's shoulder line. For the correct orientation use the scale on the label. Indicate the position of the patient's shoulder line using the numbers on the scale. Move the x-ray cassette holder in such position so that the centre of the handrail is on the respective scale number.
- ▶ Adjust parameters of the x-ray device and do the image.

NOTE This procedure is above all suitable for patients who cannot be moved due to critical conditions (e. g. internal bleeding) or unstable patients.

14.10 Nurse Call



WARNING!

Functions of the Nurse Call system depends on the local hospital information system!

- Ensure the Nurse Call system is compatible with local hospital information system!



CAUTION!

Ability to use the Nurse Call correctly increases patient's safety!

- Hospital staff should inform patient how to use the Nurse Call system!

Intended use

Nurse Call system is intended for sending signals from the bed to hospital staff.

Nurse Call system can be used by hospital staff and by patient.

Positions of the Nurse Call control elements

The buttons for activating the Nurse Call function are located on the inner and outer sides of the head siderails. Speakers and microphones are located on the inner sides of the head siderails.

Activating the Nurse Call function:

- Press Nurse Call button (1).

The patient can speak into the microphone (2) located on the inner side of the head siderails.

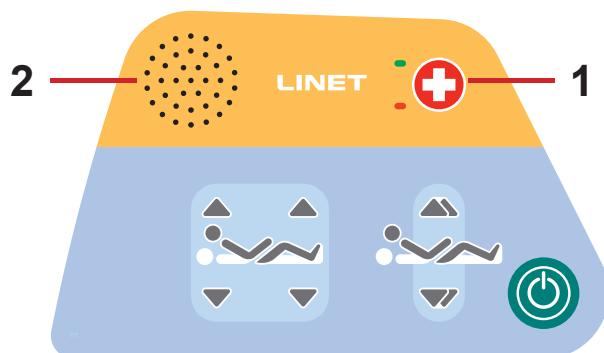


Fig. Nurse Call Control Element (inner side of head siderail) - version 1



Fig. Nurse Call Control Element (outer side of head siderail)

- 1. Nurse Call Button
- 2. Speaker and Microphone



Fig. Nurse Call Control Element (inner side of head siderail)
- version 2

14.11 USB Connector



WARNING!

Risk of injury due to incorrect use!

- Ensure accessory plugged in USB connector is in pristine condition!
User of the bed is responsible for the fact that this requirement is met.



CAUTION!

Risk of material damage due to incorrect use!

- Do not plug heating element into USB connector!
User of the bed is responsible for the fact that this requirement is met.



WARNING!

Risk of injury due to squeezed cable!

- Ensure that cable of the plugged accessory is not squeezed by the head siderail or by the foot siderail or by parts of the mattress support platform!
User of the bed is responsible for meeting of this requirement.



CAUTION!

Risk of material damage due to incorrect use!

- Ensure that plugged accessory is prevented from falling down!
User of the bed is responsible for meeting of this requirement.

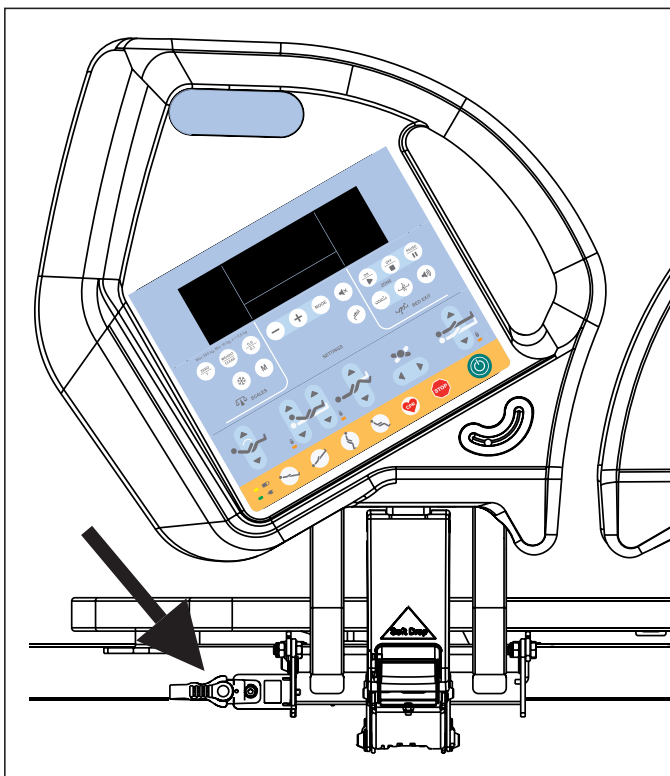


Fig. USB Connector on the right side of Backrest

USB
5V DC
I_{max}:2A

USB Connector situated on the both sides of Backrest is intended for charging mobile phones and tablets.

NOTE Maximum current for this device is 2 A.

15 Mattress

Eleganza 5 bed is designed for passive and active mattresses from LINET portfolio.



CAUTION!

Incompatibility with bed due to incorrect mattress dimensions!

- ▶ Check maximum approved mattress dimensions (chapter Technical Specification).

The manufacturer recommends the use of the following mattress systems on the Eleganza 5 bed:

PASSIVE MATTRESSES

- CliniCare 10
- CliniCare 20
- CliniCare 30

ACTIVE MATTRESSES

- Virtuoso (not integrated)
- OptiCare (integrated)

15.1 Passive Mattress

Recommended Passive Mattresses are equipped with straps (1) intended for fixing mattress on the Mattress support platform.

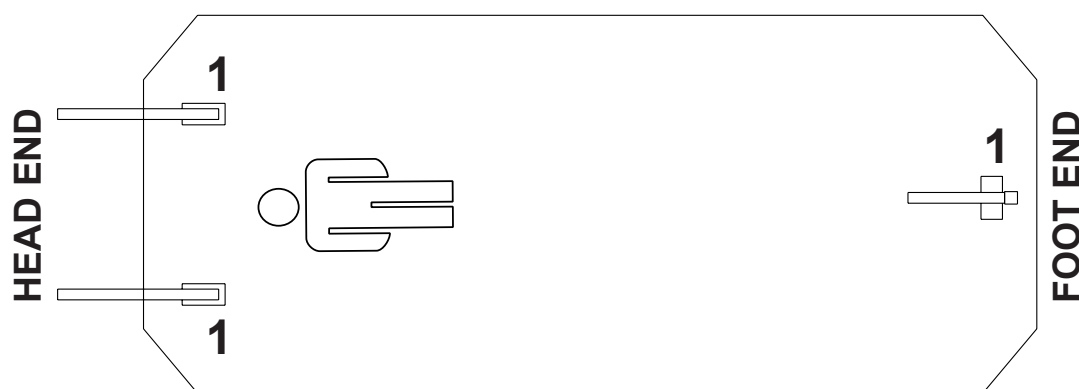


Fig. Bottom of Passive Mattress

15.1.1 Straps with side release buckles

To fix mattress on the Mattress Support Platform:

- ▶ Run three straps through the three corresponding holes in the covers of Mattress Support Platform.
- ▶ Run these three straps under the bars of the Mattress Support Platform.
- ▶ Lock the three side release buckles by connecting their male and female parts together.

To remove mattress from the Mattress Support Platform:

- ▶ Release the three buckles by pressing them from both sides and by disconnecting their male and female parts.
- ▶ Pull these three straps out of the Mattress Support Platform.
- ▶ Remove mattress from the Mattress Support Platform.

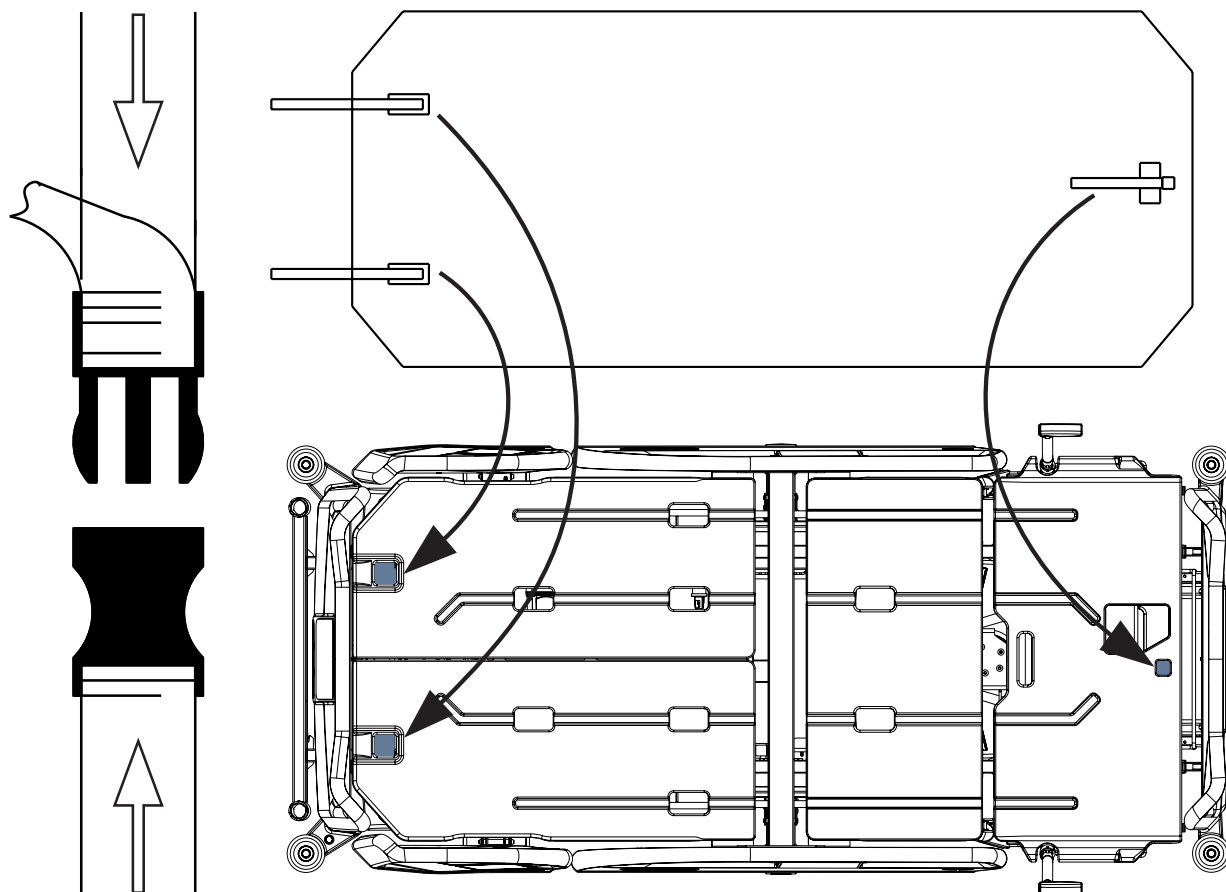


Fig. Fixation of the passive mattress with straps on the mattress support platform of Eleganza 5 bed

15.2 Active Mattress (not integrated)



WARNING!
Follow instructions for use of a compatible active mattress carefully!



CAUTION!
Risk of material damage due to an incorrect fixation of compatible active mattress on the mattress support platform!

- ▶ Adjust the bed to maximum Cardiac Chair Position before fixing all the straps of the inflated mattress to the mattress support platform!

Installation instructions:

- ▶ Remove any existing mattress.
- ▶ Observe mattress dimensions and its orientation before putting it on the Mattress support platform.
- ▶ Place SCU on the foot board of the bed or on the floor.

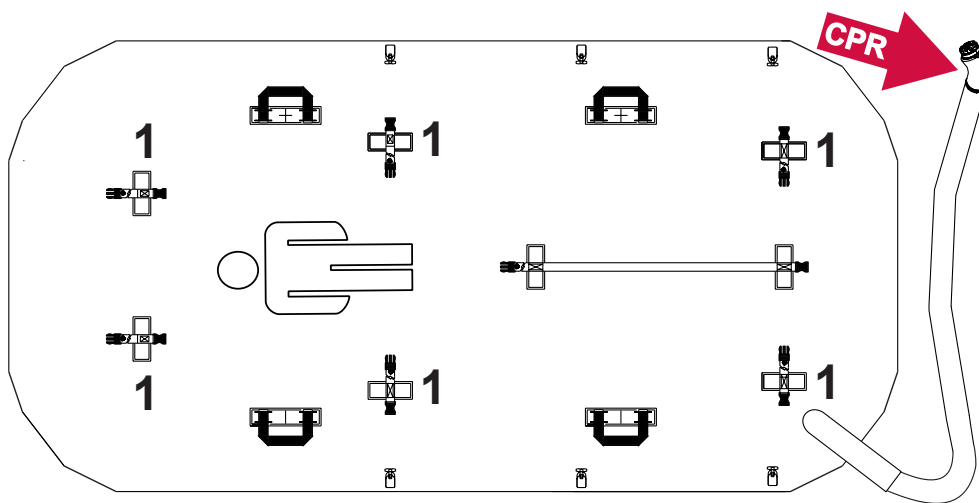


Fig. Bottom of Active Mattress (not integrated)

15.3 OptiCare (integrated mattress)



WARNING!

Follow instructions for use of the OptiCare integrated mattress replacement system carefully!



CAUTION!

Risk of material damage due to an incorrect fixation of compatible integrated mattress on the mattress support platform!

- Adjust the bed to maximum Cardiac Chair Position before fixing all the straps of the inflated mattress to the mattress support platform!



WARNING!

OptiCare mattress is compatible with System Control Unit delivered by manufacturer only!

- Do not use any other System Control Unit with OptiCare mattress!



CAUTION!

Material damage due to incorrect installation of SCU!

- If the SCU does not come factory-fitted, have it installed by a service engineer authorised by LINET ®.

15.3.1 Mattress Screen

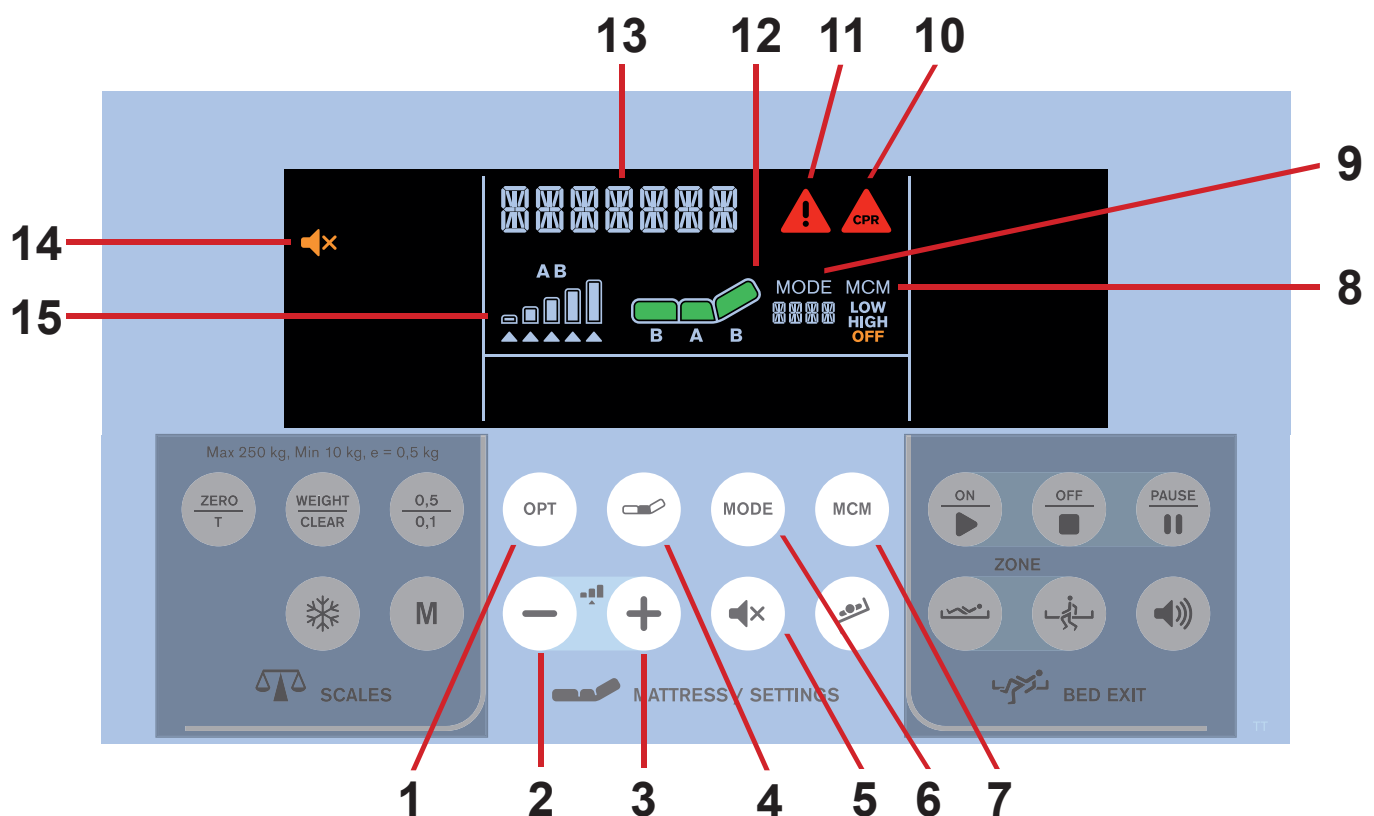


Fig. Mattress Display and Keyboard (iBoard Standard)

- | | | | |
|----|---|-----|--------------------------|
| 1. | OPT Mode Button | 9. | Mode Icon with Mode Name |
| 2. | MINUS Button | 10. | Mattress CPR Alert Icon |
| 3. | PLUS Button | 11. | Alert Icon |
| 4. | COMFORT Button | 12. | Mattress Icon |
| 5. | MUTE Button (Press to mute SCU sounds.) | 13. | Status Indicator |
| 6. | MODE Button | 14. | MUTE Icon |
| 7. | MCM Button | 15. | Pressure Level Icon |
| 8. | MCM Mode Icon (LOW/HIGH/OFF) | | |

16 Accessories


WARNING!
Risk of injury due to incompatible accessories!

- Use exclusively original accessories from the manufacturer.
- The manufacturer is not responsible for the use of unapproved accessories.


WARNING!
Risk of injury due to damaged accessories!

- Use exclusively accessories in perfect condition.

Accessory	head end	foot end	on sides
Lifting pole	✓		
with Triangular holder	✓		
with Infusion holder	✓		
Infusion Stand	✓		
with Fixation holder	✓		
with Infusion bottle basket	✓		
Writing shelf		✓	
Monitor shelf		✓	
Utility shelf		✓	
Crutches holder	✓	✓	
Name holder	✓	✓	
Horizontal oxygen bottle holder	✓		
Vertical oxygen bottle holder	✓		
with Adaptor	✓		
Ventilation circuit holder	✓		
Urinary bag holder			✓
Positioning cushion	✓	✓	✓
Traction frame	✓	✓	✓
Protector			✓
SafetyMonitor			
Pushing Handles	✓		
Pushing Handles with Holder of the i-Drive Power Control Panel	✓		

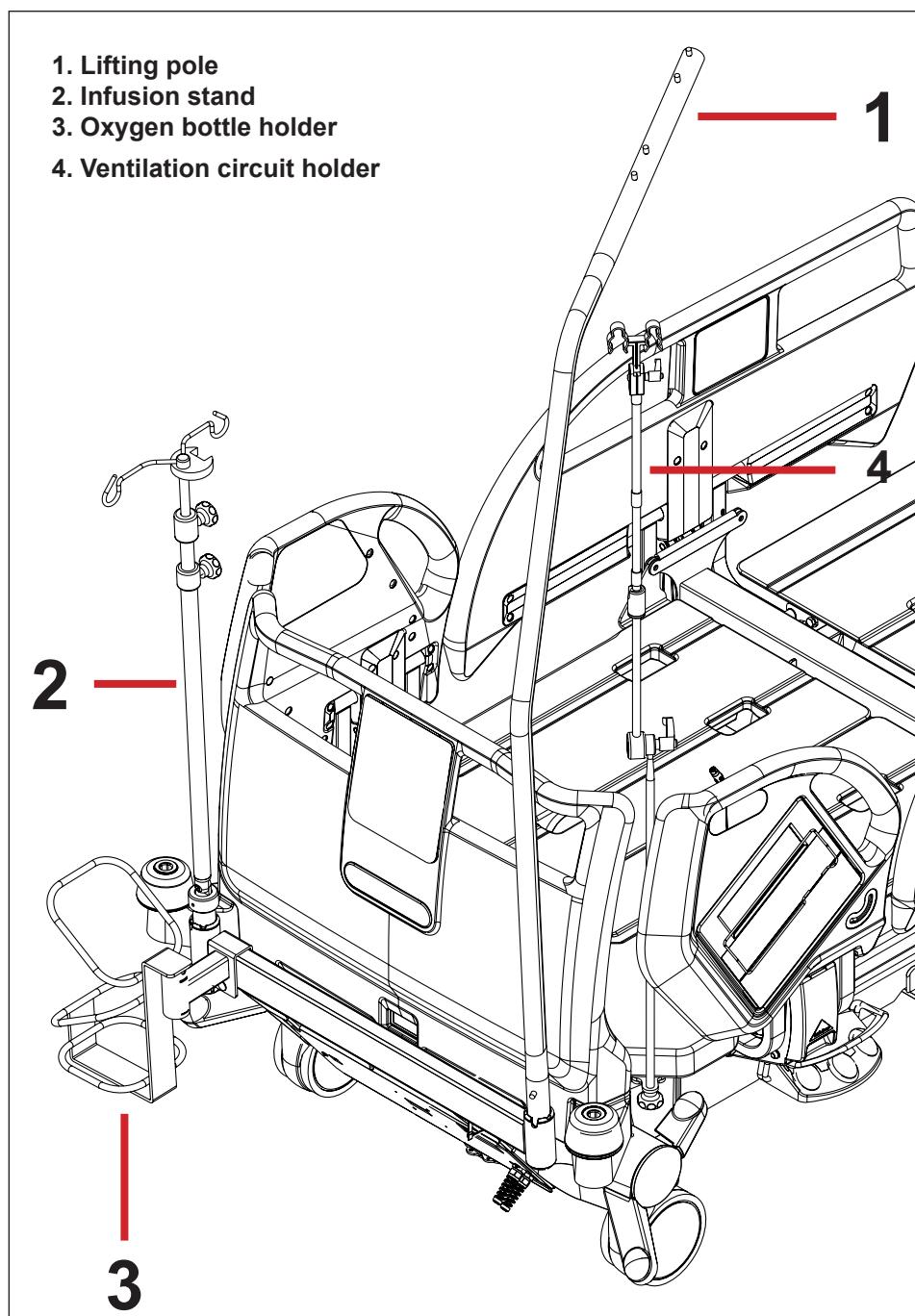


Fig. Accessories

16.1 Lifting Pole

To ensure safe use of the lifting pole:

- ▶ Never exceed the maximum load of 75 kg.
- ▶ Never use the lifting pole for rehabilitation exercises.
- ▶ To prevent the bed from tipping over, ensure that the lifting pole does not project out from the bed.
- ▶ Replace plastic handle every 4 years.

To install the lifting pole:

- ▶ Insert lifting pole in corresponding sleeve fitting on accessory adapter at head end.
- ▶ Ensure that safety pin locks into place.
- ▶ Attach a plastic grab handle with an adjustable strap to the lifting pole.

NOTE The date of manufacture is marked on the grab handle. LINET® recommends replacing the plastic grab handle every four years.

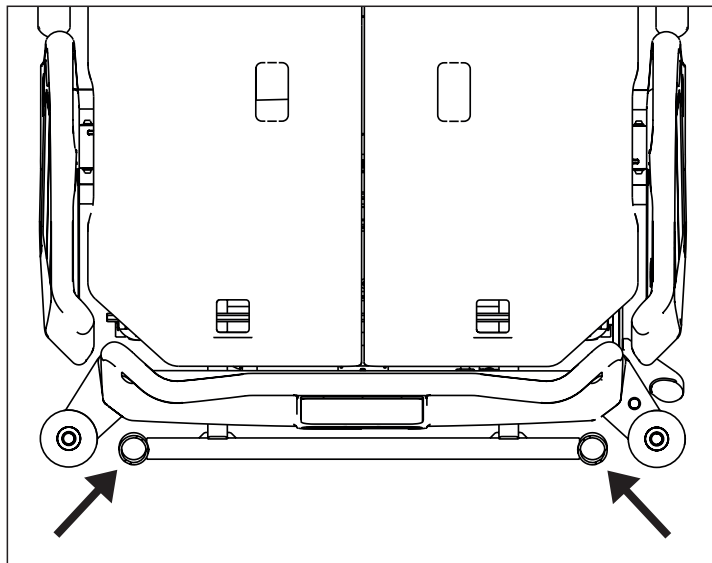


Fig. Places for lifting pole (sleeve fittings on accessory adapter)

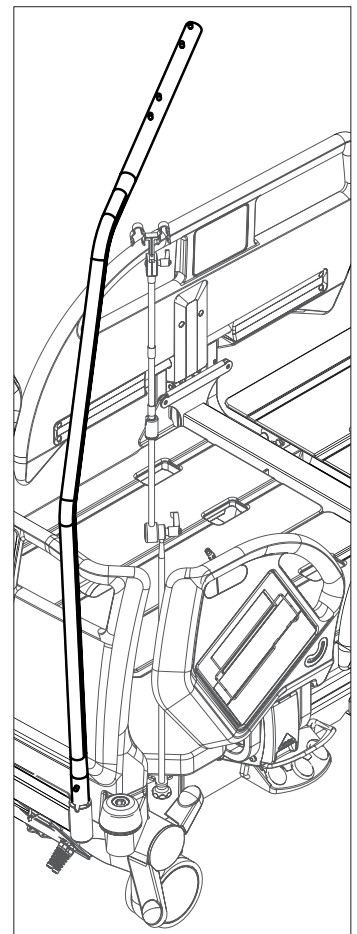


Fig. Lifting pole

16.2 Infusion Stand



WARNING!

Risk of injury due to use of incorrect accessories or because of incorrect use!

Infusion Stands must only be used for their intended use. Always read the instructions for use!

- ▶ Only mount an infusion pump to the lower (wider) telescopic section of an infusion stand above the head board / foot board.
- ▶ Never mount an infusion pump to the upper (thinner) telescopic section of an infusion stand.
- ▶ Ensure the infusion pump will not collide with any movable parts of the bed (especially Backrest part) or with the patient. This must be verified during installation.
- ▶ Do not over tighten the infusion pump clamps during fitment. Over tightening may damage the infusion stand.
- ▶ Infusion pump can be only used if the infusion stand is fitted in the accessory holder socket in the head end on the under carriage of the bed.
- ▶ Do not use the infusion stand as driving/pushing device during the bed transport.



CAUTION!

Risk of collision with oxygen bottle holder on the bed end due to incompatibility!

- ▶ Use the foldable infusion stand with adapter to avoid the collision.

Infusion stands are intended to provide a suitable support for the attachment of the infusion pumps/syringe pumps and suspension of the infusion bags or bottles.

Infusion stands can be fitted to the head and foot end of the bed by either fitting into the IV/Infusion sockets mounted on the bed or using alternative accessory holder socket in the head end on the undercarriage of the bed.

- ▶ Use exclusively infusion stands with hooks for hanging IV bags or baskets for intravenous solutions.
- ▶ Ensure the infusion stand individual hook 2kg maximum Safe Working Load is not exceeded.
- ▶ Capacity per hook: 2 kg.
- ▶ Ensure the infusion stand 20kg maximum Safe Working Load is not exceeded.
- ▶ Follow the actual price list for information about types of infusion stands.

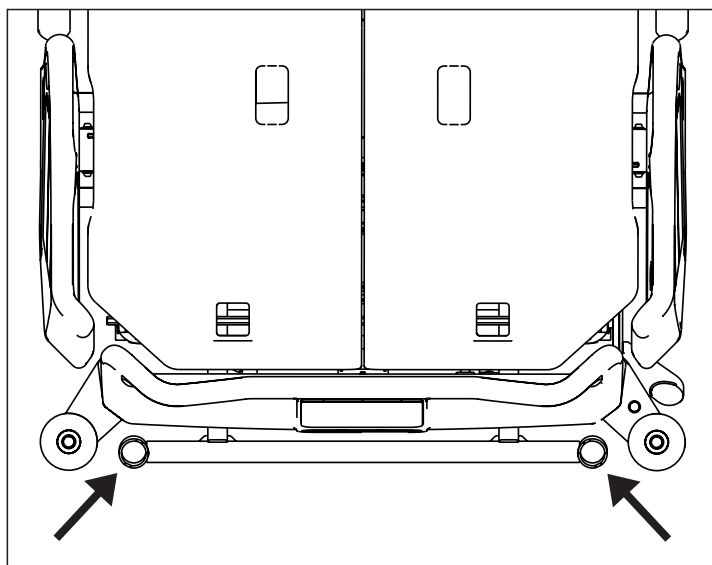


Fig. Places for infusion stand (sleeve fittings on accessory adapter)

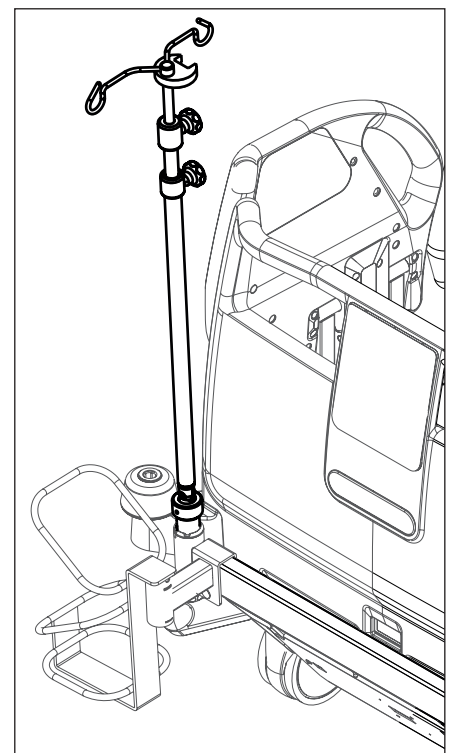


Fig. Infusion stand

16.3 Oxygen Bottle Holder



WARNING!

Risk of injury with oxygen bottle holder due to incorrect use or due to careless driving!

- ▶ Ensure the oxygen bottle holder is correctly fitted in correct position.
- ▶ It is necessary to place oxygen bottle holder (with or without O2 bottle) before transport to secure transport position.
- ▶ Be aware of people or objects in close proximity when driving or manipulating the bed equipped with oxygen bottle holder.
- ▶ Secure the oxygen bottles against falling or involuntary movement with rubber strap.
- ▶ Place the oxygen bottle holder on the bed by instructions in the following text.
- ▶ Ensure the oxygen bottle valve will not get damaged by careless or incorrect manipulation or placement.

The oxygen bottle holders are suitable for transporting oxygen bottles with a weight of up to 15 kg and a volume of 5 litres.

Horizontal version (Head End)

- ▶ Put oxygen bottle holder on transversal profile behind head end.


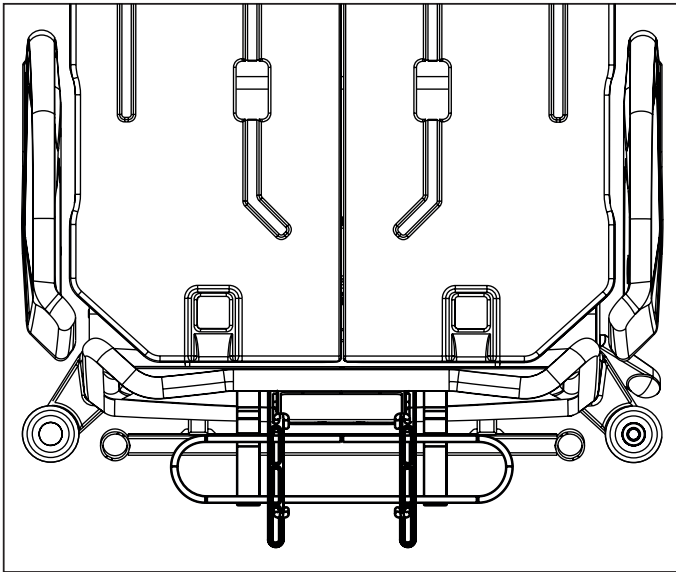


Fig. Oxygen Bottle Holder (on the head end)


Vertical version (with adapter)

On following pictures there are 4 positions of the oxygen bottle holder with adapter.


- ▶ Put holder on sleeve fittings in multifunctional accessory adapter on head end.
- ▶ Ensure the locking pin of vertical oxygen bottle holder is locked in sleeve fitting.

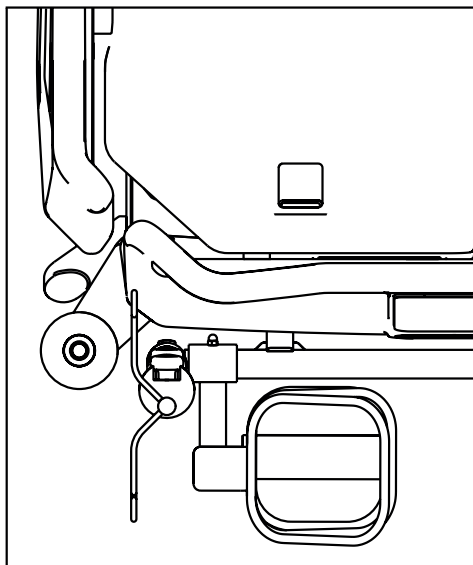


*Fig. Oxygen Bottle Holder in the adapter
(position 1)*

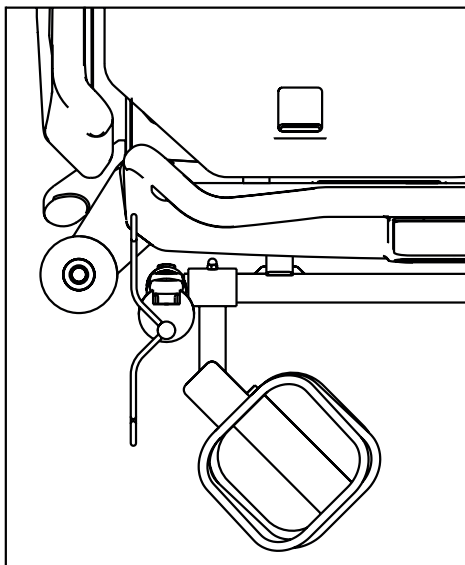

*Fig. Oxygen Bottle Holder in the adapter
(position 2)*

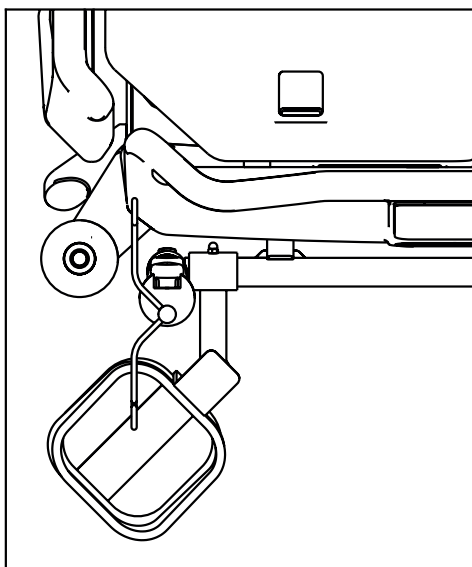

*Fig. Oxygen Bottle Holder in the adapter
(position 3)*

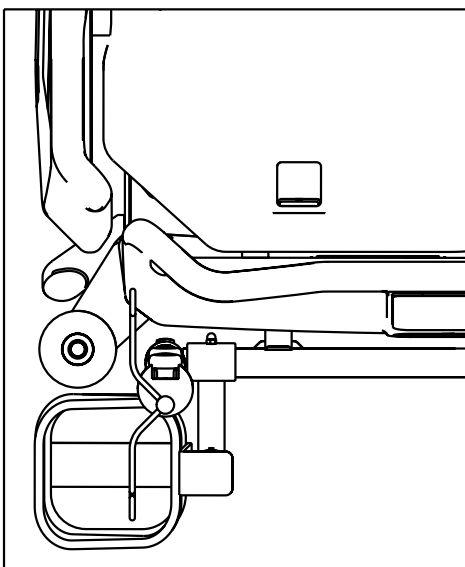

*Fig. Oxygen Bottle Holder in the adapter
(position 4)*

16.4 Ventilation Circuit Holder

The ventilation circuit holder prevents extubation of the patient connected to the ventilator.

- Always use LINET ® ventilation circuit holder to prevent extubation during any procedures.

Applying ventilation circuit holder:

- Put ventilation circuit holder in hole on right or left side of the Backrest frame.
 - Fasten ventilation circuit holder with wing screw provided.
 - Put intubation tube through plastic head of ventilation circuit holder.
 - Tilt mattress support platform left and right by 15° to check if intubation tube is fastened securely.
- The fastening is secure if no parts of the ventilation circuit are disconnected.

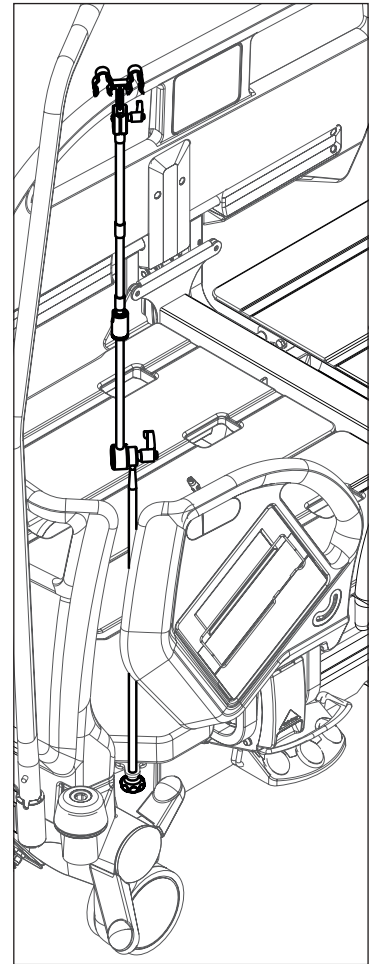


Fig. Ventilation Circuit Holder

16.5 Writing Shelf

The Writing Shelf is intended for writing of nursing staff. It is placed in the handles of the foot board (as on the picture).

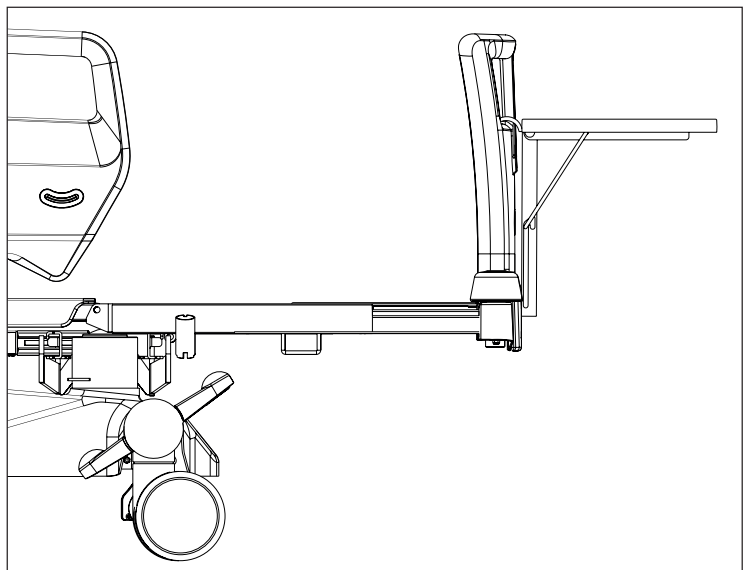


Fig. Writing Shelf

16.6 Monitor Shelf

The Monitor Shelf is suitable for transporting monitors with a weight of up to 15 kg.

Installing the monitor shelf:

- ▶ Place the Monitor Shelf on the foot board.
- ▶ Fixate monitor with safety belts in order to avoid any damage during transport.

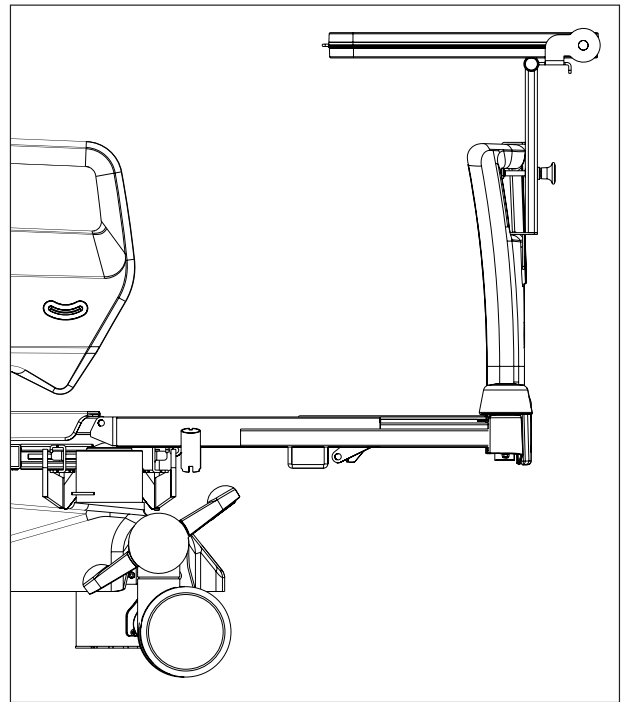


Fig. Monitor Shelf

16.7 Protector



WARNING!

Risk of injury due to the patient falling off the bed!

- ▶ Ensure that the Protector is installed securely.
- ▶ Always check that the siderails are properly locked in the „up“ position.
- ▶ Make sure the fall risk assessment was done properly before Protector use.

The Protector is an optional accessory for the Eleganza 5 bed. The main purpose of the Protector is to reduce the risk of fall especially at very risky patients (confused restless patients). The Protector can be used with extended or standard beds.

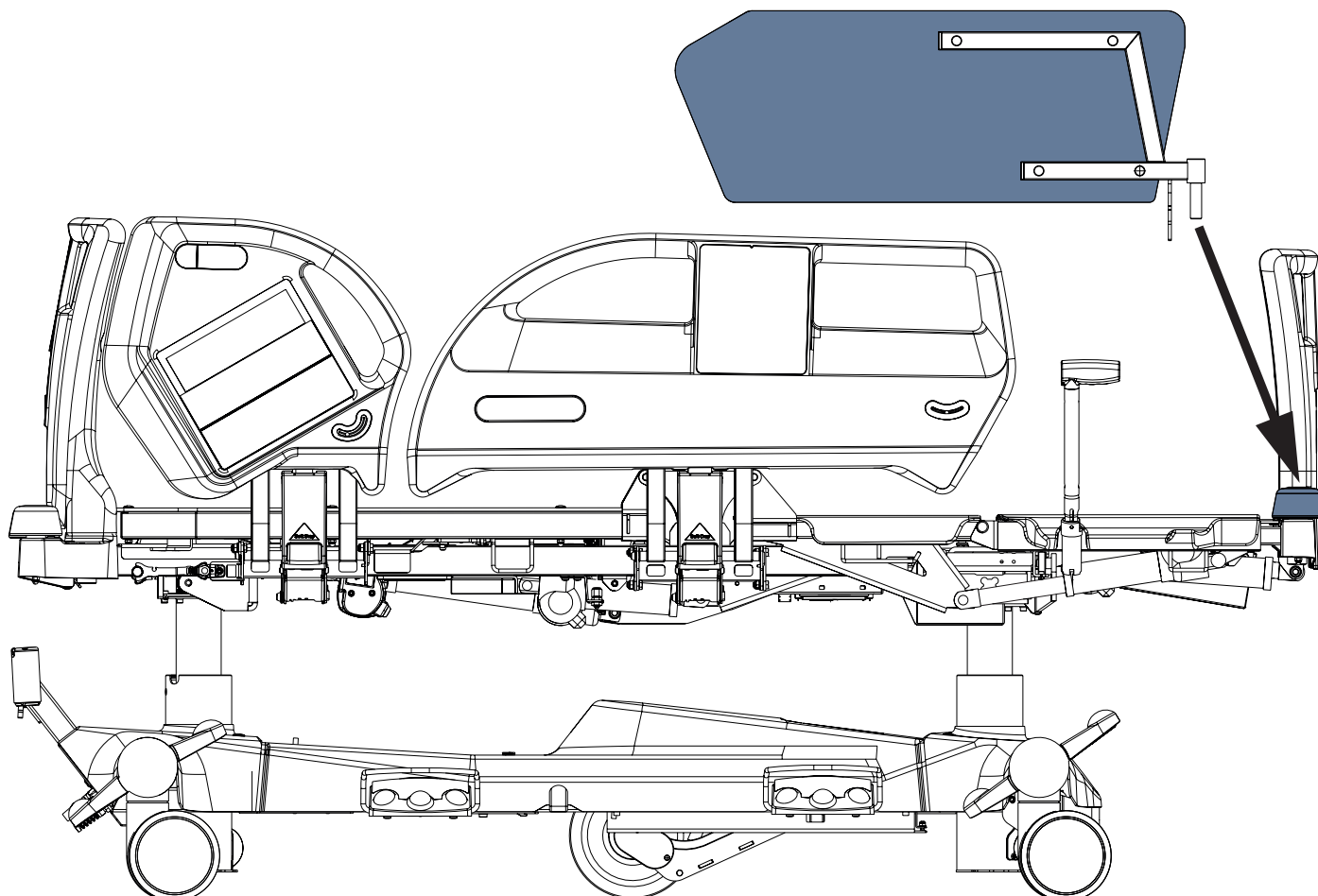


Fig. Placement of Protector

Placement

Attach the Protector to the bed as follows:

- ▶ Insert the Protector pin into the casing in the protective ring at the corner of the foot end of the bed.
- ▶ Ensure that the fixing element is secured to the telescopic profile of the bed extension.

Remove the Protector from the bed as follows:

- ▶ Grasp the upper end of the Protector.
- ▶ Remove the Protector from the casing.

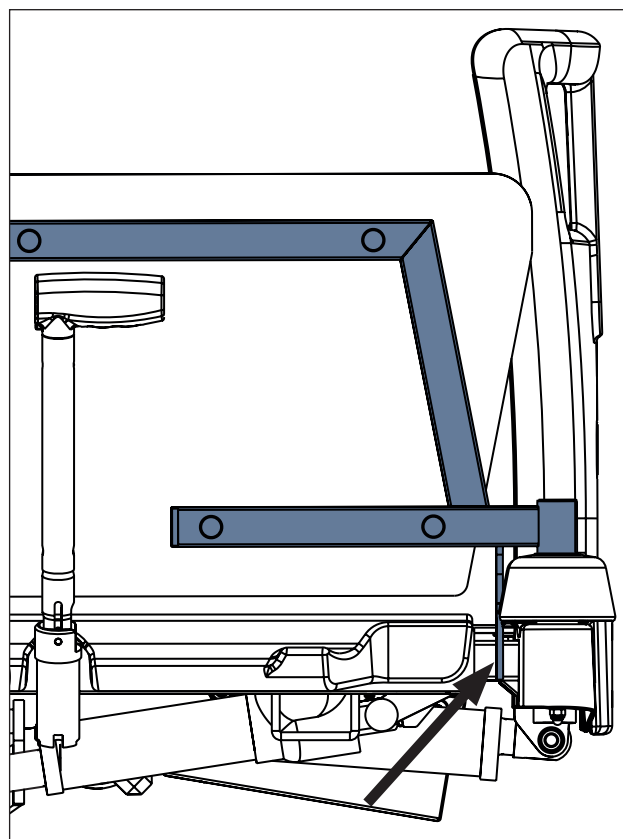


Fig. Fixing element on the telescopic profile of the bed extension

16.8 Urinary Bag Holder

Urinary Bag Holders are available on both sides of the bed at Backrest area.

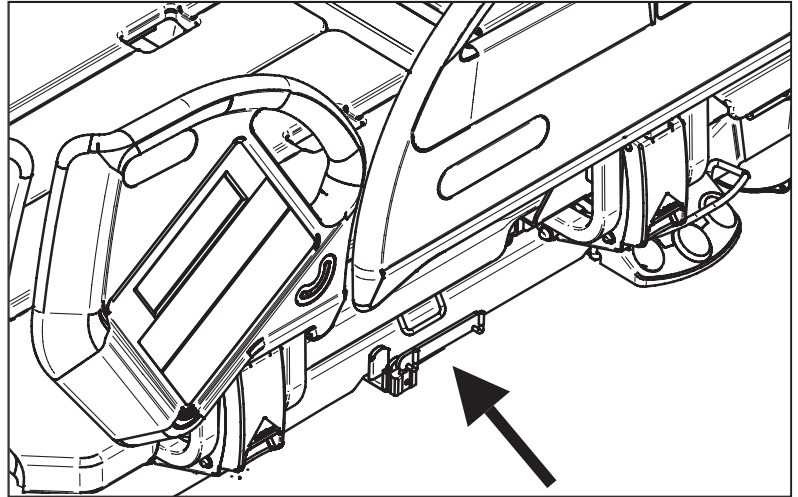


Fig. Placement of the Urinary Bag Holder

16.9 Traction Frame



WARNING!

Risk of injury due to incorrect use!

- ▶ There must be no patient on the bed when installing the Traction Frame!
- ▶ Avoid collisions between the Traction Frame and the bed (Foot Board, Backrest and Lateral Tilt) during bed positioning!
- ▶ Avoid collisions between the Traction Frame and accessories!
- ▶ Cross the thresholds with caution during transporting the bed with installed Traction Frame!
- ▶ It is allowed to transport a patient on the bed with Traction Frame just in emergency cases and with caution!
- ▶ Respect the Safe Working Load of the bed, of the Traction Frame and of its hooks and pulleys!
- ▶ Remove the Traction Frame from the bed if it is not needed for treatment!

Intended Use

Traction Frame is a supporting construction intended for fixation, traction and relief of limbs, spine and pelvis. Traction Frame is intended for orthopaedic department, surgery department, traumatologic department and for ICU.

Placement

Traction Frame is inserted to the holes in the Accessory Adaptor at head end and to the holes in the Traction Frame Holder at foot end.

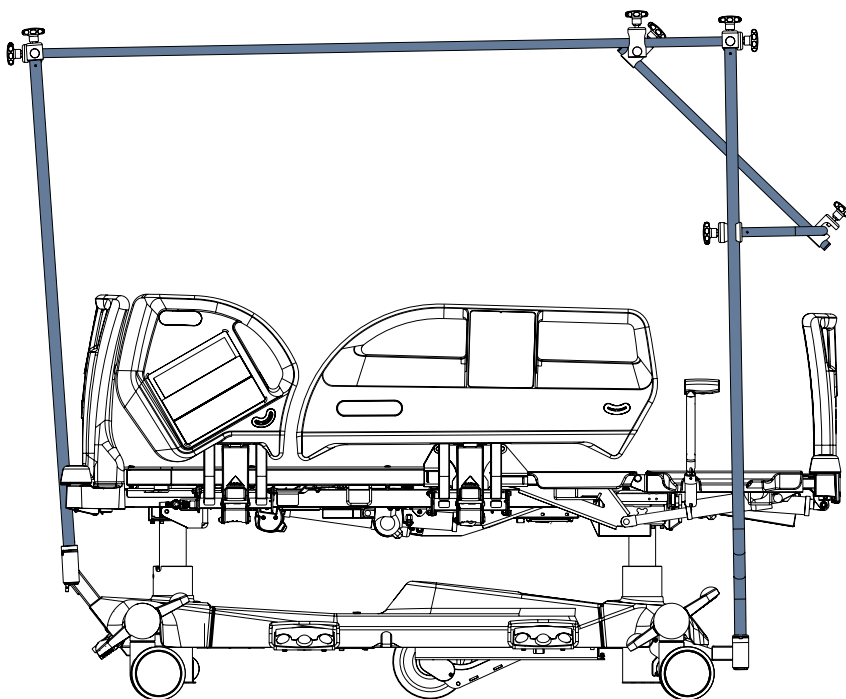


Fig. Eleganza 5 with Traction Frame (side view)

$$\begin{aligned} \text{hook} + \text{hook} &= 6 \text{ kg} \\ \text{max} \sum \text{hook} &= 12 \text{ kg} \end{aligned}$$

Fig. Safe Working Load of the hooks (Infusion Holder)

$$\text{pulley} = 20 \text{ kg}$$

Fig. Safe Working Load of the pulley

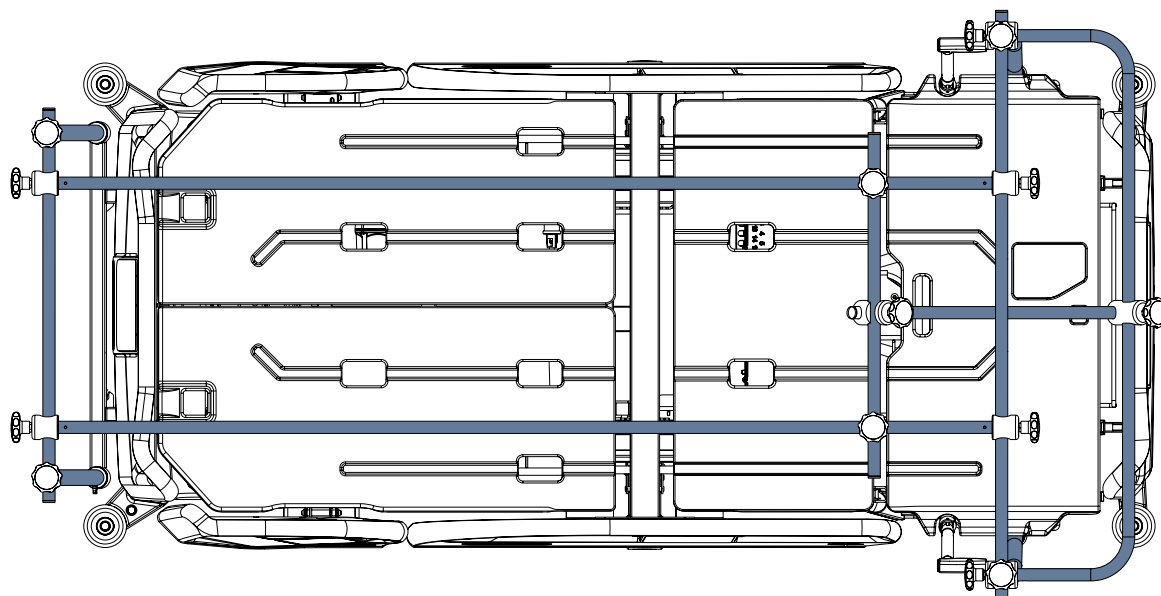


Fig. Eleganza 5 with Traction Frame (top view)

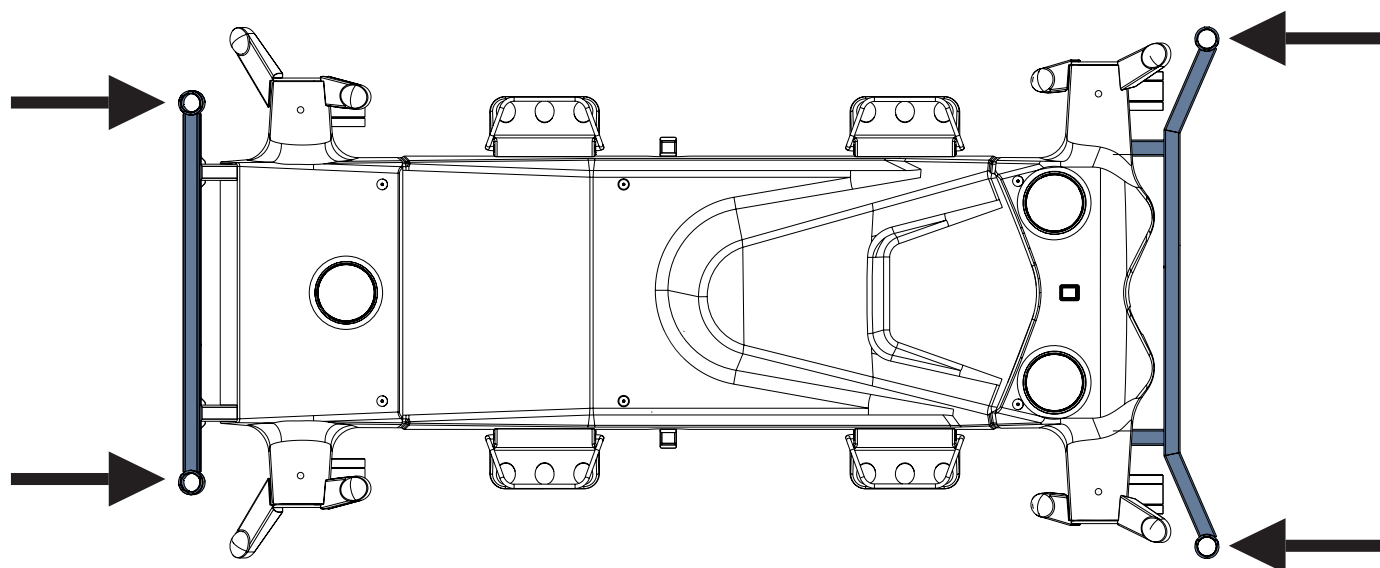


Fig. Positions to insert Traction Frame (undercarriage of Eleganza 5)

16.10 SafetyMonitor



WARNING!

Read the instructions for use for SafetyMonitor carefully before use of the Eleganza 5 bed with SafetyMonitor!

Intended Use

SafetyMonitor is a system that monitors statuses of Eleganza 5 bed: brake status (braked castors/unbraked castors), siderail status (siderails up/siderail down), bed height (bed in the lowest position/bed not in the lowest position), backrest angle (backrest in more than 30°/backrest in less than 30°), patient presence (patient on the bed/patient not on the bed) and location of the bed (where the bed is situated). Secure statuses are: braked bed, siderails up and locked, bed in the lowest position, backrest in more than 30° and patient on the bed. The system triggers notifications (notice signals for insecure statuses) and alerts (alert signals for the absence of a patient on the bed). Notifications and alerts are automatically passed to the hospital information system and displayed on a screen in the nurse station and on a smartphone/tablet. Information is transmitted via LAN or Wi-Fi connection. Acoustic Alert (Bed Exit Alert) sounds just from the bed. In this way, medical staff can be informed in a timely manner of any safety risk while saving administrative time.

System components

SafetyMonitor system consists of installed server, secure intranet infrastructure (Wi-Fi or LAN), screen in the nurse station (PC or tablet or smartphone), Tag on the wall, parked Eleganza 5 EMR ready bed with iBoard Standard or iBoard basic, Integration Module, Localisation Receiver, LAN connector and LAN cable.

iBoard Standard (Bed Exit Section)

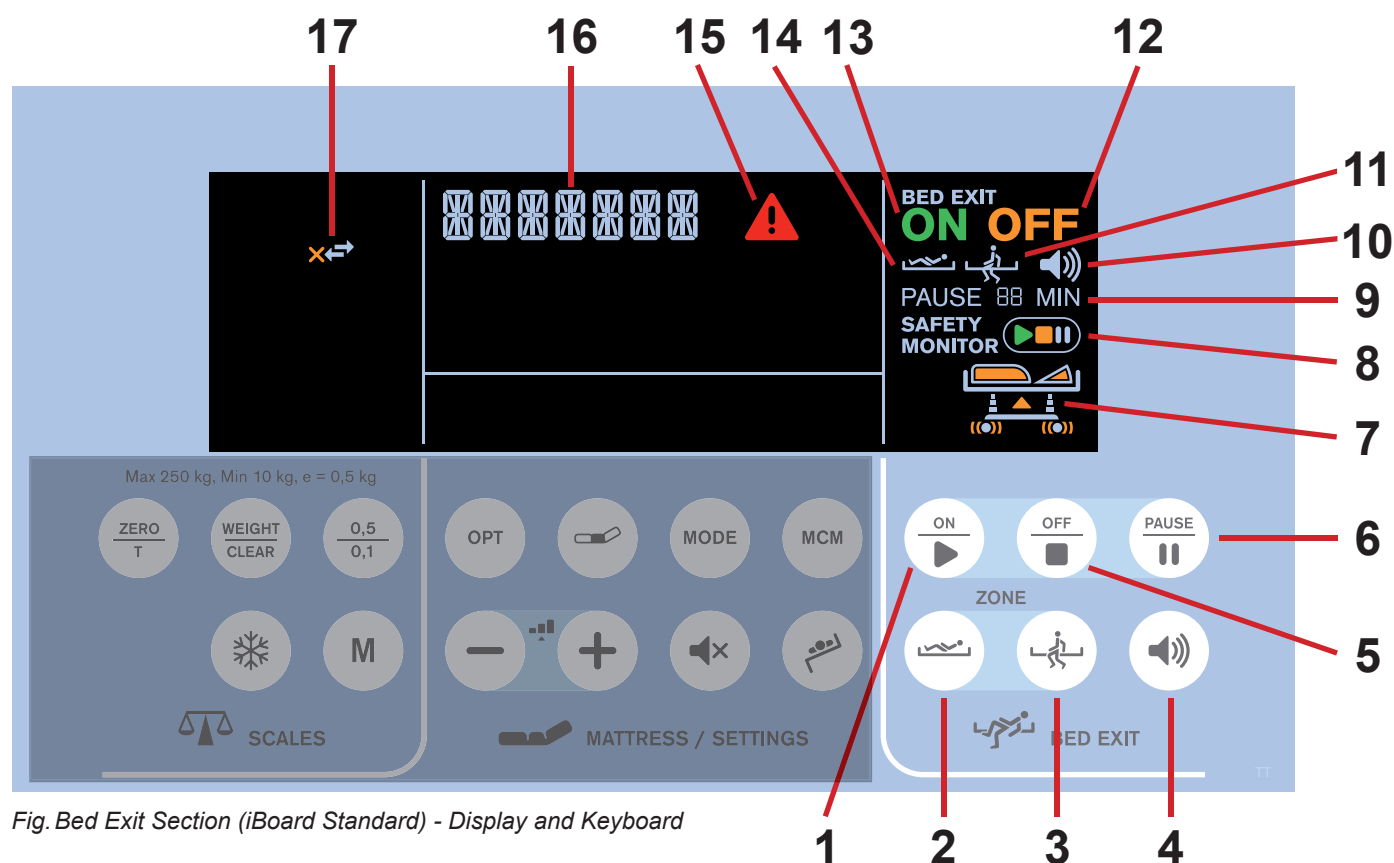


Fig. Bed Exit Section (iBoard Standard) - Display and Keyboard

1. ON button (RUN)
2. Inner Zone button (Bed Exit Monitoring)
3. Outer Zone button (Bed Exit Monitoring)
4. Volume button (3 levels of sound pressure)
5. OFF button
6. PAUSE button
7. Bed icon with bed statuses
8. SafetyMonitor system status indicator (ON/OFF/PAUSE)
9. PAUSE countdown (with remaining minutes)
10. Volume indicator (3 levels of sound pressure)
11. Bed Exit Monitoring Activated (Outer Zone Monitoring)
12. Bed Exit Monitoring OFF
13. Bed Exit Monitoring ON
14. Bed Exit Monitoring Activated (Inner Zone Monitoring)
15. WARNING icon
16. Bed status description
17. Server connection icon (arrows only – connected, arrows with cross – disconnected)

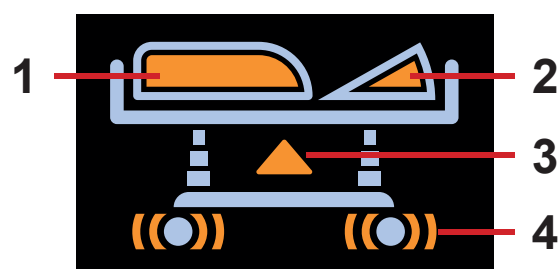


Fig. Bed icon with bed statuses (orange - alerts/insecure statuses)

1. siderail status (orange - siderail down)
2. backrest angle (orange - backrest in less than 30°)
3. bed height (orange - bed not in the lowest position)
4. brake status (orange - unbraked castors)

iBoard Basic (Bed Exit Section)

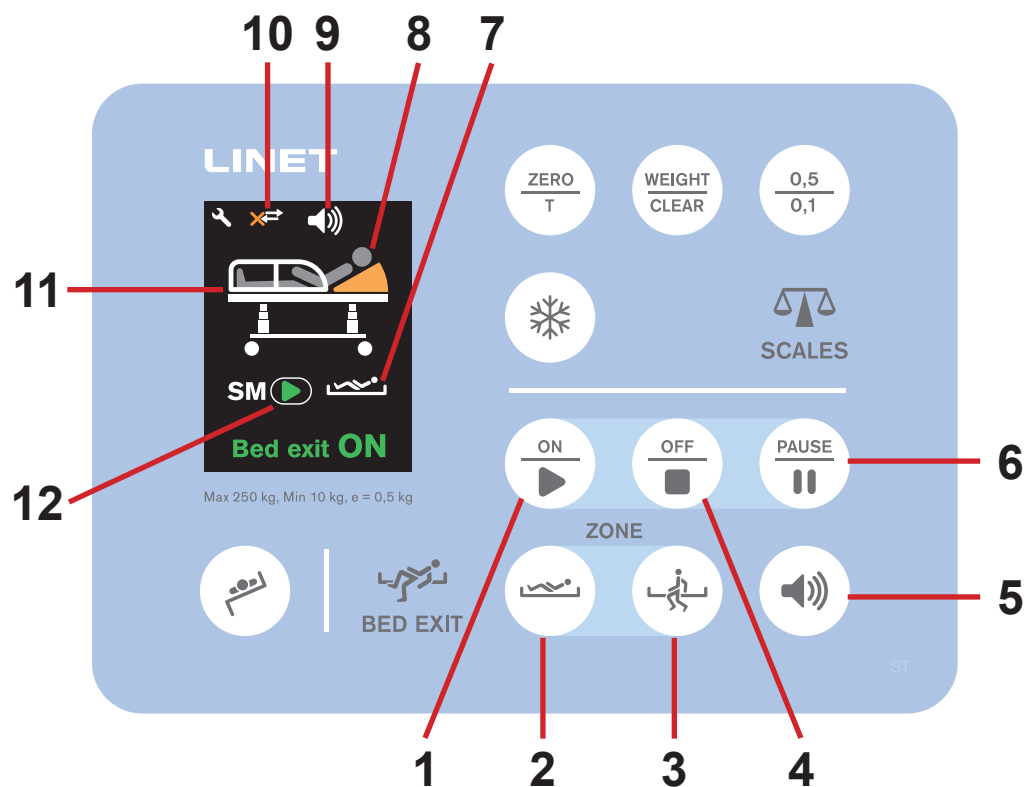


Fig. Bed Exit Section (iBoard Basic) - Display and Keyboard

1. ON button (RUN)
2. Inner Zone button (Bed Exit Monitoring)
3. Outer Zone button (Bed Exit Monitoring)
4. OFF button
5. Volume button (3 levels of sound pressure)
6. PAUSE button
7. Bed Exit Monitoring Activated (Inner Zone Monitoring)
8. Patient icon (patient is on the bed)
9. Volume indicator (3 levels of sound pressure)
10. Server connection icon (arrows only – connected, arrows with cross – disconnected)
11. Bed icon with bed statuses
12. SafetyMonitor system status indicator (ON/OFF/PAUSE)

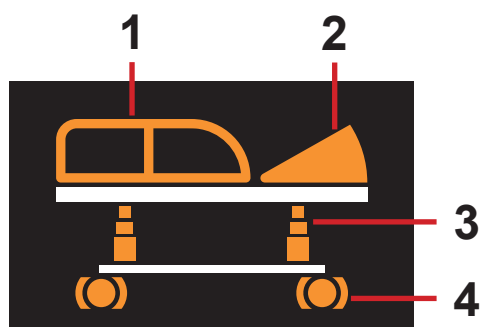


Fig. Bed icon with bed statuses
(orange - alerts/insecure statuses)

1. siderail status (orange - siderail down)
2. backrest angle (orange - backrest in less than 30°)
3. bed height (orange - bed not in the lowest position)
4. brake status (orange - unbraked castors)

16.11 Pushing Handles



WARNING!

Pushing handles are not compatible with i-Drive Power system!

- ▶ Do not use the pushing handles on the bed that is equipped with i-Drive Power system!




CAUTION!

Follow these instructions during placement of the pushing handles on the head end crossbar:

- ▶ Placement of the pushing handles must be performed by hospital technician according to these instructions for use!
- ▶ Pushing handles can only be placed on the head end crossbar!
- ▶ Positions of the pushing handles are shown on the picture below!
- ▶ Distance between bushing holder and the bar that is at right angle to the head end crossbar is 12 mm.
- ▶ Nuts of bolts must be oriented inwards!

Pair of Pushing Handles is intended for bed transport.
Pushing Handles are not compatible with head board.
Pushing Handles are located on the head end crossbar.

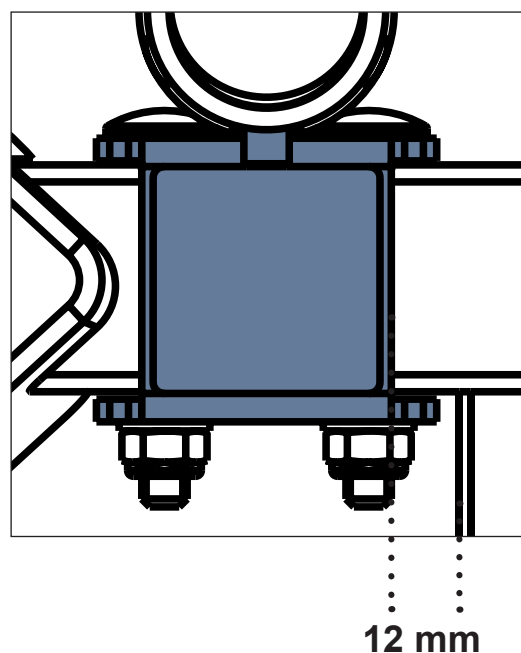

12 mm

Fig. Distance between bushing holder and the bar that is at right angle to the head end crossbar

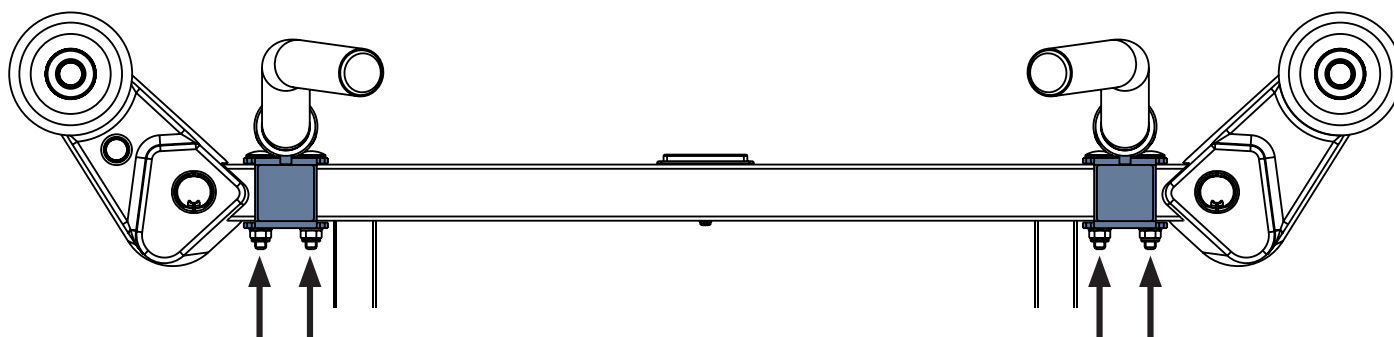


Fig. Placement of pushing handles on the head end crossbar (positions of nuts)

Removal of the Pushing Handles from the bushings:

- Pull both Pushing Handles out of the fixed bushings on the head end crossbar.

Insertion of the Pushing Handles to their fixed bushings:

- Insert both Pushing Handles to their fixed bushings on the head end crossbar.

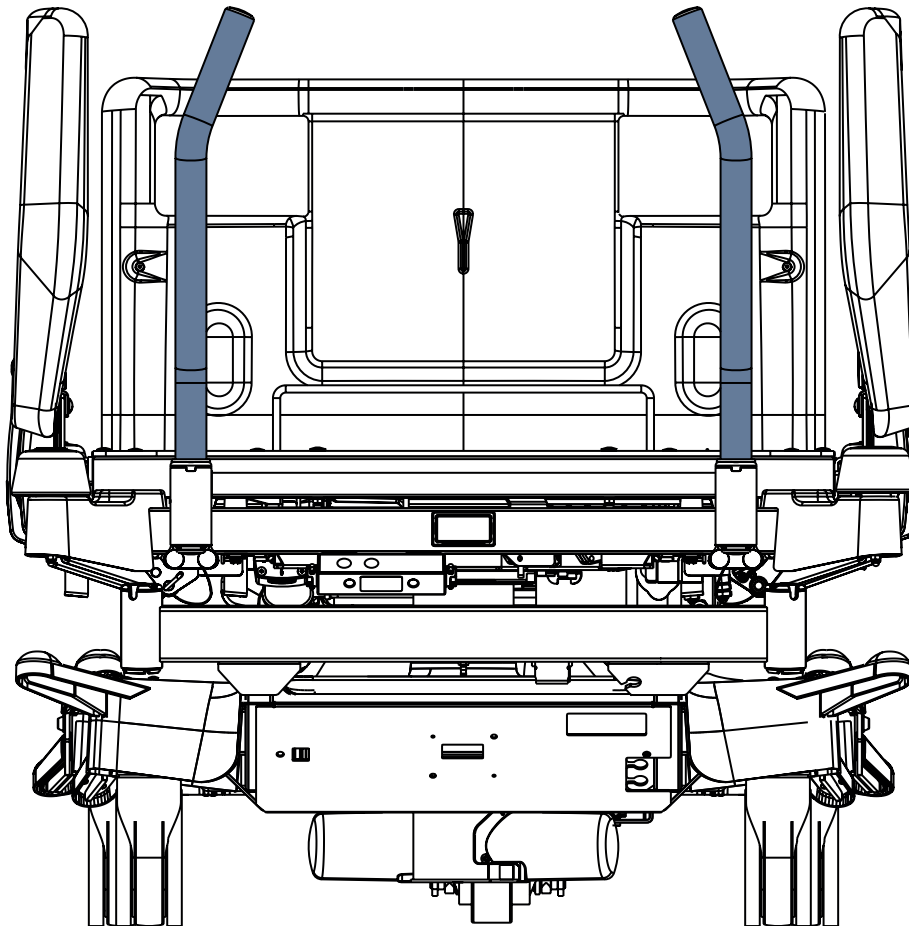


Fig. Pushing Handles on the Eleganza 5 bed


16.12 Pushing Handles with Holder of the i-Drive Power Control Panel



CAUTION!

Follow these instructions during placement of the Pushing Handles with Holder of the i-Drive Power Control Panel on the head end crossbar:

- ▶ Placement of the pushing handles must be performed by hospital technician according to these instructions for use!
- ▶ Pushing handles can only be placed on the bed head end crossbar!
- ▶ Positions of the pushing handles are shown on the picture below!
- ▶ Distance between bushing holder and the bar that is at right angle to the head end crossbar is 12 mm.
- ▶ Nuts of bolts must be oriented inwards!

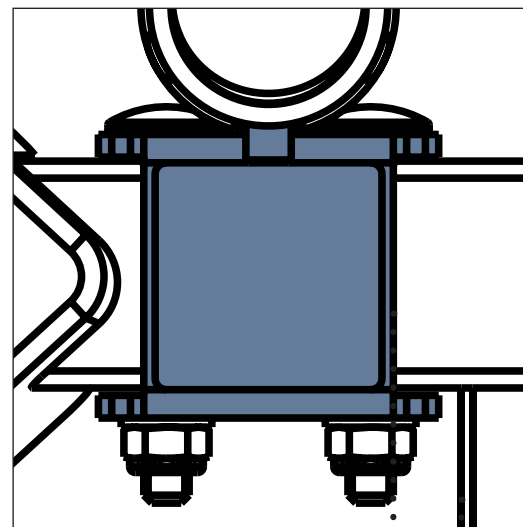

12 mm

Fig. Distance between bushing holder and the bar that is at right angle to the head end crossbar

Pair of Pushing Handles with Holder of the i-Drive Power Control Panel is intended for bed transport.

Pushing Handles with Holder of the i-Drive Power Control Panel are not compatible with bed head board.

Pushing Handles with Holder of the i-Drive Power Control Panel are located on the head end crossbar.

Pushing Handles with Holder of the i-Drive Power Control Panel are compatible with i-Drive Power system. i-Drive Power Control Panel is intended to be placed on the Holder of the i-Drive Power Control Panel.

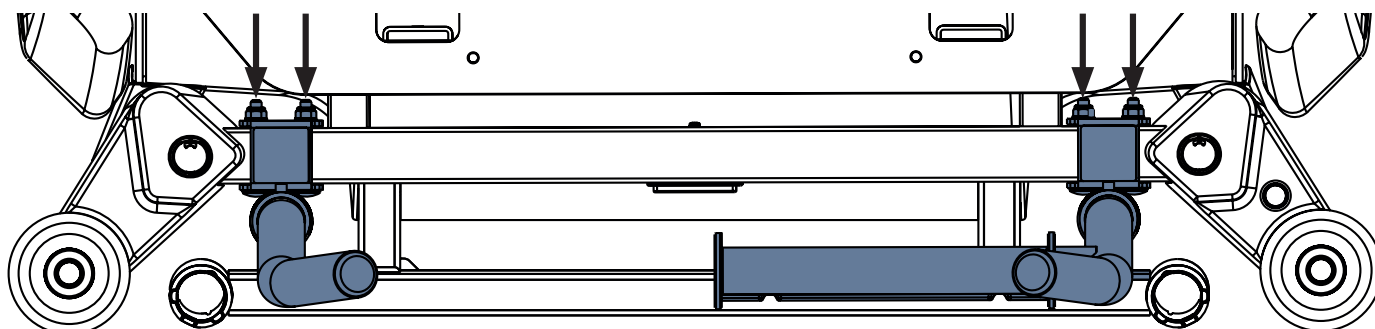


Fig. Placement of pushing handles on the head end crossbar (positions of nuts)

Removal of the Pushing Handles from the bushings:

- Pull both Pushing Handles out of the fixed bushings on the head end crossbar.

Insertion of the Pushing Handles to their fixed bushings:

- Insert both Pushing Handles to their fixed bushings on the head end crossbar.

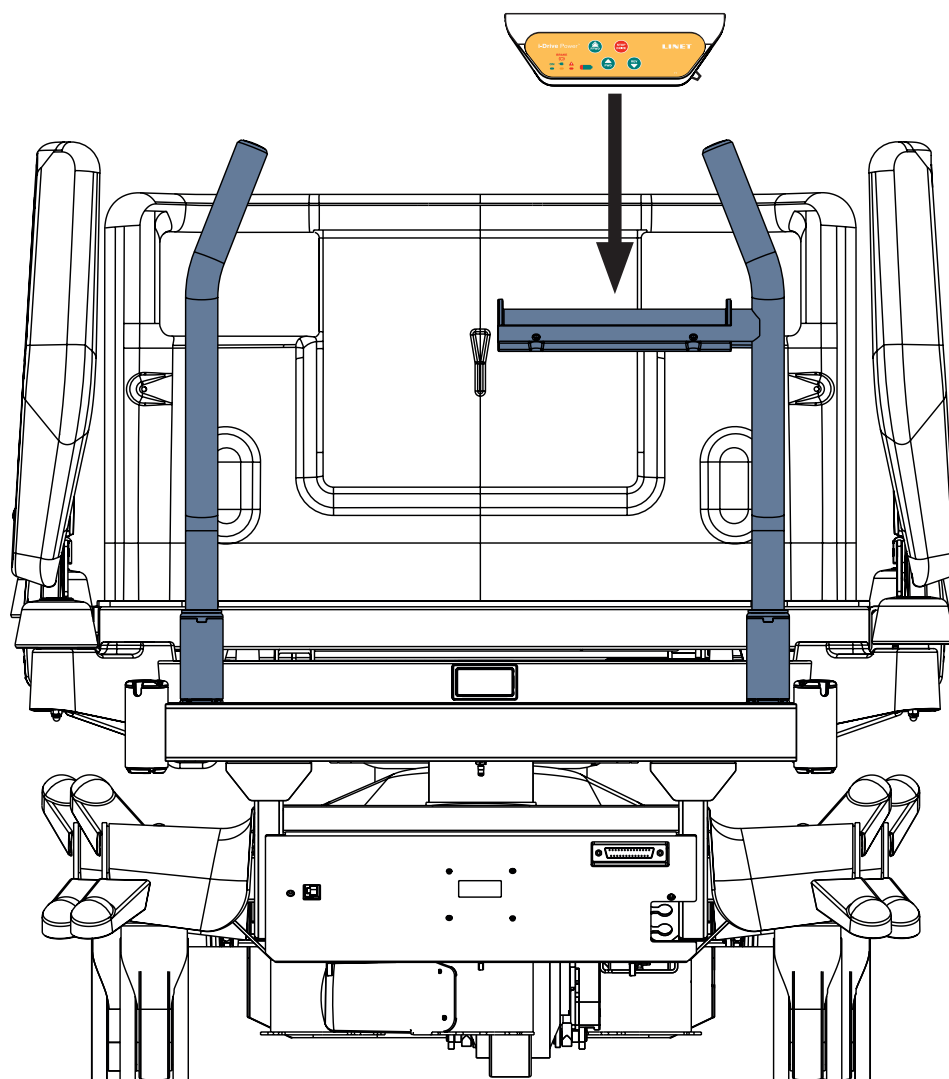


Fig. Pushing Handles with Holder of the i-Drive Power Control Panel on the Eleganza 5 bed

16.13 IDock (size S and size M)



WARNING!

Risk of injury or material damage due to incorrect use!

- ▶ Read Instructions for use of the IDock docking-cart before use of this product!
- ▶ Ensure the safe working load of the IDock docking-cart is not exceeded!



CAUTION!

Risk of collision due to incompatibility!

- ▶ Ensure the selected type of IDock is compatible with the bed undercarriage!
- ▶ IDock is not compatible with any active mattress control unit placed on the Foot Board!



CAUTION!

Risk of injury or material damage due to incorrect installation!

- ▶ The hospital personnel has to ensure the secure mounting of devices on the IDock and of the IDock on the bed!

IDock is intended to transport medical devices and oxygen bottles.

IDock is compatible with Multicare X bed, Multicare bed, Eleganza 5 bed and Eleganza 4 bed.

IDock size S and IDock size M are 2 types of the IDock docking-cart.

IDock must be connected with Foot Board via 2 hooks. If IDock is equipped with Transport Bracket (WDE2903860000) installed on the Foot Board of Multicare X bed or Multicare bed, IDock should be connected with the Transport Bracket via 2 hooks instead.



Fig. IDock docking-cart

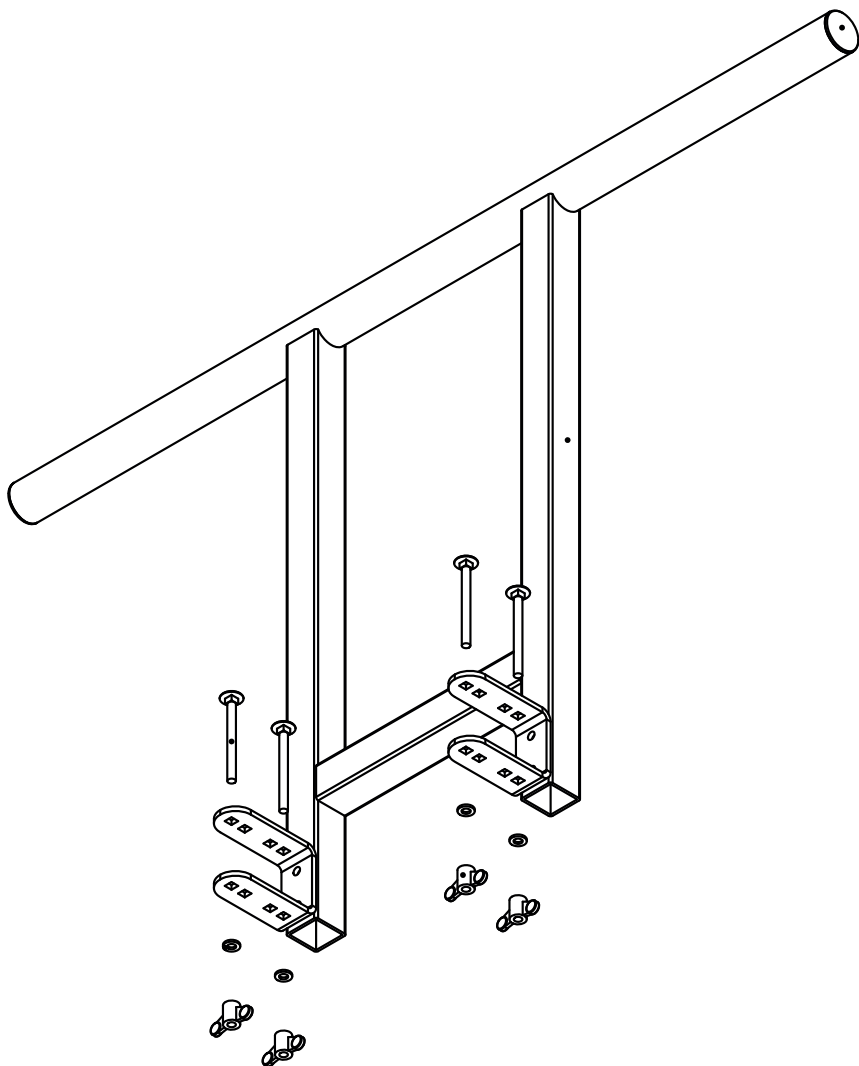


Fig. Installation of the Transport Bracket on the bed foot end crossbar

17 Cleaning/Disinfection


WARNING!
Risk of injury due to accidental bed movement!

- ▶ Always disable the function buttons when cleaning between the undercarriage and mattress support platform.


CAUTION!
Material damage due to incorrect cleaning/disinfection!

- ▶ Do not use washing machines.
- ▶ Do not use pressure or steam cleaners.
- ▶ Follow the instructions and observe the dosages recommended by the manufacturer.
- ▶ Ensure that disinfectants are selected and applied exclusively by qualified hygiene experts.
- ▶ **Check if used cleaning agents and disinfectants are compatible with materials that the product consists of! For information see the following table.**

BED COMPONENTS THAT ARE INTENDED TO BE CLEANED Do not clean what is not mentioned in this column!	MATERIALS (SURFACES OF THE MENTIONED BED COMPONENTS) Competent user is responsible for check if used cleaning agents and disinfectants are compatible with mentioned materials!	
Head board and foot board	Polypropylene (PP)	
Head siderails and foot siderails	Polypropylene (PP)	
Mattress support platform covers (Backrest)	Polypropylene (PP)	version with x-ray cassette holder: High Pressure Laminate (HPL)
Mattress support platform covers (Thighrest, Calfrest)	Polypropylene (PP)	
Seat section	Lacquered steel	
Castors	Polyurethane (PUR) + Polypropylene (PP)	
Castor control levers	Polyamide (PA6) + Lacquered steel	
Frame of the mattress support platform	Lacquered steel	
Columns	Oxidized aluminium alloy	
Undercarriage cover	Acrylonitrile butadiene styrene (ABS)	
Corner covers	Polypropylene (PP)	
Corner bumpers	Polypropylene (PP)	
Keyboards (Attendant Control Panel, Handset, control elements integrated in the siderails)	Polyethylene terephthalate (PET)	
CPR levers	Lacquered steel	
Labels	Polyethylene terephthalate (PET)	
Accessory rail	Polyoxymethylene (POM) + Lacquered steel	
Actuators	Polyamide (PA6) + Aluminium (Al)	
Decors (head board, foot board, head siderails, foot siderails)	Acrylonitrile butadiene styrene (ABS)	
Mobi-Lift® handles	Polyamide (PA66) + Lacquered steel	
Foot controllers	Acrylonitrile butadiene styrene (ABS) + rubber + Lacquered steel	

For safe and gentle cleaning:

- ▶ Do not use any strong acids or bases (optimum pH range 6 - 8).
- ▶ Exclusively use detergents that are suitable for cleaning medical equipment.
- ▶ Do not use abrasive powders, steel wool, or other materials and cleaning agents that might damage the mattress replacement system.
- ▶ Never use any corrosive or caustic detergents.
- ▶ Never use detergents that deposit calcium carbonate.
- ▶ Never use detergents with solvents that might affect the structure and consistency of the plastics (benzene, toluene, acetone, etc.).
- ▶ Clean electrical components carefully and allow them to dry completely.
- ▶ Do not immerse SCU in water or steam-clean it.
- ▶ Observe local directives regarding infection control.
- ▶ Make sure any cleaning agent used is approved by:
 - the facility in which the mattress replacement system is to be used.
 - by the environmental protection agency of the country in which the mattress replacement system is to be used.

17.1 Cleaning (Eleganza 5)

Prepare for cleaning as follows:

- ▶ Put the mattress support platform in the highest position.
- ▶ Adjust the back and thigh rests so that the reverse sides are accessible.
- ▶ Disable the function buttons on the control elements using the Attendant Control Panel.
- ▶ Disable the foot controls using the Attendant Control Panel.
- ▶ Disconnect the bed from the mains.
- ▶ Move the bed to the location where it will be cleaned.
- ▶ Lock the brakes on the bed.

17.1.1 Daily Cleaning

Clean the following bed parts:

- All control elements for adjusting the bed
- All handles
 - CPR release handle
- Head board and Foot board
- Siderails (in highest position)
- Freely accessible mattress surface
- Mobi-Lift®
- Accessory rails

17.1.2 Cleaning before Changing Patients

Clean the following bed parts:

- All control elements for adjusting the bed
- All handles
- CPR release handle
- Head board and Foot board
- Siderails (in highest position)
- Freely accessible mattress surface
- Mobi-Lift®
- Accessory rails
- All plastic mattress support platform covers
- Plastic undercarriage covers
- Telescopic columns
- Mattress on all sides
- Freely accessible metal parts of mattress support platform
- Cable ducts
- Lifting pole sleeve fitting
- Infusion stand sleeve fitting
- Bumpers
- Castors
- Brakes

17.1.3 Complete Cleaning and Disinfection

Clean the following bed parts:

- All control elements for adjusting the bed
- All handles
- CPR release handle
- Head board and Foot board
- Siderails (in highest position)
- Freely accessible mattress surface
- Mobi-Lift®
- Accessory rails
- All plastic mattress support platform covers
- Plastic undercarriage covers
- Telescopic columns
- Mattress on all sides
- Freely accessible metal parts of mattress support platform
- Cable ducts
- Lifting pole sleeve fitting
- Infusion stand sleeve fitting
- Bumpers
- Castors
- Brakes
- Interior parts (accessible after removing mattress support platform covers)

18 Troubleshooting



DANGER!

Risk of mortal injury due to electric shock!

- ▶ If a fault occurs, have the electric motor, power box or other electrical parts repaired by qualified personnel exclusively.
- ▶ Do not open the protective covers of the electric motor or the power box.

Error/Fault	Cause	Solution
Adjusting with position buttons not possible	GO Button was not pressed	Press the GO button.
	Function disabled on Attendant Control Panel	Enable disabled function.
	Actuators have no power Defective actuators Defective accumulator	Check the mains connection. Notify the manufacturer's service department.
	Mains Plug inserted incorrectly	Insert the Mains Plug correctly.
	Faulty Power Source	Notify the manufacturer's service department.
	Faulty Control Element	Notify the manufacturer's service department.
Faulty Mattress support platform Height/Tilt Adjustment	Obstacle on the undercarriage cover	Remove the obstacle.
	Function disabled on Attendant Control Panel	Enable disabled function.
	Actuators have no power Defective actuators Defective accumulator	Check the mains connection. Notify the manufacturer's service department.
	Mains Plug inserted incorrectly	Insert the Mains Plug correctly.
	Faulty Power Source.	Notify the manufacturer's service department.
	Faulty Control Element.	Notify the manufacturer's service department.
Lowering Backrest from the upright position not possible	Obstacle under the Backrest or in the drive mechanism.	Remove the obstacle
	CPR Release Handle is defective	Notify the manufacturer's service department.
Adjusting Siderails not possible	Obstacle in the Siderail Release Mechanism	Remove the obstacle.
	Siderail Release Mechanism is defective.	Notify the manufacturer's service department.
Faulty brakes	Obstacle blocking brakes mechanically	Remove the obstacle.
	The brake mechanism is defective	Notify the manufacturer's service department

19 Maintenance



WARNING!

Risk of injury when working on the bed!

- ▶ Ensure that the bed is disconnected from the mains power prior to installation, putting into service, maintenance and deinstallation.
- ▶ Ensure that the castors are locked prior to installation, putting into service, maintenance and deinstallation.
- ▶ No part of the Eleganza 5 ME equipment shall be serviced or maintained while in use with a patient.



WARNING!

Risk of injury due to defective bed!

- ▶ Have a defective bed repaired immediately.
- ▶ If the defect cannot be repaired, do not use the bed.



CAUTION!

Material damage due to incorrect maintenance!

- ▶ Ensure that maintenance is performed exclusively by manufacturer's customer service or by authorised service personnel certified by the manufacturer.
- ▶ If the defect cannot be repaired, do not use the bed.

LINET ® recommends attaching the maintenance plaque to the bed.

19.1 Regular maintenance

- ▶ Check regularly movable parts for wear.
- ▶ Perform regularly visual check of the product (with delivery note if necessary).
- ▶ Ask service department of the manufacturer for addition of the original spare parts if some product parts are missing.
- ▶ Ask service department of the manufacturer for replacement of any damaged product parts by the original spare parts.
- ▶ Check that the accumulator is working properly. Disconnect the bed from the mains power to check signalisation of accumulator indicator according to the instructions for use.
- ▶ Have the accumulator replaced if it is not working properly.
- ▶ Check regularly that all accessories are working properly.
- ▶ Replace damaged accessories immediately.

19.2 Spare Parts

The serial label is located on the frame of the mattress support platform. The serial label contains information for claims and ordering replacement parts.

Information about spare parts is available from:

- Manufacturer's customer service
- Sales department

19.3 Electrical Safety Check and Preventive Maintenance

19.3.1 Electrical Safety Check



WARNING!

Risk of injury due to incorrect electrical safety checks!

- ▶ Ensure that electrical safety checks are performed exclusively by manufacturer's customer service or by authorised service personnel certified by the manufacturer.
- ▶ Ensure that the electrical safety checks are recorded in accordance with local regulations and Manufacturer's recommendation.

Electrical safety check of the bed must be performed at least once every 12 months.

The procedure for performing the electrical safety check is stipulated in EN 62353:2014.

19.3.2 Preventive Maintenance

Preventive maintenance is the routine service operations of equipment to ensure safe use, prevent unexpected failures and extend life span.

To ensure the correct Preventive Maintenance:

- ▶ Ensure that preventive maintenance is performed exclusively by manufacturer's customer service or by authorised service personnel certified by the manufacturer.
- ▶ Ensure that preventive maintenance is performed in strict accordance with the Service Manual and the mandatory process steps described in the Preventive Maintenance – Checklist.
- ▶ Ensure that the electrical safety checks are recorded in accordance with local regulations and Manufacturer's recommendation.

Preventive maintenance of the bed must be performed at least once every 12 months.

NOTE On request, the manufacturer will provide service documentation (e.g. circuit diagrams, component part lists, descriptions, calibration instructions etc.) for service personnel for the repair of ME equipment designated by the manufacturer as repairable by service personnel.

20 Disposal

20.1 Environment Protection

The company LINET® is aware of the importance of environmental protection for future generations. Within this company the environmental management system is applied in accordance with the internationally agreed standard ISO 14001. The compliance with this standard is annually tested by the external audit executed by an authorised company. Based on the **WEEE** - Waste, Electric and Electronic Equipments Directive the company LINET, s. r. o. is registered in the List of Electric and Electronic Equipment Producers (**Seznam výrobců elektrozařízení**) on the Ministry of the Environment of the Czech Republic (Ministerstvo životního prostředí).

Materials used in this product and in the LINET® accessories are not environmentally hazardous. LINET® products and LINET® accessories meet valid requirements of national and European legislation in the areas of **RoHS** and **REACH**, so they do not contain any prohibited substances in excess quantities. None of the wooden parts is made of tropical wood (such as mahogany, rosewood, ebony, teak etc.) or made of timber from the Amazon region or from similar rainforests. Product noise (sound pressure level) meets requirements of the regulations for the protection of public health against undesirable effects of noise and vibration in protected interior spaces of buildings. Used packaging materials are in accordance with requirements of the Packaging Act (**Zákon o obalech**). For disposal of packaging materials after installation of products contact your sales representative or manufacturer's customer service about the possibility of a free take-back of packaging through an authorized company (more details on www.linet.cz).

20.2 Disposal

Materials used in this product and in the LINET® accessories burden the environment but at the same time the whole range of these materials can be very effectively reused and recycled. Mechanical disassembly of the product and material sorting into the basic waste types (plastic, metal, wooden materials) should be performed after the end of product life. The main objective of the obligations arising from the European Directive on Waste, Electric and Electronic Equipments is to increase the re-use, material recovery and recovery of electric and electronic equipment at the required level, thereby avoiding the production of waste and thereby avoiding the possible harmful effects of hazardous substances contained in electric and electronic equipment on human health and the environment. LINET® electric and electronic equipments that have a built-in battery or accumulator and LINET® accessories are designed so that the used batteries or accumulators can be safely removed by LINET® qualified service technicians. There is an information about its type on the built-in battery or accumulator.

20.2.1 Within Europe

To dispose of the electric and electronic equipment including LINET® accessories:

- ▶ The electric and electronic equipment must not be disposed of as household waste.
- ▶ Dispose of this equipment at designated collection points or take-back points.
- ▶ Dispose of the product or its components or its accessories in accordance with local laws and regulations!
- ▶ Hire an approved waste disposal company for disposal!

To dispose of the other equipment including LINET® accessories:

- ▶ The equipment must not be disposed of as household waste.
- ▶ Dispose of this equipment at designated collection points or take-back points.

LINET® participates in a collective system with take-back company REMA System. By bringing electric and electronic equipment to a take-back point, you participate in recycling and you save primary raw material resources while protecting your environment from effects of unprofessional disposal.

20.2.2 Outside Europe

- ▶ Dispose of the product or its components or its accessories in accordance with local laws and regulations!
- ▶ Hire an approved waste disposal company for disposal!

21 Warranty

LINET® will only be held responsible for the safety and reliability of products that are regularly serviced, maintained and used in accordance with the safety guidelines.

Should a serious defect arise that cannot be repaired during maintenance:

- ▶ Do not continue to use the bed.

The warranty on this product and its conditions depend on the agreement between the buyer and the seller, in addition to the legal minimum guarantee provided by law.

The warranty covers all material and manufacturing-related failures and errors. Failures and errors caused by incorrect use and external effects are not covered. Justified complaints will be fixed free of charge during the warranty period. Proof of purchase, with the date of purchase, is required for all warranty service. Our standard terms and conditions apply.

22 Standards and Regulations

Applied norms are stated on Declaration of Conformity.

The manufacturer adheres to a certified quality management system in compliance with the following standards:

- ISO 9001
- ISO 14001
- ISO 13485
- MDSAP (Medical Device Single Audit Program)