


## Instructions for Use and Technical Description



## Essenza 300 Junior<sup>+</sup>® Accessories

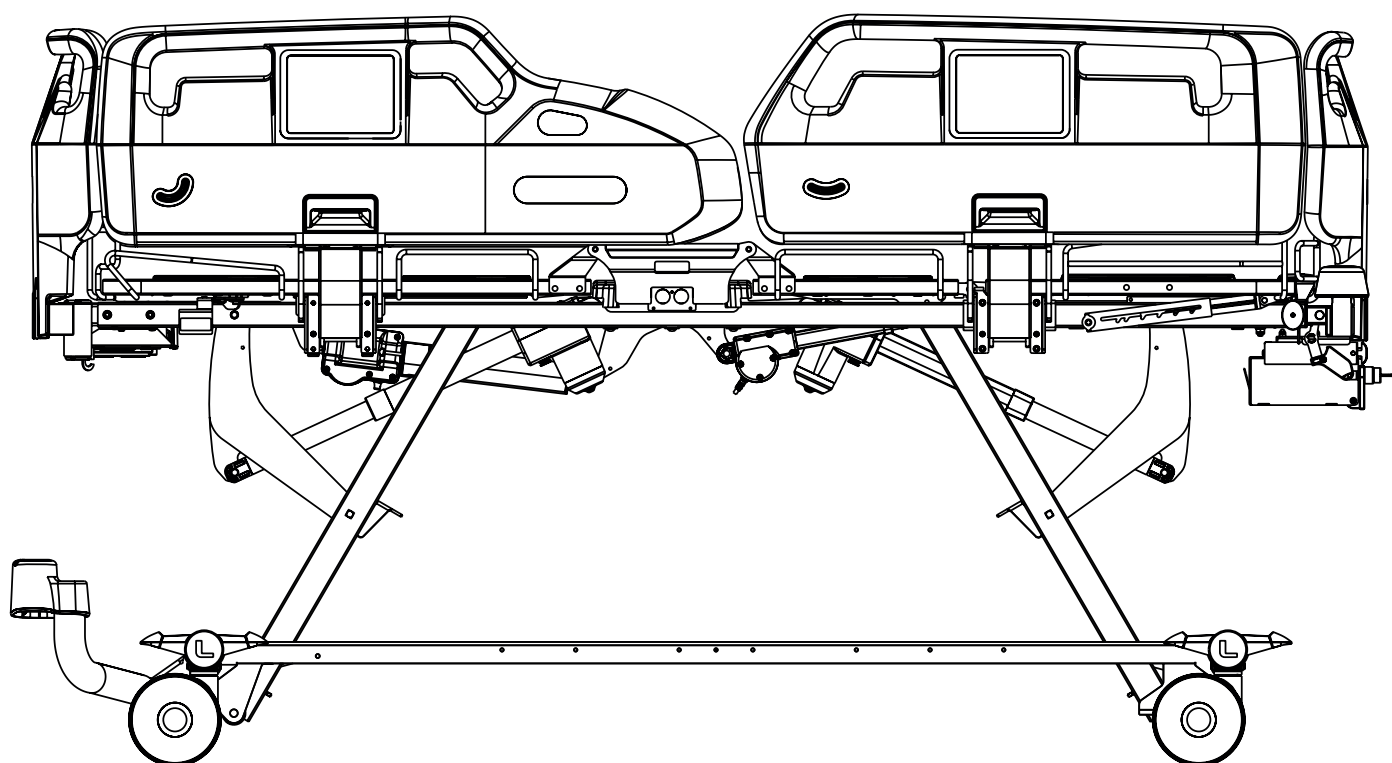

D9U00AEZJ-0101

Version: 00

Publication Date: 2026-06

## Manufacturer:

L I N E T spol. s r.o.  
Želevčice 5  
274 01 Slaný

Tel.: +420 312 576 111  
Fax: +420 312 522 668

E-mail: [info@linet.cz](mailto:info@linet.cz)  
<http://www.linet.com>  
Service department: [service@linetgroup.com](mailto:service@linetgroup.com)

## Essenza 300 Junior<sup>+</sup> Accessories

Author: L I N E T, s.r.o.  
Related links: [www.linet.com](http://www.linet.com)

### D9U00AEZJ-0101

Version: 00  
Publication Date: 2026-06

Copyright © L I N E T, s.r.o., 2026  
Translation © L I N E T, s.r.o., 2026  
All rights reserved.

All trademarks and brands are the property of the appropriate owners. The manufacturer reserves the right to changes in the contents of the instructions for use that relate to the product's technical regulations. It is for this reason that the contents of the instructions for use may indicate differences from the current manufacture of the product. Reproduction, also excerpts, only with prior permission of the publisher. Subject to changes due to technical developments. All technical data are rated data and are subject to construction and manufacturing tolerances.

# Table of Contents

|                                                                                        |           |
|----------------------------------------------------------------------------------------|-----------|
| <b>1 Symbols and Definitions .....</b>                                                 | <b>5</b>  |
| 1.1 Warning Notices .....                                                              | 5         |
| 1.1.1 Types of Warning Notices .....                                                   | 5         |
| 1.1.2 Structure of Warning Notices .....                                               | 5         |
| 1.2 Instructions .....                                                                 | 5         |
| 1.3 Lists .....                                                                        | 5         |
| 1.4 Symbols on the Package .....                                                       | 6         |
| 1.5 Symbols on the accessories .....                                                   | 7         |
| 1.6 Abbreviations .....                                                                | 12        |
| <b>2 Safety Instructions for Essenza 300 Junior<sup>®</sup> with accessories .....</b> | <b>13</b> |
| <b>3 Environment Conditions .....</b>                                                  | <b>18</b> |
| <b>4 Accessories .....</b>                                                             | <b>19</b> |
| 4.1 Lifting Pole .....                                                                 | 25        |
| 4.2 Extended Lifting Pole .....                                                        | 27        |
| 4.3 Triangular Holder for Lifting Pole .....                                           | 29        |
| 4.4 Pediatric Triangular Holder for Lifting Pole .....                                 | 30        |
| 4.5 Infusion Holder for Lifting Pole .....                                             | 31        |
| 4.6 Infusion Pump Holder (Accessory Pole) .....                                        | 32        |
| 4.7 Infusion Bottle Basket .....                                                       | 32        |
| 4.8 Telescopic Infusion Stand .....                                                    | 33        |
| 4.9 Mobile Infusion Stand .....                                                        | 35        |
| 4.10 Monitor Shelf .....                                                               | 36        |
| 4.11 Utility Shelf .....                                                               | 38        |
| 4.12 Oxygen Bottle Holder .....                                                        | 40        |
| 4.12.1 Horizontal Oxygen Bottle Holder for Plastic Head Board .....                    | 40        |
| 4.13 Urinary Bag Holder .....                                                          | 42        |
| 4.14 Urinary Bottle Holder .....                                                       | 43        |
| 4.14.1 Urinary Bottle Holder for Plastic Siderails .....                               | 43        |
| 4.15 Redon Bottle Holder .....                                                         | 44        |
| 4.16 Crutch Holders .....                                                              | 45        |
| 4.16.1 Crutch Holder for Plastic Foot Board .....                                      | 45        |
| 4.17 Towel Holder .....                                                                | 49        |
| 4.17.1 Towel Holder for Plastic Foot Board .....                                       | 49        |
| 4.18 Name Holder .....                                                                 | 50        |
| 4.18.1 Name Holder for Plastic Foot Board .....                                        | 50        |
| 4.19 Chart Holder .....                                                                | 51        |
| 4.19.1 Chart Holder for Plastic Foot Board .....                                       | 51        |
| 4.20 Holder of Small Accessories with Two Hooks .....                                  | 52        |
| 4.21 Positioning Cushion .....                                                         | 53        |
| 4.22 SafeSense 3 .....                                                                 | 53        |
| 4.23 Traction Frame Z .....                                                            | 54        |
| 4.24 Reading Lamp .....                                                                | 56        |
| 4.25 Traction Frame D Holder for Foot Board with One Safety Lock .....                 | 58        |
| 4.26 Traction Frame T73 .....                                                          | 60        |
| 4.27 Partial Traction Frame MP for Foot Board with One Safety Lock .....               | 62        |
| 4.28 Cross Infusion Pump Holder .....                                                  | 64        |
| <b>5 Disposal .....</b>                                                                | <b>65</b> |
| 5.1 Environment Protection .....                                                       | 65        |
| 5.2 Disposal .....                                                                     | 65        |
| 5.2.1 Within Europe .....                                                              | 65        |
| 5.2.2 Outside Europe .....                                                             | 65        |

## List of Figures

|                                                                                                                                                                                      |    |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----|
| Fig. Wash Label of the Plastic Siderail Cover .....                                                                                                                                  | 11 |
| Fig. Lifting Pole in the bushing at head end of the mattress support platform and Extended Lifting Pole in the Accessory Adaptor in undercarriage at bed head end .....              | 26 |
| Fig. Bushings for the Standard Lifting Pole (1) at head end of the mattress support platform .....                                                                                   | 26 |
| Fig. Extended Lifting Pole .....                                                                                                                                                     | 27 |
| Fig. Standard Lifting Pole .....                                                                                                                                                     | 27 |
| Fig. Bushings for the Extended Lifting Pole (1) at head end of the mattress support platform and Accessory Adaptors for the Extended Lifting Pole (2) in the bed undercarriage ..... | 28 |
| Fig. Triangular Holder 4ROTGERSG700-2 .....                                                                                                                                          | 29 |
| Fig. Extendable Triangular Holder 4ROTGNR100GR-2 .....                                                                                                                               | 29 |
| Fig. Pediatric Triangular Holder for Lifting Pole 4ROTKG100000 .....                                                                                                                 | 30 |
| Fig. Infusion Holder 4MAPL00N1001 .....                                                                                                                                              | 31 |
| Fig. Infusion Bottle Basket .....                                                                                                                                                    | 32 |
| Fig. Infusion Pump Holder at bed head end .....                                                                                                                                      | 32 |
| Fig. S-shaped Infusion Stand (4MA45MNA0003) .....                                                                                                                                    | 33 |
| Fig. Infusion Stand with receptacle for cannulas (4PKV26107200) .....                                                                                                                | 33 |
| Fig. Infusion Stand in the bushing at head end of the mattress support platform .....                                                                                                | 34 |
| Fig. Mobile Infusion Stand with receptacle for cannulas (4ZZ426100) .....                                                                                                            | 35 |
| Fig. Monitor Shelf placed in bed foot end corner bumpers .....                                                                                                                       | 36 |
| Fig. Manipulation with Monitor Shelf Control Lever (1) .....                                                                                                                         | 37 |
| Fig. Description of the Monitor Shelf .....                                                                                                                                          | 37 |
| Fig. Utility Shelf on the Foot Board .....                                                                                                                                           | 38 |
| Fig. Placement of plastic container to wire construction of the Utility Shelf .....                                                                                                  | 39 |
| Fig. Plastic container in the non-working position (1 - release lever of the plastic container) .....                                                                                | 39 |
| Fig. Horizontal Oxygen Bottle Holder on the plastic foot board .....                                                                                                                 | 41 |
| Fig. Urinary Bag Holder hanging from the hook on the Accessory Rail .....                                                                                                            | 42 |
| Fig. Urinary Bottle Holder hanging from the right window in the plastic foot siderail .....                                                                                          | 43 |
| Fig. Redon Bottle Holder on the Accessory Rail under the backrest of the bed .....                                                                                                   | 44 |
| Fig. Crutch Holder 1102070000000 .....                                                                                                                                               | 45 |
| Fig. Crutch placement to the Crutch Holder 1102070000000 .....                                                                                                                       | 46 |
| Fig. Crutch Holder 1102070000000 on the Plastic Foot Board .....                                                                                                                     | 46 |
| Fig. Crutch Holder 1102080000000 .....                                                                                                                                               | 47 |
| Fig. Crutch placement to the Crutch Holder 1102080000000 .....                                                                                                                       | 47 |
| Fig. Crutch Holder 1102080000000 on the Plastic Foot Board .....                                                                                                                     | 48 |
| Fig. Towel Holder on the Plastic Foot Board .....                                                                                                                                    | 49 |
| Fig. Name Holder on the Plastic Foot Board .....                                                                                                                                     | 50 |
| Fig. Chart Holder on the Plastic Foot Board .....                                                                                                                                    | 51 |
| Fig. Holder of Small Accessories with Two Hooks .....                                                                                                                                | 52 |
| Fig. Holder of Small Accessories with Two Hooks on the Plastic Foot Board with One Safety Lock .....                                                                                 | 52 |
| Fig. Positioning Cushion .....                                                                                                                                                       | 53 |
| Fig. Positions for Traction Frame Z (1) in the case of the bed with corner bumpers with accessory bushings .....                                                                     | 54 |
| Fig. Traction frame Z without two Adaptors for Traction Frame Z .....                                                                                                                | 55 |
| Fig. Reading Lamp .....                                                                                                                                                              | 56 |
| Fig. Accessory Adaptors for Reading Lamp (1) in the bed undercarriage and bushings for Reading Lamp (2) at head end of the mattress support platform .....                           | 57 |
| Fig. Positions of the Traction Frame D Holder for Foot Board with One Safety Lock (1) on the bed with corner bumpers with accessory bushings .....                                   | 58 |
| Fig. Placement of the Traction Frame D onto the Traction Frame D Holder for Foot Board with One Safety Lock .....                                                                    | 59 |
| Fig. Positions for Traction Frame T73 (1) on the bed with corner bumpers with accessory bushings .....                                                                               | 60 |
| Fig. Traction frame T73 .....                                                                                                                                                        | 61 |
| Fig. Positions for Partial Traction Frame MP for Foot Board with One Safety Lock (1) on the bed with corner bumpers with accessory bushings .....                                    | 62 |
| Fig. Partial Traction Frame MP for Foot Board with One Safety Lock .....                                                                                                             | 63 |
| Fig. Bushings for the Cross Infusion Pump Holder (1) .....                                                                                                                           | 64 |
| Fig. Cross Infusion Pump Holder .....                                                                                                                                                | 64 |

# 1 Symbols and Definitions

## 1.1 Warning Notices

### 1.1.1 Types of Warning Notices

Warning notices are differentiated by the type of danger using the following signal words:

- ▶ **CAUTION** warns about the risk of material damage.
- ▶ **WARNING** warns about the risk of physical injury.
- ▶ **DANGER** warns about the risk of fatal injury.

### 1.1.2 Structure of Warning Notices



**DANGER!**  
**Dangerous electricity!**  
 ▶ Measures to avoid the risk.



**WARNING!**  
**Type and source of risk!**  
 ▶ Measures to avoid the risk, if necessary.



**CAUTION!**  
**Type and source of risk!**  
 ▶ Measures to avoid the risk, if necessary.

## 1.2 Instructions

**Structure of instructions:**



- ▶ Perform this step.
- Results, if necessary.

## 1.3 Lists


**Structure of bulleted lists:**




- List level 1
  - List level 2
    - List level 3


1.4 Symbols on the Package
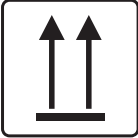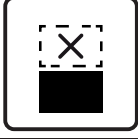
|                                                                                                                                                                     |                                                                       |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------|
|   | FRAGILE, HANDLE WITH CARE                                             |
|                                                                                    | THIS WAY UP                                                           |
|   | KEEP DRY (PROTECT FROM HUMIDITY)                                      |
|                                                                                    | PAPER RECYCLING SYMBOL                                                |
|                                                                                   | DO NOT USE HAND TRUCK HERE                                            |
|                                                                                  | DO NOT STACK DURING STORAGE                                           |
|                                                                                  | OVERSEAS PACKAGE: STACKING LIMIT BY NUMBER (4 PACKAGES FOR STORAGE)   |
|                                                                                  | OVERSEAS PACKAGE: STACKING LIMIT BY NUMBER (2 PACKAGES FOR TRANSPORT) |
|                                                                                  | READ INSTRUCTIONS FOR USE                                             |
|                                                                                  | DO NOT POLLUTE THE ENVIRONMENT                                        |

1.5 Symbols on the accessories
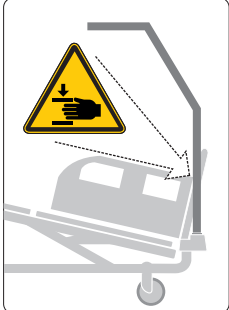
|                                                                                     |                                                                                                                                                                                  |
|-------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|    | SAFE WORKING LOAD                                                                                                                                                                |
|    | WARNING AGAINST CRUSHING OR TRAPPING                                                                                                                                             |
|    | MONITOR SHELF: MAXIMUM LOAD OF THE MONITOR SHELF 15 KG,<br>PLACE MONITOR ON THIS SIDE OF THE MONITOR SHELF                                                                       |
|    | MONITOR SHELF: MAXIMUM LOAD OF THE MONITOR SHELF 15 KG,<br>DO NOT PLACE MONITOR ON THIS SIDE OF THE MONITOR SHELF                                                                |
|  | CE MARKING (PRODUCT NORMATIVELY HARMONIZED FOR EUROPEAN ECONOMIC AREA)                                                                                                           |
|  | MANUFACTURER                                                                                                                                                                     |
|  | MANUFACTURING DATE                                                                                                                                                               |
|  | REFERENCE NUMBER<br>(PRODUCT TYPE DEPENDING ON CONFIGURATION)                                                                                                                    |
|  | LIFTING POLE: RISK OF GETTING YOUR HAND TRAPPED BETWEEN LIFTING POLE<br>AND BED DURING TRENDLENBURG TILT POSITIONING WITH LIFTING POLE IN<br>THE UNDERCARRIAGE ACCESSORY ADAPTOR |
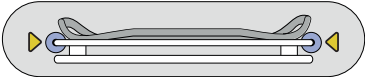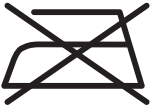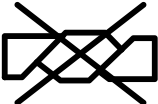
|                                                                                     |                                                                 |
|-------------------------------------------------------------------------------------|-----------------------------------------------------------------|
|    | SERIAL NUMBER                                                   |
|    | MEDICAL DEVICE (COMPATIBLE WITH MEDICAL DEVICE REGULATION)      |
|    | BATCH NUMBER                                                    |
|    | MONITOR SHELF: INSTRUCTIONS FOR CORRECT MONITOR SHELF PLACEMENT |
|   | READ INSTRUCTIONS FOR USE                                       |
|  | DO NOT IRON                                                     |
|  | DO NOT USE A CLEANER CONTAINING PHENOL                          |
|  | DO NOT WRING                                                    |
|  | REGULARLY INSPECT THE INSIDE OF THE COVER FOR CONTAMINATION     |
|  | MACHINE WASH AT 71°C WITH DETERGENT                             |
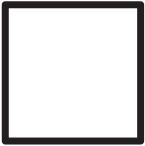
|                                                                                                                                                                                                                                         |                                                                                                       |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------|
|                                                                                                                                                        | TUMBLE DRY ON LOW HEAT SETTING (MAX. 60°C)                                                            |
|                                                                                                                                                        | PROFESSIONAL CHEMICAL CLEANING (MODERATE PROCEDURE)                                                   |
|                                                                                                                                                        | RINSE WITH WARM WATER AND NEUTRAL DETERGENT (INITIAL TEMPERATURE OF HOT WATER SHOULD NOT EXCEED 50°C) |
|  <b>NaClO</b><br>1,000 ppm<br>max                                                                                                                      | DISINFECT USING SOLUTION NO GREATER THAN 1,000 PPM OF CHLORINE                                        |
|                                                                                                                                                       | RINSE WITH WARM WATER                                                                                 |
|                                                                                                                                                      | NATURAL DRYING                                                                                        |
|                                                                                                                                                      | DO NOT BLEACH                                                                                         |
| <b>BS 7175</b><br><br> <b>5</b><br><b>MEDIUM</b><br><b>HAZARD</b> | MATTRESS CONFORMS TO BS7177: 2008 FOR MEDIUM HAZARD                                                   |
|                                                                                                                                                      | POSITIONING CUSHION: MAXIMUM WASHING TEMPERATURE 40°C                                                 |

|                                                                                     |                                                                                                                 |
|-------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------|
|    | CONSULT INSTRUCTIONS FOR USE OR CONSULT ELECTRONIC INSTRUCTIONS FOR USE                                         |
|    | DISTRIBUTOR                                                                                                     |
|    | ONLY SUITABLE FOR INDOOR USE                                                                                    |
|    | CAUTION SIGN                                                                                                    |
|   | GENERAL WARNING SIGN                                                                                            |
|  | ACCESSORY MASS                                                                                                  |
|  | APPLIED PARTS TYPE <b>B</b>                                                                                     |
|  | EAC MARKING (EURASIAN CONFORMITY)                                                                               |
|  | UK CONFORMITY ASSESSED ( <b>UKCA</b> ) MARKING (PRODUCT NORMATIVELY HARMONIZED FOR GREAT BRITAIN ECONOMIC AREA) |
|  | AUTHORIZED REPRESENTATIVE IN GREAT BRITAIN                                                                      |

|                                                 |              |     |                                          |
|-------------------------------------------------|--------------|-----|------------------------------------------|
| <table><tr><td>CH</td><td>REP</td></tr></table> | CH           | REP | AUTHORIZED REPRESENTATIVE IN SWITZERLAND |
| CH                                              | REP          |     |                                          |
|                                                 | EARTH GROUND |     |                                          |

BS 7175

5

MEDIUM HAZARD

1

2

NaClO

1,000 ppm

max

3

4

D7001340D

Fig. Wash Label of the Plastic Siderail Cover

## 1.6 Abbreviations

|                 |                                                            |
|-----------------|------------------------------------------------------------|
| <b>AC ( ~ )</b> | Alternating Current                                        |
| <b>ALT</b>      | Automatic Lateral Therapy                                  |
| <b>BMI</b>      | Body Mass Index                                            |
| <b>CE</b>       | European Conformity                                        |
| <b>CPR</b>      | Cardiopulmonary Resuscitation                              |
| <b>dB</b>       | Sound Intensity Unit                                       |
| <b>DC ( — )</b> | Direct Current                                             |
| <b>CUC</b>      | Configuration number                                       |
| <b>EMC</b>      | Electromagnetic Compatibility                              |
| <b>FET</b>      | Field-effect transistor                                    |
| <b>HF</b>       | High Frequency                                             |
| <b>HPL</b>      | High Pressure Laminate                                     |
| <b>ICU</b>      | Intensive Care Unit                                        |
| <b>INT.</b>     | Duty Cycle                                                 |
| <b>IP</b>       | Ingress Protection                                         |
| <b>IV</b>       | Intravenous                                                |
| <b>LCD</b>      | Liquid Crystal Display                                     |
| <b>LED</b>      | Light Emitting Diodes                                      |
| <b>ME</b>       | Medical Electrical (Equipment)                             |
| <b>ON</b>       | Activation                                                 |
| <b>OFF</b>      | Deactivation                                               |
| <b>ppm</b>      | Parts per million, millionth (1000 ppm = 0,1%)             |
| <b>REF</b>      | Reference Number (product type depending on configuration) |
| <b>SCU</b>      | System Control Unit (integrated mattress)                  |
| <b>SN</b>       | Serial Number                                              |
| <b>SWL</b>      | Safe Working Load                                          |
| <b>UDI</b>      | Unique Device Identification (for medical devices)         |
| <b>USB</b>      | Universal Serial Bus                                       |
| <b>WEEE</b>     | Waste Electrical and Electronic Equipment                  |

## 2 Safety Instructions for Essenza 300 Junior<sup>+</sup> with accessories


**WARNING!**

Risk of injury due to incorrect use!

- ▶ Essenza 300 Junior<sup>+</sup> bed should be left in its lowest position when the patient is unattended in order to reduce risk of injury due to falls!


**WARNING!**

Risk of injury due to incorrect use!

- ▶ Siderails of Essenza 300 Junior<sup>+</sup> should be located in the „up“ position to reduce the risk of the patient accidentally slipping or rolling off the mattress!


**WARNING!**

Ensure any button of the control elements is not involuntarily pressed by persons, by the bed accessories, by the mattress or by the other objects!

- ▶ To avoid unintended activation of moving parts during any use of the bed, always check that none of the buttons on the control elements of the bed is pressed!
- ▶ Always ensure any control panel keyboard is not covered by anything, loaded by anything or pressed by anything!


**WARNING!**

Risk of injury due to incorrect use!

- ▶ Staff expert assessment is needed to consider all individual cases of contraindications!


**WARNING!**

Risk of trapping or squeezing because of patient's body constitution disproportionate to the size of mattress support platform!


**WARNING!**

Risk of injury due to incorrect use!

- ▶ Certain bed positions are not suitable for specific diagnosis/medical conditions. Fowler position is not suitable for spinal cord injuries! Trendelenburg position is not suitable for patients with higher intracranial pressure!


**WARNING!**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established!


**WARNING!**

This medical device is not intended for oxygen enriched environment!



**WARNING!**

This medical device is not intended for use with flammable substances!



**WARNING!**

This medical device is not portable medical electrical equipment!



**WARNING!**

Adult patient is allowed to use selected control elements only if hospital personnel had assessed that the patient's physical and psychological state is in accordance with use of them and only if the hospital personnel had trained the patient in accordance with the instructions for use!



**WARNING!**

An additional multiple socket-outlet or extension cord shall not be connected to the medical electrical system of Essenza 300 Junior<sup>+</sup>.



**WARNING!**

Inappropriate handling of the power supply cord of Essenza 300 Junior<sup>+</sup>, e. g. by kinking, shearing or other mechanical damages is hazardous!



**WARNING!**

During specific investigations or treatments, the significant risks of reciprocal interference posed by Essenza 300 Junior<sup>+</sup> (ME equipment) may occur.



**WARNING!**

To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.



**WARNING!**

Incompatible siderails and mattresses can cause an entrapment hazard!



**WARNING!**

Incompatible mattresses can create hazards!



**WARNING!**

The bed with mattress is intended for one adult or for one child or for one adult with atypical anatomy.

- ▶ Follow the Intended use chapter.
- ▶ Essenza 300 Junior<sup>+</sup> bed is not intended for children having a physical size less than 125 cm.



**WARNING!**

When routing cables from other equipment in the Essenza 300 Junior<sup>+</sup> bed avoid squeezing those between parts of the Essenza 300 Junior<sup>+</sup> bed!



**WARNING!**

No modification of this equipment is allowed.  
Do not modify this equipment without authorization of the manufacturer.



**WARNING!**

If this Essenza 300 Junior<sup>+</sup> medical bed (ME equipment) is modified, appropriate inspection and testing must be conducted to ensure continued safe use of the equipment.



**WARNING!**

Only authorised and trained person using an appropriate tool is allowed to change fuses, accumulators, control units and power supplies!



**WARNING!**

Make sure the duty cycle (2 min ON/18 min OFF) is not exceeded during bed positioning!



**WARNING!**

No part of the Essenza 300 Junior<sup>+</sup> ME equipment shall be serviced or maintained while in use with a patient!



**WARNING!**

Adult patient is allowed to use selected control elements of the ME equipment unless servicing and maintenance of the ME equipment is performed!



**WARNING!**

Avoid collisions of the accessories reaching above the bed Head Board or the bed Foot Board with things above the bed (e.g. doorframe)!



**WARNING!**

Do not exceed the stipulated safe working load!



**WARNING!**

Do not leave child or adult with atypical anatomy unattended when head board or foot board is removed!



**WARNING!**

Child or adult with atypical anatomy is not allowed to use control elements of the bed!



**WARNING!**

Ensure that the power supply cord is not accessible to child or adult with atypical anatomy!



**WARNING!**

Equipments in close proximity or attached to the Essenza 300 Junior<sup>+</sup> medical bed can cause a hazard, e.g. entrapment or tipping over.



**WARNING!**

The Essenza 300 Junior<sup>+</sup> medical bed shall not be used for transportation of a patient in any kind of vehicle.



**WARNING!**

Risk of injury due to incorrect use!

- Remove the Handset Control Panel from the bed when a child or adult with atypical anatomy occupies the bed.



**WARNING!**

Risk of injury due to incorrect use!

- Compatible mattresses intended for adults are not intended for children or adults with atypical anatomy!

## Additional Instructions for correct use:

- ▶ Follow the instructions for use carefully.
- ▶ Use the bed exclusively if it is in perfect working order.
- ▶ If necessary, check the bed functions daily or at each shift change.
- ▶ Ensure any user has read and understood this manual completely before operating the product.
- ▶ Use the bed exclusively with the correct mains supply.
- ▶ Ensure that the bed is operated exclusively by qualified personnel who have been trained according to the instructions for use.
- ▶ Ensure the bed is operated by qualified personnel trained in using the bed in accordance with the instructions for use by manufacturer or by person approved by the manufacturer, eventually by authorized representative or by person approved by the authorized representative.
- ▶ Ensure that the patient (health permitting) has been informed about the operation of the bed and all applicable safety instructions.
- ▶ Contact service department of the manufacturer immediately to let it replace any damaged parts by the original spare parts.
- ▶ Move the bed exclusively on even, hard-surfaced floors.
- ▶ Replace any damaged parts immediately with original spare parts.
- ▶ Ensure that maintenance and installation are performed exclusively by qualified personnel who have been trained by the manufacturer.
- ▶ Before peak loads or unavoidable excess loads (CPR), place Mattress Support Platform to the lowest position.
- ▶ Ensure that only one patient lies on the bed at any time.
- ▶ Take care to avoid injuries or squeezing when operating moving parts.
- ▶ When using lifting pole or infusion stand, ensure that nothing will be damaged when you move or adjust the bed.
- ▶ Brake the castors when the bed is occupied.
- ▶ Keep the Mattress Support Platform in the lowest position when the patient is unattended by healthcare personnel in order to minimize risk of patient falls.
- ▶ Ensure that siderails are operated exclusively by healthcare personnel.
- ▶ Never use the bed in areas where there is a hazard of explosion.
- ▶ Enable or disable functions on control elements as appropriate for the patient's physical and mental state. Afterwards ensure the lock function was used as required.
- ▶ Ensure that parts of the bed intended for movement are not blocked.
- ▶ Never handle the mains plug with wet hands.
- ▶ Disconnect the product from the mains exclusively by pulling the mains plug.
- ▶ When pulling the mains plug, always hold the plug, not the cable.
- ▶ Position the mains cable so that there are no loops or kinks in the cable. Protect the cable from mechanical wear and tear.
- ▶ Improper handling of mains cable can cause an electric shock hazard, other serious injuries or damage to the integrated mattress system.
- ▶ Ensure that the stipulated duty cycle of motor is not exceeded.
- ▶ To change fuses or cables contact service organisation authorized by manufacturer.
- ▶ To prevent failures, use exclusively the manufacturer's original accessories and mattresses.
- ▶ Ensure that the stipulated safe working load is not exceeded.
- ▶ If the patient's condition could lead to an entrapment, leave the mattress support platform in the flat position whilst unattended.
- ▶ Adjust bed height when transporting the bed in order to facilitate overcoming possible obstacles.
- ▶ Do not exceed maximum load of 80 kg for mattress support platform extension.
- ▶ Do not hang anything on any cable.
- ▶ Do not modify bed and its components without the manufacturer's approval.
- ▶ Do not exceed the maximum patient weight limit (see Mechanical Specifications).
- ▶ Do not use the bed in the case its parts have been removed unless these parts are designed to be removed.
- ▶ To avoid injury or crushing, take extra caution when operating any moving parts of the bed.
- ▶ Never use the mattress replacement system compatible with the Essenza 300 Junior+ bed near radiators or other heat sources.
- ▶ Select a suitable location for the placement of bed accessories and other objects to prevent involuntary activation of buttons or controls which may result in the adjustment of bed positioning.
- ▶ Never place any accessories or Handset Control Panel on the siderails where keyboards are located.
- ▶ After each emergency situation always check if any of the control elements is not involuntarily pressed by the bed accessories or by the mattress.
- ▶ To avoid unintended activation of moving parts during any use of the bed always check that none of the control elements of the bed is pressed by persons, mattress or other objects.

### 3 Environment Conditions



**WARNING!**  
Risk of damaging the product due to incorrect environment conditions!  
► Do not use the Essenza 300 Junior\* bed with the compatible accessories under the environmental conditions outside of those specified in the Environment Conditions chapter!



**CAUTION!**  
Risk of damaging the product if its packaging is exposed to environmental conditions outside of those specified in the Environment Conditions chapter!

| Parameter                        | Value          |
|----------------------------------|----------------|
| Use Conditions                   |                |
| Ambient Temperature              | 10°C — 40°C    |
| Relative Humidity                | 30% — 75 %     |
| Atmospheric Pressure             | 795 — 1060 hPa |
| Storage and Transport Conditions |                |
| Ambient Temperature              | -20°C — 50°C   |
| Relative Humidity                | 20% — 90 %     |
| Atmospheric Pressure             | 795 — 1060 hPa |

## 4 Accessories

**WARNING!**

Risk of injury due to incompatible accessories!

- ▶ Use exclusively original accessories from the manufacturer.
- The manufacturer is not responsible for the use of unapproved accessories.

**WARNING!**

Risk of injury due to damaged accessories!

- ▶ Use exclusively accessories in perfect condition.

**WARNING!**

Risk of injury or material damage due to incorrect use!

- ▶ Compatible accessories manufactured by different manufacturers have their own instructions for use. It is necessary to read instructions for use of a compatible accessory with instructions for use of the compatible LINET product to respect especially technical parameters, warning notifications, cleaning and maintenance instructions of LINET products and their compatible accessories!

**WARNING!**

Risk of injury or material damage when adjusting the bed!

- ▶ Ensure that there are no installed accessories that may collide with moving parts of the bed.
- ▶ Extra care must be taken to avoid collisions of the accessories with bed parts and with the other accessories during bed positioning.
- ▶ Extra care must be taken to avoid collisions of the accessories installed on the bed and things in the vicinity of the bed.

**WARNING!**

Adult patient is allowed to use selected accessories only if hospital personnel had assessed that the patient's physical and psychological state is in accordance with use of them and only if the hospital personnel had trained the patient in accordance with the instructions for use!

## Compatible Accessories for Essenza 300 Junior<sup>+</sup> bed

| Compatible Accessories for Essenza 300 Junior <sup>+</sup> | Manufacturers and Identification Numbers                                                               | Compatibility                                   |
|------------------------------------------------------------|--------------------------------------------------------------------------------------------------------|-------------------------------------------------|
| Lifting Pole                                               | LINET: 11011410A0000, 11011410B0000, 1101141000000                                                     | bed without Accessory Adaptor                   |
| Extended Lifting Pole                                      | LINET: 11011460A0000, 11011460B0000, 1101146000000                                                     | bed with Accessory Adaptor                      |
| Triangular Holder for Lifting Pole                         | Rolko Kohlgruber: 4ROTGERSG700-2, 4ROTGNR-100GR-2, Schnieder: 4SN4290111, 4SN4360311                   | all bed variants                                |
| Pediatric Triangular Holder for Lifting Pole               | Rolko Kohlgruber: 4ROTGK100000                                                                         | all bed variants                                |
| Infusion Holder for Lifting Pole                           | MARO - Mader: 4MAPL00N1001, LINET: 16010700A0000, 16010700B0000, wissner-bosserhoff: WDE5006220000     | all bed variants                                |
| Infusion Pump Holder (Accessory Pole)                      | LINET: 1102520000000                                                                                   | all bed variants                                |
| Infusion Bottle Basket                                     | DRÁTĚNÝ PROGRAM: 4DR426101                                                                             | all bed variants                                |
| Telescopic Infusion Stand (D34)                            | Provita Medical: 4PV340290000, 4PV17744500L, 4PV17587200L, 4PV322717000, 4PV297475000, 4PV33449300L    | all bed variants                                |
| Telescopic Infusion Stand (D19)                            | Provita Medical: 4PKV26107200                                                                          | all bed variants                                |
| S-shaped Telescopic Infusion Stand                         | Provita Medical: 4PV17604800L, 4PV50-0742-0000, 4PKV50-0693, 4PV33466000L, MARO - Mader: 4MA45M-NA0003 | all bed variants                                |
| Mobile Infusion Stand                                      | Provita Medical: 4ZZ426100                                                                             | all bed variants                                |
| Monitor Shelf                                              | LINET: 11026500B0000                                                                                   | bed with corner bumpers with accessory bushings |
| Utility Shelf for Plastic Head Board and Foot Board        | LINET: 11026000B0000                                                                                   | bed with Plastic Head Board and Foot Board      |
| Horizontal Oxygen Bottle Holder for Plastic Head Board     | MARO - Mader: 4MAS0001653                                                                              | bed with Plastic Head Board and Foot Board      |
| Urinary Bag Holder                                         | LINET: 11023700B0000                                                                                   | all bed variants                                |
| Urinary Bottle Holder for Plastic Siderails                | LINET: 11027900B0000                                                                                   | bed with plastic siderails                      |
| Redon Bottle Holder                                        | LINET: 11008200B0000, 11008200A0000                                                                    | all bed variants                                |
| Crutch Holder for Plastic Foot Board                       | LINET: 1102070000000, 1102080000000                                                                    | bed with Plastic Head Board and Foot Board      |
| Towel Holder for Plastic Foot Board                        | LINET: 11025002B0000                                                                                   | bed with Plastic Head Board and Foot Board      |
| Name Holder for Plastic Foot Board                         | Grafiteam: 4GT006000000                                                                                | bed with Plastic Head Board and Foot Board      |
| Chart Holder for Plastic Foot Board                        | LINET: 11023900A0000                                                                                   | bed with Plastic Head Board and Foot Board      |

| Compatible Accessories for Essenza 300 Junior <sup>+</sup>    | Manufacturers and Identification Numbers | Compatibility                                   |
|---------------------------------------------------------------|------------------------------------------|-------------------------------------------------|
| Holder of Small Accessories with Two Hooks                    | LINET: 1102490000000                     | bed with corner bumpers with accessory bushings |
| Positioning Cushion                                           | PERCEPT-FUN: 4SET17500001                | bed with Lateral Tilt                           |
| SafeSense 3                                                   | LINET: 1SFS300001, 1SFS300032            | SafeSense 3 ready bed                           |
| Traction Frame Z                                              | MARO - Mader: 4MAEXZ000000               | bed with corner bumpers with accessory bushings |
| Reading Lamp                                                  | Provita Medical: 4PV383435X00            | all bed variants                                |
| Traction Frame D Holder for Foot Board with One Safety Lock   | LINET: 1103470000000                     | bed with corner bumpers with accessory bushings |
| Set of weight bags for children (5,75 kg)                     | ROUSEK: 4RKZVSD001                       | MARO - Mader traction frame                     |
| Set of weight bags for adults (6,5 kg)                        | ROUSEK: 4RKZVSD006                       | MARO - Mader traction frame                     |
| Set of weights (5 kg)                                         | MARO - Mader: 4MAEX2500580               | MARO - Mader traction frame                     |
| Traction Frame T73                                            | Kinetec SAS: 4KI4655006152               | bed with corner bumpers with accessory bushings |
| Partial Traction Frame MP for Foot Board with One Safety Lock | Kinetec SAS: 4KI4655006178               | bed with corner bumpers with accessory bushings |
| Arm Suspension Set                                            | Kinetec SAS: 4KI4655003124               | Kinetec traction frame                          |
| Cervical Traction Set                                         | Kinetec SAS: 4KI4655002548               | Kinetec traction frame                          |
| Lumbar Suspension Set                                         | Kinetec SAS: 4KI4655002530               | Kinetec traction frame                          |
| Boppe Traction Set                                            | Kinetec SAS: 4KI4655002556               | Kinetec traction frame                          |
| Cross Infusion Pump Holder                                    | LINET: 1102480000000                     | all bed variants                                |

## Specifications of the Compatible Accessories for Essenza 300 Junior<sup>+</sup> bed



### CAUTION!

Material damage due to incorrect cleaning or disinfection!

► Check if used cleaning agents and disinfectants are compatible with materials that the accessory consists of! For information see the following tables.

| Compatible Accessories for Essenza 300 Junior*            | Materials (surfaces of the accessories mentioned for the sake of cleaning and disinfection) | Accessory Mass | SWL                   |
|-----------------------------------------------------------|---------------------------------------------------------------------------------------------|----------------|-----------------------|
| Lifting Pole 11011410A0000                                | Lacquered steel                                                                             | 5,76 kg        | 75 kg                 |
| Lifting Pole 11011410B0000                                | Lacquered steel                                                                             | 5,76 kg        | 75 kg                 |
| Lifting Pole 1101141000000                                | Chrome steel                                                                                | 5,76 kg        | 75 kg                 |
| Extended Lifting Pole 11011460A0000                       | Lacquered steel                                                                             | 6,1 kg         | 75 kg                 |
| Extended Lifting Pole 11011460B0000                       | Lacquered steel                                                                             | 6,1 kg         | 75 kg                 |
| Extended Lifting Pole 1101146000000                       | Chrome steel                                                                                | 6,1 kg         | 75 kg                 |
| Triangular Holder for Lifting Pole 4ROTGERSG700-2         |                                                                                             | 0,19 kg        | 75 kg                 |
| Triangular Holder for Lifting Pole 4ROTGNR100GR-2         |                                                                                             | 0,37 kg        | 75 kg                 |
| Triangular Holder for Lifting Pole 4SN4290111             | Acrylonitrile butadiene styrene (ABS)                                                       | 0,21 kg        | 75 kg                 |
| Triangular Holder for Lifting Pole 4SN4360311             | Acrylonitrile butadiene styrene (ABS)                                                       | 0,3 kg         | 75 kg                 |
| Pediatric Triangular Holder for Lifting Pole 4ROTGK100000 |                                                                                             |                | 45 kg                 |
| Infusion Holder for Lifting Pole 4MAPL00N1001             |                                                                                             | 0,15 kg        | 9 kg                  |
| Infusion Holder for Lifting Pole 16010700A0000            | Lacquered steel                                                                             | 0,15 kg        | 9 kg (3 kg per hook)  |
| Infusion Holder for Lifting Pole 16010700B0000            | Lacquered steel                                                                             | 0,15 kg        | 9 kg (3 kg per hook)  |
| Infusion Holder for Lifting Pole WDE5006220000            | Lacquered steel                                                                             | 0,41 kg        | 20 kg (5 kg per hook) |
| Infusion Pump Holder (Accessory Pole) 1102520000000       | Chrome steel                                                                                | 1,7 kg         | 3 kg                  |
| Infusion Bottle Basket 4DR426101                          | Stainless steel                                                                             | 0,08 kg        | 2 kg                  |
| Telescopic Infusion Stand (D34) 4PV340290000              | Chrome steel                                                                                | 0,78 kg        | 15 kg (2 kg per hook) |
| Telescopic Infusion Stand (D34) 4PV17744500L              | Chrome steel                                                                                | 1,58 kg        | 15 kg (2 kg per hook) |
| Telescopic Infusion Stand (D34) 4PV17587200L              | Chrome steel                                                                                | 1,57 kg        | 8 kg (2 kg per hook)  |
| Telescopic Infusion Stand (D34) 4PV322717000              | Chrome steel                                                                                | 1,51 kg        | 15 kg (2 kg per hook) |
| Telescopic Infusion Stand (D34) 4PV297475000              | Chrome steel                                                                                | 1,63 kg        | 10 kg (5 kg per hook) |
| Telescopic Infusion Stand (D34) 4PV33449300L              | Stainless steel                                                                             | 1,52 kg        | 2 kg per hook         |
| Telescopic Infusion Stand (D19) 4PKV26107200              | Chrome steel                                                                                | 1,54 kg        | 8 kg (2 kg per hook)  |
| S-shaped Telescopic Infusion Stand 4PV17604800L           | Chrome steel                                                                                | 1,45 kg        | 4 kg (2 kg per hook)  |
| S-shaped Telescopic Infusion Stand 4PV50-0742-0000        | Stainless steel                                                                             | 1,37 kg        | 4 kg (5 kg per hook)  |
| S-shaped Telescopic Infusion Stand 4PKV50-0693            | Stainless steel                                                                             | 2,3 kg         | 2 kg per hook         |
| S-shaped Telescopic Infusion Stand 4PV33466000L           | Stainless steel                                                                             | 1,5 kg         | 2 kg per hook         |
| S-shaped Telescopic Infusion Stand 4MA45MNA0003           | Stainless steel                                                                             | 2,3 kg         | 20 kg (5 kg per hook) |

| Compatible Accessories for Essenza 300 Junior <sup>+</sup>                | Materials (surfaces of the accessories mentioned for the sake of cleaning and disinfection) | Accessory Mass | SWL    |
|---------------------------------------------------------------------------|---------------------------------------------------------------------------------------------|----------------|--------|
| Mobile Infusion Stand 4ZZ426100                                           | Chrome steel                                                                                | 4 kg           | 9,5 kg |
| Monitor Shelf 11026500B0000                                               | Lacquered steel + HPL                                                                       | 8,84 kg        | 15 kg  |
| Utility Shelf for Plastic Head Board and Foot Board 11026000B0000         | Lacquered steel + Acrylonitrile butadiene styrene (ABS)                                     | 2 kg           | 10 kg  |
| Horizontal Oxygen Bottle Holder for Plastic Head Board 4MAS0001653        |                                                                                             | 1,39 kg        | 15 kg  |
| Urinary Bag Holder 11023700B0000                                          | Lacquered steel                                                                             | 0,1 kg         | 5 kg   |
| Urinary Bottle Holder for Plastic Siderails 11027900B0000                 | Lacquered steel                                                                             | 0,42 kg        | 5 kg   |
| Redon Bottle Holder 11008200B0000                                         | Lacquered steel                                                                             | 0,47 kg        | 5 kg   |
| Redon Bottle Holder 11008200A0000                                         | Lacquered steel                                                                             | 0,47 kg        | 5 kg   |
| Crutch Holder for Plastic Foot Board 1102070000000                        | Lacquered steel                                                                             | 0,2 kg         | 3 kg   |
| Crutch Holder for Plastic Foot Board 1102080000000                        | Lacquered steel                                                                             | 0,2 kg         | 3 kg   |
| Towel Holder for Plastic Foot Board 11025002B0000                         | Galvanised steel + Acrylonitrile butadiene styrene (ABS)                                    | 0,84 kg        | 3 kg   |
| Name Holder for Plastic Foot Board 4GT006000000                           |                                                                                             | 0,04 kg        |        |
| Chart Holder for Plastic Foot Board 11023900A0000                         | Acrylonitrile butadiene styrene (ABS)                                                       | 0,26 kg        |        |
| Holder of Small Accessories with Two Hooks 1102490000000                  | Polymethyl methacrylate (PMMA)                                                              |                | 3 kg   |
| Positioning Cushion 4SET17500001                                          | Washable cover                                                                              | 1,47 kg        |        |
| SafeSense 3 Stand-alone Version 1SFS300001                                | Consult the Instructions for Use of this product!                                           | 2,6 kg         |        |
| SafeSense 3 for Essenza 300 1SFS300032                                    | Consult the Instructions for Use of this product!                                           |                |        |
| Traction Frame Z 4MAEXZ000000                                             | Stainless steel + Polyamide (PA) + Brass + Duralumin                                        | 35 kg          | 75 kg  |
| Reading Lamp 4PV383435X00                                                 | Consult the Instructions for Use of this product!                                           | 1,2 kg         |        |
| Traction Frame D Holder for Foot Board with One Safety Lock 1103470000000 | Chrome steel + Polyamide (PA) + Polyethylene (PE)                                           | 6 kg           | 20 kg  |
| Set of weight bags for children (5,75 kg) 4RKZVSD001                      |                                                                                             | 5,75 kg        |        |

| Compatible Accessories<br>for Essenza 300 Junior <sup>+</sup>               | Materials (surfaces of the accessories mentioned<br>for the sake of cleaning and disinfection) | Accessory<br>Mass | SWL   |
|-----------------------------------------------------------------------------|------------------------------------------------------------------------------------------------|-------------------|-------|
| Set of weight bags for adults (6,5 kg) 4RKZVSD006                           |                                                                                                | 6,5 kg            |       |
| Set of weights (5 kg) 4MAEX2500580                                          |                                                                                                | 5 kg              |       |
| Traction Frame T73 4KI4655006152                                            | Consult the Instructions for Use of this product!                                              | 24 kg             | 75 kg |
| Partial Traction Frame MP for Foot Board with One Safety Lock 4KI4655006178 | Consult the Instructions for Use of this product!                                              | 4,5 kg            | 20 kg |
| Arm Suspension Set 4KI4655003124                                            |                                                                                                | 24 kg             |       |
| Cervical Traction Set 4KI4655002548                                         | Aluminium (Al)                                                                                 | 7 kg              |       |
| Lumbar Suspension Set 4KI4655002530                                         | Aluminium (Al)                                                                                 | 7 kg              |       |
| Boppe Traction Set 4KI4655002556                                            | Aluminium (Al)                                                                                 | 16 kg             |       |
| Cross Infusion Pump Holder 1102480000000                                    | Chrome steel + Stainless steel + Plastic                                                       | 1,11 kg           | 3 kg  |

## 4.1 Lifting Pole

**WARNING!****Risk of injury due to incorrect use!**

- ▶ Ensure the Lifting Pole is correctly inserted to a bushing for Lifting Pole in inner corners of the mattress support platform at bed head end!
- ▶ Ensure the Lifting Pole is fixed above the mattress support platform to avoid any risk of the bed tipping over!

Lifting Pole is intended for supporting a patient who uses it to sit or lift by himself / herself or with the help of a caregiver.

Standard Lifting Pole is not compatible with the Essenza 300 Junior\* with Accessory Adaptor.

Lifting Pole must be placed into a bushing at head end of the mattress support platform.


Maximum Load of the Lifting Pole is 75 kg.

**To ensure safe use of the Lifting Pole:**


- ▶ Never exceed the maximum load of 75 kg.
- ▶ Never use the Lifting Pole for demanding rehabilitation exercises.
- ▶ To prevent the bed from tipping over, ensure that the Lifting Pole does not project out from the bed.

**To install the Lifting Pole:**

- ▶ Insert Lifting Pole into a bushing at head end of the mattress support platform.
- ▶ Ensure the Lifting Pole is safely installed above the mattress support platform.
- ▶ Ensure the safety pin of the Lifting Pole is locked in a compatible bushing.
- ▶ Attach a Triangular Holder with an adjustable strap to the Lifting Pole.

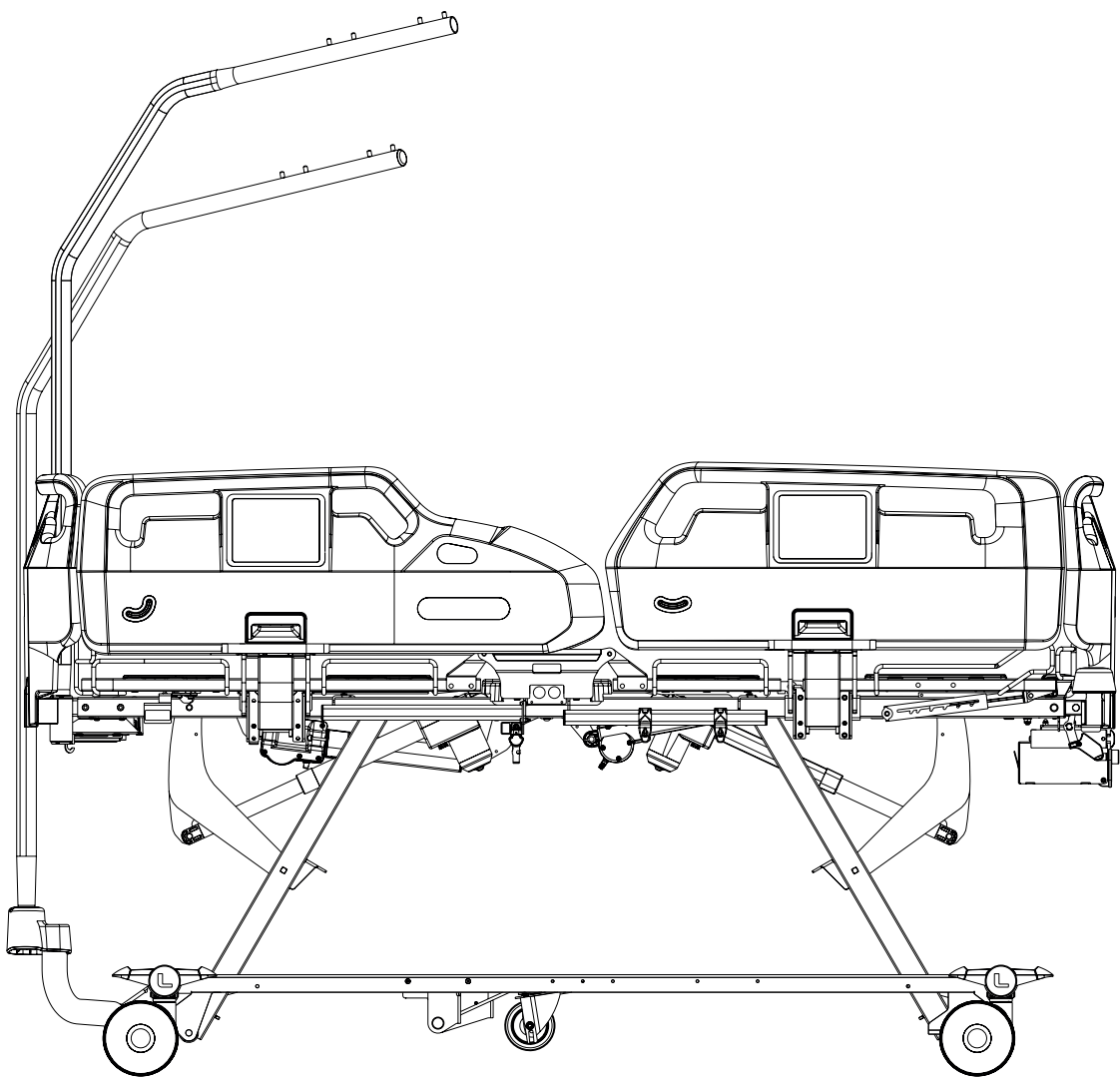

*Fig. Lifting Pole in the bushing at head end of the mattress support platform and Extended Lifting Pole in the Accessory Adaptor in undercarriage at bed head end*

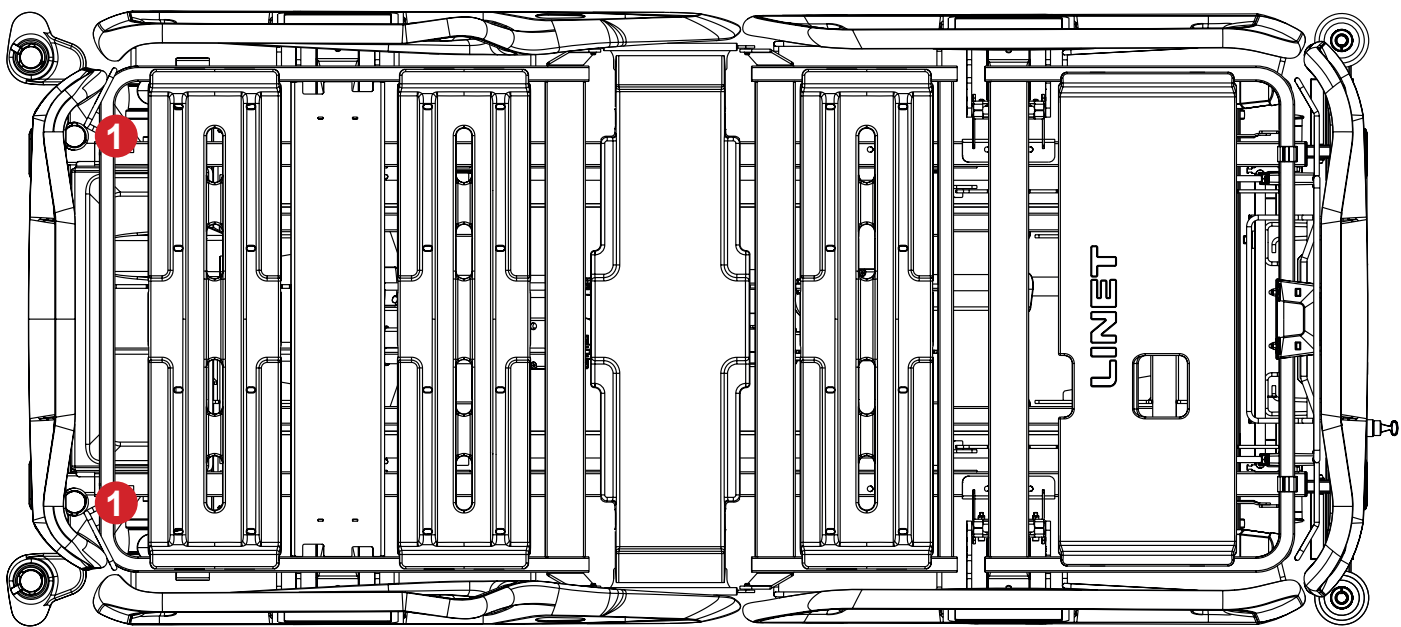

*Fig. Bushings for the Standard Lifting Pole (1) at head end of the mattress support platform*

## 4.2 Extended Lifting Pole



### **WARNING!**

#### **Risk of injury due to incorrect use!**

- ▶ Ensure the Extended Lifting Pole is correctly inserted to a bushing for Extended Lifting Pole in inner corners of the mattress support platform at bed head end or to a bushing in the Accessory Adaptor at bed head end!
- ▶ Ensure the Extended Lifting Pole is fixed above the mattress support platform to avoid any risk of the bed tipping over!

Extended Lifting Pole is intended for supporting a patient who uses it to sit or lift by himself / herself or with the help of a caregiver. Extended Lifting Pole is compatible with the Essenza 300 Junior+ with Accessory Adaptor. Extended Lifting Pole must be placed into a bushing at head end of the mattress support platform or into a bushing on the Accessory Adaptor at bed head end.


Maximum Load of the Extended Lifting Pole is 75 kg.

#### **To ensure safe use of the Extended Lifting Pole:**


- ▶ Never exceed the maximum load of 75 kg.
- ▶ Avoid collision between Extended Lifting Pole and the Head Board during Backrest adjustment or tilting to the Trendelenburg Position.
- ▶ Never use the Extended Lifting Pole for demanding rehabilitation exercises.
- ▶ To prevent the bed from tipping over, ensure that the Extended Lifting Pole does not project out from the bed.

#### **To install the Extended Lifting Pole:**


- ▶ Insert Extended Lifting Pole into a bushing at head end of the mattress support platform.
- ▶ Ensure the Extended Lifting Pole is safely installed above the mattress support platform.
- ▶ Ensure the safety pin of the Extended Lifting Pole is locked in a compatible bushing.
- ▶ Attach a Triangular Holder with an adjustable strap to the Extended Lifting Pole.

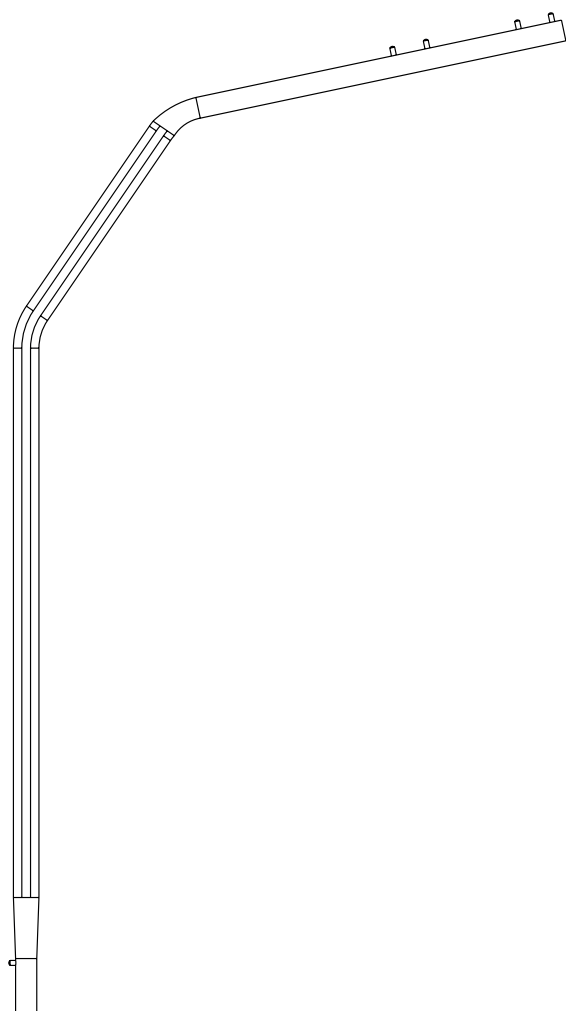

*Fig. Extended Lifting Pole*

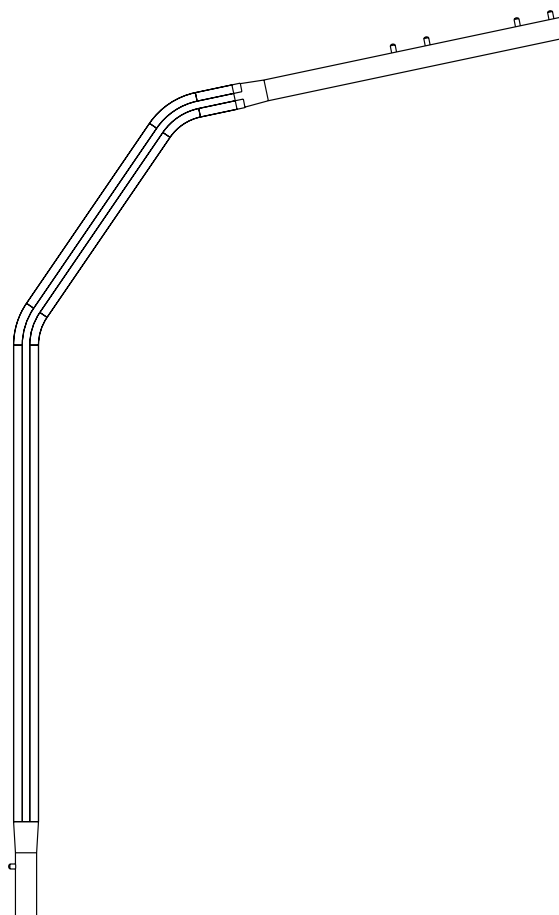

*Fig. Standard Lifting Pole*

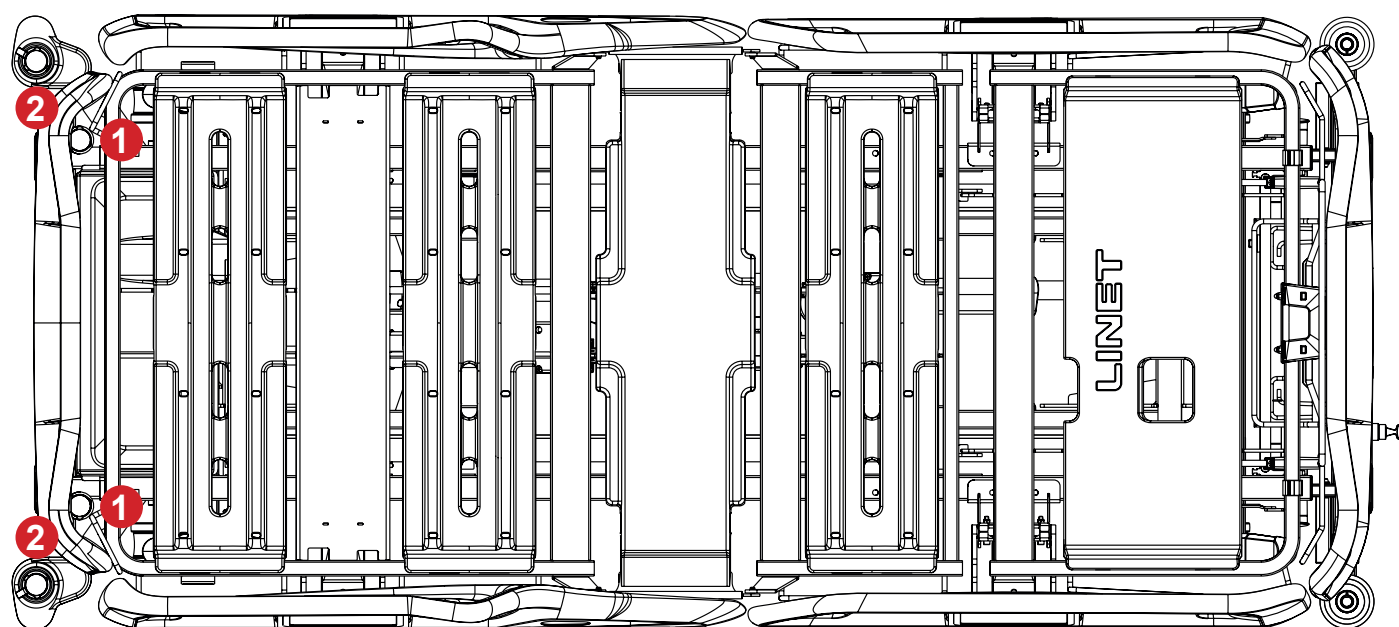

*Fig. Bushings for the Extended Lifting Pole (1) at head end of the mattress support platform and Accessory Adaptors for the Extended Lifting Pole (2) in the bed undercarriage*

## 4.3 Triangular Holder for Lifting Pole



**WARNING!**

Read Instructions for use of the Triangular Holder before use of this product!



**WARNING!**

Replace Triangular Holder when this replacement is recommended by its manufacturer! Read Instructions for use of the Triangular Holder before use of this product!

Triangular Holder is intended for supporting a patient who uses it to sit or lift by himself / herself or with the help of a caregiver. Triangular Holder is intended to be placed on the Lifting Pole. Maximum Load of the Triangular Holder is 75 kg.

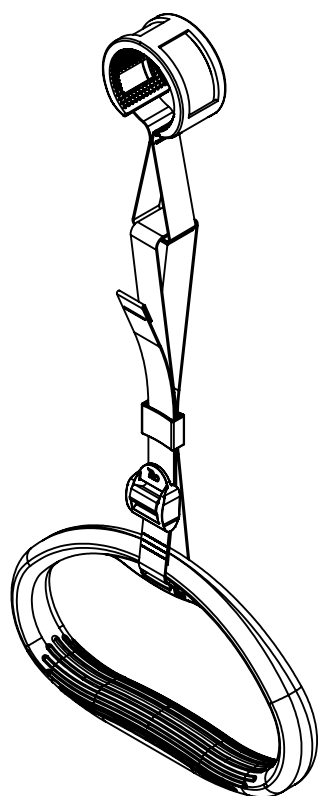

*Fig. Triangular Holder*  
4ROTGERSG700-2

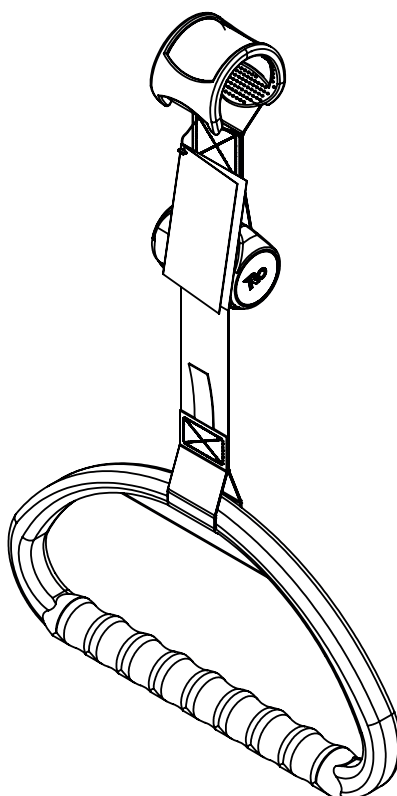

*Fig. Extendable Triangular Holder*  
4ROTGNR100GR-2

## 4.4 Pediatric Triangular Holder for Lifting Pole



**WARNING!**

Read Instructions for use of the Pediatric Triangular Holder before use of this product!



**WARNING!**

Replace Pediatric Triangular Holder when this replacement is recommended by its manufacturer! Read Instructions for use of the Pediatric Triangular Holder before use of this product!

Pediatric Triangular Holder is intended for supporting a child or adult with atypical anatomy who uses it to sit or lift by himself / herself or with the help of a caregiver.

Pediatric Triangular Holder is intended to be placed on the Lifting Pole.

Maximum Load of the Pediatric Triangular Holder is 45 kg.

Pediatric Triangular Holder is not a single-use accessory.

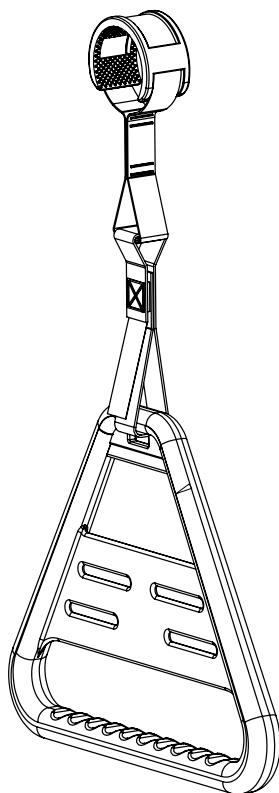


Fig. Pediatric Triangular Holder for Lifting Pole 4ROTGK100000

## 4.5 Infusion Holder for Lifting Pole

Infusion Holder is intended for holding IV bags with intravenous solutions.  
Infusion Holder is intended to be placed on the Lifting Pole.  
Maximum Load of the Infusion Holders is 10 kg.

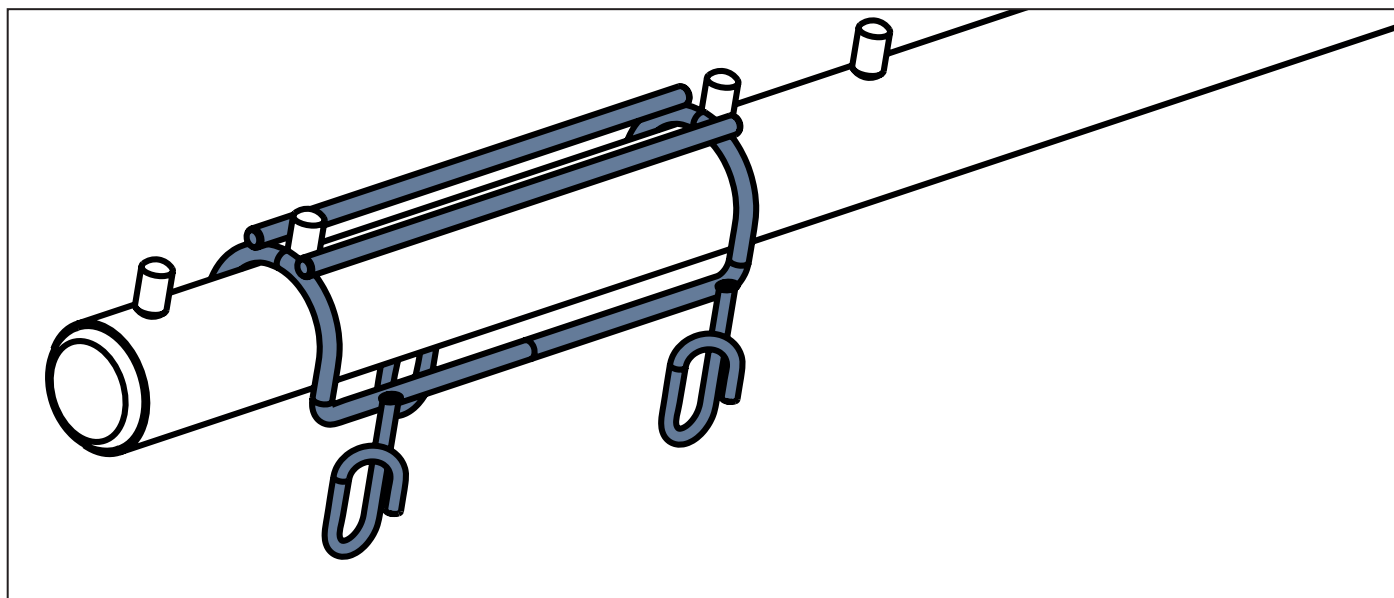


Fig. Infusion Holder 4MAPL00N1001

## 4.6 Infusion Pump Holder (Accessory Pole)



### WARNING!

#### Risk of injury due to incorrect use!

- ▶ Ensure the Infusion Pump Holder will not collide with any movable parts of the bed (especially Backrest bed part) or with the patient. This must be verified during installation.
- ▶ Do not overtighten the infusion pump clamps during fitment. Overtightening may damage the Infusion Pump Holder.
- ▶ Infusion pump can only be used if the Infusion Pump Holder is fitted into a bushing at head end of the mattress support platform or into a bushing on the Accessory Adaptor at bed head end.
- ▶ Do not use the Infusion Pump Holder as driving/pushing device during the bed transport.

Infusion Pump Holder is intended for holding a compatible infusion pump.

Infusion Pump Holder must be placed into a bushing at head end of the mattress support platform or into a bushing on the Accessory Adaptor at bed head end.

## 4.7 Infusion Bottle Basket

Infusion Bottle Basket is intended for carrying bottles with intravenous solutions.

It is intended to be suspended on a compatible Infusion Stand.

Maximum Load of the Infusion Bottle Basket is 2 kg.

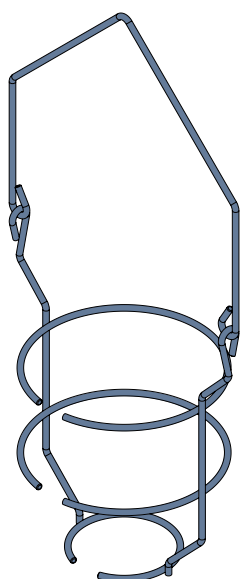


Fig. Infusion Bottle Basket

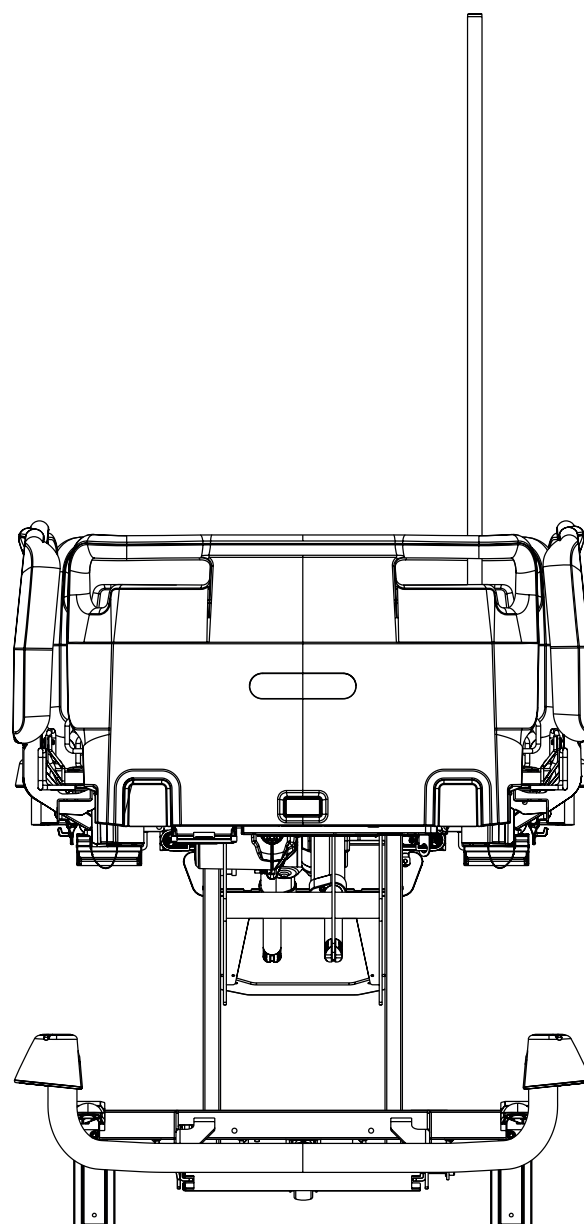


Fig. Infusion Pump Holder at bed head end

## 4.8 Telescopic Infusion Stand



### WARNING!

**Risk of injury due to incorrect use!**


**Infusion Stands must only be used for their intended use. Always read the instructions for use of the corresponding infusion stand!**

- ▶ Only mount an infusion pump to the lower (wider) telescopic section of an infusion stand above the Head Board or Foot Board.
- ▶ Never mount an infusion pump to the upper (thinner) telescopic section of an infusion stand.
- ▶ Ensure the infusion pump will not collide with any movable parts of the bed (especially Backrest bed part) or with the patient. This must be verified during installation!
- ▶ Do not overtighten the infusion pump clamps during installation. Overtightening may damage the infusion stand.
- ▶ Infusion pump can only be used if the infusion stand is fitted into a bushing at head end of the mattress support platform or into a bushing on the Accessory Adaptor at bed head end or into a bushing in the bed corner bumper depending on the compatibility.
- ▶ Do not use the infusion stand as driving/pushing device during the bed transport!
- ▶ Use exclusively infusion stands with hooks for hanging IV bags or baskets for intravenous solutions.
- ▶ Do not install the Infusion Stand 4PV50-0742-0000 to the bushings on the Accessory Adaptor at bed head end!
- ▶ Ensure the Safe Working Load of the infusion stand individual hook is not exceeded!
- ▶ Ensure the Safe Working Load of the whole infusion stand is not exceeded!
- ▶ Fix a required position of the Infusion Stand Telescopic Part by turning the knob and ensure the Telescopic Part of the Infusion Stand is secured! Inform patient whether he or she is allowed to adjust the turning knob!

Infusion stands are intended to provide a suitable support for the attachment of the infusion pumps/syringe pumps and suspension of the IV bags, bottles or baskets for intravenous solutions. Maximum Load of the Infusion Stand individual hook and Maximum Load of the whole Infusion Stand are stated on the corresponding Infusion Stand.



*Fig. S-shaped Infusion Stand (4MA45MNA0003)*

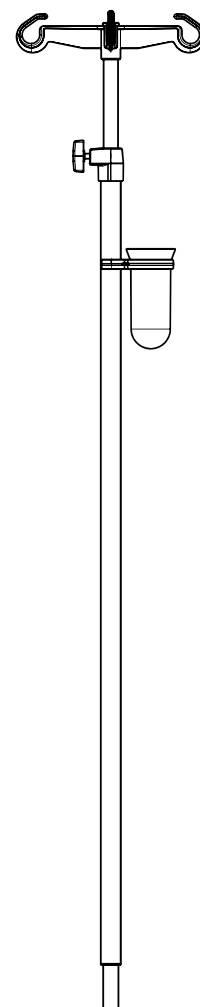

*Fig. Infusion Stand with receptacle for cannulas (4PKV26107200)*

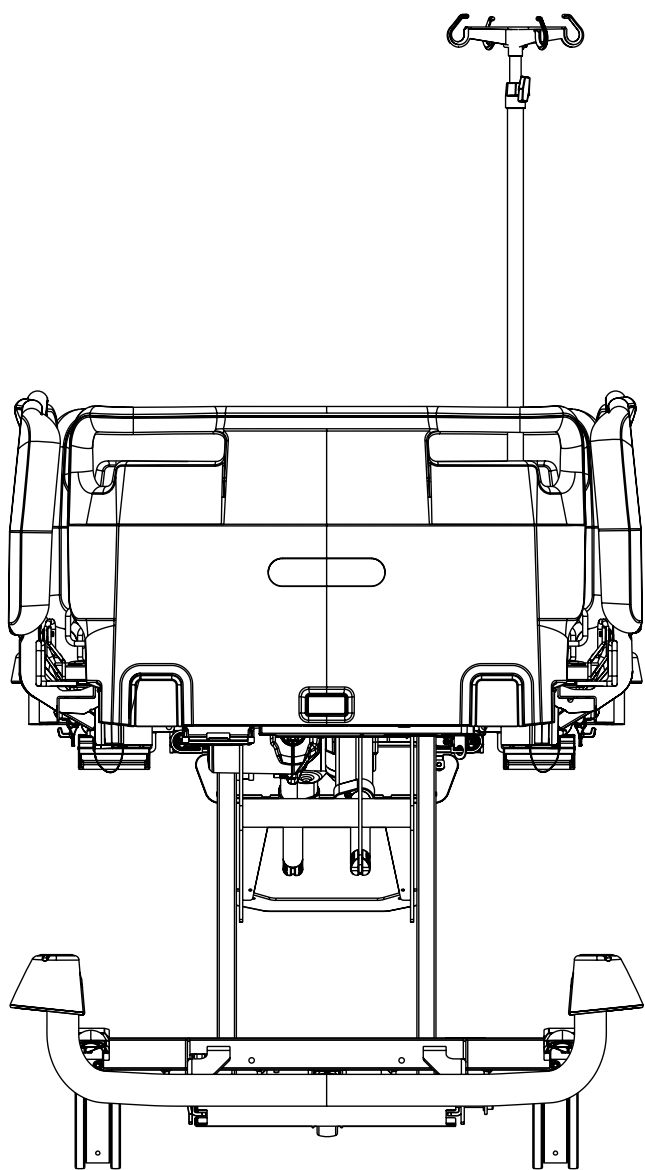


Fig. Infusion Stand in the bushing at head end of the mattress support platform


| Infusion Stand Identification Number | Placement                                                                                                    |
|--------------------------------------|--------------------------------------------------------------------------------------------------------------|
| 4PV340290000                         | 2 bushings at head end of the mattress support platform, 2 bushings on the Accessory Adaptor at bed head end |
| 4PV17744500L                         |                                                                                                              |
| 4PV17587200L                         |                                                                                                              |
| 4PV322717000                         |                                                                                                              |
| 4PV297475000                         |                                                                                                              |
| 4PV17604800L                         |                                                                                                              |
| 4MA45MNA0003                         |                                                                                                              |
| 4PV33449300L                         |                                                                                                              |
| 4PV30862900L-A                       |                                                                                                              |
| 4PV33466000L                         |                                                                                                              |
| 4PV33449300L                         |                                                                                                              |
| 4PKV50-0693                          |                                                                                                              |
| 4PV33466000L                         |                                                                                                              |
| 4PV50-0742-0000                      | 2 bushings at head end of the mattress support platform                                                      |
| 4PKV26107200                         | bushings in the bed corner bumpers                                                                           |

## 4.9 Mobile Infusion Stand



**WARNING!**

Read Instructions for use of the Mobile Infusion Stand before use of this product!

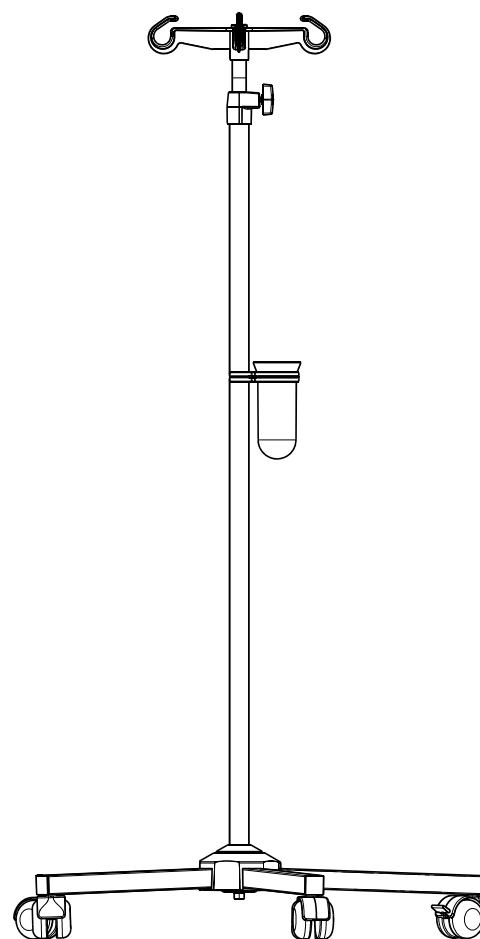

*Fig. Mobile Infusion Stand with receptacle for cannulas (4ZZ426100)*

## 4.10 Monitor Shelf



### WARNING!

#### Risk of injury due to incorrect use!

- ▶ Monitor must be fixed with straps on the Monitor Shelf before use!
- ▶ Avoid collision with Monitor Shelf when lifting the Calfrest!
- ▶ It is forbidden to install the Monitor Shelf into the bed corner bumpers at bed head end!

Monitor Shelf is intended for carrying a monitor when the Monitor Shelf is folded inwards the bed (3). Monitor Shelf must be located at bed foot end in the bed corner bumpers only. When the Monitor Shelf is folded outwards the bed (2) it serves for writing. Monitor Shelf is equipped with straps to fix a monitor on the Monitor Shelf.

Monitor Shelf is compatible with the bed equipped with corner bumpers with accessory bushings only.

Maximum load of the Monitor Shelf in position for carrying a monitor (3) and for writing (2) is 15 kg.

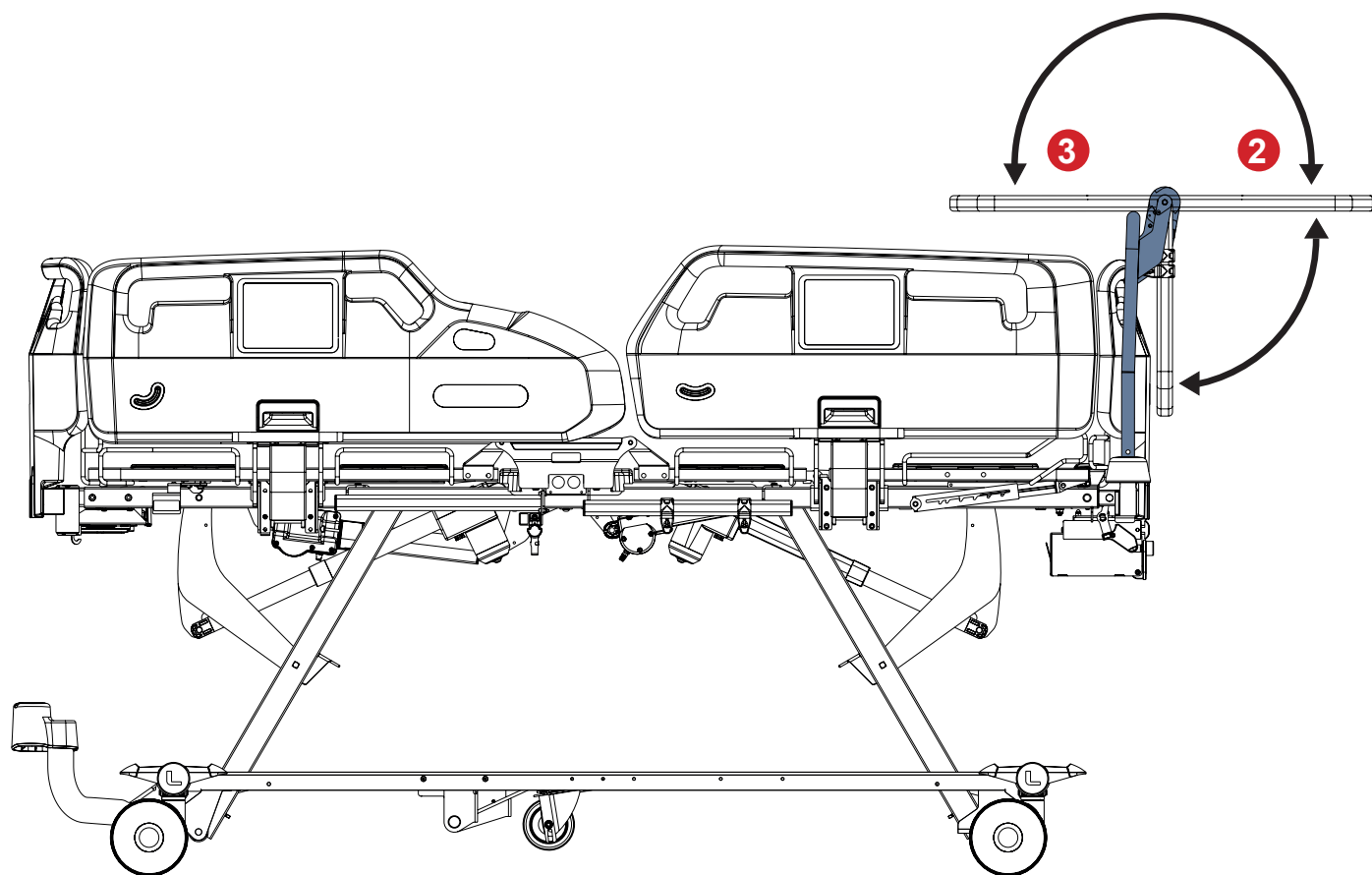


Fig. Monitor Shelf placed in bed foot end corner bumpers

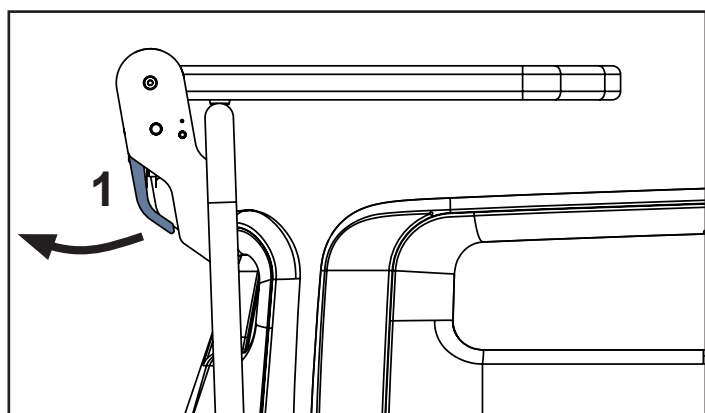


Fig. Manipulation with Monitor Shelf Control Lever (1)

- 1. Monitor Shelf Control Lever
- 4. Monitor Shelf Board
- 5. Straps for fixing a monitor
- 6. Hooks for accessories

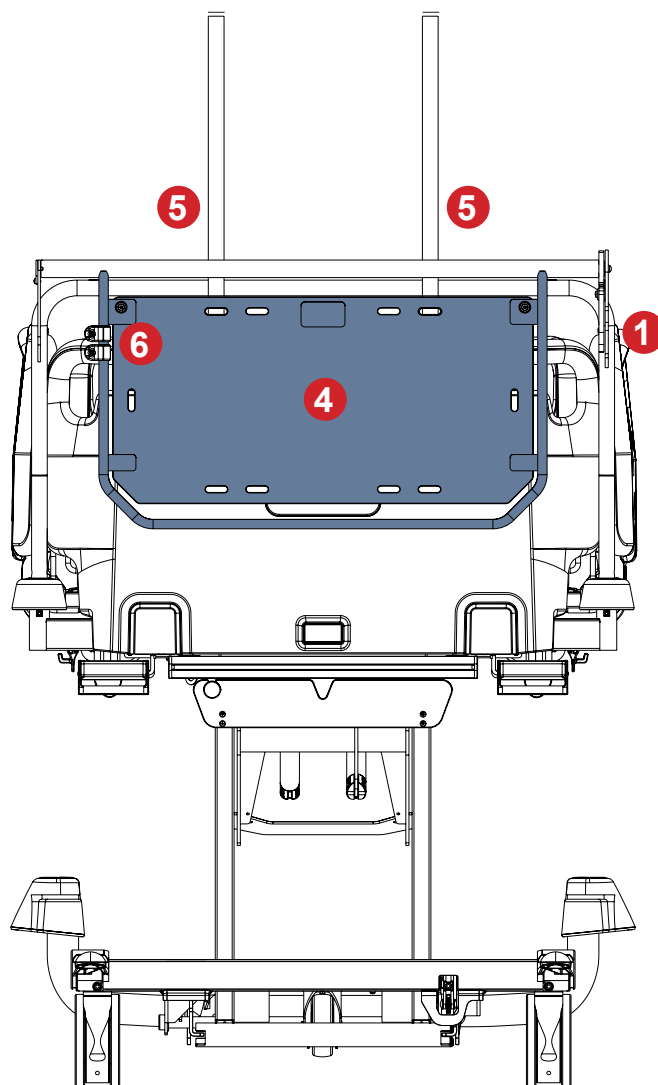


Fig. Description of the Monitor Shelf

## To change position of the board:

- Pull a control lever (1).
- Change position of the Monitor Shelf Board.
- Release the control lever (1) in order to lock the position of the Monitor Shelf board.
- Move the board up and down to ensure the board is fixed.

## HOOKS FOR ACCESSORIES


The position of hooks placed on the frame of Monitor Shelf can be changed to a more convenient position. Respect maximum load of the Monitor Shelf when hanging things on the hooks! Maximum load of the plastic hook is 2 kg.

## 4.11 Utility Shelf

The Utility Shelf is intended for linens or sheets.

It is intended to be placed on the Plastic Head Board or the Plastic Foot Board.

Maximum Load for the Utility Shelf is 10 kg.

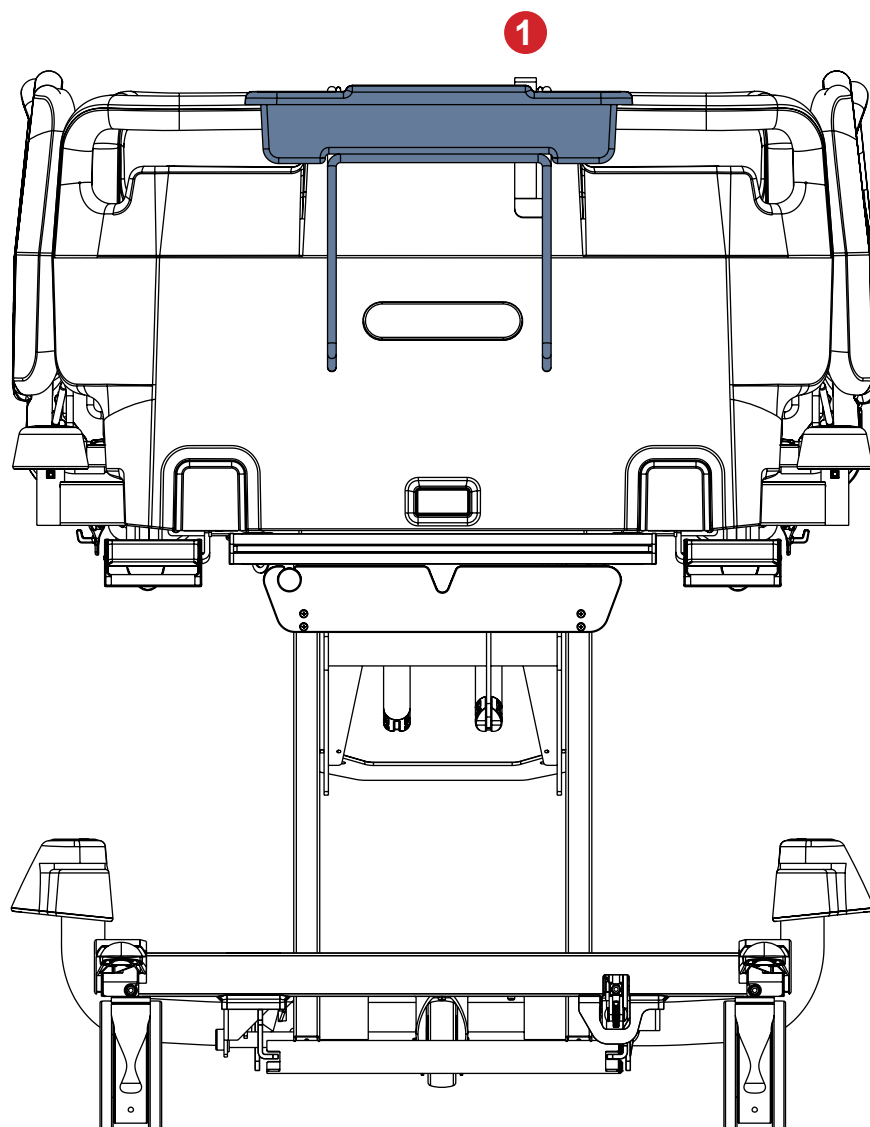


*Fig. Utility Shelf on the Foot Board*

### **To place Utility Shelf on the Plastic Head Board or Foot Board:**


- ▶ Place wire construction on the Head Board or Foot Board.
- ▶ Insert upper edge of the plastic container under the upper holder of the wire construction as on the picture above.
- ▶ Lower the rest of the plastic container to hang it on the wire construction.

### **To tilt the plastic container to non-working position:**

- ▶ Press release lever (1) and tilt the plastic container into the wire construction.

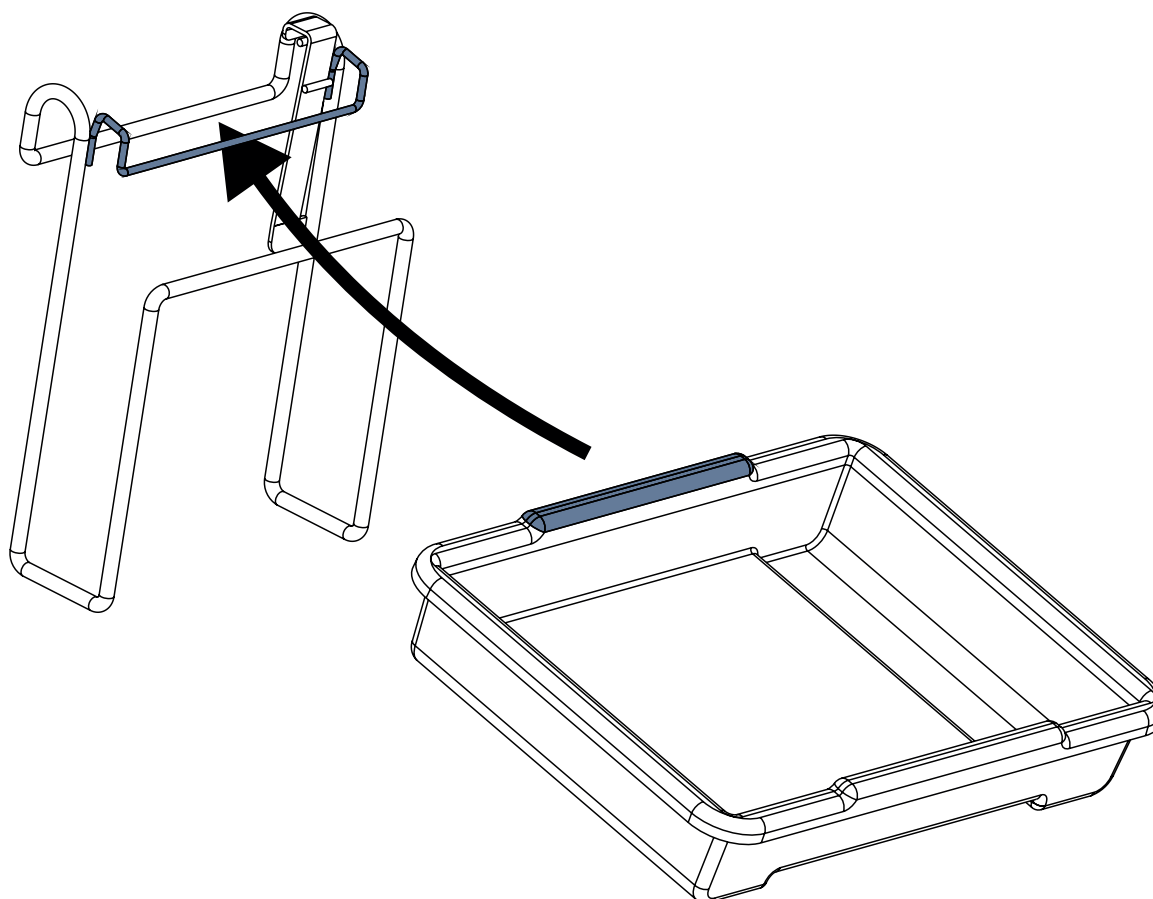

*Fig. Placement of plastic container to wire construction of the Utility Shelf*

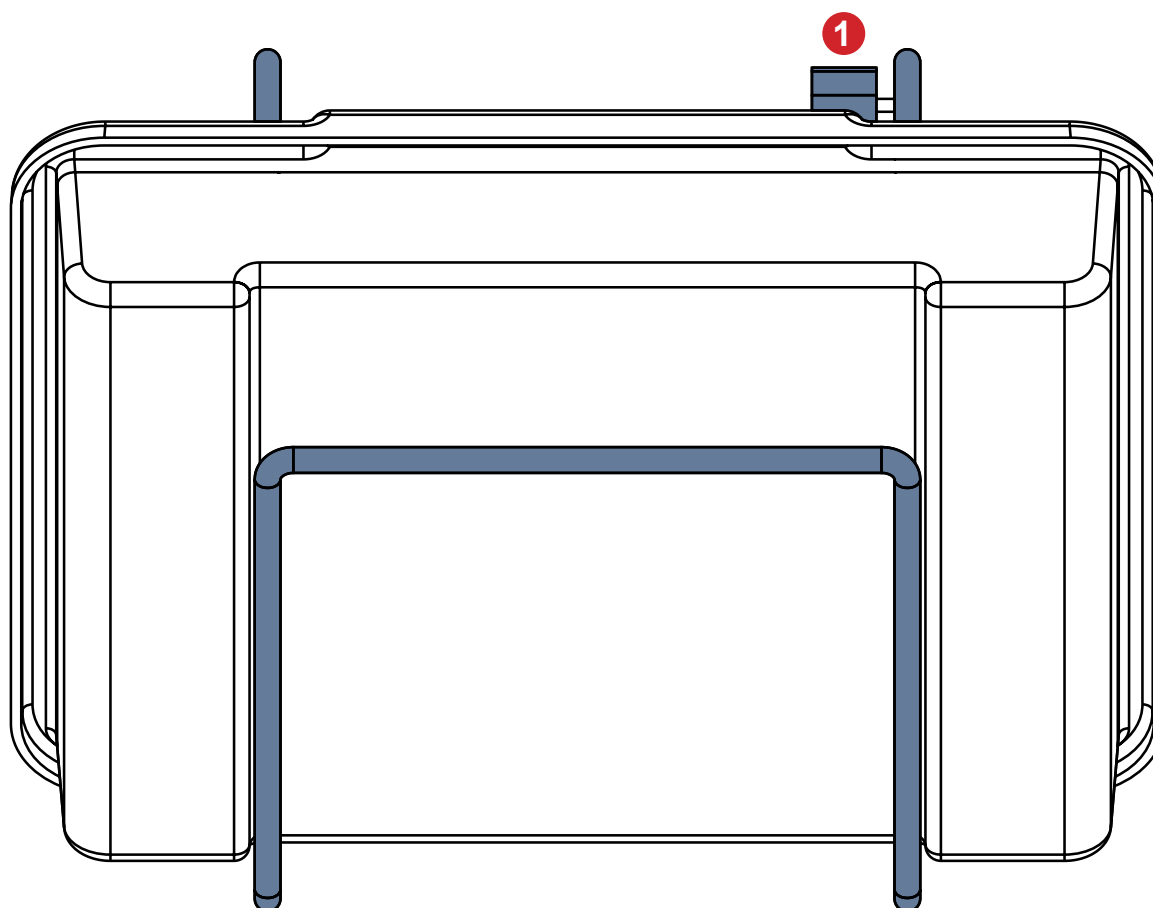

*Fig. Plastic container in the non-working position (1 - release lever of the plastic container)*

4.12 Oxygen Bottle Holder




- WARNING!**  
**Risk of injury with Oxygen Bottle Holder due to incorrect use or due to careless driving!**
- ▶ Ensure the Plastic Head Board is correctly inserted into the bed before placing the Oxygen Bottle Holder onto the Plastic Head Board.
  - ▶ Ensure the Oxygen Bottle Holder is correctly fitted in correct position on the Plastic Head Board.
  - ▶ It is necessary to check the position of the Oxygen Bottle Holder (with or without oxygen bottle) before transport to secure its transport position.
  - ▶ Be aware of people or objects in close proximity when driving or manipulating the bed equipped with Oxygen Bottle Holder.
  - ▶ Secure any oxygen bottle in the Horizontal Oxygen Bottle Holder against falling or involuntary movement with rubber strap. Do not use the Horizontal Oxygen Bottle Holder without fixing rubber strap!
  - ▶ Ensure the oxygen bottle valve will not get damaged by careless or incorrect manipulation or placement.

Oxygen Bottle Holders are intended for holding and transporting oxygen bottles with a weight of up to 15 kg and a volume of 5 litres.

| Oxygen Bottle Holder Identification Number | Compatible oxygen bottle dimensions (length x diameter) |
|--------------------------------------------|---------------------------------------------------------|
| 4MAS0001653                                | Max: 52 cm x 14 cm<br>Min: 29 cm x 9 cm                 |

4.12.1 Horizontal Oxygen Bottle Holder for Plastic Head Board

- To place Horizontal Oxygen Bottle Holder on the bed:**
- ▶ Put Horizontal Oxygen Bottle Holder onto the Plastic Head Board so that Horizontal Oxygen Bottle Holder is hanging from both windows in the Plastic Head Board.
- To insert any compatible oxygen bottle into the Horizontal Oxygen Bottle Holder safely:**
- ▶ Secure the oxygen bottle against falling out of the Horizontal Oxygen Bottle Holder or involuntary movement in the Horizontal Oxygen Bottle Holder with rubber strap.
  - ▶ Ensure the Horizontal Oxygen Bottle Holder and a compatible oxygen bottle are fitted correctly.

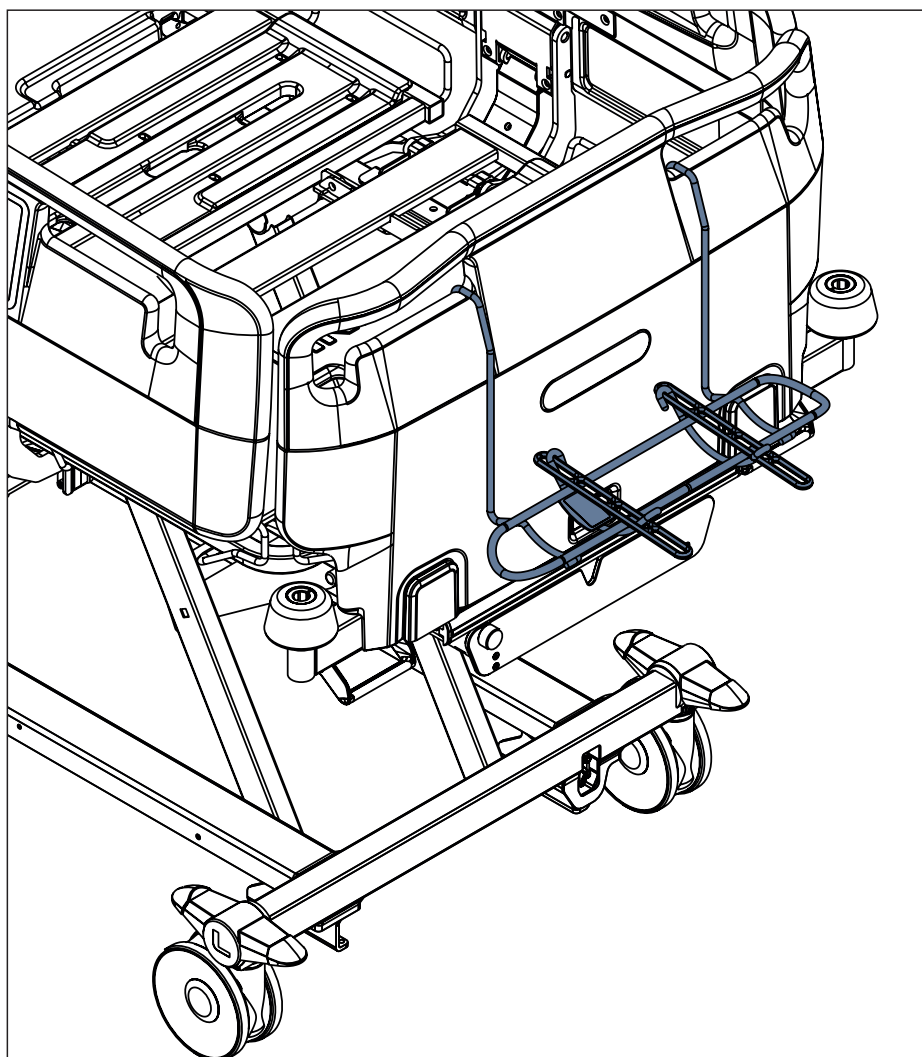


*Fig. Horizontal Oxygen Bottle Holder on the plastic foot board*

## 4.13 Urinary Bag Holder

Urinary Bag Holder is intended for holding the urinary bags.

Urinary Bag Holder is intended to be placed on the Accessory Rail towards the centre of the bed.

Maximum Load of the Urinary Bag Holder is 5 kg.

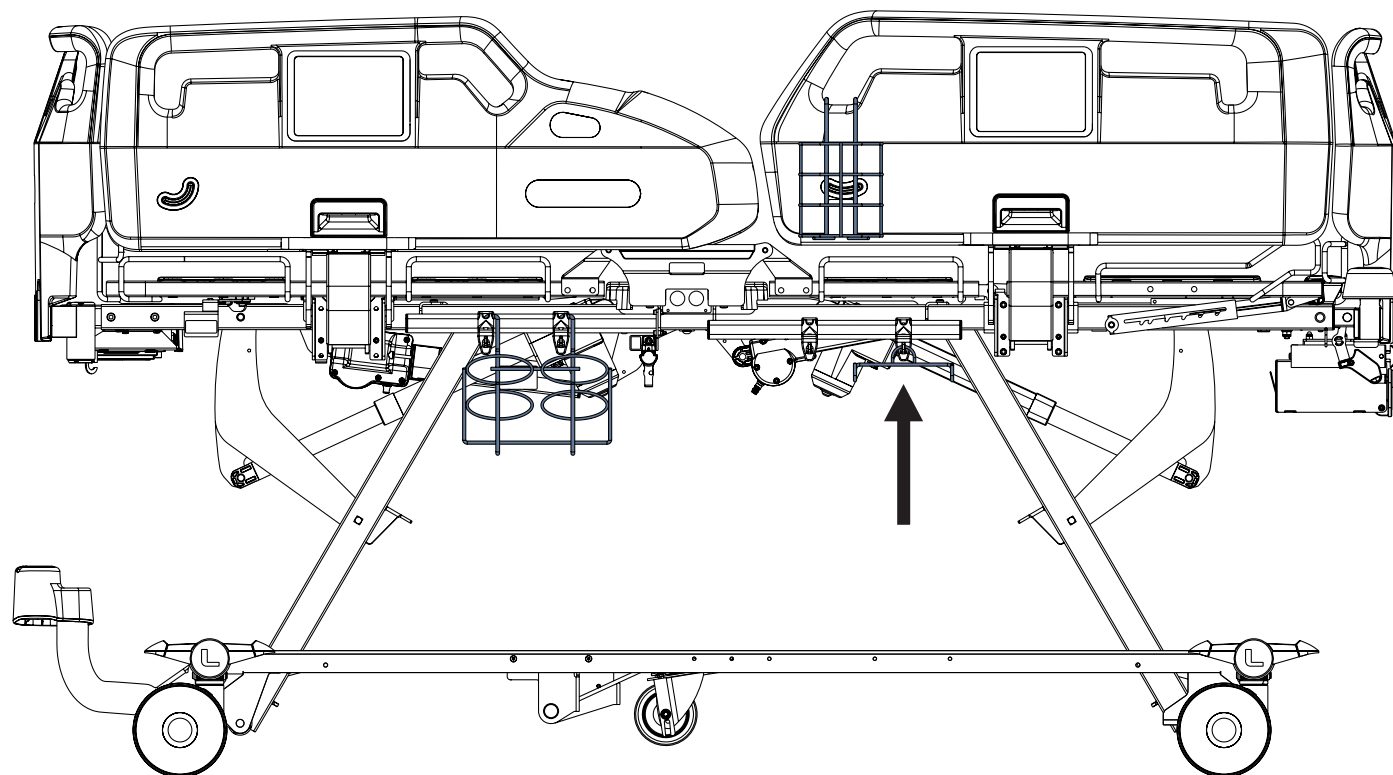


*Fig. Urinary Bag Holder hanging from the hook on the Accessory Rail*

## 4.14 Urinary Bottle Holder

Urinary Bottle Holder is intended for holding the compatible urinary bottles.

### 4.14.1 Urinary Bottle Holder for Plastic Siderails

Urinary Bottle Holder for Plastic Siderails is intended to be placed on the Accessory Rail or on the plastic Foot Siderail.

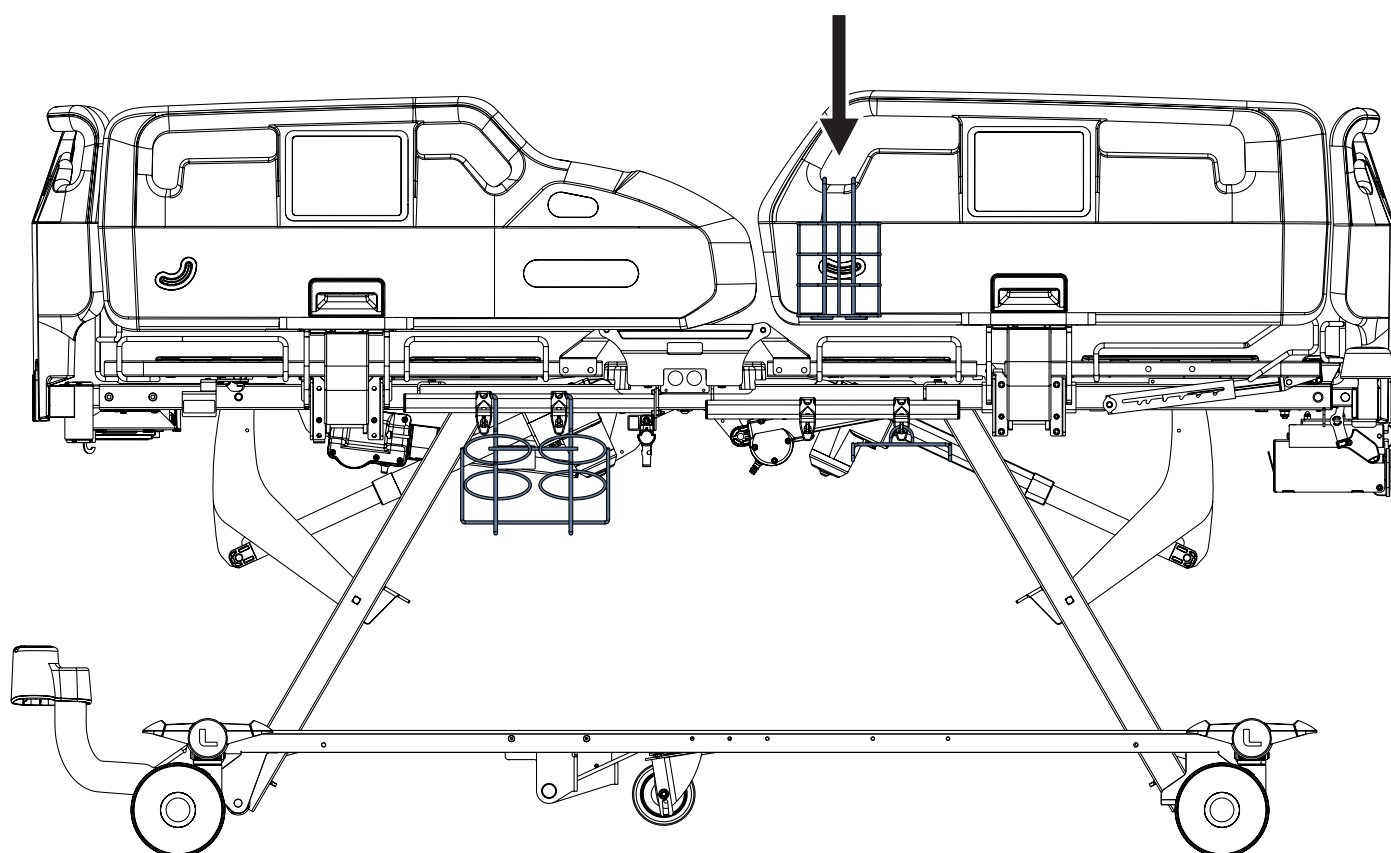


*Fig. Urinary Bottle Holder hanging from the right window in the plastic foot siderail*

## 4.15 Redon Bottle Holder

Redon Bottle Holder is intended for holding redon bottles.

Redon Bottle Holder in configuration with Plastic Siderails is intended to be placed on the Accessory Rail under the Backrest only.

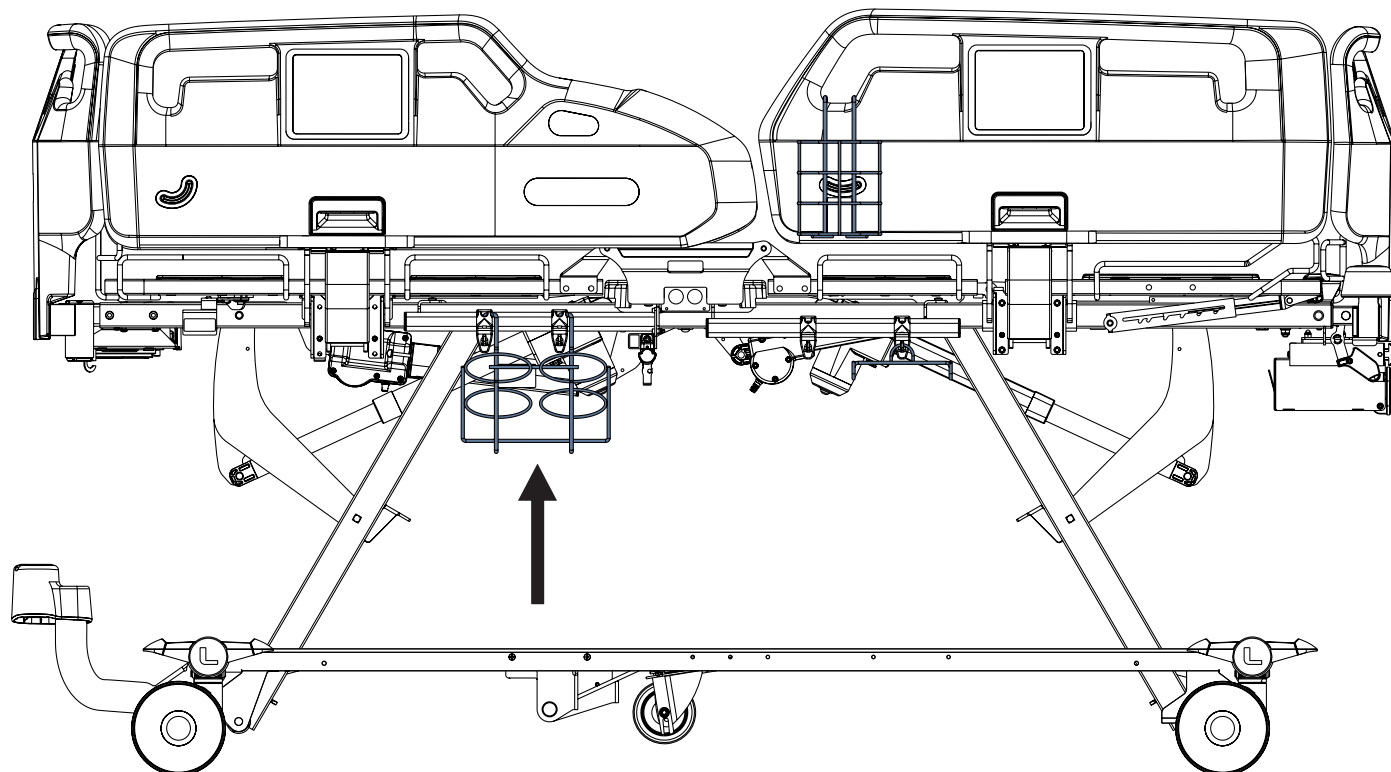

*Fig. Redon Bottle Holder on the Accessory Rail under the backrest of the bed*

## 4.16 Crutch Holders

Crutch Holder is intended for holding the pair of crutches only.

Crutch Holder is intended to be placed on the bed plastic foot board or bed plastic head board only.

Maximum diameter of a compatible crutch tube under the crutch handle is 3 cm. Minimum diameter of a compatible crutch tube under the crutch handle is 2 cm. Maximum even load of the Crutch Holder is 3 kg.

### To place a compatible crutch to the Crutch Holder:

- Insert a compatible crutch tube under the crutch handle to the Crutch Holder (1).

### To remove a compatible crutch from the Crutch Holder:

- Take the crutch out of the Crutch Holder by moving it outwards towards you (2).

### 4.16.1 Crutch Holder for Plastic Foot Board

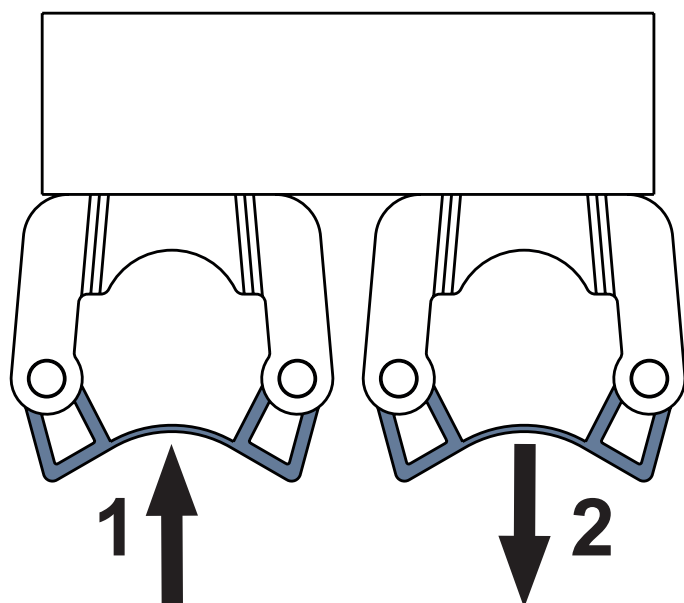


Fig. Crutch Holder 1102070000000

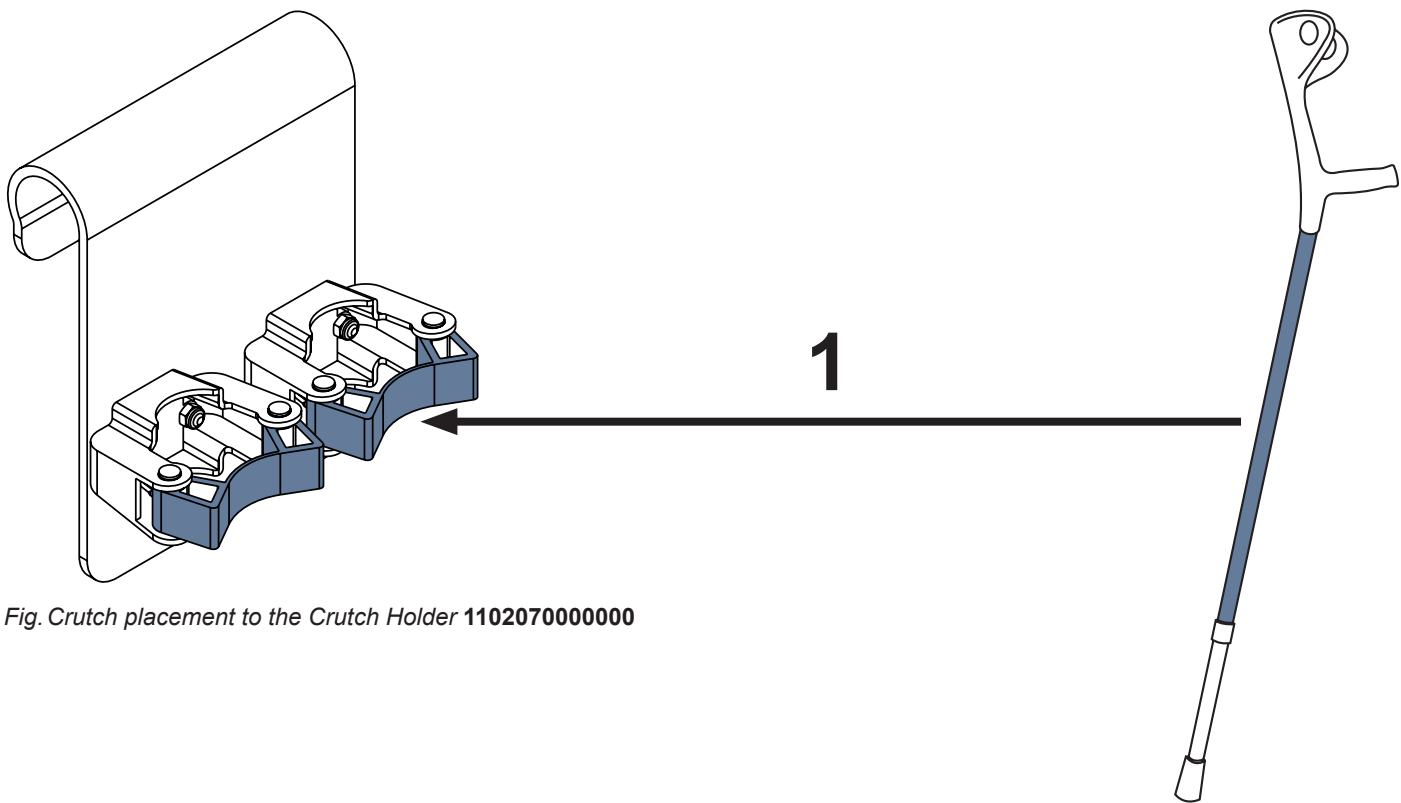


Fig. Crutch placement to the Crutch Holder 1102070000000

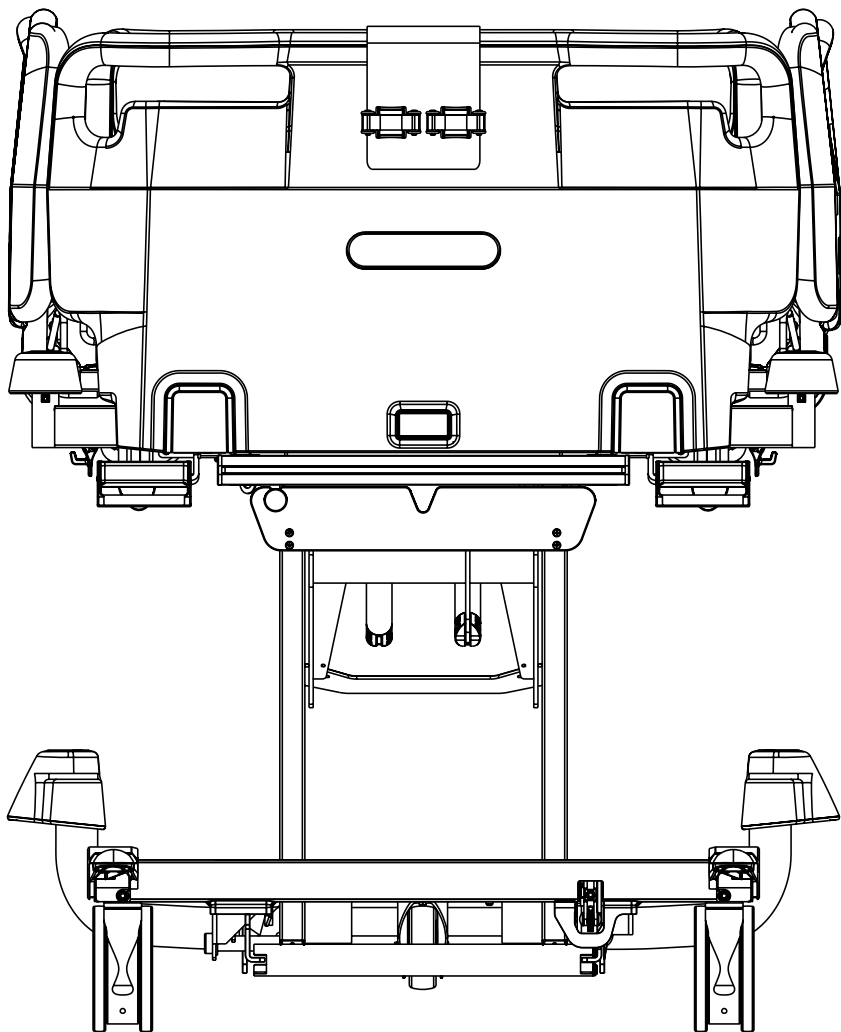


Fig. Crutch Holder 1102070000000 on the Plastic Foot Board

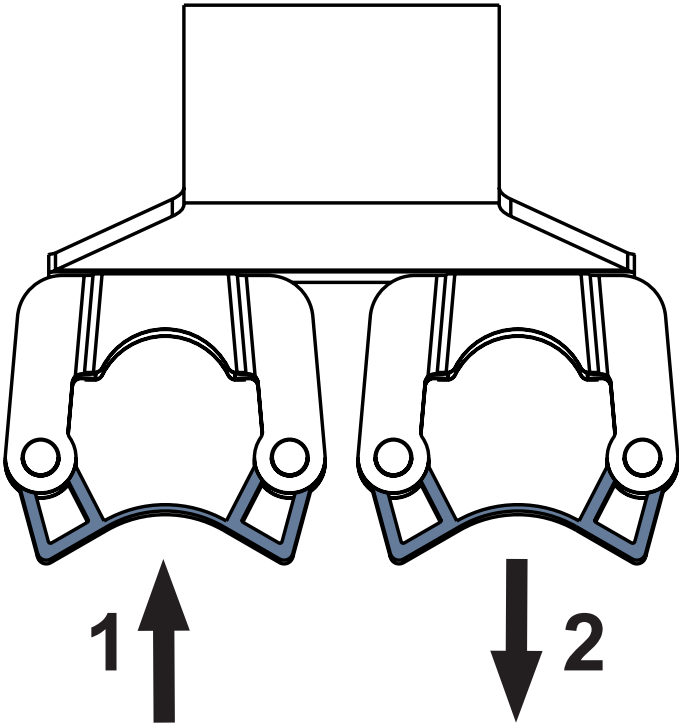


Fig. Crutch Holder 1102080000000


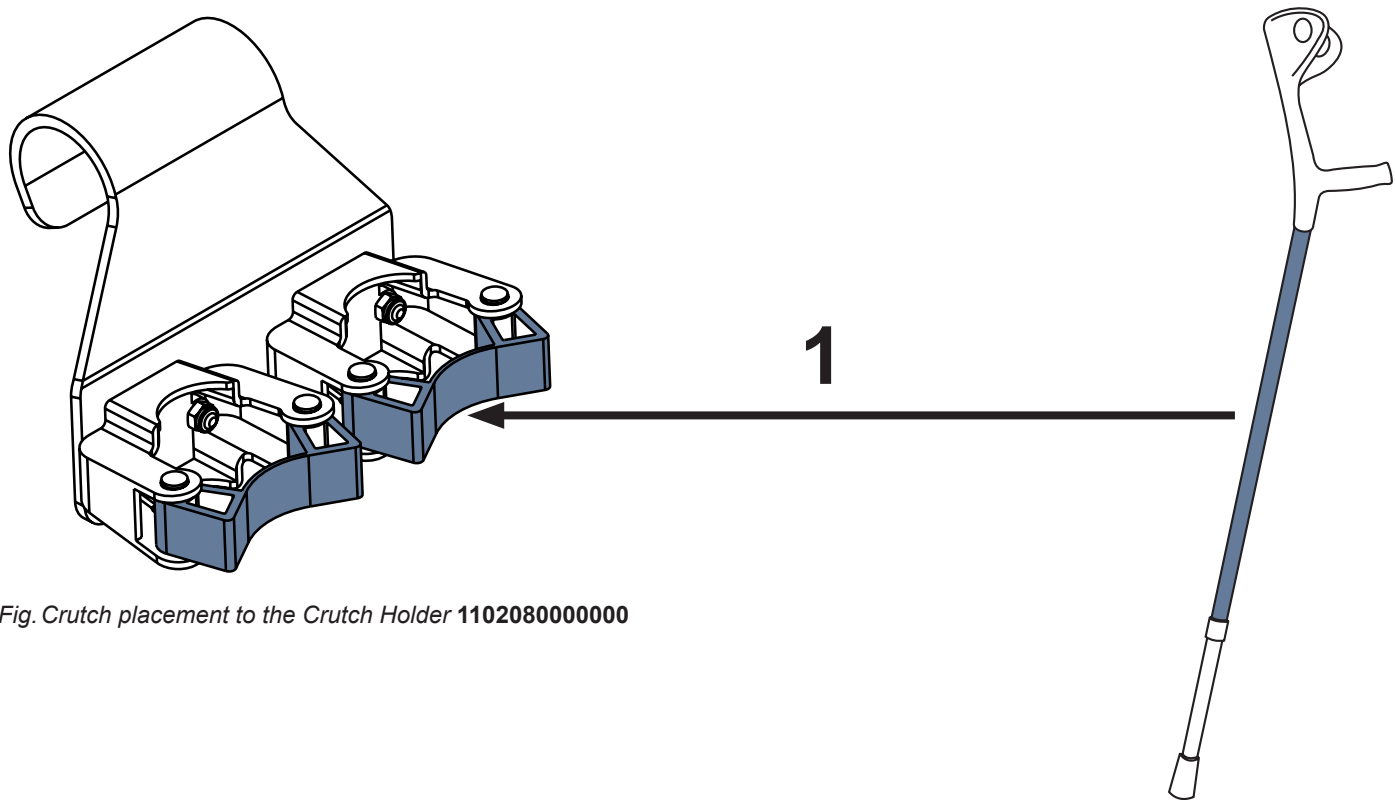


Fig. Crutch placement to the Crutch Holder 1102080000000

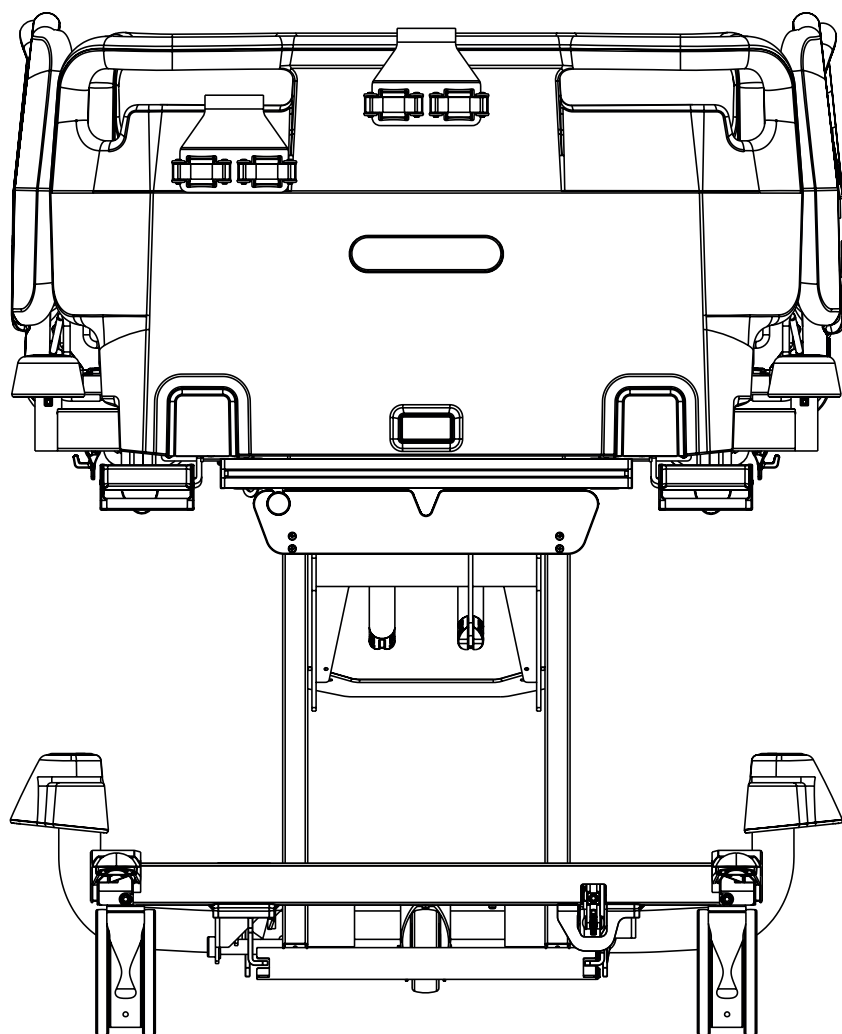


*Fig. Crutch Holder 1102080000000 on the Plastic Foot Board*

## 4.17 Towel Holder

Towel Holder is intended for holding towels or linens.

### 4.17.1 Towel Holder for Plastic Foot Board

Towel Holder for Plastic Foot Board is intended to be suspended from both windows in the Plastic Foot Board or Plastic Head Board.

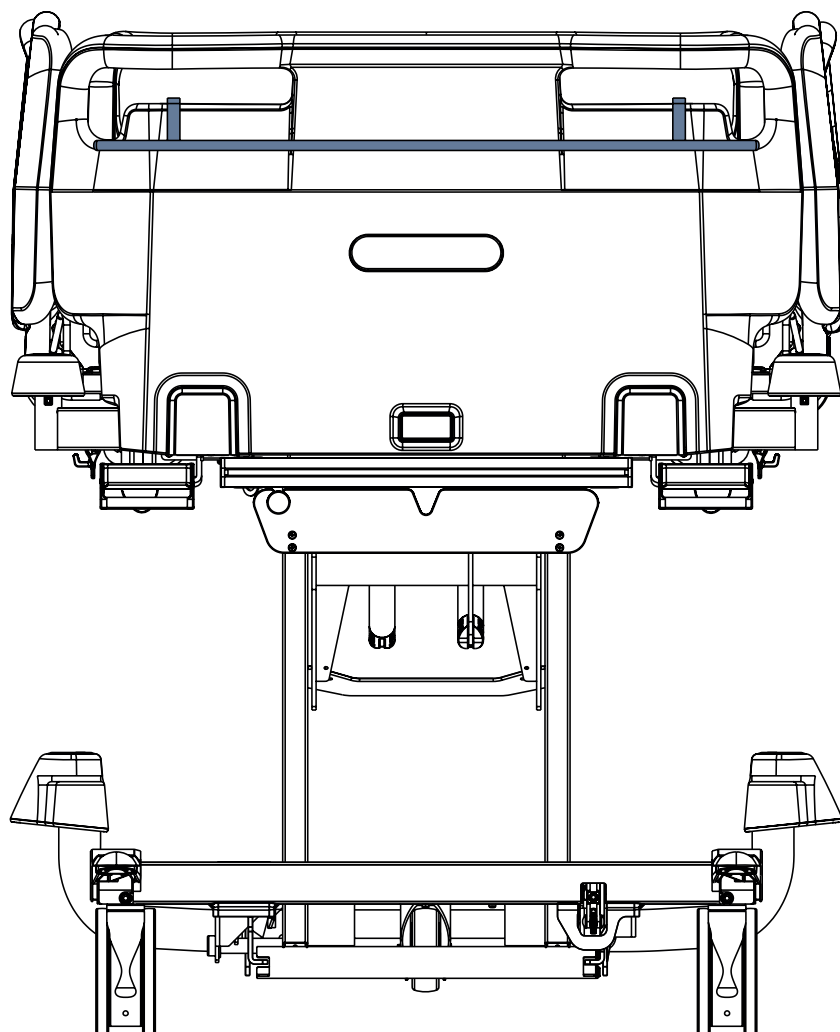


*Fig. Towel Holder on the Plastic Foot Board*

## 4.18 Name Holder

Name Holder is intended for holding a card with patient's name.

### 4.18.1 Name Holder for Plastic Foot Board

Name Holder for Plastic Foot Board is intended to be placed on the upper edge of the Plastic Foot Board or Plastic Head Board.

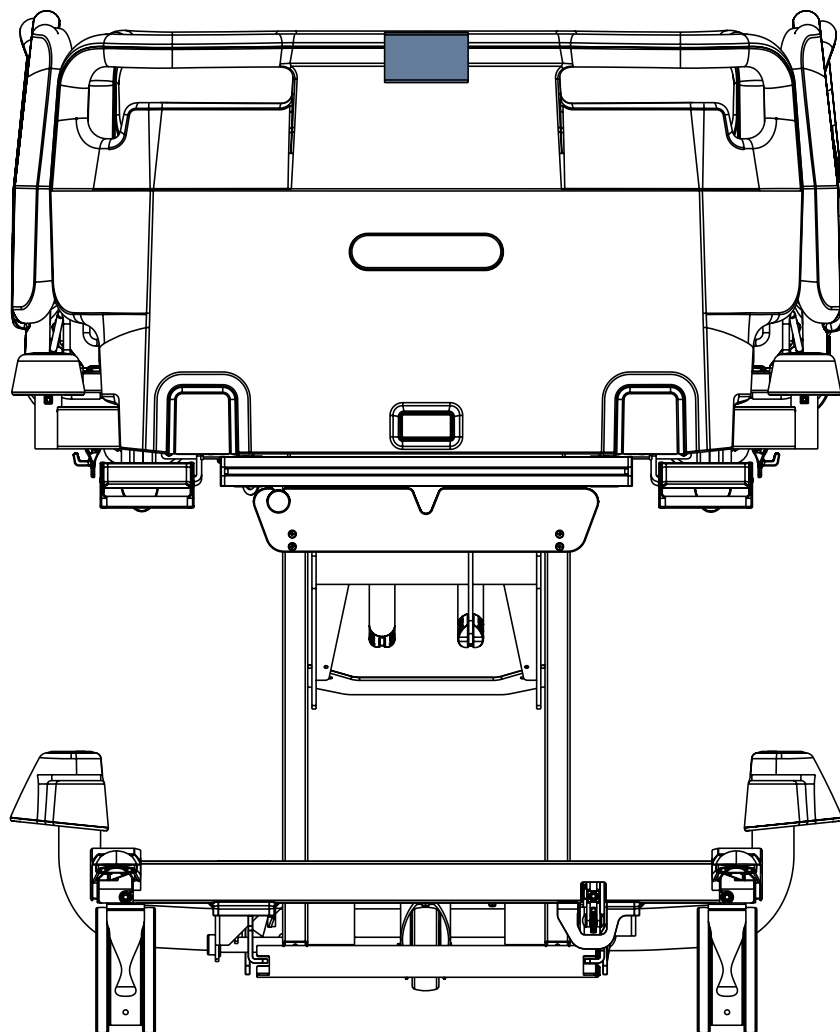


*Fig. Name Holder on the Plastic Foot Board*

## 4.19 Chart Holder

Chart Holder is intended for holding a chart that is registering the development of health condition of the patient.

### 4.19.1 Chart Holder for Plastic Foot Board


Chart Holder for Plastic Foot Board is intended to be placed on the upper edge of the Plastic Foot Board or Plastic Head Board.

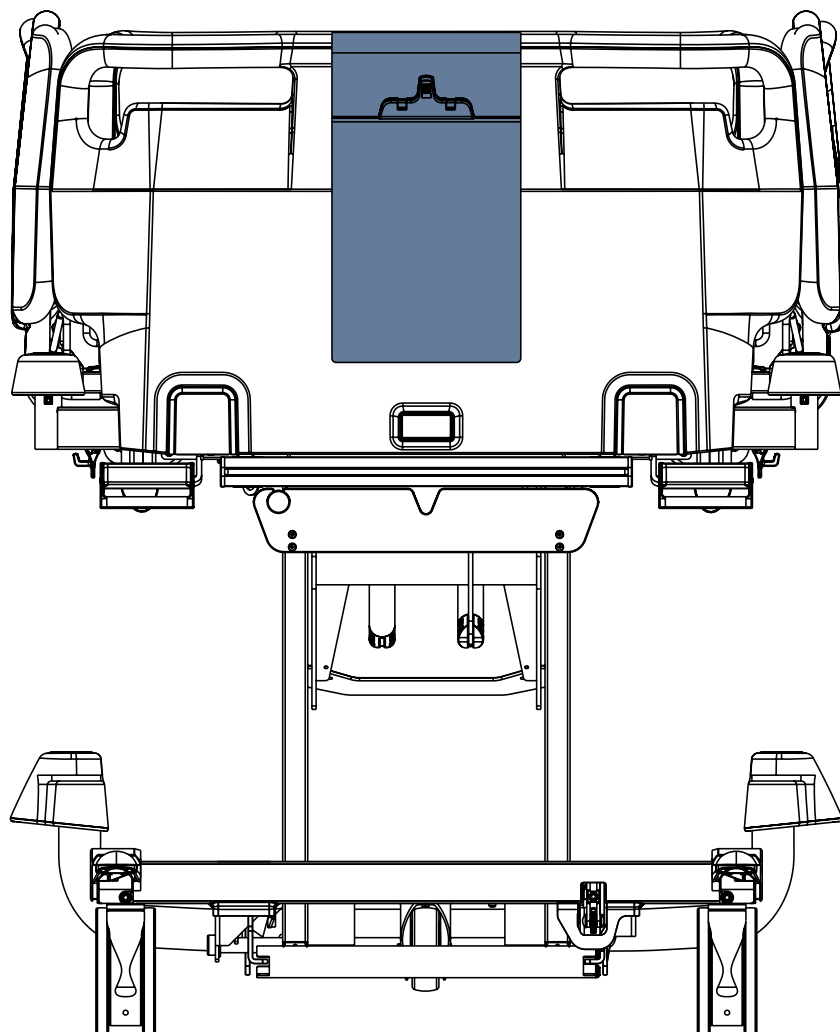


*Fig. Chart Holder on the Plastic Foot Board*

## 4.20 Holder of Small Accessories with Two Hooks

Holder of Small Accessories with Two Hooks is intended to be placed on the upper edge of the Plastic Foot Board with One Safety Lock or Plastic Head Board with One Safety Lock. This holder is intended for a card with patient's name and for patient's things like handbag.

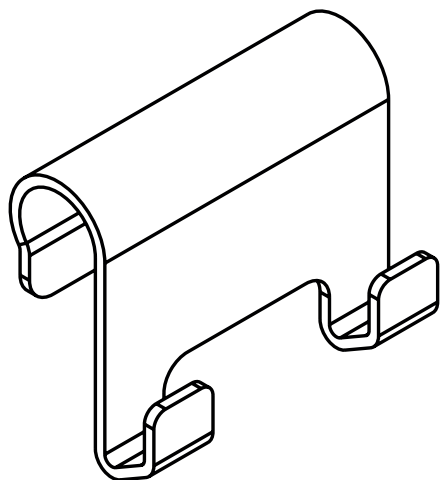

*Fig. Holder of Small Accessories with Two Hooks*

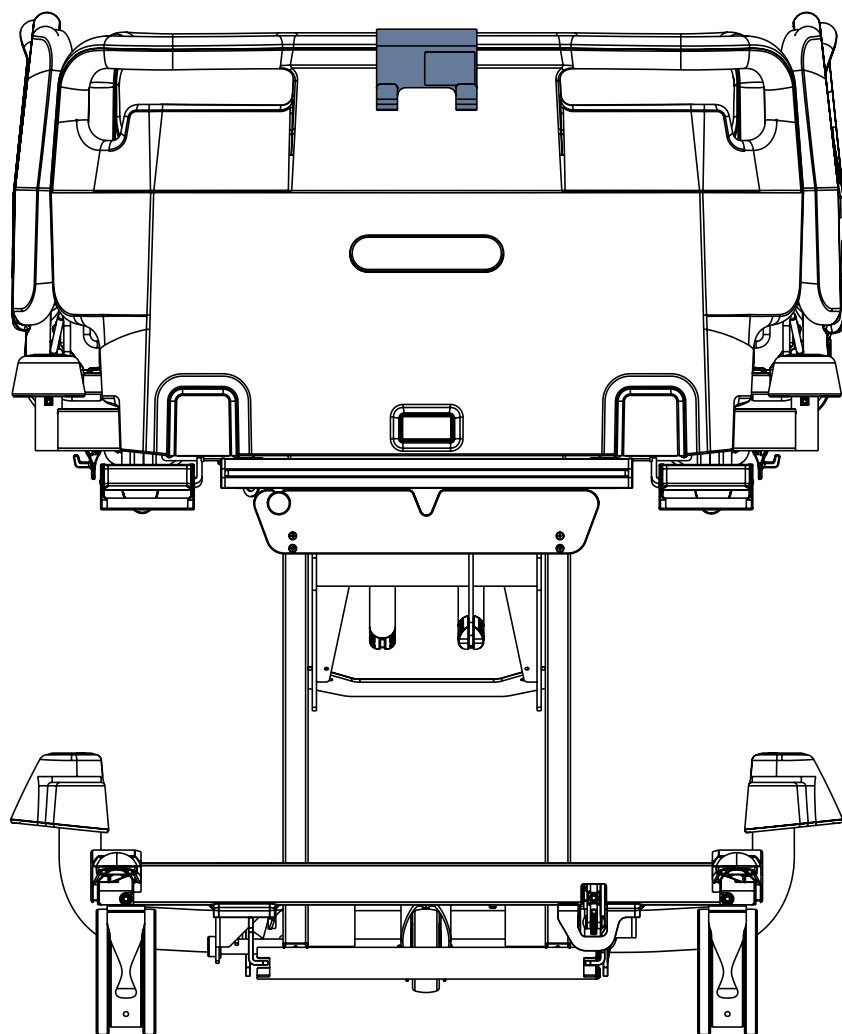

*Fig. Holder of Small Accessories with Two Hooks on the Plastic Foot Board with One Safety Lock*

## 4.21 Positioning Cushion

Positioning Cushion is intended to provide additional support for a patient during lateral tilting of the Essenza 300 Junior<sup>+</sup> bed with Lateral Tilt. Positioning Cushion is intended to be placed on the mattress.



*Fig. Positioning Cushion*

## 4.22 SafeSense 3



### **WARNING!**

**Risk of injury due to incorrect use!**

- ▶ Follow instructions for use of the SafeSense 3 Monitoring System carefully!



### **CAUTION!**

**Risk of incorrect installation due to incompatibility!**

- ▶ **Essenza 300 Junior<sup>+</sup> bed is compatible with Stand-alone version of the SafeSense® 3 Monitoring System and with SafeSense® 3 Monitoring System in Version for Essenza 300. Essenza 300 Junior<sup>+</sup> bed must be SafeSense® 3 ready for use with SafeSense® 3 in Version for Essenza 300 so the compatibility depends on configuration of the bed.**

### **Purpose:**

The SafeSense® 3 Monitoring System is intended to use for the automated, contactless and unobtrusive continuous mobility detection of patients in bed, bed exit and continuous wetness detection in the context of healthcare and nursing facilities.

### **Description of the SafeSense® 3 Monitoring System in Version for Essenza 300:**

- SafeSense® 3 Hub
- Comb-like part securing connectors connected to the Hub
- Hub Holder for Essenza 300
- SafeSense® 3 Air Pad
- Air Hose
- Cables (Nurse Call cable, LAN cable)
- Wetness Pad with Wetness Pad Cable (optionally)

## 4.23 Traction Frame Z



### WARNING!

Read Instructions for use of the Traction Frame Z before use of this product!



### WARNING!

#### Risk of injury due to incorrect use!

- ▶ There must be no patient in bed when installing the Traction Frame Z!
- ▶ Avoid collisions between the Traction Frame Z and the bed (Foot Board and Backrest) during bed positioning!
- ▶ Avoid collisions between the Traction Frame Z and the other bed accessories!
- ▶ Do not transport the bed equipped with Traction Frame Z with patient or without patient!
- ▶ Do not adjust Lateral Tilt when the bed is equipped with the Traction Frame Z!
- ▶ Adjust bed extension only before placement of the Traction Frame Z into the bed!
- ▶ Ensure the positioning functions are locked not to be used by a patient in the bed with the Traction Frame Z!
- ▶ Respect the Safe Working Load of the bed, of the Traction Frame Z and of its hooks and pulleys!
- ▶ Remove the Traction Frame Z from the bed if it is not needed for treatment!
- ▶ Ensure and check regularly the fixing nodes of the Traction Frame Z are secured in intended positions!
- ▶ Essenza 300 Junior<sup>+</sup> bed with corner bumpers without accessory bushings is compatible with Traction Frame Z only if the bed is equipped with two Adaptors for Traction Frame Z!

### Intended Use

Traction Frame Z is a supporting construction intended for fixation, traction and relief of limbs, spine and pelvis.  
Traction Frame Z is intended for orthopaedic department, surgery department, traumatologic department and for ICU.

### Placement

Traction Frame Z is inserted to the bushings for Lifting Pole at bed head end and to the accessory bushings in the corner bumpers at bed foot end in the case of the Essenza 300 Junior<sup>+</sup> bed with corner bumpers with accessory bushings.  
Traction Frame Z is inserted to the bushings for Lifting Pole at bed head end and to the holes in the two Adaptors for Traction Frame Z at bed foot end in the case of the Essenza 300 Junior<sup>+</sup> bed with corner bumpers without accessory bushings. The Adaptors for Traction Frame Z are inserted and fixed in the bushings for accessories at bed foot end in the case of the Essenza 300 Junior<sup>+</sup> bed with corner bumpers without accessory bushings.


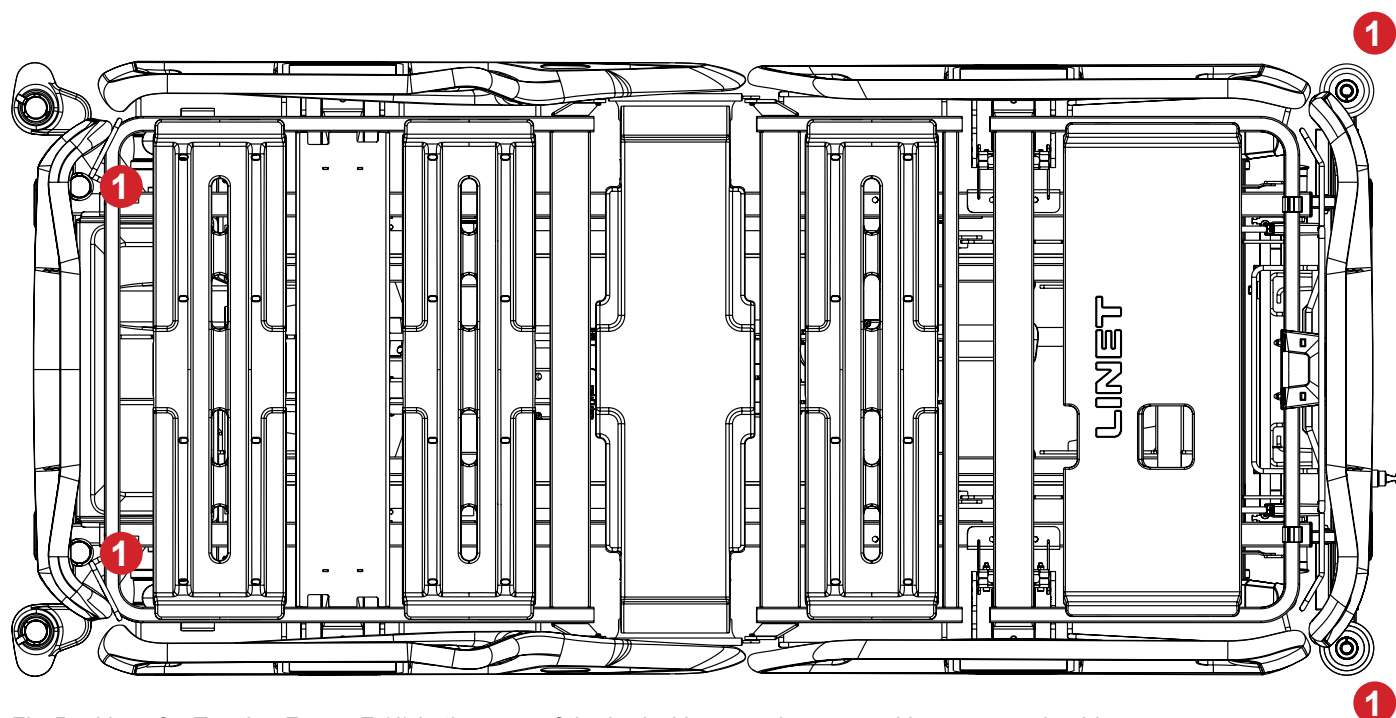


Fig. Positions for Traction Frame Z (1) in the case of the bed with corner bumpers with accessory bushings



*Fig. Traction frame Z without two Adaptors for Traction Frame Z*

## 4.24 Reading Lamp

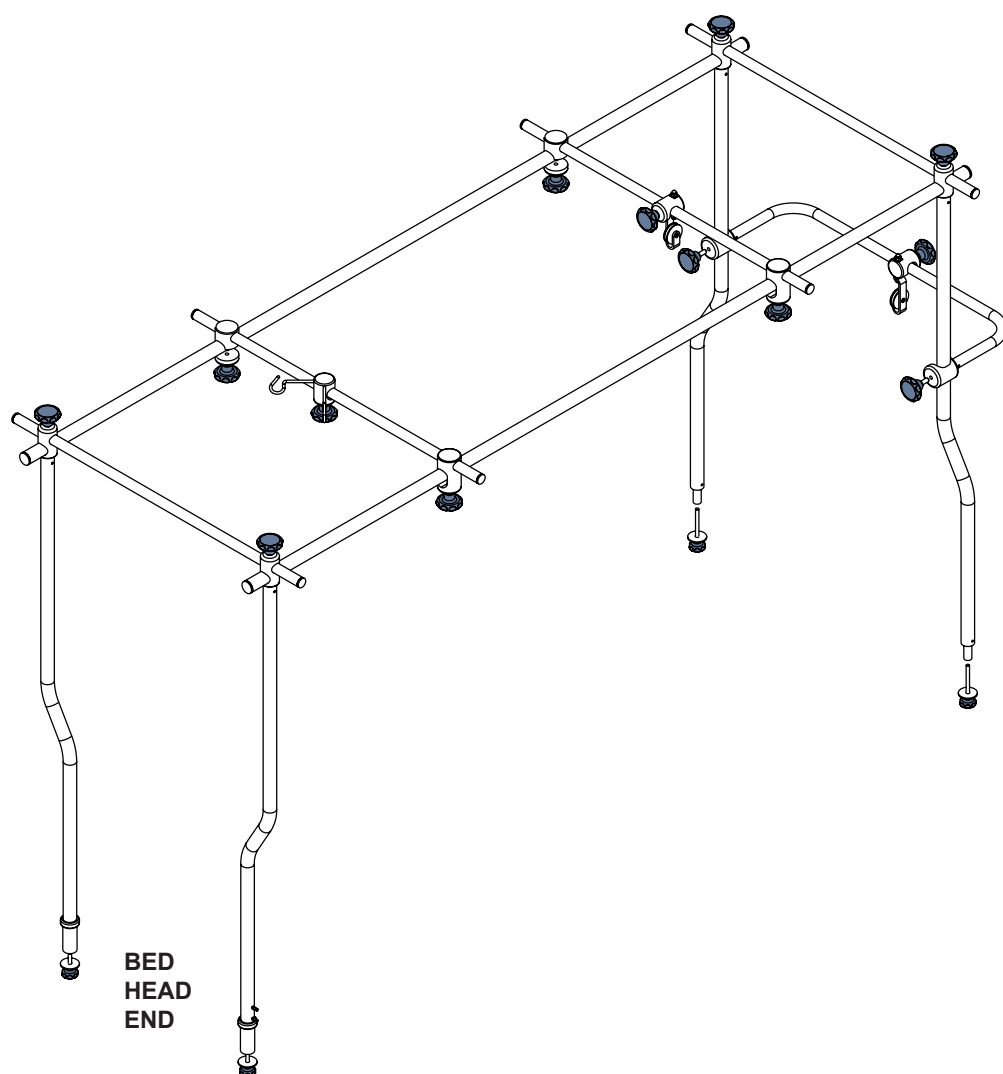

### WARNING!

Read Instructions for use of the Reading Lamp (Series 8) before use of this product!



### WARNING!

#### Risk of injury due to incorrect use!

- ▶ Patient is allowed to use the Reading Lamp only if hospital personnel had assessed that the patient's physical and psychological state is in accordance with use of it and only if the hospital personnel had trained the patient in accordance with the instructions for use! Hospital personnel is responsible for plugging the Reading Lamp into the mains.
- ▶ Never cover the Reading Lamp up!
- ▶ Never look directly into the light! Bright light can damage your eyes.
- ▶ Do not use any explosive or flammable substances near the Reading lamp to avoid the fire and explosion hazard! Ensure the top of the Reading Lamp is at least 15 cm away from the other objects!



### CAUTION!

#### Risk of material damage due to the incorrect power cable placement!

- ▶ Hospital personnel is responsible for running the Reading Lamp power cable between bed head board and siderail. Do not place the power cable under the head board or over the head board! Ensure the power cable is not squeezed by siderail or by parts of the mattress support platform!

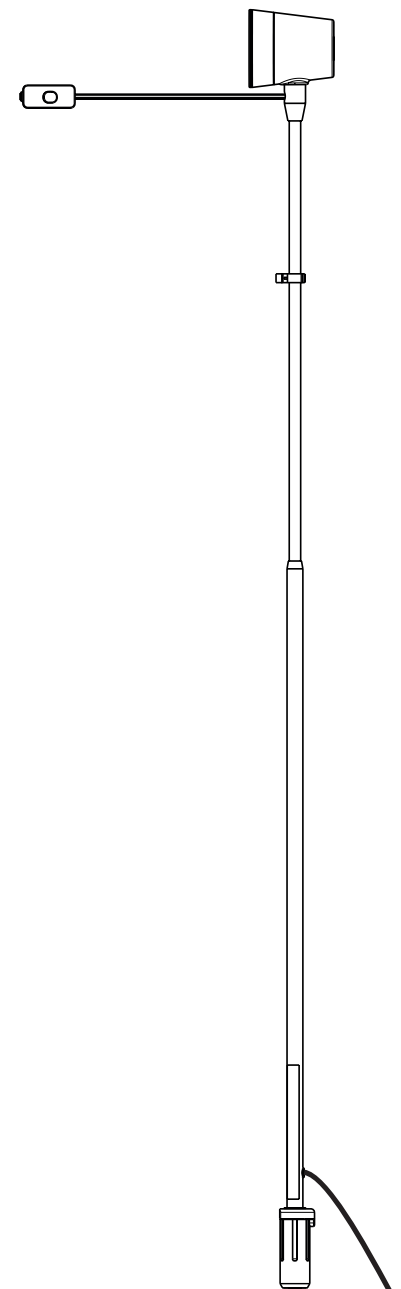


Fig. Reading Lamp

Reading Lamp is intended for patients to enable them to read in bed.

Reading Lamp must be placed in the undercarriage Accessory Adaptors or in the bushings at head end of the mattress support platform.

D9U00AEZJ-0101\_00

## 4.25 Traction Frame D Holder for Foot Board with One Safety Lock



### WARNING!

Read Instructions for use of the Traction Frame D before use of this product!



### WARNING!

#### Risk of injury due to incorrect use!

- ▶ Do not transport the bed equipped with Traction Frame D with patient or without patient!
- ▶ Do not adjust Reverse-Trendelenburg Tilt when the bed is equipped with the Traction Frame D Holder!
- ▶ Ensure the bed Foot Board is secured in the selected bed extension position to avoid a risk of any unintended movement of the Foot Board!
- ▶ Ensure the positioning functions are locked not to be used by a patient in the bed with the Traction Frame D!
- ▶ Remove the Traction Frame D and Traction Frame D Holder from the bed if it is not needed for treatment!
- ▶ Ensure and check regularly the fixing nodes of the Traction Frame D and of the Traction Frame D Holder. The rosette screws must be secured in intended positions!

Traction Frame D Holder for Foot Board with One Safety Lock is intended for correct placement of the Traction Frame D on the Essenza 300 Junior\* bed with corner bumpers with accessory bushings.

Traction Frame D Holder for Foot Board with One Safety Lock is inserted to the accessory bushings in the corner bumpers at bed foot end.


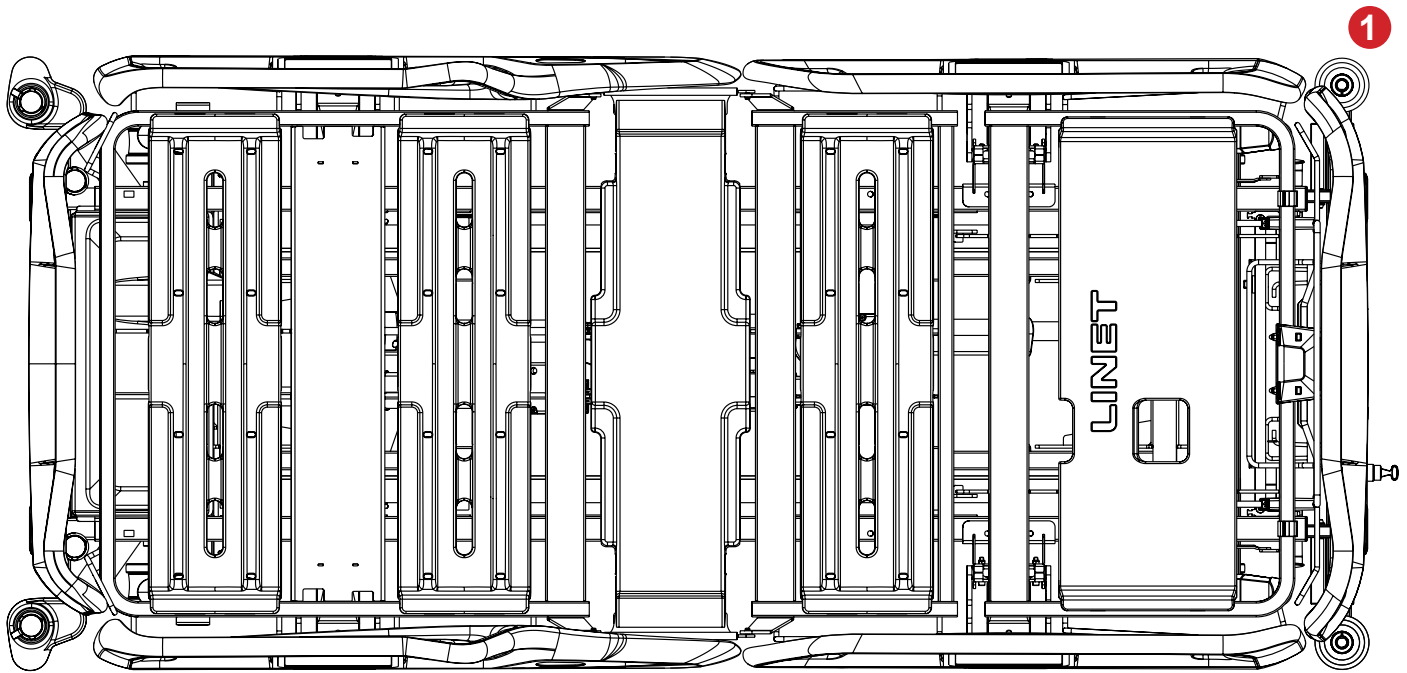


Fig. Positions of the Traction Frame D Holder for Foot Board with One Safety Lock (1) on the bed with corner bumpers with accessory bushings

### To place the Traction Frame D Holder on a compatible Essenza 300 Junior\* bed:

- ▶ Ensure the Traction Frame D Holder is correctly oriented so that both horizontal bars are accessible from the outer side of the bed foot end.
- ▶ Insert both pins of the Traction Frame D Holder into the corner bumpers with accessory bushings at bed foot end.
- ▶ Tighten both rosette screws to fix the pins.

Correctly installed Traction Frame D Holder is leant out of the bed foot end.

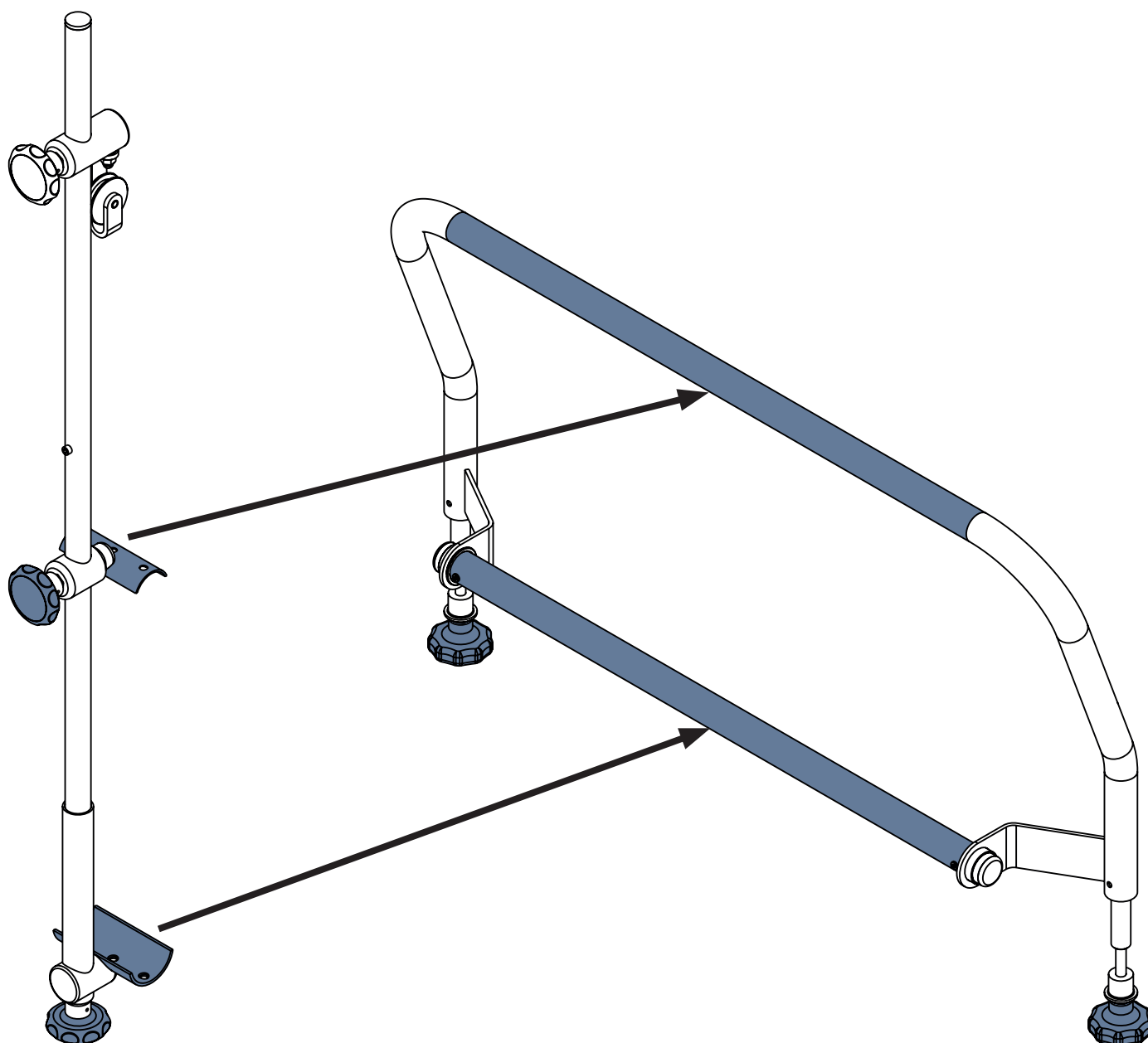

*Fig. Placement of the Traction Frame D onto the Traction Frame D Holder for Foot Board with One Safety Lock*

**To place the Traction Frame D onto the Traction Frame D Holder:**

- ▶ Release the lower rosette screw of the Traction Frame D.
- ▶ Fix the lower Traction Frame D part to the lower horizontal bar of the Traction Frame D Holder by the lower rosette screw of the Traction Frame D.
- ▶ Release the upper rosette screw of the Traction Frame D.
- ▶ Fix the upper Traction Frame D part to the upper horizontal bar of the Traction Frame D Holder by the upper rosette screw of the Traction Frame D.
- ▶ Ensure the selected position of the Traction Frame D is in accordance with an intended support of the patient.

## 4.26 Traction Frame T73



### WARNING!

Read Instructions for use of the Traction Frame T73 before use of this product!



### WARNING!

#### Risk of injury due to incorrect use!

- ▶ There must be no patient in bed when installing the Traction Frame T73!
- ▶ Avoid collisions between the Traction Frame T73 and the bed (Foot Board and Backrest) during bed positioning!
- ▶ Avoid collisions between the Traction Frame T73 and the other bed accessories!
- ▶ Do not transport the bed equipped with Traction Frame T73 with patient or without patient!
- ▶ Do not adjust Lateral Tilt when the bed is equipped with the Traction Frame T73!
- ▶ Adjust bed extension only before placement of the Traction Frame T73 into the bed!
- ▶ Ensure the positioning functions are locked not to be used by a patient in the bed with the Traction Frame T73!
- ▶ Respect the Safe Working Load of the bed, of the Traction Frame T73 and of its hooks, pulleys or handles!
- ▶ Remove the Traction Frame T73 from the bed if it is not needed for treatment!
- ▶ Ensure and check regularly the fixing nodes of the Traction Frame T73 are secured in intended positions!

### Intended Use

Traction Frame T73 is a supporting construction intended for fixation, traction and relief of limbs, spine and pelvis. Traction Frame T73 is intended for orthopaedic department, surgery department, traumatologic department and for ICU.

### Placement

Traction Frame Z is inserted to the bushings for Lifting Pole at bed head end and to the accessory bushings in the corner bumpers at foot end of the Essenza 300 Junior<sup>+</sup> bed with corner bumpers with accessory bushings.


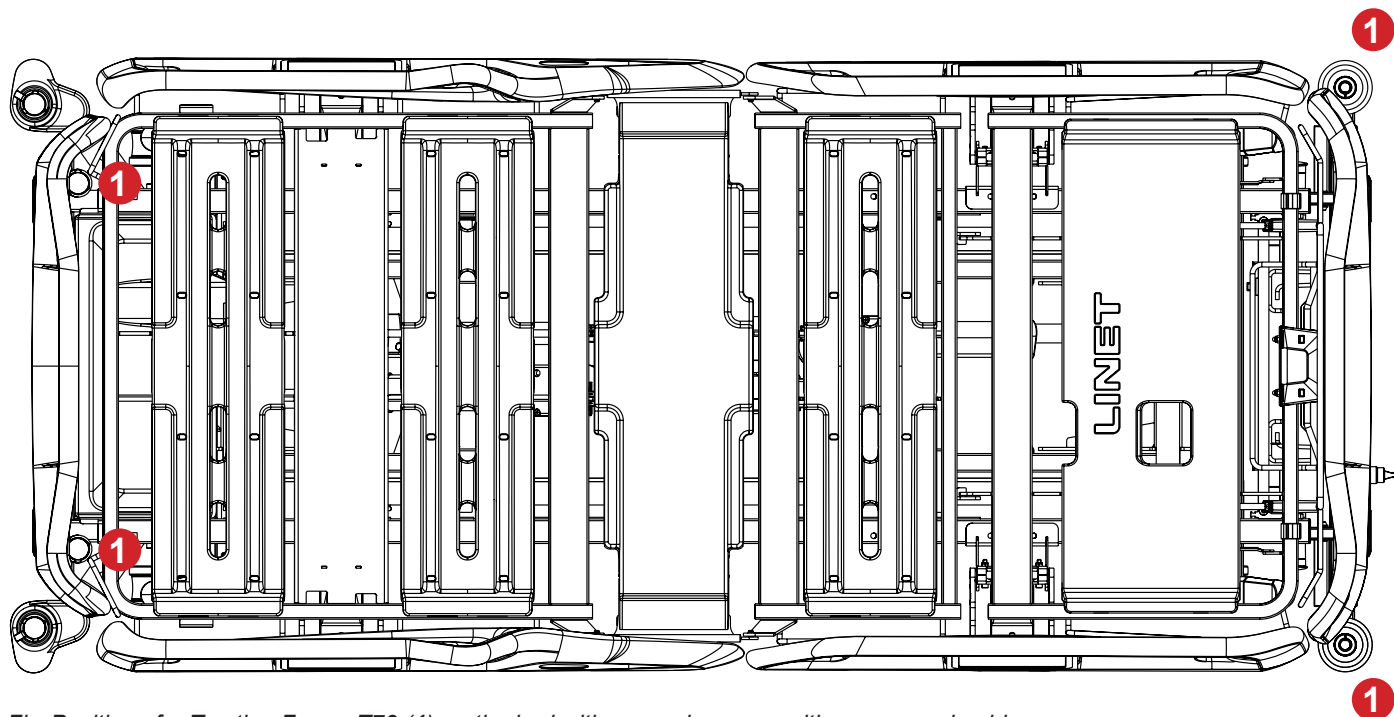


Fig. Positions for Traction Frame T73 (1) on the bed with corner bumpers with accessory bushings



*Fig. Traction frame T73*

## 4.27 Partial Traction Frame MP for Foot Board with One Safety Lock

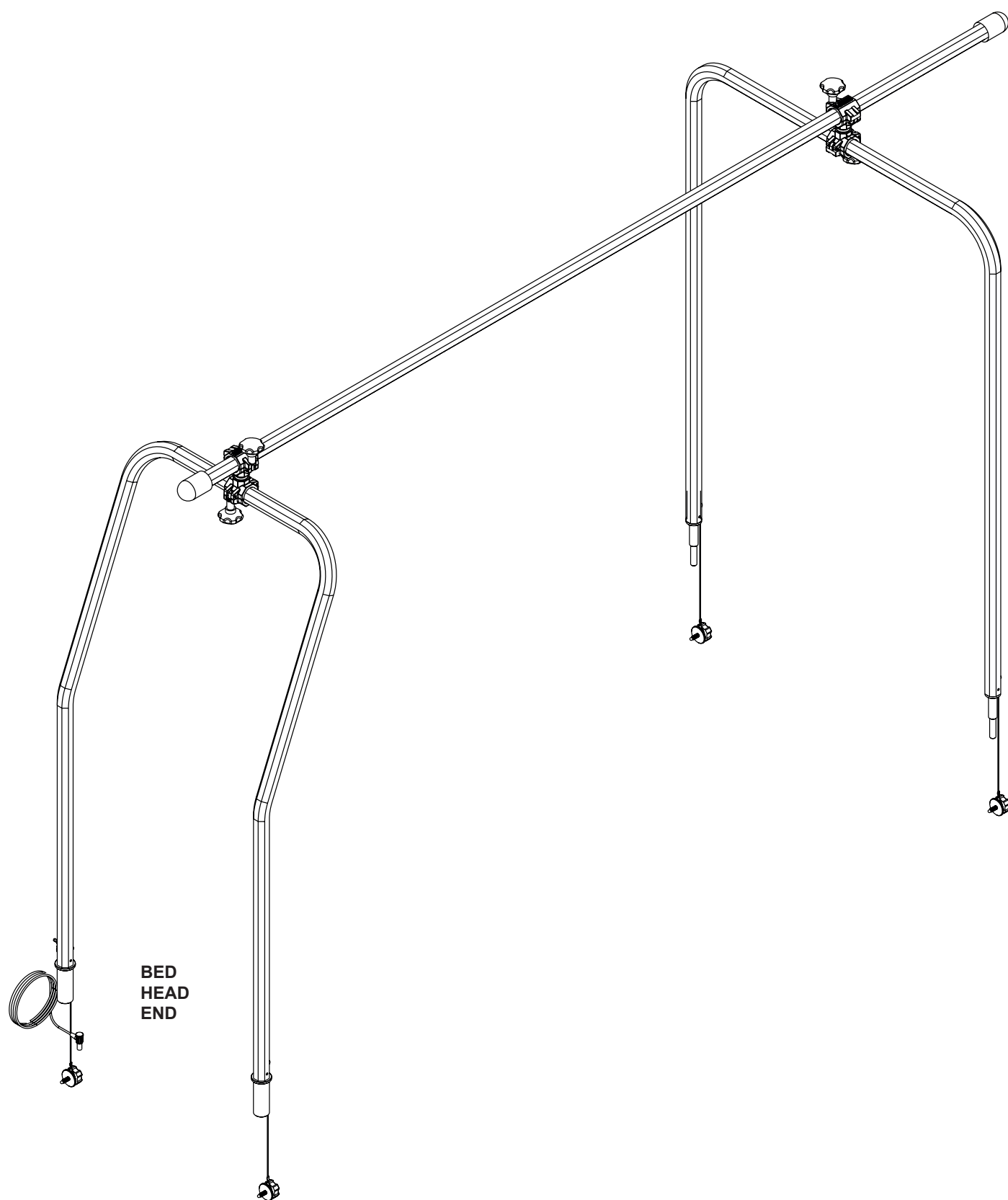

### WARNING!

Read Instructions for use of the Partial Traction Frame MP for Foot Board with One Safety Lock (MP Essenza 300 S0002789 - Partial assembling) before use of this product!



### WARNING!

#### Risk of injury due to incorrect use!

- ▶ Avoid collisions between the Partial Traction Frame MP and the other bed accessories!
- ▶ Do not transport the bed equipped with Partial Traction Frame MP with patient or without patient!
- ▶ Do not adjust Lateral Tilt when the bed is equipped with the Partial Traction Frame MP!
- ▶ Adjust bed extension only before placement of the Partial Traction Frame MP into the bed!
- ▶ Ensure the positioning functions are locked not to be used by a patient in the bed with the Partial Traction Frame MP!
- ▶ Respect the Safe Working Load of the bed and of the Partial Traction Frame MP!
- ▶ Remove the Partial Traction Frame MP from the bed if it is not needed for treatment!
- ▶ Ensure and check regularly the fixing nodes of the Partial Traction Frame MP are secured in intended positions!

### Intended Use

Partial Traction Frame MP for Foot Board with One Safety Lock is a supporting construction intended for fixation, traction and relief of limbs, spine and pelvis.

Partial Traction Frame MP for Foot Board with One Safety Lock is intended for orthopaedic department, surgery department, traumatologic department and for ICU.

### Placement

Partial Traction Frame MP for Foot Board with One Safety Lock is inserted to the accessory bushings in the corner bumpers at foot end of the Essenza 300 Junior<sup>®</sup> bed with corner bumpers with accessory bushings.


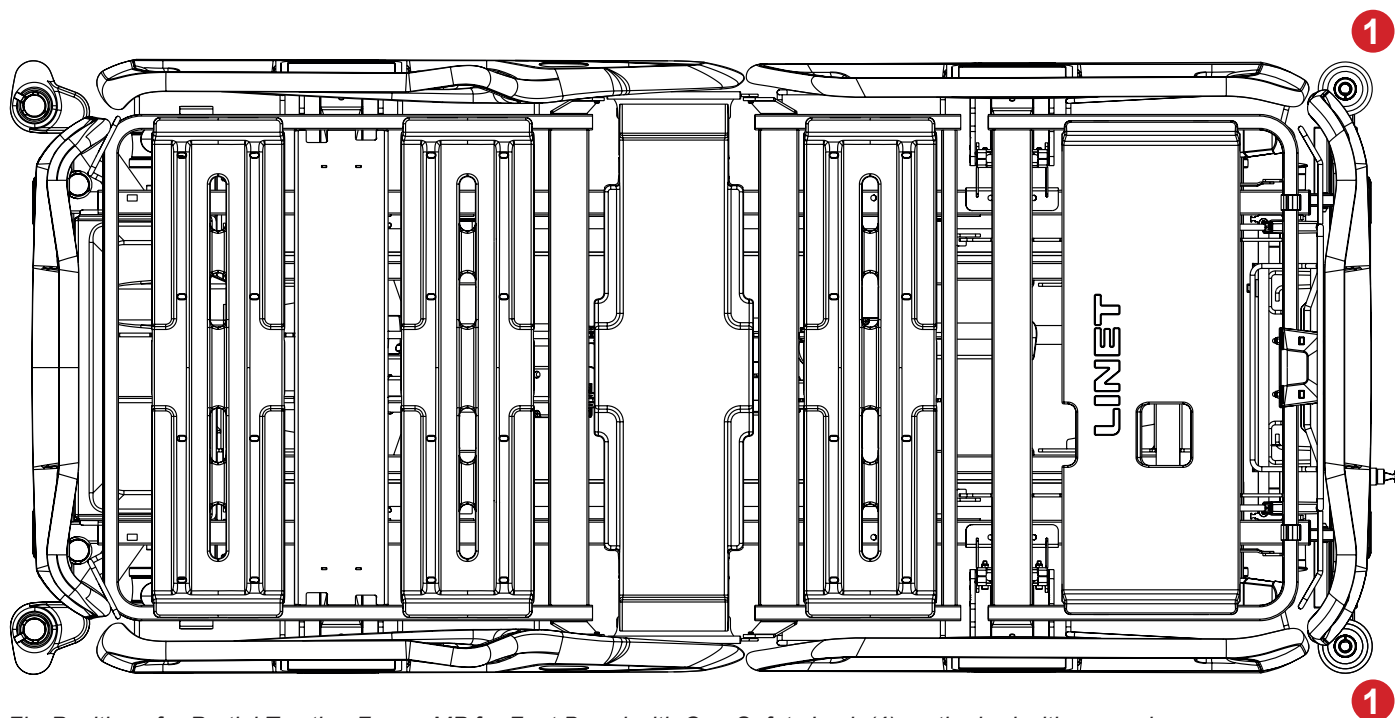


Fig. Positions for Partial Traction Frame MP for Foot Board with One Safety Lock (1) on the bed with corner bumpers with accessory bushings

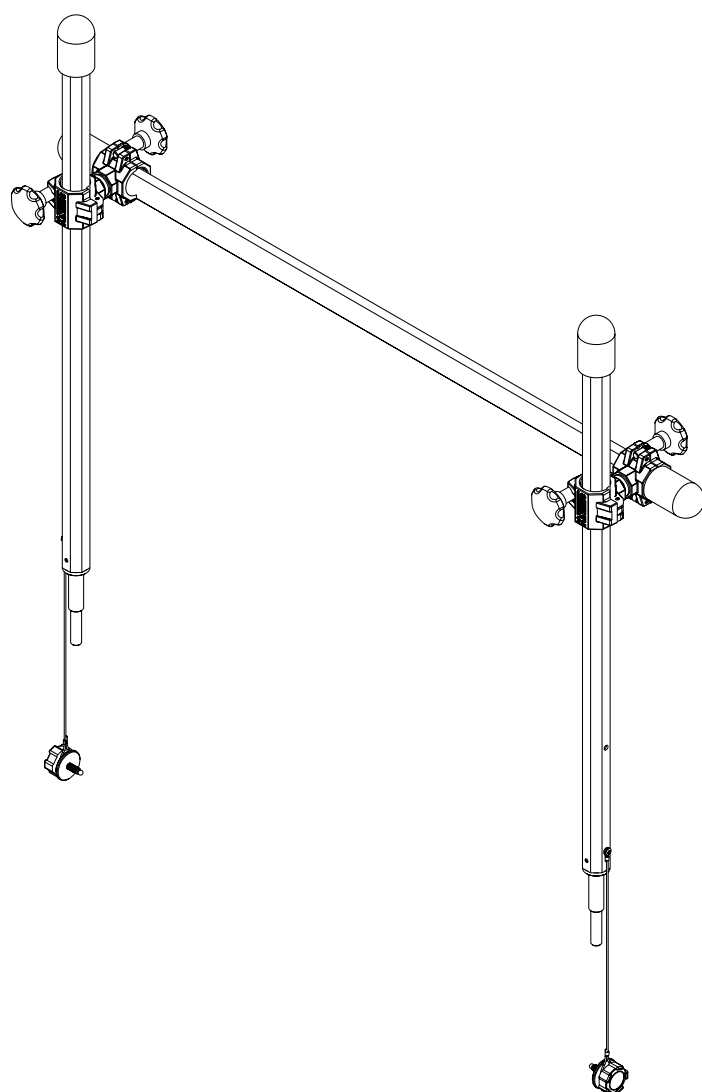

**To place the Partial Traction Frame MP for Foot Board with one Safety Lock at bed foot end:**

- ▶ Ensure the Partial Traction Frame MP is correctly oriented so that the horizontal bar is fixed before the both vertical bars on the outer side of the bed foot end.
- ▶ Insert both pins of the Partial Traction Frame MP into the corner bumpers with accessory bushings at bed foot end.
- ▶ Tighten both rosette screws to fix the pins.

*Fig. Partial Traction Frame MP for Foot Board with One Safety Lock*

## 4.28 Cross Infusion Pump Holder



### WARNING!

#### Risk of injury due to incorrect use!

- ▶ Ensure and check regularly the fixing nodes of the Cross Infusion Pump Holder are secured in intended positions!
- ▶ Avoid collisions between the Cross Infusion Pump Holder and the other bed accessories!
- ▶ Do not overtighten the infusion pump clamps during installation. Overtightening may damage the Cross Infusion Pump Holder.
- ▶ Infusion pump can only be used if the Cross Infusion Pump Holder is fitted into a bushing at head end of the mattress support platform and the infusion pump is correctly attached to the Cross Infusion Pump Holder.
- ▶ Do not use the Cross Infusion Pump Holder as driving or pushing device during the bed transport.
- ▶ Ensure the Cross Infusion Pump Holder will not collide with any movable parts of the bed (especially bed Backrest) or with the patient. This must be verified during installation.

Cross Infusion Pump Holder is intended for holding a compatible infusion pump.  
Cross Infusion Pump Holder must be placed into a bushing for Lifting Pole at bed head end.

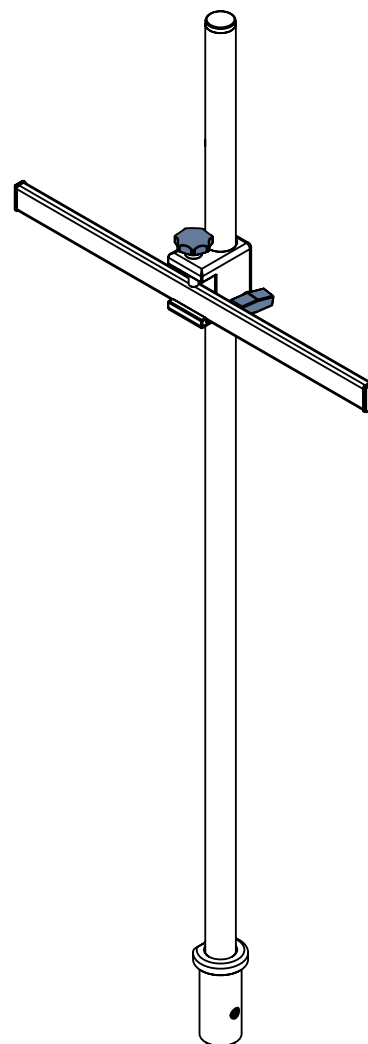


Fig. Cross Infusion Pump Holder

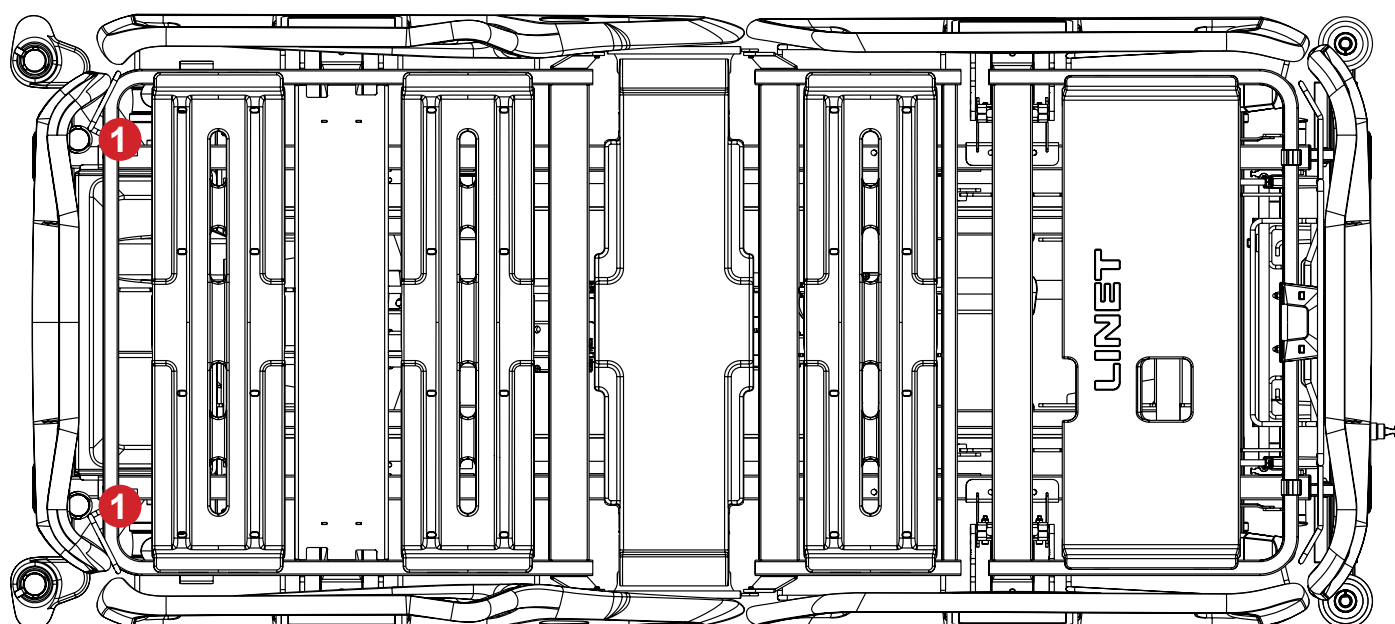


Fig. Bushings for the Cross Infusion Pump Holder (1)

## 5 Disposal

### 5.1 Environment Protection

The company LINET® is aware of the importance of environmental protection for future generations. Within this company the environmental management system is applied in accordance with the internationally agreed standard ISO 14001. The compliance with this standard is annually tested by the external audit executed by an authorised company. Based on the **WEEE** - Waste, Electric and Electronic Equipments Directive the company LINET, s. r. o. is registered in the List of Electric and Electronic Equipment Producers (**Seznam výrobců elektrozařízení**) on the Ministry of the Environment of the Czech Republic (Ministerstvo životního prostředí).

Materials used in this product and in the LINET® accessories are not environmentally hazardous. LINET® products and LINET® accessories meet valid requirements of national and European legislation in the areas of **RoHS** and **REACH**, so they do not contain any prohibited substances in excess quantities. None of the wooden parts is made of tropical wood (such as mahogany, rosewood, ebony, teak etc.) or made of timber from the Amazon region or from similar rainforests. Product noise (sound pressure level) meets requirements of the regulations for the protection of public health against undesirable effects of noise and vibration in protected interior spaces of buildings. Used packaging materials are in accordance with requirements of the Packaging Act (**Zákon o obalech**). For disposal of packaging materials after installation of products contact your sales representative or manufacturer's customer service about the possibility of a free take-back of packaging through an authorized company (more details on **www.linnet.cz**).

### 5.2 Disposal

Materials used in this product and in the LINET® accessories burden the environment but at the same time the whole range of these materials can be very effectively reused and recycled. Mechanical disassembly of the product and material sorting into the basic waste types (plastic, metal, wooden materials) should be performed after the end of product life. The main objective of the obligations arising from the European Directive on Waste, Electric and Electronic Equipments is to increase the re-use, material recovery and recovery of electric and electronic equipment at the required level, thereby avoiding the production of waste and thereby avoiding the possible harmful effects of hazardous substances contained in electric and electronic equipment on human health and the environment. LINET® electric and electronic equipments that have a built-in battery or accumulator and LINET® accessories are designed so that the used batteries or accumulators can be safely removed by LINET® qualified service technicians. There is an information about its type on the built-in battery or accumulator.

#### 5.2.1 Within Europe

**To dispose of the electric and electronic equipment including LINET® accessories:**

- ▶ The electric and electronic equipment must not be disposed of as household waste.
- ▶ Dispose of this equipment at designated collection points or take-back points.
- ▶ Dispose of the product or its components or its accessories in accordance with local laws and regulations!
- ▶ Hire an approved waste disposal company for disposal!

**To dispose of the other equipment including LINET® accessories:**

- ▶ The equipment must not be disposed of as household waste.
- ▶ Dispose of this equipment at designated collection points or take-back points.

LINET® participates in a collective system with take-back company REMA System. By bringing electric and electronic equipment to a take-back point, you participate in recycling and you save primary raw material resources while protecting your environment from effects of unprofessional disposal.

#### 5.2.2 Outside Europe

- ▶ Dispose of the product or its components or its accessories in accordance with local laws and regulations!
- ▶ Hire an approved waste disposal company for disposal!