

USER'S GUIDE

JORDAN A



RESI[®]
www.resi.cz

Manufacturer: RESI Třeboň spol. s r.o., Novohradská 1153, 379 01 Třeboň

Medical device options:
JORDAN A1, A3, A wood, AS

Medical device dedicated purpose: The JORDAN A tables are intended for usage in medical facilities as examination couches, for basic examination. They can also be used as rehabilitation couches for patients in rehabilitation facilities, they are used for physical therapy under continuous supervision of medical personnel. It is suitable for massage and spa procedures. The JORDAN tables can be used for ambulance surgery operations without narcosis, when the patient is treated in the state of consciousness.

Important notice - the JORDAN table is not suitable for explosive environment. The device is not suitable for usage with the presence of inflammable anaesthetics and air mixtures, or inflammable anaesthetics and oxygen mixtures, or monoxide nitrogen. Usage of explosive anaesthetics is dangerous.

Risk class: In terms of § 6, par. 1, letter a) of Act No. 268/2014 Coll., on medical devices, and in terms of Act No. 634/2004 Coll., on administrative fees, as amended, the medical device set forth above is classified in compliance with Attachment No. 9 of the Government Regulation No. 54/2014 Coll. in the Risk Class I.

Medical device sterility: The medical device indicated above is put in the market as a NON-STERILE device.

Measuring features: The medical device indicated above is put in the market as a NON-MEASURING device.

Specifications:	
Table weight	approx. 35 kg
Basic dimensions *	200 x 65 cm
Maximum couch load	180 kg

* The basic dimensions may differ according to requirements of customer.

Couch installation steps:

- 1. Remove the packaging materials.
- 2. Put the head section raiser to horizontal position by holding the control lever. (JORDAN A3).



- 3. Check the functionality of the headrest by moving it upwards. The headrest has 5 different positions.



The headrest can be returned to the horizontal position by raising it to the top dead centre, then lowering it (JORDAN A1, A wood). The headrest is designed for a maximum load of 60 kg. If overloaded, the piston may sustain damage or impairment.



- 4. Adapt to the floor unevenness using the levelling screw.

Couch operation:

JORDAN A1




JORDAN A3



JORDAN A wood




JORDAN AS

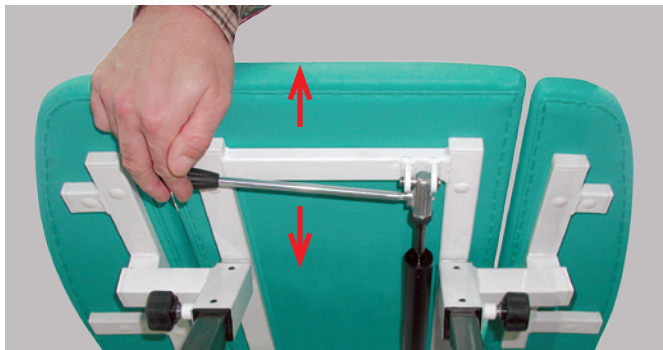


The head section raiser is set to the required position by lifting and holding the lever mechanism. After the lever is released, the head section raiser is locked in the current position.


The head section raiser side parts are lowered by releasing the fly screw, setting the side part to the required position, and tightening the fly screw.

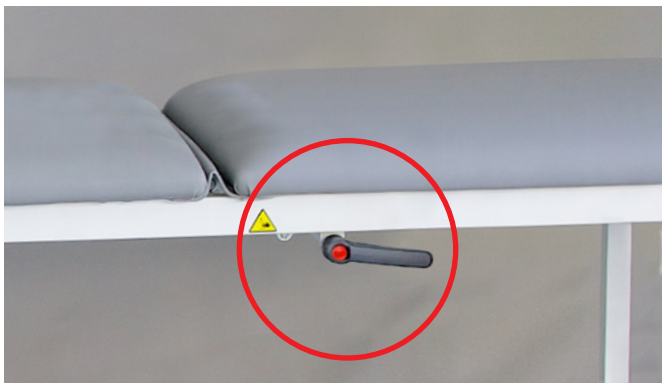

The foot segment slope is set by rotating the lever aside, and by releasing the lever after the required position is set. Maximum load of the set foot part is 60 kg. (Do not sit down!)

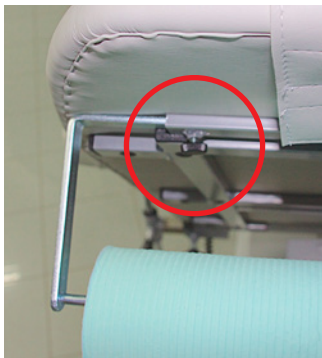

Release the holder slider by loosening the fly nut, and adjust the paper size.

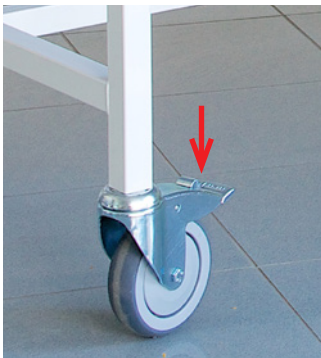

The rolling casters can be secured by stepping on the brake.

Safety

Safety instructions:

- Do not allow children to use and operate the device.
- Do not overload above the maximum limit of 180 kg.
- Do not sit on the head section raiser with the maximum load of 60 kg, in order to prevent from injury or couch permanent damage.

Potential failures and troubleshooting:

- The head section raiser does not keep its position even under a small load: you can set the spring pressure and the spring resistance force by rotating the gas piston body clockwise. When the failure persists, contact the service.

Special conditions for storage and maintenance:

Good quality coaches require proper care to prevent from preliminary devaluation.

Keep the coaches away from excessive moisture and heat.

Protect the upholstery against shocks from sharp edges of other furniture, and against sunshine.

Clean the dust away with wet cloth.

You can disinfect the couch parts with INCIDIN LIQUID disinfection agent, which is delivered with the couch including a sprayer. You can also order these disinfection agents later from RESI Třeboň spol. s r.o. company.

You can remove contamination from all parts of the couch using detergents, such as Corona, Universal, etc., with sponge or cloth. Do not use any organic solvents for the cleaning (benzol, trichlorethylene, acetone, etc.).

In order to treat chromed parts, use Neoxyd, Silichrom, etc.

Alerts and other important warnings:

Guarantee, guarantee conditions:

Providers (of health care) are bound to assure the following 2-year guarantee conditions:

- Install, use, and maintain medical devices for the intended purpose, and in compliance with operating rules (user's guide) of the manufacturer, and regulations for occupational and health protection.
- Keep the operating documentation of the medical device.
- The manufacturer sets the obligation to perform a safety/technical inspection at least every 12 months, provided by the manufacturer.

 The couch lifetime is max. 10 years since the delivery date. After this period, complete audit of the couch must be performed by the manufacturer. The couch may not be used within the health service without the audit!! The manufacturer performs the audit based on the order by the medical device user. Based on the complete audit, the couch lifetime is extended by 5 years.

Caution!

If the instructions included in this user's guide are not observed, the manufacturer is not liable for damages, if any, resulting from incompetent operation or treatment.

User's guide version number: 1.1

User's guide publication date: 1. 4. 2015



RESI[®]
www.resi.cz

RESI Třeboň spol. s r. o.

Novohradská 1153, 379 01 Třeboň, Czech Republic

Tel.: +420 384 724 292, Fax: +420 384 724 293